# FIT KIT for WalkOn / WalkOn Trimable

Quality for life

# FIT KIT for WalkOn / WalkOn Trimable

<b>DE</b>	Gebrauchshinweise .....	4
<b>EN</b>	Instruction folder .....	6
<b>IT</b>	Vedere allegato .....	8
<b>ES</b>	Carpeta de instrucciones .....	10
<b>NL</b>	Instructiefolder .....	12
<b>NO</b>	Bruksanvisning .....	14
<b>SV</b>	Instruktionsfolder .....	16
<b>DA</b>	Brugsanvisning .....	18
<b>CS</b>	Pokyny pro použití .....	20
<b>FI</b>	Ohjekansio .....	22

Best.-Nr. 28T1N

## WalkOn FIT KIT für 28U11 WalkOn / 28U23 WalkOn Trimable

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Art. no. 28T1N

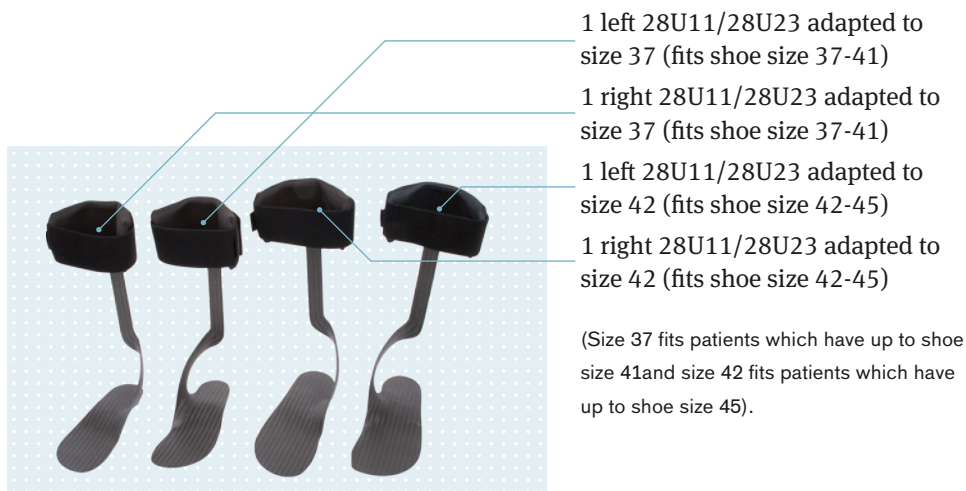
## WalkOn FIT KIT for 28U11 WalkOn / 28U23 WalkOn Trimable

This FIT KIT is testing equipment designed to facilitate the correct choice of a WalkOn AFO (ankle foot orthoses). The objective is to assess if a WalkOn or a WalkOn Trimable is appropriate for the patient. The purpose is not to evaluate the correct size.

The FIT KIT is suited for both WalkOn 28U11 (our classic model) and 28U23 WalkOn Trimable (the classic model designed with a rubber sole attached on the footplate, which makes it easy to customize with the use of a simple pair of scissors).

### The FIT KIT includes four products:

## How to use the FIT KIT?

---

- ▶ Choose a suitable product in terms of size and side for your patient.
  - ▶ Place the AFO in the patient's shoe, preferably under any removable insole.
  - ▶ Apply the shoe and product onto the patient.
  - ▶ Let the patient walk a short distance and evaluate the effectiveness of the AFO.
  - ▶ If the result is positive, fit the patient with the appropriate size of a WalkOn or WalkOn Trimable, depending on whether you prefer to customize the sole with a grinding machine or a pair of scissors.
- 

If the patient prefers a more flexible (softer) AFO, choose a WalkOn Flex, 28U22. (WalkOn Flex FIT KIT, article number. 28T2).

### Note

As the size of the test-model is often not entirely correct, the patient may experience some pressure at the medial malleolus. This is remedied by a correct customized product, grinding or cutting according to our fitting instructions, (see the products IFU, instructions for use).

Art. no	Side	Shoe Size	Height
28U11/28U23=L/R 36-39	Left/Right	36-39	35 cm / 13,8 inch
28U11/28U23=L/R 39-42	Left/Right	39-42	36 cm / 14,2 inch
28U11/28U23=L/R 42-45	Left/Right	42-45	37,5 cm / 14,8 inch
28U11/28U23=L/R 45-48	Left/Right	45-48	39 cm / 15,4 inch

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Otto Bock HealthCare GmbH