



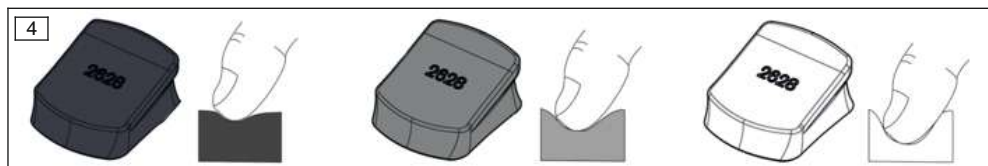
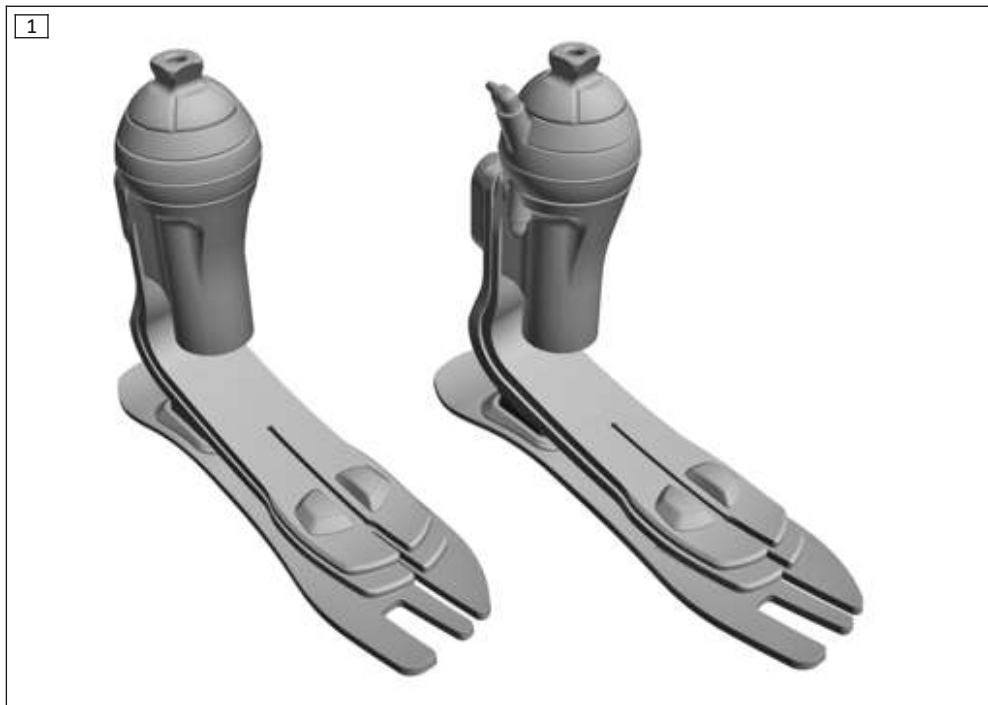
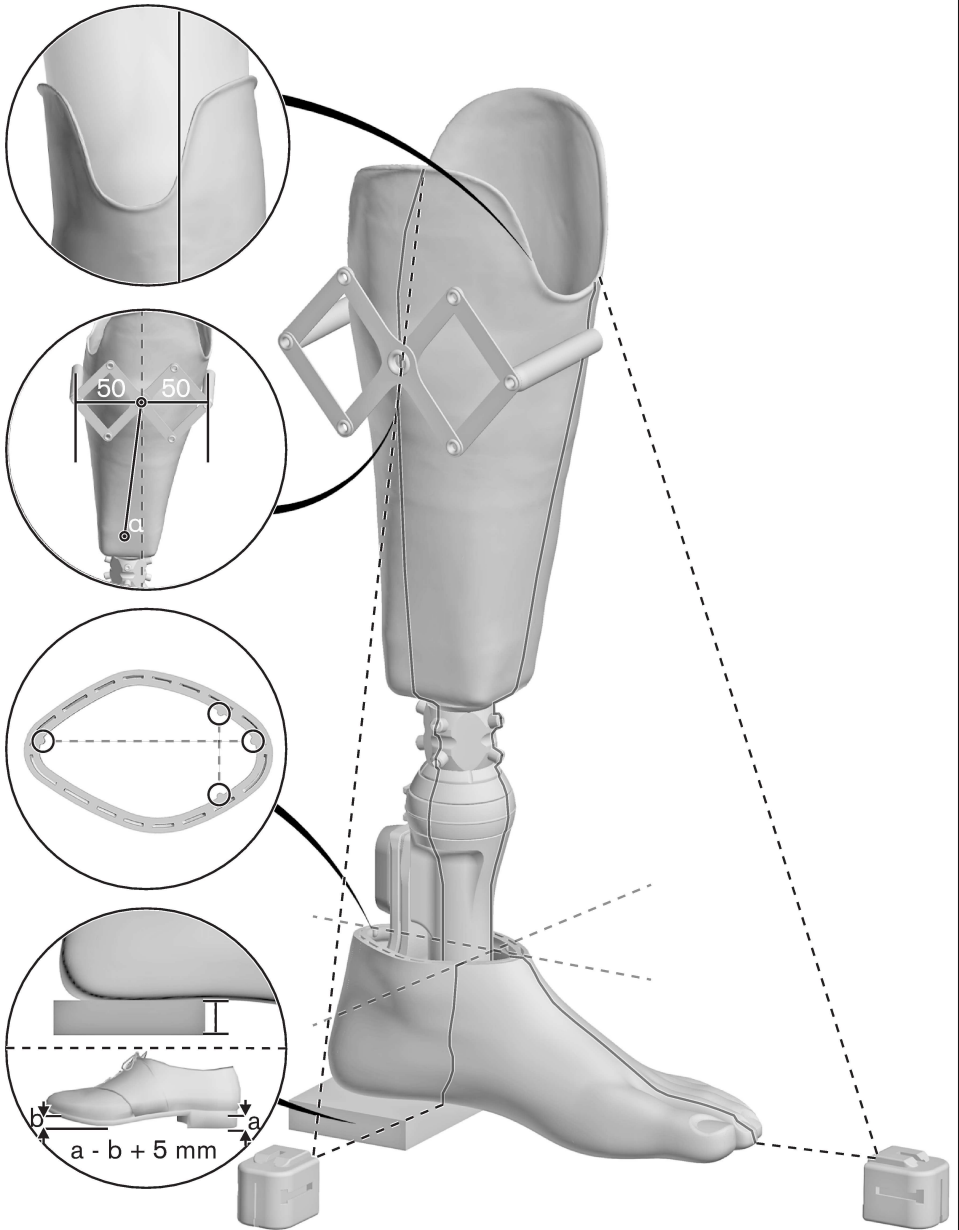
## 1C51 Taleo Vertical Shock, 1C52 Taleo Harmony

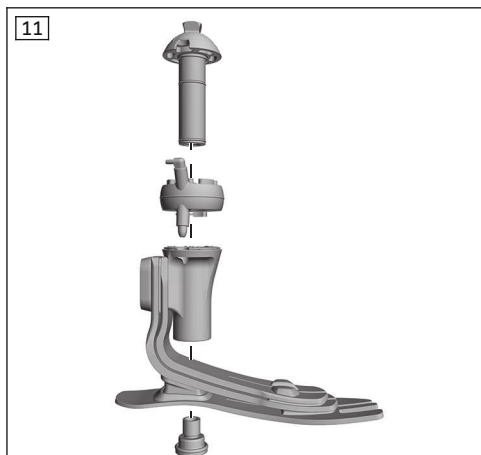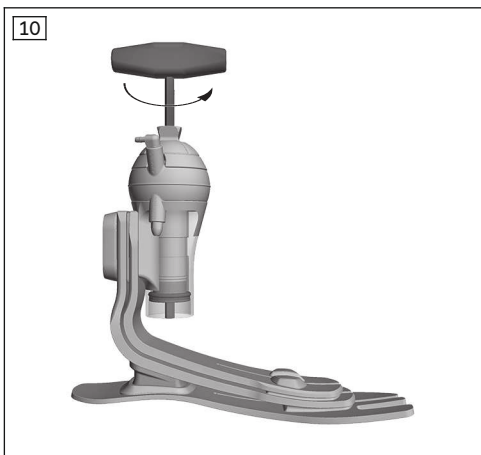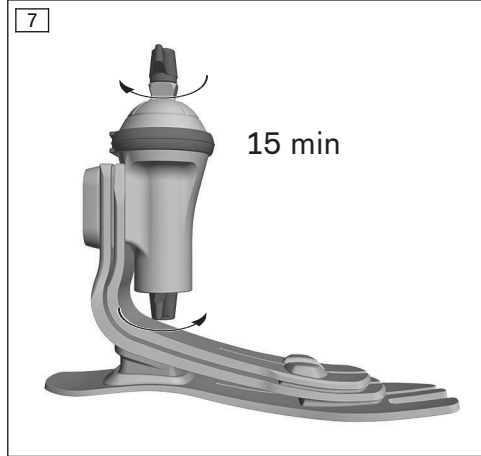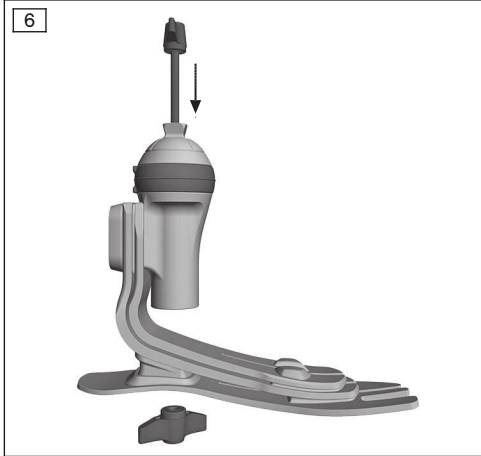
<b>DE</b> Gebrauchsanweisung .....	6
<b>EN</b> Instructions for use .....	16
<b>FR</b> Instructions d'utilisation .....	26
<b>IT</b> Istruzioni per l'uso .....	37
<b>ES</b> Instrucciones de uso .....	48
<b>PT</b> Manual de utilização .....	59
<b>NL</b> Gebruiksaanwijzing .....	70
<b>SV</b> Bruksanvisning .....	81
<b>DA</b> Brugsanvisning .....	91
<b>NO</b> Bruksanvisning .....	101
<b>FI</b> Käyttöohje .....	110
<b>PL</b> Instrukcja użytkowania .....	121
<b>HU</b> Használati utasítás .....	131
<b>CS</b> Návod k použití .....	142
<b>RO</b> Instrucțiuni de utilizare .....	152
<b>HR</b> Upute za uporabu .....	163
<b>SL</b> Navodila za uporabo .....	173
<b>SK</b> Návod na používanie .....	183
<b>BG</b> Инструкция за употреба .....	193
<b>TR</b> Kullanma talimatı .....	205
<b>EL</b> Οδηγίες χρήσης .....	214
<b>RU</b> Руководство по применению .....	226
<b>JA</b> 取扱説明書 .....	238
<b>ZH</b> 使用说明书 .....	247
<b>KO</b> 사용 설명서 .....	256

12
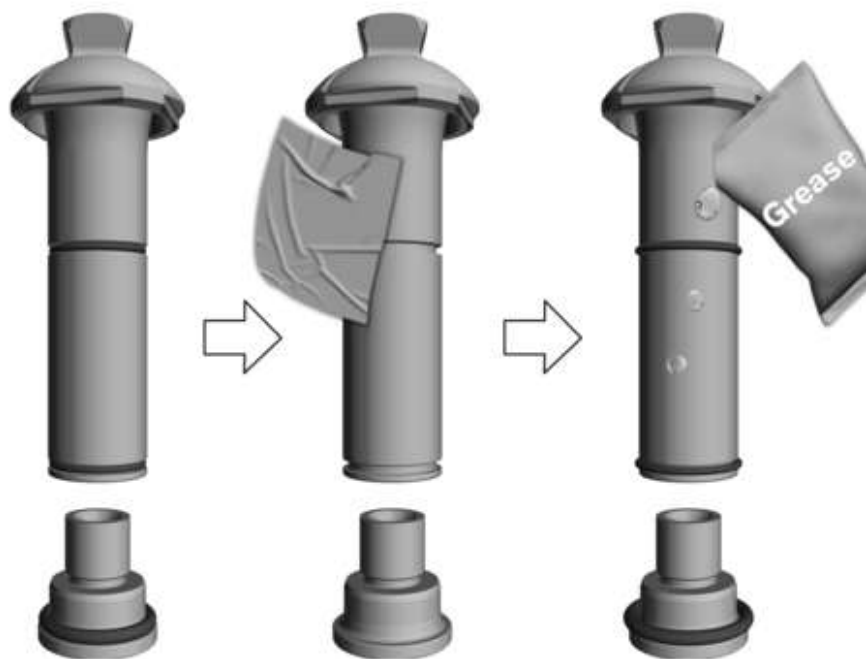
## INFORMATION

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### INFORMATION

Date of last update: 2022-03-16

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

### 1.1 Construction and Function

The 1C51 Taleo Vertical Shock and 1C52 Taleo Harmony prosthetic feet are suitable for walking on various surfaces and at a wide range of speeds. The heel strike is absorbed by the large, replaceable heel wedge.

Carbon and polymer spring elements permit perceptible plantar flexion at heel strike, a natural rollover movement and high energy return.

The prosthetic feet have a functional ring. The functional ring absorbs vertical impact loads and enables slight torsional movements.

The Taleo Harmony is additionally equipped with a mechanical vacuum pump.

While walking, the vacuum pump is compressed and released again, thus generating the vacuum. The elevated vacuum in the prosthetic socket improves the connection between the user and the prosthesis.

### 1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

## 2 Intended use

### 2.1 Indications for use

The product is to be used **solely** for lower limb exoprosthetic fittings.

### 2.2 Area of application

Our components perform optimally when paired with appropriate components based upon weight and mobility grades identifiable by our MOBIS classification information and which have appropriate modular connectors.

The product is recommended for mobility grade 2 (restricted outdoor walker), mobility grade 3 (unrestricted outdoor walker) and mobility grade 4 (unrestricted outdoor walker with particularly high demands).

The table that follows shows the suitable spring stiffness of the prosthetic foot, matching the patient's body weight and activity.

Spring stiffness relative to body weight and activity level		
Body weight [kg]	Normal activity level	High activity level
up to 51	1	2
52 to 58	2	3
59 to 67	3	4
68 to 77	4	5
78 to 88	5	6
89 to 100	6	7
101 to 115	7	8
116 to 130	8	9
131 to 150	9	–

### 2.3 Environmental conditions

Storage and transport
Temperature range –20 °C to +60 °C (–4 °F to +140 °F), relative humidity 20 % to 90 %, no mechanical vibrations or impacts

<b>Allowable environmental conditions</b>
<b>Temperature range:</b> -10 °C to +45 °C (14 °F to 113 °F)
<b>Moisture:</b> relative humidity: 20% to 90%, non-condensing
<b>Chemicals/liquids:</b> fresh water as dripping water, occasional contact with salty air (e.g. near the ocean)
<b>Solids:</b> dust

<b>Prohibited environmental conditions</b>
<b>Chemicals/moisture:</b> salt water, perspiration, urine, acids, soapsuds, chlorine water
<b>Solids:</b> dust in high concentrations (e.g. construction site), sand, highly hygroscopic particles (e.g. talcum)

## 2.4 Lifetime

### Prosthetic foot

Depending on the patient's activity level, the maximum lifetime of the product is 3 years.

### Footshell, protective sock, functional ring and O-rings

The product is a wear part, which means it is subject to normal wear and tear.

## 2.5 Qualification

### Taleo Harmony

The product may only be integrated into the prosthesis by O&P professionals who have completed training as certified Harmony experts.

- **Fabricating the total surface weight-bearing socket:** Fabricating a total surface weight-bearing socket is required for the use of the product.

## 2.6 Absolute Contraindications

### Taleo Harmony

- Interim prosthesis
- Effectively non-weight-bearing residual limb end
- Neuromas in the area of the residual limb
- User limitations which prevent the safe use of the product

## 3 Safety

### 3.1 Explanation of warning symbols

 <b>CAUTION</b>	Warning regarding possible risks of accident or injury.
 <b>NOTICE</b>	Warning regarding possible technical damage.

### 3.2 General safety instructions

#### Risk of injury and risk of product damage

- ▶ Note the combination possibilities/comboination exclusions in the instructions for use of the products.
- ▶ Comply with the product's field of application and do not expose it to excessive strain (see page 17).
- ▶ To avoid the risk of injury and product damage, do not use the product beyond the tested lifetime.
- ▶ To avoid the risk of injury and product damage, only use the product for a single patient.
- ▶ To prevent mechanical damage, use caution when working with the product.
- ▶ If you suspect the product is damaged, check it for proper function and readiness for use.

- ▶ Do not use the product if its functionality is restricted. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).
- ▶ Only use suitable liners (without a textile coating or with a partial textile coating) for prostheses with Harmony vacuum pumps. This prevents skin irritation due to vacuum on the skin.

**NOTICE!**

**Risk of product damage and limited functionality**

- ▶ Check the product for damage and readiness for use prior to each use.
- ▶ Do not use the product if its functionality is restricted. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).
- ▶ Do not expose the product to prohibited environmental conditions.
- ▶ Check the product for damage if it has been exposed to prohibited environmental conditions.
- ▶ Do not use the product if it is damaged or in a questionable condition. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).

**Signs of changes in or loss of functionality during use**

Reduced spring effect (e.g. decreased forefoot resistance or changed rollover behaviour) or delamination of the spring are indications of loss of functionality. Unusual noises can indicate a loss of functionality.

**4 Scope of Delivery and Accessories**

Quantity	Designation	Reference number
1	Instructions for use	-
1	Prosthetic foot	-
1	Protective sock	SL=Spectra-Sock-7
1	Heel wedge set	2F50*
1	Pre-compression kit	-
<b>Only for Taleo Vertical Shock</b>		
1	Functional ring set	2Z362=*
<b>Only for Taleo Harmony</b>		
1	Functional ring set	2Z360=*
1	Socket connector	2R117=0
1	Extension hose for socket connector	755Z39

<b>Additional accessories/spare parts (not included in the scope of delivery)</b>	
Designation	Reference number
Footshell	2C15*
Connection cover	2C19*, 2C20*
Screw cover	2F51=26-30
Pressure gauge with T-piece (Harmony only)	755Z37
One-way valve with filter, five pieces (Harmony only)	2Z361
One-way valve, straight, five pieces (Harmony only)	4R142
One-way valve, 90° angle, five pieces (Harmony only)	4R143
Service set, Harmony P3 (Harmony only)	4X148
Exhaust flange set for foam cover (Harmony only)	4Y383

## 5 Preparing the product for use

### ⚠ CAUTION

#### **Incorrect alignment, assembly or adjustment**

Injury due to incorrectly installed or adjusted as well as damaged prosthetic components

- ▶ Observe the alignment, assembly and adjustment instructions.

### NOTICE

#### **Use of talcum**

Impairment of the vacuum pump due to plugged valves

- ▶ Do not use talcum on the product or other prosthetic components.

### NOTICE

#### **Grinding the prosthetic foot or footshell**

Premature wear resulting from damage to the product

- ▶ Do not grind the prosthetic foot or footshell.

### 5.1 Pre-compressing the functional ring

In order to achieve its full flexibility, the functional ring has to be compressed before the first use.

- 1) Using the pre-compression kit, compress the functional ring for **15 minutes** (see fig. 6, see fig. 7).
- 2) Disassemble the pre-compression kit.

### 5.2 Alignment

#### INFORMATION

There is a plastic protector on the product's pyramid. It protects the connecting section from scratches during the alignment and fitting of the prosthesis.

- ▶ Remove the protector before the patient leaves the workshop/fitting area.

#### 5.2.1 Applying/removing the footshell

#### INFORMATION

- ▶ Pull the protective sock over the prosthetic foot to prevent noises in the footshell.
- ▶ Always use the prosthetic foot with the footshell.

- ▶ Apply or remove the footshell as described in the footshell instructions for use.

#### 5.2.2 Bench Alignment

#### INFORMATION

**Use the prosthetic foot only with an installed heel wedge.**

#### INFORMATION

**For users with a greater need for safety (e.g. mobility grade 2), shift the foot approx. 1 cm further in the anterior direction. This extends the forefoot and increases the knee securing effect.**

## TT bench alignment

Bench alignment process	
<b>Required materials:</b> 662M4 goniometer, 743S12 heel height measuring apparatus, 743A80 50:50 gauge, alignment apparatus (e.g. 743L200 L.A.S.A.R. Assembly or 743A200 PROS.A. Assembly)	
Assemble and align the prosthetic components in the alignment apparatus according to the following specifications:	
①	Heel height: <b>effective heel height</b> (shoe heel height - sole thickness of forefoot) + 5 mm
②	Sagittal alignment: <b>alignment reference line on a-p markings of the footshell (see fig. 5)</b>
③	Frontal alignment: <b>alignment reference line on m-l markings of the footshell (see fig. 5)</b>
④	Connect the prosthetic foot and prosthetic socket using the selected adapters. Follow the adapter instructions for use.
⑤	Sagittal alignment: Determine the centre of the prosthetic socket with the 50:50 gauge. Align the prosthetic socket centrally to the alignment reference line. Socket flexion: <b>individual residual limb flexion + 5°</b>
⑥	Observe the abduction or adduction position.

## TF bench alignment

► Observe the information in the prosthetic knee joint instructions for use.

### 5.2.3 Static Alignment

- Ottobock recommends checking the alignment of the prosthesis using the L.A.S.A.R. Posture and adapting it as needed.
- If necessary, the alignment recommendations (TF modular leg prostheses: **646F219\***, TT modular leg prostheses: **646F336\***) may be requested from Ottobock.

### 5.2.4 Installing the socket connector (Harmony only)

The vacuum pump is connected to the prosthetic socket by a socket connector. The hose can either be considerably shortened or wrapped around the adapter to secure it closely on the prosthesis.

- 1) Install the 2R117 socket connector according to the corresponding instructions for use.
- 2) Shorten the hose of the socket connector to the optimal length and connect it to the vacuum pump (see fig. 8, see fig. 9).

### 5.2.5 Dynamic Trial Fitting

- Adapt the alignment of the prosthesis in the frontal plane and the sagittal plane (e.g. by making angle or slide adjustments) to ensure an optimum gait pattern.
- **TT fittings:** Make sure that physiological knee movement in the sagittal and frontal plane is achieved when the leg begins to bear weight after the heel strike. Avoid medial movement of the knee joint. If the knee joint moves in the medial direction in the first half of the stance phase, move the prosthetic foot in the medial direction. If the medial movement occurs in the second half of the stance phase, reduce the exterior rotation of the prosthetic foot.
- Remove the plastic protector from the pyramid after completing the dynamic fitting and the walking exercises.

#### 5.2.5.1 Optimising the heel characteristics

The behaviour of the prosthetic foot at heel strike and during heel contact in the mid-stance phase can be adapted by replacing the heel wedge. Heel wedges of various hardness are included in the scope of delivery.

**Heel wedge hardness:** The colour of the heel wedge indicates the hardness (see fig. 4). Ottobock recommends starting with the preinstalled heel wedge.

- 1) Pull the prosthetic foot apart slightly and remove the existing heel wedge.
- 2) Align the other heel wedge so the Ottobock lettering is the right way up and the tip points to anterior.
- 3) Insert the heel wedge into the prosthetic foot (see fig. 2).

### 5.2.5.2 Checking the stiffness of the functional ring

The stiffness of the functional ring influences the level of comfort while walking. With the Harmony vacuum pump, the functional ring also determines the efficiency of vacuum generation. The stiffness is specified on the functional ring (the higher the number, the greater the stiffness).

Functional ring while walking	Cause	Correction/solution
Soft damping for the user. The functional ring is not compressed to the stop.	The functional ring has the appropriate stiffness	–
User hits the stop during damping. The functional ring is fully compressed.	Functional ring too soft	Install a harder functional ring (see page 25)
No damping for the user. The functional ring is not compressed.	Functional ring too hard	Install a softer functional ring (see page 25)
	<b>Possible:</b> Heel wedge too soft	Install a harder heel wedge (see page 21)

### 5.2.5.3 Checking vacuum generation (Harmony only)

Vacuum generation is tested during the dynamic fitting.

> **Recommended materials:** 755Z37 pressure gauge (with T-piece)

- 1) Disconnect the hose from the suction connector of the vacuum pump.
- 2) Connect the hose to the T-piece of the pressure gauge.
- 3) Connect the pressure gauge with the T-piece to the intake valve on the vacuum pump.
- 4) Attach the pressure gauge to the prosthesis so that the patient is able to move freely.
- 5) Have the patient walk **50** steps to activate the vacuum pump.
- 6) Check whether the vacuum is between **340 hPa (10 inHG)** and **680 hPa (20 inHG)** and is maintained for **20 seconds**.

If a sufficient vacuum is built up and the vacuum does not decrease within **20 seconds**, the vacuum system is functioning correctly.

If a sufficient vacuum is built up but the vacuum decreases within **20 seconds**, determine the cause (see page 24).

- 7) Remove the pressure gauge after completing the test and reinstall the hose on the suction connection of the vacuum pump.

## 5.3 Installing the outlet hose (Harmony only)

### CAUTION

#### Improper installation of the outlet hose

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ Run the outlet hose in such a way that the hose opening is not directed towards parts of the body or into the cosmetic cover.

A valve with sound suppressor is installed on the outlet of the vacuum pump. The valve has to be removed in order to install an outlet hose (available separately).

- 1) Pull the outlet valve with sound suppressor off the vacuum pump.
- 2) Insert an outlet valve with hose connector into the functional ring. An arrow on the valve indicates the flow direction.
- 3) Install the desired outlet hose according to its instructions for use.

## 5.4 Optional: Installing the foam cover

The foam cover sits between the prosthetic socket and prosthetic foot. It is cut longer in order to compensate for the movements of the prosthetic foot and prosthetic knee joint. During flexion of the prosthetic knee joint, the foam cover undergoes posterior compression and anterior elongation. The foam cover should be stretched as little as possible in order to increase its service life. There is a connecting element (such as a connection plate, connection cap or connection cover) on the prosthetic foot.

> **Required materials:** Degreasing cleaner (e.g. 634A58 isopropyl alcohol), 636N9 contact adhesive or 636W17 plastic adhesive, 4Y383 foam cover connection set

1) **INFORMATION: Add as little length as possible so the functional ring can be compressed.**

Measure the length of the foam cover on the prosthesis and add the length allowance.

**TT prostheses:** Distal allowance for movement of the prosthetic foot.

**TF prostheses:** Allowance proximal of the knee rotation point for flexion of the prosthetic knee joint and distal allowance for movement of the prosthetic foot.

- 2) Cut the pre-shaped foam cover to length and fit it in the proximal area on the prosthetic socket.
- 3) Pull the pre-shaped foam cover over the prosthesis.
- 4) Set the connection cover onto the footshell. The connection cover engages in the edge of the footshell. The connection cover must be cut into for sizes 22 through 25.
- 5) Install the prosthetic foot on the prosthesis.
- 6) Mark the outer contours of the connection cover on the distal face of the foam cover.
- 7) Detach the prosthetic foot and remove the connection cover.
- 8) Clean the connection cover using a degreasing cleaner.
- 9) Glue the connection cover onto the distal face of the pre-shaped foam cover according to the marked outer contour.
- 10) Let the glue dry (approx. **10 minutes**).
- 11) Install the prosthetic foot and adapt the exterior cosmetic shape. Take into account compression caused by cosmetic stockings or SuperSkin.
- 12) Install the foam cover connection set.

## 6 Cleaning

### 6.1 Cleaning the prosthetic foot

> **Allowable cleaning agent:** pH neutral soap (e.g. 453H10 Derma Clean)

- 1) **NOTICE! To avoid product damage, only use the allowable cleaning agents.**  
Clean the product with clear fresh water and a pH neutral soap.
- 2) **If present:** Remove dirt from water drainage contours using a toothpick and rinse.
- 3) Rinse the soap away with clear fresh water. In doing so, rinse the footshell until all dirt has been removed.
- 4) Dry the product with a soft cloth.
- 5) Allow to air dry in order to remove residual moisture.

### 6.2 Flushing the vacuum pump (Harmony only)

 **CAUTION**

**Improper cleaning**

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ Clean the product regularly.

- > **Required materials:** 30 ml to 60 ml distilled water, vessel
- 1) Fill distilled water into the prosthetic socket.  
Alternatively, a residual limb sock can be soaked with distilled water to flush the vacuum pump while walking.
  - 2) Position the vessel under the outlet of the vacuum pump.
  - 3) Alternately load and unload the prosthesis to pump all the water through the vacuum system.
  - 4) Clean and dry the interior surface of the prosthetic socket.

## 7 Maintenance

### **⚠ CAUTION**

#### **Failure to follow the maintenance instructions**

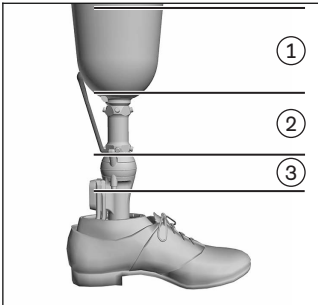
Risk of injuries due to changes in or loss of functionality and damage to the product

► Observe the following maintenance instructions.

- A visual inspection and functional test of the prosthetic components should be performed after the first 30 days of use.
- Inspect the entire prosthesis for wear during normal consultations.
- Conduct annual safety inspections.
- Check the condition of the functional ring as part of each inspection. If the user frequently compresses the functional ring to the stop while walking, or if the functional ring exhibits visible damage, replace the functional ring (see page 25).

### 7.1 Troubleshooting leaks (Harmony only)

This section gives a methodical troubleshooting procedure for when the vacuum system leaks.

To check for leaks, the prosthesis is divided into areas.

- **Area 1:** Prosthetic socket and sealing sleeve
- **Area 2:** Socket connector and connection to the vacuum pump
- **Area 3:** Vacuum pump

#### Checking area 1

First check the donning process and the sealing sleeve. No pressure gauge is needed. The vacuum suction the sealing sleeve against the socket brim. The socket brim is clearly visible under the sealing sleeve during the stance phase. If no vacuum is generated or the vacuum decreases, this can be identified on the sealing sleeve.

- Ask the user to put on the prosthesis. Check the process to exclude errors during donning.
- Get the user to walk in order to build up a vacuum. Get the user to stand and check whether the vacuum is maintained.
- **If the vacuum is not maintained:** Replace the sealing sleeve with a new one.
- Get the user to walk in order to build up a vacuum. Get the user to stand and check whether the vacuum is maintained.
- **If the vacuum is still not maintained:** The fault is in area 2 or 3.

#### Checking area 3

- Connect the pressure gauge directly to the vacuum pump.
- Activate the vacuum pump two to four times (vacuum of **340 hPa to 680 hPa**).

- **If the vacuum is maintained:** The vacuum pump is OK. The fault is in area 2.
- **If the vacuum is not maintained:** Replace the valves of the vacuum pump. Then check whether this measure was successful.
- **If this measure was not successful:** Replace the functional ring. Then check whether this measure was successful.
- **If this measure was not successful:** Send the vacuum pump to the manufacturer for inspection.

### Checking area 2

- Connect the pressure gauge with a T-piece between the socket connector and the vacuum pump.
- Seal the opening to the socket connector on the inside of the prosthetic socket with air-tight adhesive tape (e.g. PVC tape).
- Activate the vacuum pump until a vacuum of **340 hPa** to **680 hPa** is reached.
- **If the vacuum is not maintained:** The socket connector is defective. Replace the socket connector. Then check whether this measure was successful.
- **If the vacuum is maintained:** The glued joint of the socket connector is likely defective.
- Seal the opening to the socket connector on the inside of the prosthetic socket with a small piece of paper and air-tight adhesive tape (e.g. PVC tape).
- Activate the vacuum pump until a vacuum of **340 hPa** to **680 hPa** is reached.
- **If the vacuum is not maintained:** Glue the socket connector again. Then check whether this measure was successful.

### 7.2 Replacing valves (Harmony only)

- 1) Pull the old valves off the functional ring, noting the flow direction (arrows on the valves).
- 2) Clean the functional ring with compressed air.
- 3) Plug the new valves into the openings of the functional ring in the correct flow direction.

### 7.3 Replacing the functional ring

To replace the functional ring, the adapter with pyramid is removed from the base body. The O-rings on the adapter and the mounting screw are also replaced together with the functional ring.

> **Required materials:** Pre-compression kit, torque wrench (e.g. 710D20), degreasing cleaner (e.g. 634A58 isopropyl alcohol), Loctite 243 or Loctite 246

- 1) Remove the footshell and the prosthetic foot from the prosthesis.
- 2) Insert an Allen key (**6 mm**) into the hole in the pyramid and unscrew the mounting screw in the distal end of the base body (see fig. 10).
- 3) Pull the adapter with pyramid up and out of the base body (see fig. 11).
- 4) Remove the functional ring.
- 5) **If the O-rings are worn:** Remove the O-rings from the adapter (see fig. 12).
- 6) **If the O-ring is worn:** Remove the O-ring from the mounting screw (see fig. 12).
- 7) Clean the mounting screw and adapter including thread with degreasing cleaner (see fig. 12).
- 8) Slide new O-rings onto the adapter and apply silicone grease. Make sure that no silicone grease gets into the threads (see fig. 12).
- 9) Slide a new O-ring onto the mounting screw (see fig. 12).
- 10) Apply Loctite to the thread of the mounting screw.
- 11) Slide the new functional ring onto the adapter of the prosthetic foot. The structure of the functional ring only permits installation with the correct alignment.
- 12) Insert the adapter of the prosthetic foot into the base body and attach it with the mounting screw (tightening torque: **15 Nm**). To do so, insert the torque wrench through the hole in the pyramid.
- 13) Pre-compress the new functional ring (see page 20).
- 14) Reinstall the prosthetic foot on the prosthesis and pull on the footshell.

## 8 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

## 9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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