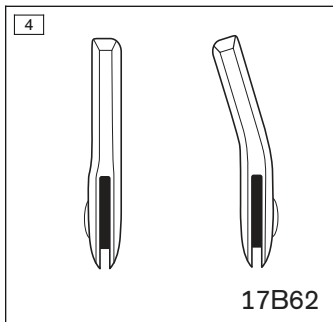
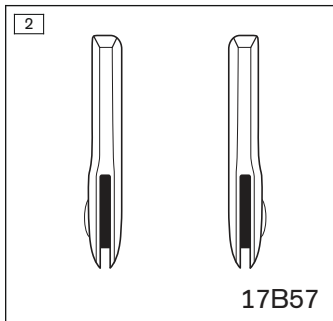
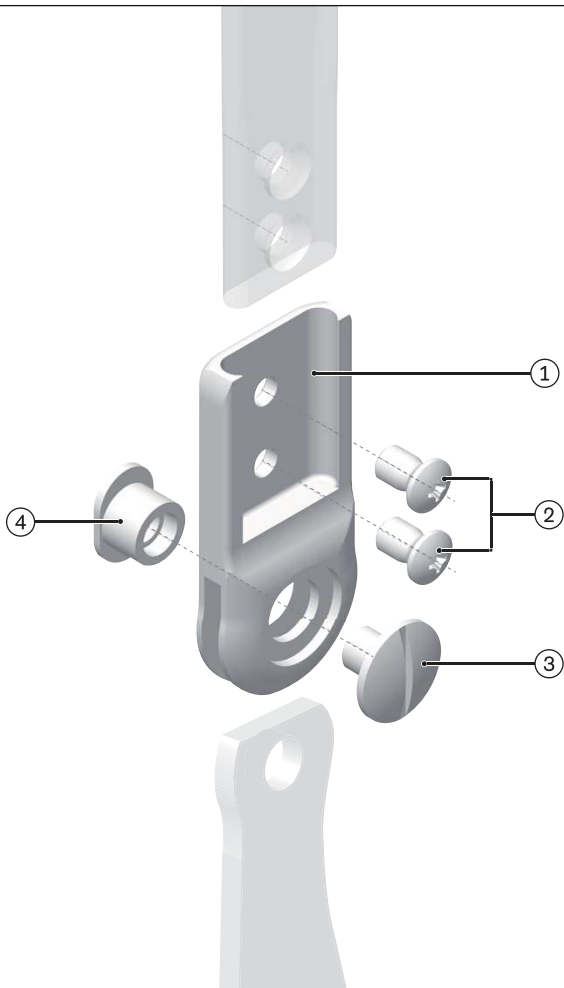
## 17B57\*, 17B62\*

|  |    |
|--|----|
| <b>DE</b> Gebrauchsanweisung .....         | 4  |
| <b>EN</b> Instructions for use .....       | 9  |
| <b>FR</b> Instructions d'utilisation ..... | 14 |
| <b>IT</b> Istruzioni per l'uso .....       | 20 |
| <b>ES</b> Instrucciones de uso .....       | 26 |
| <b>PT</b> Manual de utilização .....       | 32 |
| <b>NL</b> Gebruiksaanwijzing .....         | 38 |
| <b>SV</b> Bruksanvisning .....             | 43 |
| <b>DA</b> Brugsanvisning .....             | 49 |
| <b>NO</b> Bruksanvisning .....             | 54 |
| <b>PL</b> Instrukcja użytkowania .....     | 59 |
| <b>CS</b> Návod k použití .....            | 65 |
| <b>TR</b> Kullanma talimatı .....          | 71 |
| <b>EL</b> Οδηγίες χρήσης .....             | 76 |
| <b>RU</b> Руководство по применению .....  | 82 |
| <b>ZH</b> 使用说明书 .....                      | 89 |

5

## INFORMATION

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### INFORMATION

Date of last update: 2021-05-07

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide you with important information on the use of the 17B57\* and 17B62\* orthotic ankle joints.

## 2 Product description

### 2.1 Available sizes

| Article number | Medial and lateral joint           | System width | Joint head diameter | Slot width | Length from joint centre | Material        |
|----------------|------------------------------------|--------------|---------------------|------------|--------------------------|-----------------|
| 17B57=16       | Inside, outside straight           | 16 mm        | 22 mm               | 2.5 mm     | 41 mm                    | Stainless steel |
| 17B57=20       |                                    | 20 mm        | 24 mm               | 3 mm       |                          |                 |
| 17B62=16       | Inside contoured, outside straight | 16 mm        | 22 mm               | 2.5 mm     |                          |                 |
| 17B62=20       |                                    | 20 mm        | 24.5 mm             | 3 mm       |                          |                 |

### 2.2 Components/design (see Fig. 1)

| Scope of delivery (see fig. 5) |                      |             |                |
|--------------------------------|----------------------|-------------|----------------|
| Item                           | Quantity [Piece (s)] | Component   | Article number |
| 1                              | 1                    | Joint body  |                |
| 2                              | 2                    | Screws      | 501T7*         |
| 3                              | 1                    | Joint screw | 501S32*        |
| 4                              | 1                    | Bearing nut | 17Y93*         |

### Accessories

| Not included in scope of delivery |                      |                                |  |
|-----------------------------------|----------------------|--------------------------------|--|
| Item                              | Quantity [Piece (s)] | Component                      | Article number                                 |
| not illustrated                   | 1                    | Lamination bar                 | 17Y128*, 17Y129*                               |
|                                   | 1                    | Side bars                      | 17B38*, 17B39*, 17B40*, 17B41*, 17B84*, 17B85* |
|                                   | 1                    | System foot stirrup            | 17B55*, 17B60*, 17F61*                         |
| not illustrated                   | 1                    | System lamination foot stirrup | 17B100*  |

| Not included in scope of delivery |                            |                     |                |
|-----------------------------------|----------------------------|---------------------|----------------|
| Item                              | Quantity<br>[Piece<br>(s)] | Component           | Article number |
| not illustrated                   | 1                          | System shoe stirrup | 17F36*         |
|                                   | 1                          | System shoe plate   | 17F35*         |

### 3 Intended use

#### 3.1 Indications for use

The product is intended **exclusively** for use in pairs for orthotic treatment of the lower limbs.

#### 3.2 Indications

- Partial or total paralysis of the leg muscles
  - Orthopaedic diseases of the lower limbs
- Indications must be determined by the physician.

#### 3.3 Lifetime

The product is designed for a lifetime of **3 years** when used as intended and assembled professionally.

#### 3.4 Qualification

Patients may be fitted with the product only by trained qualified personnel. The qualified personnel must be familiar with the handling of the various techniques, materials, machines and tools.

### 4 Safety

#### 4.1 Explanation of warning symbols

|   |   |
|---|---|
|  <b>CAUTION</b> | Warning regarding possible risks of accident or injury. |
|  <b>NOTICE</b>  | Warning regarding possible technical damage.            |

#### 4.2 Safety instructions

|   |
|---|
|  <b>CAUTION</b> |
| <b>Excessive strain due to use on more than one patient</b>                                       |
| Risk of injury and loss of functionality as well as damage to the product                         |
| ▶ Use the product on only one patient.  |
| ▶ Observe the maintenance recommendations.  |

**⚠ CAUTION**

**Excessive strain on load-bearing components**

Injuries due to changes in or loss of functionality

- ▶ Only use the product for the defined area of application.
- ▶ If the product has been exposed to extreme strain (e.g. due to falling), take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

**⚠ CAUTION**

**Mechanical damage to the product**

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

**NOTICE**

**Thermal overloading of the product**

Damage due to improper thermal treatment

- ▶ Do not carry out any heat treatment at temperatures above **300 °C (570 °F)**.
- ▶ Prior to thermal treatment, remove all temperature-critical components (such as plastic parts).

**NOTICE**

**Exposure of the product to unsuitable environmental conditions**

Damage, brittleness or destruction due to improper handling

- ▶ Avoid storage in condensing ambient humidity.
- ▶ Avoid contact with abrasive substances (e.g. sand, dust).
- ▶ Do not expose the product to temperatures below **-10 °C (14 °F)** or above **+60 °C (140 °F)** (e.g. sauna, excessive sunlight, drying on a radiator).

## 5 Preparing the product for use

**⚠ CAUTION**

**Incorrect alignment or assembly**

Injuries due to changes in or loss of functionality

- ▶ The product may only be installed by trained, qualified personnel.
- ▶ Observe the alignment and assembly instructions.

### INFORMATION

#### Parallel alignment of the orthotic joints

Use the 743R6 alignment fixture to position the orthotic joints on the plaster positive.

#### Mounting the foot stirrup

- > **Prerequisite:** The orthotic ankle joints are disassembled. To do so, remove the joint screw and the bearing nut from the joint.
- 1) Insert the contoured foot stirrup into the orthotic ankle joint.
- 2) Assemble the foot stirrup and the orthotic joint with the bearing nut and the joint screw.
- 3) For final assembly, secure the screw connection with Loctite 241.

#### Improving corrosion protection

Polish or sinter the product for improved corrosion protection. Ottobock recommends the 618T40\* sintering powder for surface coating.

- ▶ Do not heat the material longer than **5 minutes** at max. **150 °C** for surface coating.

#### Gluing in bars

**Required materials:** 636W28\* special adhesive, degreasing cleaner

The bars must be glued in the insertion zones to ensure stability.

- 1) Clean the connecting surfaces with a degreasing cleaner.
- 2) Apply the special adhesive to the connecting surfaces.
- 3) Insert the bars.
- 4) Secure with the enclosed screws.
- 5) Allow to cure for at least **4 hours**.

**INFORMATION: Final bonding strength will be reached after 16 hours.**

#### Using protective devices

- ▶ Use joint protectors to protect the product against soiling.

## 6 Cleaning

Promptly clean the product after contact with:

- Water containing salt, chlorine or soap, perspiration, urine or splashed water (e.g. after rain) and soiling.
- 1) Rinse the product with clear fresh water.

- 2) Dry the product with a cloth or allow it to air dry. Avoid exposure to direct heat (e.g. from an oven or radiator).

## 7 Maintenance

### INFORMATION

**The product may be exposed to increased loads by the patient.**

- ▶ Shorten the maintenance intervals according to the expected loads.

The manufacturer requires at least a semi-annual inspection of the product to verify functionality and check for wear.

Only use 633F7 special lubricant.

Spare parts are listed in the section "Components/design".

If you have any doubts, please contact the manufacturer.

## 8 Disposal

Dispose of the product in accordance with national regulations.

## 9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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