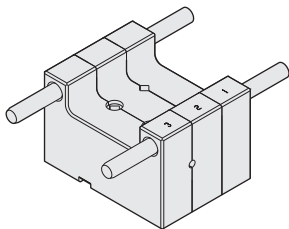
736Y6

DE Gebrauchsanleitung (Benutzer / Techniker)	6
EN Instructions for use (user / technician)	8
FR Mode d'emploi (Utilisateur / Technicien)	11
IT Istruzioni d'uso (utilizzatore / tecnici)	14
ES Instrucciones de uso (usuario / técnico)	17
PT Manual de instruções (Usuário / Técnico)	20
NL Gebruiksaanwijzing (gebruiker/technicus)	23
SV Bruksanvisning (användare/tekniker)	26
DA Brugsvejledning (bruger/tekniker)	28
PL Instrukcja użytkowania (Użytkownik / Technik)	31
HU Használati utasítás (használóknak / műszerészeknek)	34
CS Návod k použití (pro uživatele / techniky)	37
TR Kullanım kılavuzu (Kullanıcı / Teknisyen)	40
EL Οδηγίες χρήσης (χρήστης/τεχνικός)	42

1 i 4 Lieferumfang/4 Scope of delivery

736Y6

647H13

2 i 5 Technische Daten/5 Technical data

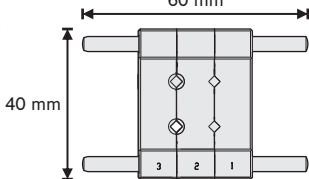
26,5 mm

60 mm

215 g

40 mm

3 **i** **5 Gebrauch/5 Use**

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INFORMATION

Date of last update: 2017-03-23

- ▶ Please read this document carefully before using the product.

- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Please keep this document in a safe place.

This document is valid for:

- Crimping tool (736Y6)

The product and this document are intended for qualified personnel in an orthopaedic workshop.

1.1 Explanation of warning symbols

CAUTION

Warning regarding possible risks of accident or injury.

Warning regarding possible technical damage.

1.2 Meanings of pictograms in the illustration

①	Numbering for the illustrations	i	Reference to the number of the corresponding section
②	Numbering for a defined sequence	③	Numbering for the parts of an illustration

2 Product description

The crimping tool is intended for crimping a component with a steel cable. The Figures 3 and 4 on pages 3 and 4 illustrate which component and which steel cable may be crimped with each other.

3 Scope of delivery

The scope of delivery is shown in Figure 1 on page 2.

Only product components with an article number can be reordered separately.

4 Technical data

The technical data are shown in Figure 2 on page 2.

5 Use

CAUTION

Carelessness when working with the product

Injuries (for instance to the hands) due to pinching or cutting

- ▶ Use caution and maintain awareness of safety when working with the product.

NOTICE

Failure to follow the instructions for use

Damage to components and inadequate strength of the crimped connection

- ▶ Observe all instructions for use.

3 + **4** (Pages 3 + 4): **1** Inserting the component

- ▶ Only use the product in clean condition (section "Cleaning" - see Page 11).
- ▶ Insert the component for crimping as shown in the figures.

5 (Page 5): **1** Crimping (10Y31=2)

- ▶ **NOTICE! Never load multiple components and crimp them simultaneously.**

Steps for crimping (example with 10Y31=2):

1 Insert component; **2** slide tool together; **3** insert tool into vice; **4** strip steel cable (e.g. with a knife); **5** insert steel cable into component; **6** with the clamping jaws of the vice, compress the tool fully to crimp the component with the steel cable

- ▶ Follow the crimping procedure.
As an exception for components 10Y23 and 10Y28=1, the steps **4** and **5** are carried out before step **2**.
- ▶ In step **3**, only clamp the complete crimping tool with components 1, 2 and 3 in the vice.
- ▶ In step **4**, strip the steel cable far enough so it is fully crimped with the component.

- ▶ After crimping, check whether the component is firmly crimped with the steel cable.

6 Cleaning

- ▶ Clean the product if it gets dirty (e.g. chips, wires, oil and grease).

7 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

8 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

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Otto Bock HealthCare GmbH