
8361 Genu Therma

8364 Genu Therma Patella

DE Gebrauchsanweisung	4
EN Instructions for use	7
FR Instructions d'utilisation	10
IT Istruzioni per l'uso	13
ES Instrucciones de uso	16
NL Gebruiksaanwijzing	19
SV Bruksanvisning	22
DA Brugsanvisning	25
NO Bruksanvisning	28
FI Käyttöohje	31
PL Instrukcja użytkowania	34
SK Návod na používanie	37
BG Инструкции за употреба	40
TR Kullanma talimatı	43
ET Kasutusjuhhis	46
RU Руководство по применению	49
ZH 使用说明书	53

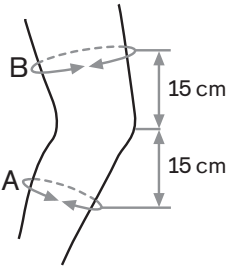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INFORMATION

Last update: 2017-06-09

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

These instructions for use provide important information on the application of the 8361 Genu Therma and 8364 Genu Therma Patella knee supports.

2 Intended use

2.1 Indications for use

The support is intended **exclusively** for fittings of the lower limbs and **exclusively** for contact with intact skin.

The support must be used according to the indication.

2.2 Indications

- Knee pain without ligament laxity or feeling of instability
- Irritation and swelling of the knee joint
- Tendomyopathy
- Insertional tendinosis
- Knee pain caused by strain
- Soft tissue irritation following injuries and/or surgeries
- Minor knee osteoarthritis

The indication must be determined by the physician.

2.3 Contraindications

2.3.1 Absolute Contraindications

None known.

2.3.2 Relative Contraindications

The following indications require consultation with a physician: skin diseases/injuries, inflammation, prominent scars that are swollen, reddening and hyperthermia of the fitted limb/body area; lymphatic flow disorders, including unclear soft tissue swelling distant to the body area to which the medical device will be applied; sensory and circulatory disorders in the leg area.

2.4 Mechanism of Action

The support improves thermo-regulation, proprioception and circulation, promotes the resorption of edemas by compressing the soft tissues and can relieve pain.

3 Safety

3.1 Explanation of warning symbols

CAUTION Warning regarding possible risks of accident or injury.

NOTICE Warning regarding possible technical damage.

3.2 General safety instructions

CAUTION

Use of a product with neoprene

Allergic reactions due to neoprene or heat allergies

- ▶ Do not wear products with neoprene for more than 3 – 4 hours without interruption (unless explicitly instructed otherwise by the doctor).
- ▶ Do **not** use the product in case of a known neoprene or heat allergy.

CAUTION

Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

CAUTION

Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

NOTICE

Improper use and changes

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

NOTICE

Contact with ointments, lotions, or other products that contain oils or acids

Insufficient stabilization due to loss of material functionality

- ▶ Do not expose the product to ointments, lotions, or other products that contain oils or acids.

4 Handling

INFORMATION

- ▶ The daily duration of use and period of application are generally determined by the physician.
- ▶ The initial fitting and application of the product must be carried out by qualified personnel.
- ▶ Instruct the patient in the handling and care of the product.
- ▶ Instruct the patient to see a physician immediately if any exceptional changes are noted (e.g. worsening of the complaint).

4.1 Selecting the Size

- 1) Measure the circumference of the thigh 15 cm above the center of the patella.
- 2) Measure the circumference of the lower leg 15 cm below the center of the patella.
- 3) Determine the size of the support (see size chart).

4.2 Application

⚠ CAUTION

Incorrect or excessively tight application

Skin irritation due to overheating, local pressure points due to tight application to the body

- ▶ Ensure that the product is applied properly and fits correctly.
- ▶ When in doubt, do not continue to use the product if skin irritation occurs.

NOTICE

Use of a worn or damaged product

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

- > The patient is sitting on the front edge of a chair.
- > Open all hook-and-loop closures on the support.
- 1) Hold the support by the top edge on both sides and pull it over the knee (see fig. 1).
- 2) **For 8364 Genu Therma Patella only:** Center the patella cutout over the patella (see fig. 3).
- 3) Fasten all hook-and-loop closures (see fig. 2 and see fig. 3).
- The red side strap of the support lies on the inner/outer side of the knee.

4.3 Cleaning

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

- 1) Fasten all hook-and-loop closures.
- 2) Recommendation: Use a laundry bag or net.
- 3) Wash the support in warm water at **40 °C** (104 °F) with standard mild detergent. Do not use fabric softener. Rinse thoroughly.
- 4) Allow to air dry. Do not expose to direct heat sources (e.g. sunlight, stove or radiator).

5 Disposal

Dispose of the product in accordance with national regulations.

6 Legal Information

All legal conditions are subject to the respective national law of the country of use and may vary accordingly.

6.1 Liability

The manufacturer shall be liable in the event that the product is used in accordance with the descriptions and instructions in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorized modification of the product.

6.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The manufacturer therefore drew up the declaration of conformity on its own responsibility in accordance with Annex VII of the directive.

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Otto Bock HealthCare GmbH