743G5

DE Gebrauchsanleitung (Benutzer/Techniker)	6
EN Instructions for Use (User/Technician)	8
FR Mode d'emploi (Utilisateur/Technicien)	10
ES Instrucciones de uso (usuario/técnico).....	13
ZH 使用说明书（用户/技术人员）	15

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English

Date of the last update: 2014-07-01

- Please read this document carefully.
- Follow the safety instructions.

1 Intended Use

The 743G5 Hip Cast Casting Apparatus in combination with the 743A11 Casting Apparatus is to be **exclusively** used for the plaster cast technique on patients with hip disarticulation. It is used for the fabrication of a plaster negative under load, for a hip disarticulation socket. By using the proven 743Y26 Pressure Blocks, the stop position for a modular hip joint is predetermined and taking a plaster cast of the pelvis is standardized.

In conformity with the proven SIT Cast casting technique, special silicone casting forms encompass the ramus and position it in the middle of the support plate.

The Hip Cast Casting Apparatus is a practical alternative to the manual plaster cast technique and can also be used on site, e.g. in the clinic, at the patient's home, etc.

2 Safety Instructions

Attention!

The patient must have sufficient standing ability and use a standing aid such as a standing support or two forearm crutches during the entire procedure.

It is recommended that two orthopedic technicians take the plaster cast.

During assembly of the 743G5 Hip Cast Casting Apparatus in combination with the 743A11 Casting Apparatus, make sure that the plug connection is inserted up to the stop and secured with the clamping screw.

Adjust the height of the 743A11 Casting Apparatus to the distance between the ischial tuberosity and the floor measured on the patient and secure it with the set screw.

Then, using the attached bubble level, adjust the height-adjustable support of the 743G5 Hip Cast Casting Apparatus so that the support plate is horizontal. Make sure to tighten the clamping screw of the support after completing the adjustment.

3 Scope of Delivery of the 743G5 Hip Cast Casting Apparatus (Fig. 1)

- 1 height-adjustable base unit
- 1 support plate (743Y447)
- 1 silicone pad, semi-circular, left (743Y448=L)

- 1 silicone pad, semi-circular, right (743Y448=R)
- 1 silicone pad, semi-circular, flat (743Y448=F)
- 2 pressure blocks (743Y26)
- 1 caliper (for measuring the hip circumference and ramus angle)
- 1 tension belt

The required 743A11 Casting Apparatus is not included in the scope of delivery. It must be ordered separately.

4 Preparing to Take a Plaster Cast

- The patient must pull on a 641T3=* Body Cover.
- Insert the plug connection of the 743G5 Hip Cast Casting Apparatus into the receiver of the 743A11 Casting Apparatus up to the stop (Fig. 2).
- Secure the plug connection with the clamping screw.
- Measure the ramus angle, hip circumference, and the distance between the ischial tuberosity and the floor (Fig. 3 and 5).
- Adjust the 743A11 Casting Apparatus to the distance between the ischial tuberosity and floor measured on the patient and secure it with the set screw (Fig. 4).
- Using the attached bubble level, adjust the height-adjustable support of the 743G5 Hip Cast Casting Apparatus so that the support plate is horizontal.
- Make sure to tighten the clamping screw of the support after completing the adjustment.
- Insert the 743Y447 Support Plate in the 743G5 Base Unit.
- Check all settings, connections, and the height before positioning the patient on the support plate.
- Select the appropriate 743Y448=* Silicone Pad and insert it in the recess of the support plate.
- Adjust the position of the silicone pad to the measured ramus angle (Fig. 6).

5 Casting Technique

- Test the fit of the patient on the silicone pad and check whether the position of the pelvis is level using a 743Y32 Hip Leveling Guide (Fig. 7).
- Check whether the support plate is horizontal using the attached bubble level (Fig. 8).
- Attach the tension belt and mount it to the 743G5 Base Unit (Fig. 9 and 10).
- Apply the tension belt on the patient and check its position and course above the iliac crests (Fig. 11 and 12).
- Mark the center of the body on the body cover (Fig. 13).
- Cover the support plate and upper part of the base unit with plastic foil for protection and then apply 640Z5=* Plaster Parting Agent Cream to the surface of the support plate and the pressure blocks (Fig. 14).
- To fabricate the plaster negative, you need the following:
 - 8-fold longuette of 240 mm wide elastic plaster bandage (699G1=24);
 - 350 mm wide elastic plaster bandage (699G1=35);
 - three other 150 or 240 mm wide elastic plaster bandages (699G1=15 or 699G1=24).
- Take the plaster cast as usual. More information on taking plaster casts is contained on the enclosed user CD (646C47).

- Position the patient on the longuette and shape the longuette against the body contours.
- Hold the pressure blocks against the patient's body at the front and rear, parallel to the support plate. Then apply targeted pressure with both hands on the pressure blocks while ensuring proper fit (Fig. 15).
- Wrap around elastic plaster bandages.
- Position the tension belt and check the fit above the iliac crests (Fig. 16).
- Tighten the tension belt and keep it taut until the elastic plaster bandages have started to harden.
- Loosen and remove the tension belt.
- Cut the shape of the plaster cast on the body and cut it open.

6 Liability

The manufacturer's warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use.

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Otto Bock HealthCare GmbH