# ottobock.

## 10A25, 10A37

<b>DE</b>	Gebrauchsanweisung .....	2
<b>EN</b>	Instructions for use .....	5
<b>FR</b>	Instructions d'utilisation .....	9
<b>IT</b>	Istruzioni per l'uso .....	13
<b>ES</b>	Instrucciones de uso .....	17
<b>PT</b>	Manual de utilização .....	20
<b>NL</b>	Gebruiksaanwijzing .....	24
<b>SV</b>	Bruksanvisning .....	28
<b>DA</b>	Brugsanvisning .....	31
<b>NO</b>	Bruksanvisning .....	35
<b>PL</b>	Instrukcja użytkowania .....	38
<b>HU</b>	Használati utasítás .....	42
<b>CS</b>	Návod k použití .....	45
<b>RO</b>	Instrucțiuni de utilizare .....	48
<b>TR</b>	Kullanma talimatı .....	52
<b>EL</b>	Οδηγίες χρήσης .....	56
<b>RU</b>	Руководство по применению .....	60
<b>JA</b>	取扱説明書 .....	64
<b>ZH</b>	使用说明书 .....	67

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English

Last update: 2016-01-18

- Please read this document carefully before using the product.
- Follow the safety instructions to avoid injuries and damage to the product.
- Instruct the user in the proper and safe use of the product.
- Please keep this document in a safe place.

The scope of delivery is shown on the front page.

## **1 Single components**

### **1.1 Single components**

For available single components refer to catalog or [www.ottobock-shop.de](http://www.ottobock-shop.de).

## 1.2 Accessories

21A13=3 Connection Piece for Hook (for 10A25 only)

21A14 Connection Piece

21A44 Connection Piece for Hook

The accessories are not included. They must be ordered separately.

## 2 Description

### 2.1 Indications for use

The Ottobock Cable Control Hook for Children 10A25=\* Ottobock Cable Control Hook for Adolescents 10A37=\* are to be used **exclusively** for exoprosthesis fittings of the upper extremities.

### 2.2 Field of application

The Ottobock Cable Control Hook for Children and Cable Control Hook for Adolescents are intended for the fabrication of a cable-controlled arm prosthesis.

The Cable Control Hook for Children 10A25 is intended for children from about 1 to 5 years.

The Cable Control Hook for Adolescents 10A37 is intended for children from above 5 years.

### 2.3 Safety instructions

Please forward the following safety instructions to your patients or their guardians:

#### **Attention!**

Use caution when handling the cable control hooks.

If used incorrectly, the hooks can cause injuries to the user or others.

**Attention!**

Careful handling of the prosthesis and its components not only increases their service life but, above all, ensures the patient's safety!

Should the prosthesis be subjected to unusual stresses (such as a fall), immediately contact your prosthetist and have the prosthesis inspected for any damage. If necessary, the responsible prosthetist will pass the prosthesis on to the Ottobock Service.

**Caution!**

Please avoid exposing prosthetic components to surroundings that corrode metal parts, for example, freshwater, saltwater, acids and other liquids.

Using this medical product in such environmental conditions will render all claims against Otto Bock HealthCare null and void.

**Caution!**

The prosthesis should not be subjected to intense smoke, dust, vibrations, shocks or high temperatures. Do not allow debris or liquids to get into the prosthesis.

Failure to follow these instructions can lead to malfunctioning and damage of the prosthesis.

**Caution!**

Do not use any aggressive cleaning agents since they could cause damage to bearings, seals and plastic parts.

**2.4 Function**

These cable control hooks are available in the following versions:

Cable Control Hook	Thread Type		Closing Rubber	Finger Cover
	Metric	Inch		
<b>10A25=* for children</b>	<b>*=LM12×1.5 left</b>	<b>*=L1/2"-20 left</b>	10Y13	10Y8
<b>10A37=* for adolescents</b>	<b>*=RM12×1.5 right</b>	<b>*=R1/2"-20 right</b>		10Y1

The cable control hooks are opened with a control cable and closed automatically via Closing Rubber 10Y13.

The branches are provided with plastic finger covers allowing for increased grip through edgeless surface.

### 3 Technical data

Article number	10A25=*	10A37=*
Weight [g/oz]	40 / 1.41	75 / 2.65
Total height [mm/in]	95 / 3.74	110 / 4.33
Opening width [mm/in]	43 / 1.69	43 / 1.69
Grip force [N]	15	15

### 4 Handling

#### 4.1 Assembly

The cable control hooks for children and adolescents are to be used exclusively in combination with Ottobock wrist units.

The 10A25 is connected to the control cable through the Connection Piece 21A13=3 and the 10A37 is connected to the control cable through the Connection Pieces 21A14 or 21A44.

To make the replacement of the finger covers easier, warm up the finger covers in lukewarm water 40° C beforehand.

### 5 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

#### 5.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

#### 5.2 CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device

according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

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Otto Bock HealthCare GmbH