4X78, 4X79

DE Gebrauchsanweisung	3
EN Instructions for use	6
FR Instructions d'utilisation	8
IT Istruzioni per l'uso	11
ES Instrucciones de uso.....	13
PT Manual de utilização	16
NL Gebruiksaanwijzing	18
SV Bruksanvisning	21
DA Brugsanvisning	23
PL Instrukcja użytkowania	26
HU Használati utasítás.....	29
CS Návod k použití	31
TR Kullanma talimatı	33
EL Οδηγίες χρήσης.....	36
RU Руководство по применению.....	39
JA 取扱説明書	42
ZH 使用说明书	44

INFORMATION

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INFORMATION

These instructions for use were last updated on 2015-06-23.

Explanation of symbols

NOTICE Warnings regarding possible technical damage.

INFORMATION

Please read these Instructions for Use thoroughly. Please follow the safety instructions.

1 Application

The 4X78 Extension Cable for Charging is to be used **solely** for transferring the charging receptacle of the electronic knee joint systems C-Leg 3C98 / 3C88, 3C98-1 / 3C88-1, 3C98-2 / 3C88-2, C-Leg compact 3C96 / 3C86, 3C96-1 / 3C86-1 and Ottobock Compact 3C95 / 3C85 into the ankle area.

2 Safety precautions

NOTICE

Mechanical overloading. Exterior mechanical impact or stress such as shocks and vibrations can lead to faulty control or malfunction of the charger extension cable. This makes impossible to ensure an appropriate charging function.

Do not use the Battery Extension Cable if any damage is visible.

NOTICE

Penetration of dirt and humidity. Penetration of dirt and humidity can lead to faulty control or malfunction of the charger extension cable. This makes impossible to ensure an appropriate charging function.

Do not allow objects or liquids to fall onto or spill into the charging receptacle.

NOTICE

Installing the charger extension cable. In case of improper installation of the cable, constant movement of the cable during gait can result in a broken wire. This makes impossible to ensure an appropriate charging function.

The cable should not be installed to the prosthetic socket, but must be guided via the tube adapter.

INFORMATION

This product may not be disposed of with regular domestic waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in your country regarding return and collection processes.

3 Scope of delivery

- 1 pc. 4X78 Battery Extension Cable and/or
- 1 pc. 4X79 Plug Holder
- 1 pc. 647H370 Instructions for Use

4 Instructions for installation

The charging receptacle can either be glued directly into the cosmetic foam cover (in this case we recommend using rigid foam for the area around the charging receptacle) or it can be firmly attached to the C-Leg / C-Leg compact / Ottobock Compact tube adapter (fig. 2, right) using the 4X79 Plug Holder (fig. 1).

1. The extension cable's battery sleeve can be shortened and thus adapted to different ankle circumferences (shortening range, see fig. 2).
2. Plug the extension cable into the knee joint.
3. Coil the cable around the tube adapter (fig. 1). When doing so, make sure the cable lies flat on the tube and does not twist around itself. With shorter tubes it helps to affix adhesive tape to each point of the joint and / or tube adapter.

4. Use double-stick tape to attach the plug holder to the C-Leg/ C-Leg compact Tube Adapter. Note: We recommend doing this only after the cosmetic foam cover has been pulled over.
5. Attach the extension cable's battery sleeve to the plug holder's receptacle using fast-setting adhesive or two-component adhesive.

5 Liability

The manufacturer's warranty applies only if the product has been used under the conditions and for the purposes described. The manufacturer recommends that the product be used and maintained according to the instructions for use.

The product may only be opened and repaired by authorized Ottobock technicians.

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Otto Bock Healthcare Products GmbH