

# CUSTOM SILICONE – INSTRUCTIONS FOR TAKING PICTURES

## THE FOLLOWING PICTURES ARE REQUIRED:

### HANDS (sound and affected sides):

- Dorsal
- Palmar
- Medial
- Lateral
- Close up of nails without nail polish

### FEET (sound and affected sides):

- Dorsal
- Plantar
- Medial
- Lateral
- Close up of nails without nail polish

## INSTRUCTIONS FOR TAKING PICTURES:

- **NO FLASH**
- Take pictures in indirect day light
- Use a grey background if possible
- Pictures should be focused, consistent in colour and light
- Place a reference colour swatch next to the sound side hand/foot if possible and send swatch along with casts
- JPEG format recommended

Pictures should reflect the natural skin colouring of the client. Colour and nail samples will be sent once satisfactory pictures are received.

