
How to use your new Rehband Knee Support - 7051*, 7084*, 7953*, 7751*, 105*

*different colors, sizes, thicknesses, sides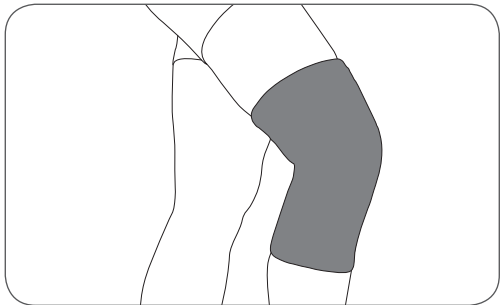

(EN)

Function

Gives support, compression and warmth to the knee joint. Increases blood circulation, relieves pressure and improves muscle coordination in the area. Provides pain relief. Strong compression for stability. * 7mm (7051*, 7084*, 1054*)

Indications

Over-exertion and soft tissue overload in the knee area. Knee pain, inflammation, wear and tear injuries.

Application

- 1) Pull the support up to the knee.
- 2) The narrowest part at the front must be centred with the kneecap.

Important Information

- 1) Use this product with care and only for its intended purpose. Do not make any improper changes to the product.
- 2) If you have questions regarding application, please contact your local dealer or the prescriber.
- 3) Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
- 4) We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
- 5) Keep the product away from open flame, embers and others sources of heat. Avoid temperature above 120 °C/248 °F.
- 6) Do not continue using a product that is no longer functional, or that is worn or damaged.

Washing Instructions

- 1) Wash in warm water at 40 °C (104 °F) with similar colours.
- 2) Use laundry bag or net.
- 3) Stretch product while wet and allow to air dry.
- 4) Clean the product regularly according to care instructions on the product care label.

Size selection

- 1) Do the measurements as instructed on the packaging.
- 2) Determine the size of the support (see size chart).

Conformity declaration

This product meets the requirements of European Directive 93/42/EEC for medical devices.

Disposal

Dispose the product in accordance with national regulations.

Material

Foam: SBR/neoprene or neoprene (see textile label). Cover: polyamide and/or polyester (see on textile label)

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