
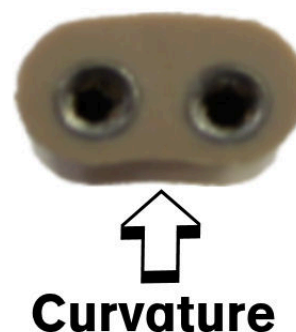


## Academy Tech Tips

### Pediatric System 2000 Battery Cable to the 4-in-1 Controller

The Tech Bulletin is designed to quickly and easily provide useful information regarding Ottobock Products and Services. In this issue, we will focus on the Pediatric System 2000 Battery Cable Attachment to the 4-in-1 Controllers.

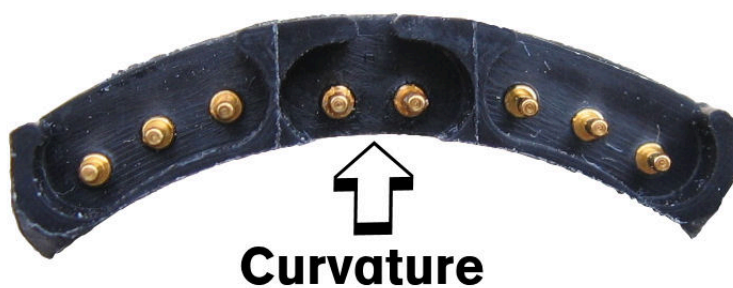

To avoid blowing a fuse in each Battery Pack inserted into the receptacle, match the slight curvature of the Plug to the curvature on the Controller. This is to be done when installing the Battery Cable into the 9E169 or 9E170 4-in-1 Controller.

If you have any further questions, please contact Ottobock at 800 328 4058 and ask to speak with a member of our Professional and Clinical Services department.