

## CASTING/FITTING MATERIAL

### Plaster Bandage

Cellona Plaster Bandages  
699G3

### Fibreglass Bandage

Cellacast Xtra Rigid Bandages  
699G30

### Alginat

87A1

### Impresil RTV silicone (casting and fitting)

#### Impresil Kit – 642V15=1

Includes: 1 dispenser, 5 cartridges  
and 20 static mixers

#### Refill Pack – 87A5

5 cartridges and 10 static mixers

### Pastasil RTV silicone (fitting)

#### Small package –

85H11=1 (2x0.5 kg)

#### Big Package –

85H11=5 (2x2.5 kg)

## CASTING INSTRUCTIONS

### HANDS

- Natural position
- Indicate any sensitive areas or bony prominences
- Cast proximal to styloid process
- Alginat for the affected and sound side
- Impresil can be used for partial fingers

### FEET

- Neutral alignment
- Semi-weight bearing
- Indicate any sensitive areas or bony prominences
- Alginat for the sound side
- Plaster or fiberglass bandage for the affected side
- If there are invaginations or scar tissue, use alginat

### LINERS/MYOELECTRIC PROSTHESES

- Slightly in flexion, around 10-15 degrees
- Cast 1" proximal to the desired trim line
- Indicate any sensitive areas, bony prominences, alignment and electrode sites
- Plaster or fiberglass bandage
- If there are invaginations, use Impresil or Pastasil in those areas

### ORTHOTICS

#### Upper extremity

- Cast in a desired final position
- Suggestion – Impresil between fingers if incorporated in the device
- Plaster or fiberglass bandage

#### Lower extremity

- Semi-weight bearing
- Plaster or Fiberglass bandage

## Do you have questions?

For assistance or more information please contact Custom Silicone Services 800.665.3327 x4842