

# Academy Tech Tips

## 4R110 Pediatric Three Prong Lamination Anchor

The Tech Bulletin is designed to quickly and easily provide useful information regarding Otto Bock products and services. In this issue, we will focus on the 4R110 Pediatric three prong lamination anchor with pyramid receiver. We will illustrate the rotation guidelines for this product.



Figure 1: Range of rotation

The threaded insert is **not designed to be removed** from the three prong adapter. This component has a security punch designed to limit internal or external rotation up to 45 degrees. The lamination protection cap has been provided and covers the red threaded insert during lamination.