

Taleo Vertical Shock 1C51

Ready for everyday life.

Taleo Vertical Shock was designed for active individuals who navigate varied indoor and outdoor environments and place a high value on effortless walking and the ability to go wherever life takes them.

Designed for optimal outcomes:

- The functional ring unit effectively absorbs torsion (+/- 10°) to relieve the residual limb
- The vertical shock unit provides shock absorption (up to 15mm) when walking
- Dual springs and a long carbon base enable a smooth rollover and efficient energy return at varying speeds
- The unique flexible connection of the carbon springs in the forefoot result in optimal adaption to varying ground conditions
- Three different heel wedges to optimize gait dynamics and reduce impact forces

More than a foot. A foundation.



©2020 Otto Bock HealthCare LP · 18596 9/20

Technical data

K-Level	K3
Max. body weight	330 lbs / 150 kg
Sizes	22 – 30 cm
Footshell	Slim shape (S) with 15 +/- 5 mm heel height (size 22 – 25 cm) Normal shape (N) with 10 +/- 5 mm heel height (size 24 – 30 cm)
Footshell color	beige (4), light brown (15)
Weight without footshell*	26.49 oz / 751 grams
Weight with normal footshell*	34.5 oz / 980 grams
Build height with normal footshell*	7 1/4 in / 185 mm
Other features	+/- 10° torsion movement; up to 15 mm vertical shock absorption

*Reference size is 26 cm.



Taleo Vertical Shock 1C51



Order form


Customer		Shipping address (if different from customer address)	
Customer no.	<input type="text"/>	Customer no.	<input type="text"/>
Company	<input type="text"/>	Company	<input type="text"/>
Street	<input type="text"/>	Street	<input type="text"/>
Postal code/city	<input type="text"/>	Postal code/city	<input type="text"/>
O&P professional	<input type="text"/>	PO no.	<input type="text"/>

Selection of the spring stiffness relative to

1 body weight and activity as well as

Body weight lbs [kg]	Normal activity level	High activity level
Up to 113 [51]	1	2
114 – 129 [52 – 58]	2	3
130 – 149 [59 – 67]	3	4
150 – 170 [68 – 77]	4	5
171 – 195 [78 – 88]	5	6
196 – 220 [89 – 100]	6	7
221 – 254 [101 – 115]	7	8
255 – 287 [116 – 130]	8	9
288 – 330 [131 – 150]	9	-

2 foot size

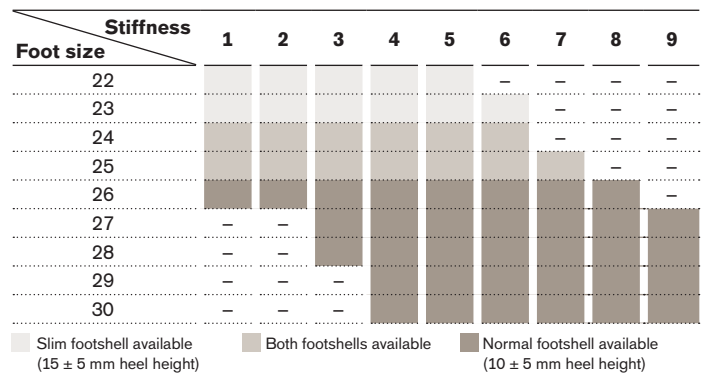

Example for ordering

1C51=R26-4-P/4N

Article no.	Side	Size	Stiffness	P /	Color	Shape
1C51=				P /		
	Right	22, 23, ..., 30		4	Beige	Slim
	Left			15	Light brown	Normal



The scope of delivery includes the foot module 1C51, the 2C15 footshell including the connection cap, the 2F50 heel wedge set (comprising three different degrees of hardness), a 2Z362 functional ring set for replacement, a pre-compression kit and a black Spectra sock.

Recommended combinations**

TT	TF
6Y95=* Caleo 3D liner & 4R140 One-way valve & 453A30/40=* ProFlex Plus knee sleeve or 6Y512=* Uneo 3D liner & 4R136 One-way valve & 453A30/40=* ProFlex Plus knee sleeve	7T451=1 SiOXC TF Pro or Laminated socket & 6Y110 Skeo Sealing liner & 21Y21 ClickValve
Aluminum line adapters	3R80 3C98* C-Leg 3B1* Genium 3B5* Genium X3
	4R10=111 Quickchange 4R57=WR Waterproof rotation adapter
	3F1=1 + 99B120=* Functional Cosmesis for C-Leg 4 3F1=2 + 99B120=* Functional Cosmesis for Genium

** All components are sold separately and are available Ottobock products that are compatible with the 1C51 Taleo Vertical Shock foot, which helps ensure optimal performance. Practitioners need to select components based upon individual patient criteria.