

4R10=111 Quickchange™

Just Click, Twist and Go



The Quickchange adapter allows the user to easily and quickly remove the foot from the prosthesis or be able to switch to a secondary foot for specialty applications. For example, they may need a prosthetic foot that has additional functions or walking properties or is waterproof. Or they may want a different heel height or a wider selection of shoes, or they may also want to play sports. With the Quickchange adapter, amputees can change their prosthetic foot by themselves when needed in just one step. Removing the foot can also help in tight situations or when changing clothes.

1 The upper section

is mounted below the prosthetic socket or knee joint. Magnets help with positioning and establish an initial connection between the lower section of the adapter and the distal end of the prosthesis.

2 The lower section

is mounted above the prosthetic foot. Two lower sections are included in the scope of delivery so different feet can be used for one prosthesis. If additional prosthetic feet are to be used, the lower section is available as an optional spare part (4R10=01).

3 The locking pin

establishes the final connection between the upper and lower sections. When switching feet, the user simply hand-tightens the pin. It is available as a spare part (4X345).



Advantages

- Free choice
Prosthesis users can use feet with different functions.
- Self-Aligning
The prosthetic alignment is not altered by using the Quickchange.
- Easy removal
Removing the foot can also help in tight situations or when changing clothes.
- Easy to use
Prosthesis users can handle the Quickchange on their own with no tools required.
- Safety
The Quickchange has been tested according to ISO 10328. The locking pin ensures a secure hold until the next foot change.

Technical data

Mobility grade	2 – 4
Max. body weight*	275 lbs (125 kg)
Field of application	TT, TF/KD/HD below the knee joint
Proximal connection	Pyramid adapter
Distal connection	4-hole connection
Weight	8 oz (220 g)
Part Clearance	1¼" (31 mm)
Material	Aluminium
Waterproof	Fresh and Salt waterproof
Selection of possible combinations**	Genium X3, Genium, Triton family, Challenger

* in combination with certain prosthetic feet, the maximum allowable body weight can differ.

** All components sold separately. These available Ottobock products are compatible with the Quickchange, to help ensure optimal performance. Practitioners need to select components based upon individual patient criteria.