
**ottobock.**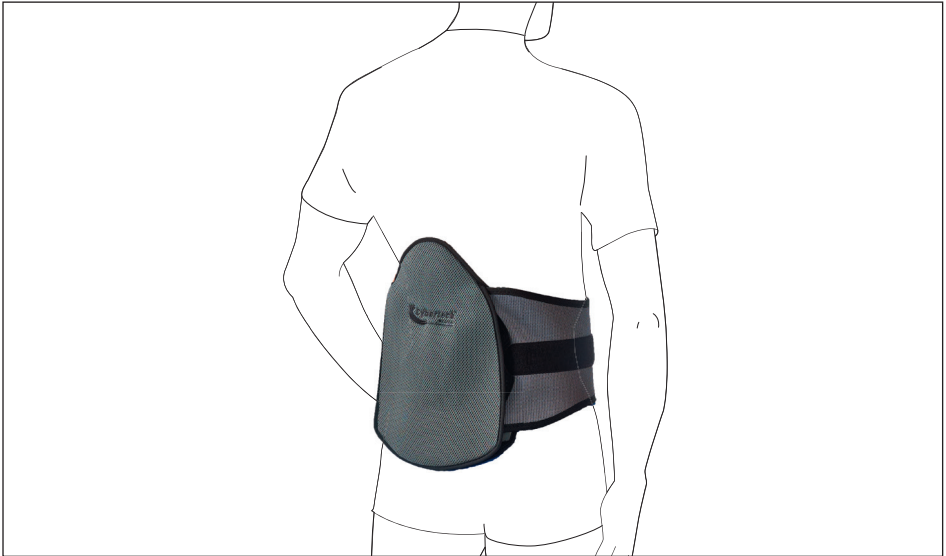

**50R215=\* Passport LSO**

<b>EN</b> Instructions for Use .....	2
<b>FR</b> Instructions d'utilisation .....	4

**50R215=\* Passport LSO**

Thank you for using this Ottobock device. The following information will help you fit and care for this device. Please review it before using this device and ask your physician, your healthcare provider, or your practitioner for clarification if needed.

**Indications for Use:**

Indicated for degenerative and bulging discs, lumbar sacral mechanical back pain, Spinal Stenosis, Spondylolisthesis, Facet Syndrome, herniated/bulging discs, post-operative stabilization protocol following Laminotomy, Foraminotomy, IDET procedures, anterior laproscopic fusion, posterior lateral fusion, and multi-level decompression.

**Warning**

This device does not prevent injury and is not intended to reduce or eliminate the risk of injury.

**Warning**

Always consult with your physician or healthcare provider before making changes to the device.

**Warning**

Carefully read all instructions for use before using or fitting this device.

**Warning**

Proper rehabilitation and activity modification are also a part of a safe treatment program.

**Warning**

If you experience pain, swelling, or any unusual reaction, contact your physician or healthcare provider immediately.

**CAUTION**

This device is offered under order of a physician or other qualified healthcare provider.

**CAUTION**

This device is intended for single patient use and is not intended to be reused on a second patient.

## **Application Instructions**

Please follow all prescribing physician's orders for frequency and duration of use

1. Locate the handle on the left side of the device and fasten the handle near the pulley unit.
2. While standing in an upright position, hold the side panels apart, and center the back panel of the device along the spine just above the tailbone.
3. Wrap the left side panel to the front to position the front panel. Then wrap the right side panel on top of the left side panel to close. Attach both sides in front. Adjust the front or back panels as needed.
4. Pull the handle out and away from the pulley unit to reach desired compression and support. Place the handle on the fabric fastener band securely.

## **Cleaning and Care**

Remove the rigid components. The rigid components can be wiped clean with a damp cloth. Softgoods can be hand washed in warm water (90°F - 110°F or 32°C - 43°C) with mild detergent. Triple rinse, squeeze and air dry. Avoid twisting the material to remove water.

## **Liability/Warranty**

This device is warranted for a period of 90 days. The manufacturer's warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use. Details of the warranty are also available at [www.ottobockus.com](http://www.ottobockus.com).

## **Manufacturer's Limited Warranty**

Otto Bock HealthCare, LP (OttoBock) warrants all of its devices, to the original purchaser, to be free from defects in materials and workmanship. This warranty applies, subject to normal wear and tear, when the devices are used as intended, without unapproved modifications, following all OttoBock instructions and requirements; and when they are fitted by or under the direct supervision of certified/licensed practitioners. This Limited Warranty does not cover device damage caused by accidents, neglect, misuse or operation beyond capacity, parts damaged by improper installation, substitution of parts not approved by OttoBock, or any alteration or repair by others that, in OttoBock's judgment, materially or adversely affect the device.

The duration of this Limited Warranty varies by product types and is effective from the date of delivery to the end-user. Please refer to [www.ottobockus.com](http://www.ottobockus.com) or call 800 328 4058 for questions. OttoBock's sole obligation under this Limited Warranty shall be to repair, replace, refabricate the device at no charge, or refund the cost of the device to the original purchaser, at OttoBock's sole discretion.

THE EXPRESS WARRANTIES SET FORTH ABOVE ARE IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, AND ALL SUCH OTHER WARRANTIES ARE HEREBY DISCLAIMED AND EXCLUDED BY OTTOBOCK. IN NO EVENT SHALL OTTOBOCK'S LIABILITY OF ANY KIND INCLUDE ANY SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, EVEN IF OTTOBOCK SHALL HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH POTENTIAL LOSS OR DAMAGE.

Some states do not allow the exclusion of incidental or consequential damages, thereby rendering the aforementioned limitation in applicable to certain original purchasers.

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