

EasyFit

The donning sheath makes donning a prosthesis easier

A blue donning sheath and a yellow elastic strap are shown against a light blue background with a white dot pattern. The sheath is a long, rectangular piece of fabric with a loop at one end. The strap is a long, thin, yellow elastic band with a textured surface.

Quality for life



EasyFit

We want you to be able to put on your prosthesis every day as easy as possible. Therefore, our EasyFit donning sheath facilitates donning of transfemoral prostheses featuring a valve opening. Its lightweight material is especially smooth. And if you want donning to be even easier, you can extend the donning sheath simply with the pull-through aid included.

Quick and effortless Donning sheath and pull-through aid

Your residual limb slides effortlessly into the socket with the EasyFit donning sheath. You then pull the donning sheath outwards through the valve opening. It is even easier using the pull-through aid. The strengthened strap can be easily guided through the valve opening. It also extends the donning sheath, so that you have to bend over less when donning the prosthesis.

Benefits at a glance

- Easy, time and energy saving donning
- Comfortable feeling when donning
- Pull-through aid for additional help
- High-quality, durable material
- Suitable for machine washing
- Lightweight and foldable, includes a sealable bag for easy transportation

The EasyFit donning sheath is available in the following sizes

Required length (in cm) up to ...	Residual limb circumference (in cm)		Size	Article number
	distally up to ...	proximally up to ...		
40	25	39	XS	OC1560=XS
43	27	43	S	OC1560=S
43	29	56	M	OC1560=M
43	36	68	L	OC1560=L
43	36	78	XL	OC1560=XL
60	36	80	XXL	OC1560=XXL

How it works

1. Spread out the EasyFit donning sheath and the included pull-through aid.
2. Turn one half of the EasyFit donning sheath in to the other half to the lower seam. The strap should face outwards.
3. For additional help attach the pull-through aid: pull the strap on the donning sheath through the loop on the pull-through aid.
4. Pull the top half of the donning sheath through the lower gap between the strap and the loop.
5. The pull-through aid and the strap of the donning sheath are now tied to each other.
6. Now pull the EasyFit donning sheath with both hands like a sock over the transfemoral residual limb, avoiding any wrinkles. Position the socket over the residual limb so that after getting into the socket the strap of the EasyFit donning sheath or the pull-through aid can be put through the socket's valve hole.
7. Finally, use the strap to slowly pull the EasyFit donning sheath out of the prosthetic socket through the valve opening.

