

## 21B37=\*

<b>DE</b> Gebrauchsanweisung .....	3
<b>EN</b> Instructions for use .....	5
<b>FR</b> Instructions d'utilisation .....	8
<b>IT</b> Istruzioni per l'uso .....	11
<b>ES</b> Instrucciones de uso .....	14
<b>PT</b> Manual de utilização .....	17
<b>NL</b> Gebruiksaanwijzing .....	20
<b>SV</b> Bruksanvisning .....	23
<b>DA</b> Brugsanvisning .....	26
<b>NO</b> Bruksanvisning .....	28
<b>FI</b> Käyttöohje .....	31
<b>PL</b> Instrukcja użytkowania .....	34
<b>HU</b> Használati utasítás .....	37
<b>CS</b> Návod k použití .....	40
<b>RO</b> Instrucțiuni de utilizare .....	43
<b>HR</b> Upute za uporabu .....	46
<b>SL</b> Navodila za uporabo .....	48
<b>SK</b> Návod na používanie .....	51
<b>BG</b> Инструкция за употреба .....	54
<b>TR</b> Kullanma talimatı .....	57
<b>EL</b> Οδηγίες χρήσης .....	60
<b>RU</b> Руководство по применению .....	63
<b>JA</b> 取扱説明書 .....	66
<b>ZH</b> 使用说明书 .....	68
<b>KO</b> 사용 설명서 .....	71


1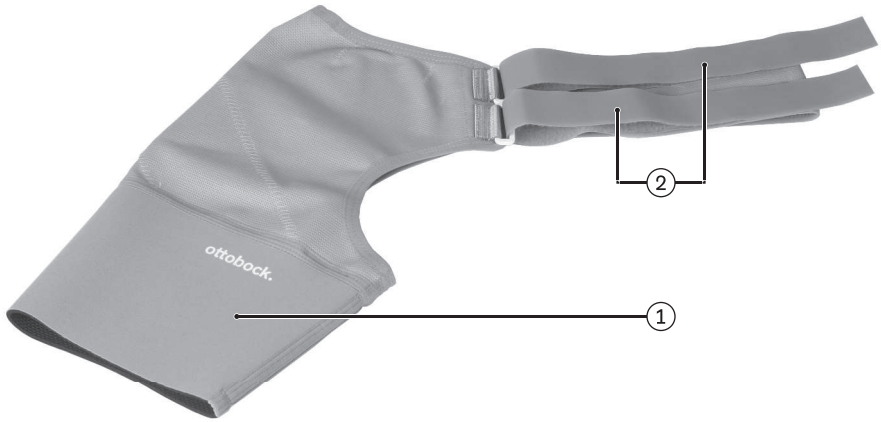

2

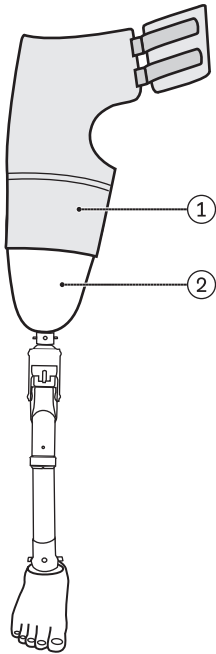3
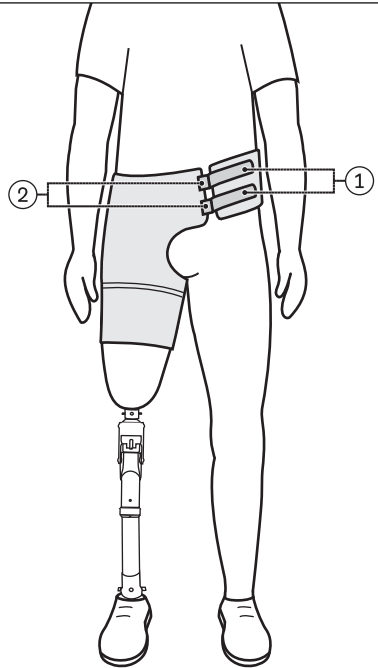
## INFORMATION

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### INFORMATION

Last update: 2017-03-29

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

The 21B37=\* Thigh Support Bandage secures the TF prosthesis to the patient.

It reduces rotational and lift movements between the prosthetic socket and the residual limb. Guiding and control of the prosthesis, the transfer of force and fixation to the body are all improved.

## 2 Safety

### 2.1 Explanation of warning symbols

**CAUTION** Warning regarding possible risks of accident or injury.

**NOTICE** Warning regarding possible technical damage.

### 2.2 General safety instructions

**CAUTION**

#### Reuse on other persons and improper cleaning

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

**NOTICE**

#### Improper use and changes

Change in or loss of functionality as well as damage to the product

- ▶ Use this product with care and only for its intended purpose.
- ▶ Do not make any improper changes to the product.

## 3 Scope of delivery

The following single components and accessories are included in the scope of delivery in the specified quantities and are available for reordering as single components (■), single components with minimum order quantities (▲) or single-component packs (●):

Fig.		Item no.	Quantity	Designation	Reference number
1	■	①	1	Thigh support bandage	21B37=*
1	-	②	2	Pelvic belt with hook and loop closure	-
-	■	-	1	Instructions for use	647G1072

## 4 Use

### 4.1 Information for everyday use

**CAUTION**

#### Use of the product on skin sensitive to heat

Skin irritation due to over-sensitivity to heat

- ▶ Do not use the product for more than 4 hours without a break.

### **⚠ CAUTION**

#### **Contact with heat, embers or fire**

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

### **NOTICE**

#### **Contact with salves, lotions or other products that contain oils or acids**

Insufficient stabilization due to loss of material functionality

- ▶ Do not expose the product to salves, lotions or other products that contain oils or acids.

### **NOTICE**

#### **Use of a worn or damaged product**

Limited effectiveness

- ▶ Before each use, check the product for functional reliability and for possible wear or damage.
- ▶ Do not continue using a product that is no longer functional, or that is worn or damaged.

## **4.2 Putting on to the prosthetic socket**

- 1) **NOTICE! Do not damage the product or other prosthetic components when putting the bandage on.**

**INFORMATION: The standard direction to put the bandage on is from the distal end over the prosthetic foot.**

Pull the support bandage ① on to the prosthetic socket ② (see fig. 2).

- 2) Check the positioning on the prosthetic socket and correct it if necessary.

## **4.3 Donning the prosthesis**

### **⚠ CAUTION**

#### **Incorrect or excessively tight application to the body**

Falling due to incorrect application, pressure points due to tight application

- ▶ Ensure that the product is applied properly and fits correctly.
- ▶ **Inform the patient.**

- 1) Slide the residual limb into the prosthetic socket.
- 2) Pull the pelvic belts ① through the eyelets ② (see fig. 3).
- 3) Tighten the hook and loop closures and secure them to the pelvic belt such that the support bandage sits securely above the hip.
- 4) **CAUTION! Adjust the positioning of the hook and loop closures immediately if pressure points appear or the support bandage sits too loosely when worn.**

## **4.4 Cleaning**

### **NOTICE**

#### **Use of improper cleaning agents**

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

- ▶ **NOTICE! Do not damage the product or other prosthetic components when taking the bandage off.**

**INFORMATION: The standard direction to take the bandage off is towards the distal end over the prosthetic foot.**

Remove the product from the prosthetic socket.

- ▶ Do not bleach the product.
- ▶ Do not dry clean the product.
- ▶ Do not use fabric softener.
- ▶ **NOTICE! Fasten all hook and loop closures before washing.**
- ▶ **NOTICE! Use a laundry bag or net when washing.**
- ▶ Use a mild detergent when washing.
- ▶ Wash the product at 40 °C/104 °F in a washing machine (setting: easy care or delicates).
- ▶ Do not dry the product in a tumble dryer.
- ▶ Do not expose to direct heat (e.g. sunshine, stove or radiator) in order to dry the product.
- ▶ Allow the product to air dry.
- ▶ Do not iron the product.
- ▶ Put the dried product on to the prosthetic socket (section "Putting on to the prosthetic socket" - see Page 7).

## 5 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

## 6 Legal information

### 6.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 6.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

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Otto Bock HealthCare GmbH