FIT KIT for WalkOn Reaction Line

28U24=*/28U25=*

Quality for life

FIT KIT for WalkOn Reaction Line

DE	Gebrauchshinweise	4
EN	Instruction folder	6
IT	Vedere allegato	8
ES	Carpeta de instrucciones	10
NL	Instructiefolder	12
NO	Bruksanvisning	14
SV	Instruktionsfolder	16
DA	Brugsanvisning	18
CS	Pokyny pro použití	20
FI	Ohjekansio	22

Best.-Nr. 28T3/28T4/28T5

FIT KIT für WalkOn Reaction Line

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Art. no. 28T3/28T4/28T5

FIT KIT for WalkOn Reaction Line

This FIT KIT is testing equipment designed to facilitate the correct choice of a WalkOn AFO (ankle foot orthoses). The objective is to use the test equipment to assess whether the orthosis is appropriate for the patient. The test orthoses are intended to evaluate whether the size is correct and the desired effect is achieved.

The FIT KIT includes eight orthoses:

One orthosis is included for each side in the shoe sizes 36, 39, 42 and 45 respectively.

The Junior version includes shoe sizes 24–27; 27–30; 30–33 and 33–36.

Notes

The Orthotic devices in the Fit Kit are not allowed to use in private environment of the user without attendance of professionals. To use the devices as an interim solution is permitted.

How to use the FIT KIT?

- ▶ Choose a suitable product in terms of size and side for your patient.
 - ▶ Place the AFO in the patient's shoe, preferably under any removable insole.
 - ▶ Apply the shoe and product onto the patient.
 - ▶ Let the patient walk a short distance and evaluate the effectiveness of the AFO.
 - ▶ If the desired effectiveness is not yet achieved, you can work with the enclosed heel or forefoot elevating elements in order to obtain better results. In case of supination or inversion of the forefoot, a lateral pronation cable which is included may be advisable in addition.
 - ▶ If the result is positive, fit your patient with an orthosis in the appropriate size.
-

If the patient needs an orthosis with less support, choose the 28U24 WalkOn Reaction with a slightly lower knee extension moment, or a classic 28U11 WalkOn or a 28U23 WalkOn Trimable (a classic WalkOn but with a rubber sole attached to the footplate), or a more flexible orthosis such as the 28U22 WalkOn Flex.

(WalkOn / WalkOn Trimable FIT KIT, article number 28T1N; WalkOn Flex FIT KIT, article number 28T2)

Note

Since the test model is often not fully adapted to the patient, some pressure may be experienced at the medial malleolus during use. This is remedied by an individually adapted orthosis, grinding or cutting according to the fitting instructions, (see the product instructions for use).

Art. no	Side	Shoe Size	Height
28U25=L/R 24-27	Left/Right	24-27	20.5 cm / 8 inch
28U25=L/R 27-30	Left/Right	27-30	24.5 cm / 9.6 inch
28U25=L/R 30-33	Left/Right	30-33	27 cm / 10.6 inch
28U25=L/R 33-36	Left/Right	33-36	29.4 cm / 11.6 inch
28U25=L/R 36-39	Left/Right	36-39	35 cm / 13.8 inch
28U25=L/R 39-42	Left/Right	39-42	36 cm / 14.2 inch
28U25=L/R 42-45	Left/Right	42-45	37.5 cm / 14.8 inch
28U25=L/R 45-48	Left/Right	45-48	39 cm / 15.4 inch

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Otto Bock HealthCare GmbH