## 4X350, 4X350-X3

<b>DE</b>	Gebrauchsanweisung (Benutzer) .....	3
<b>EN</b>	Instructions for use (user) .....	23
<b>FR</b>	Instructions d'utilisation (Utilisateur) .....	45
<b>IT</b>	Istruzioni per l'uso (Utilizzatore) .....	69
<b>ES</b>	Instrucciones de uso (Usuario) .....	90
<b>PT</b>	Manual de utilização (Usuário) .....	111
<b>NL</b>	Gebruiksaanwijzing (Gebruiker) .....	132
<b>SV</b>	Bruksanvisning (Användare) .....	154
<b>DA</b>	Brugsanvisning (Bruger) .....	174
<b>NO</b>	Bruksanvisning (Bruker) .....	193
<b>FI</b>	Käyttöohje (Käyttäjä) .....	212
<b>PL</b>	Instrukcja użytkowania (Użytkownik) .....	231
<b>HU</b>	Használati utasítás (használó) .....	252
<b>CS</b>	Návod k použití (Uživatel) .....	272
<b>SK</b>	Návod na používanie (Užívateľ) .....	292
<b>TR</b>	Kullanma talimatı (Kullanıcı) .....	312

## INFORMATION

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### **INFORMATION**

Last update: 2015-03-04

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Receive instruction in the correct and safe use of the product by qualified personnel.

- ▶ Please contact the qualified personnel if you have questions about the product (e.g. regarding the start-up, use, maintenance, unexpected operating behaviour or circumstances). Contact information can be found on the back page.
- ▶ Please keep this document in a safe place.

## 2 Product Description

It is possible to switch from basic mode to the pre-configured MyModes using the remote control. In addition, information about the prosthesis (step counter, battery charge level, etc.) can be called up.

The behaviour of the prosthesis can be changed to a certain extent on a day-to-day basis using the remote control (e.g. while becoming accustomed to the prosthesis). At the next appointment, the prosthetist can trace the changes using the adjustment software.

### **INFORMATION**

The language of the user interface (menus) and the configuration and number of MyModes can only be changed/adapted using the adjustment software.

### **INFORMATION**

The connection to the original remote control can no longer be established if the prosthesis is connected to another remote control. As a result, the message "Device disconnected" appears in the display of the original remote control when the connection is established.

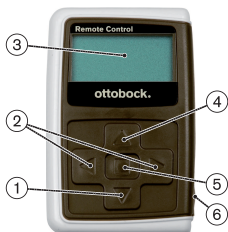
### **INFORMATION**

Bluetooth on the prosthesis must be turned on in order to use the remote control. For more information about the Bluetooth function and whether it can be turned on/off, see the instructions for use of the respective prosthesis.


## 2.1 Design

### 2.1.1 Controls on the remote control

- 1) ▼ Navigation key  
Select menu options/change settings/start to establish connection
- 2) ◀, ▶ Navigation keys  
Call up additional menu options/set, change parameters
- 3) **Display**  
Display of battery charge level of the remote control and of the last connected prosthesis
- 4) ▲ Navigation key  
Select menu options/change settings
- 5) ■ Enter key  
Switch on remote control unit/confirm selection, menu option
- 6) **Connections (under cover cap)**  
Charging receptacle for power supply connection

### 2.1.2 Information shown on remote control display

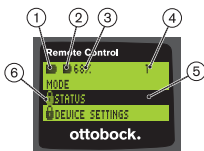1. ■ – Remote control battery charge level
2. ■ – Charge level of battery for currently connected prosthesis
3. 68% – Charge level of battery for currently connected prosthesis, as a percentage
4. 1 – Mode currently set  
"! – Service interval of the prosthesis elapsed
5. Black background – currently selected menu option, e.g.: "Status" or selected function
6. 🔒 – Menu locked  
Certain menus are disabled if, for example, the battery for the prosthesis is being charged.

To leave a menu without saving changes, use the ▼ key to select the symbol ▲, and confirm this with the ■ key.

## 2.2 Combination Possibilities

### Prosthetic feet

- 1B1 Meridium

### Knee joints

- 3C98-3, 3C88-3 C-Leg
- 3B1, 3B1=ST Genium
- 3B1-2, 3B1-2=ST Genium
- 3B5-X3, 3B5-X3=ST Genium X3
- 3B5-2, 3B5-2=ST Genium X3

## 3 Application

### 3.1 Indications for use

The remote control is to be used **exclusively** for setting up and configuring Ottobock prostheses and prosthetic components. The connection is established via Bluetooth.

### 3.2 Conditions of use

- The battery of the prosthesis must be charged.

Permissible ambient conditions are described in the technical data (see Page 44).

## 4 Safety

### 4.1 Explanation of Warning Symbols

**CAUTION** Warning regarding possible risks of accident or injury.

**NOTICE** Warning regarding possible technical damage.

### 4.2 Structure of the safety instructions

**CAUTION**

#### **The heading describes the source and/or the type of hazard**

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in case of failure to observe the hazard
- > E.g.: Consequence 2 in case of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

### 4.3 General Safety Instructions

#### CAUTION

##### **Improper use of remote control**

Falling due to altered damping behaviour as a result of unexpected switching into MyMode.

- ▶ Receive instruction in the proper use of the remote control unit.

#### CAUTION

##### **Independent user changes or modifications made to remote control unit**

Falling due to changed damping behaviour caused by unexpected switching to a MyMode.

- ▶ Have any changes or modifications to the remote control carried out only by Ottobock authorised, qualified personnel.

#### CAUTION

##### **Improper mode switching using remote control**

Falling due to unexpected behaviour of the prosthesis because of changed damping behaviour.

- ▶ Ensure that you stand securely during all switching processes.
- ▶ Verify the changed damping characteristics after switching, and observe the feedback from the acoustic signal emitter and the display of the remote control.
- ▶ Switch back to basic mode once the activities in MyMode have been completed.

#### NOTICE

##### **Remote control damage due to water penetration (warranty lapses)**

Malfunction of the remote control.

- ▶ If necessary, let the remote control unit dry at room temperature (at least 1 day).

- The remote control unit should be inspected by the prosthetist before it is used again. If necessary, the prosthetist will send the unit to an authorised Ottobock Service Centre.

## 5 Scope of Delivery and Accessories

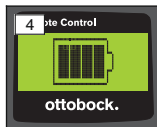
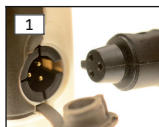
### Scope of Delivery

- 1 pc. 4X350 Remote Control or  
1 pc. 4X350-X3 Remote Control
- 1 pc. 647G1137 instructions for use (user)

## 6 Charging the battery

The following points must be observed when charging the battery:

- The battery should be charged for at least 3 hours prior to initial use.
- Use the 757L16\* power supply to charge the battery.

- 1) Slide the country-specific plug adapter onto the power supply until it locks into place.
- 2) Open the charging receptacle cover of the remote control unit.
- 3) Plug the power supply cable into the charging receptacle of the remote control until the plug connector of the cable snaps into place (see fig. 1).  
**INFORMATION: Ensure correct polarity (guide lug). Do not use force when connecting the cable plug to the charging receptacle of the remote control.**
- 4) Plug the power supply unit into the outlet (see fig. 2).
  - The green LED on the back of the power supply lights up.
  - The charging process starts.  
The current charge level of the remote control battery is shown on the display.  
Battery empty (see fig. 3), battery fully charged (see fig. 4). This symbol flashes as the battery is charging.
  - If the green LED on the power supply does not light up, there is an error (see Page 44).
  - The battery is fully charged after approx. three or four hours, after which the symbol in the display remains permanently lit up.

- When the charging symbol on the display remains permanently lit up (see fig. 4), the battery is fully charged (3–4 hours). Unplug the charging cable from the charging receptacle.
- Close the charging receptacle cover.

### 6.1 Display of the current charge level

The current charge level is displayed in the status bar when the remote control is switched on. If the remote control is connected to a prosthesis, the charge level of the prosthesis battery is shown as well.

- Remote control battery charge level
  - Charge level of battery for currently connected prosthesis
  - Prosthesis is being charged
- 68% – Charge level of battery for currently connected prosthesis, as a percentage

## 7 Preparation for Use

### 7.1 Initial connection between remote control and prosthesis

#### INFORMATION

The prosthesis serial number is needed to connect the remote control to the prosthesis. This begins with the letters SN and is found in the following position depending on the prosthesis:

Knee joints: On the inside of the frame.

Prosthetic feet: On the spherical cap.

- Use the key to activate the remote control unit.
  - The start-up screen appears and shows the battery charge level of the remote control, as well as the date and time (see fig. 5).
- Press the key to start the search for prostheses (see fig. 6).

**INFORMATION: The remote control automatically switches off after 5 seconds if no other keys are pressed.**

- The serial number of the prosthesis that was found appears in the display (see fig. 7).

- If names of prostheses appear in the display, prostheses are already saved in the remote control. Continue with the steps listed in section "Managing prostheses (see Page 35), part 'Adding new (prosthesis) connection' (menu item 'Pairing')".
- 3) Compare the serial number that is displayed to the serial number of the prosthesis you want to connect to.  
If the serial number 00000000 is displayed (multiple prostheses in range), use the ◀, ▶, ▲, ▼ keys to enter the serial number of the prosthesis you want to connect to.
- 4) Confirm with the ■ key.
- Once the connection has been established successfully, the data are read from the prosthesis (see fig. 8). This process may take a few minutes.
  - The main menu is then displayed on the remote control and includes the name of the connected prosthesis, the battery charge level of the remote control, the charge level of the prosthesis and the mode that is currently set (1 = basic mode).
- INFORMATION: The name of the prosthesis can only be entered or changed using the adjustment software.**

#### **INFORMATION**

After the initial connection to the prosthesis has been established successfully, the remote control will connect automatically each time it is turned on (■ key followed by ▼ key). No further steps are required.

## **8 Use**

### **8.1 MyModes**

With the help of adjustment software, the prosthetist can activate and configure MyModes in addition to the basic mode. These modes are intended for specific motion patterns or postures (e.g. inline skating, ...) and can be accessed using the remote control.

Changes to the prosthesis settings can be made as well.

The number of MyModes that can be used depends on the configuration using the adjustment software and on the respective prosthesis.

### 8.1.1 Switching MyModes using the remote control unit

#### INFORMATION

When the menu option "Mode" has the  symbol next to it in the display of the remote control unit, it means that MyMode settings cannot be retrieved.

Once a connection to a prosthesis has been established, the remote control can be used to switch between modes.

- 1) Use the ,  keys to select the menu option "Mode" in the main menu and confirm this with the  key.
- 2) Use the ,  keys to select the mode you want and confirm this with the  key.

**INFORMATION: If the list of modes was called up accidentally, the active mode can be selected again.**

→ A beep signal sounds to confirm the switch. The message "Mod-  
eswitch successful" appears on the remote control unit.

If necessary, switch off the remote control unit by selecting the menu option "RC power off" using the ,  keys, and confirm this with the  key.

### 8.2 Changing prosthesis settings

Once an active connection to a prosthesis has been established, the settings of the MyMode that is currently active can be changed using the remote control.

#### Information for changing the prosthesis settings

- Before changing settings, always check the display of the remote control unit to make sure the correct prosthesis has been selected. Otherwise, parameters could be changed for the wrong prosthesis.
- It is not possible to change prosthesis settings nor to switch to a different mode while the prosthesis battery is being charged. Instead of the  symbol, the  symbol appears in the status bar.
- The prosthetist's setting is located in the middle of the scale. After making adjustments, you can restore this setting by moving the slider control into the middle.

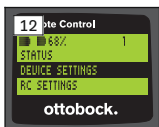
- Prosthesis settings should be optimised using the adjustment software. The remote control is not intended for use by the prosthetist to set up the prosthesis. The behaviour of the prosthesis can be changed to a certain extent on a day-to-day basis using the remote control (e.g. while becoming accustomed to the prosthesis). The prosthetist can use the adjustment software to trace these changes at the next appointment.
- If the settings of a MyMode are to be modified, one must first switch to this MyMode.

## 8.2.1 Using remote control to change prosthesis settings

### INFORMATION

The number and names of the configurable parameters depend on the respective prosthesis. Therefore, the illustrations of menus in this section are by way of example only.

Further information on the parameters is found in the instructions for use of the respective prosthesis.

- 1) Use the ▼, ▲ keys to select the menu option "Device settings" in the main menu and confirm this with the ■ key.
  - A list appears with the parameters for the currently active mode (see fig. 13).
- 2) Use the ▼, ▲ keys to select the parameter you want and confirm this with the ■ key.
  - A scale will appear in the display of the remote control and shows the value currently set for the parameter as well as the parameter's limit values (see fig. 14).
- 3) Use the ◀, ▶ keys to change the parameter setting and confirm this with the ■ key.
 

To leave the menu without saving changes, use the ▼, ▲ keys to select the ◀ symbol and confirm this with the ■ key.

  - The message "Successful" appears briefly to confirm these changes were saved in the prosthesis.
- 4) If necessary, switch off the remote control unit by selecting and confirming the menu option "RC power off".

In order to change the settings of another mode, you first have to switch into that mode.

### 8.3 Querying the prosthesis status

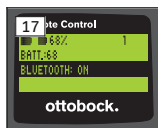
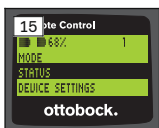
#### INFORMATION

The entries and names in the **Status** menu depend on the respective prosthesis. Therefore, the illustration of the menu and the description in this section is by way of example only.

Further information is found in the instructions for use of the respective prosthesis.

#### INFORMATION

This function is not available while the battery of the prosthesis is being charged. The  symbol appears in front of the "Status" menu option.



- 1) Once a connection has been established, use the ▲, ▼ keys to select the menu option "Status" in the main menu and confirm this with the ■ key.
- 2) Use the ▲, ▼ keys to select the following information:

Menu option	Description	Possible actions
Trip:1747	Daily step counter	Reset the counter by using the ■ key to confirm the menu option
Step:1747	Total step counter	For informational purposes only
Batt.:68	Current prosthesis charge level, as a percentage	For informational purposes only
Bluetooth: On	Switch Bluetooth function of the prosthesis on or off	The Bluetooth function of the prosthesis can be turned on or off (see Page 38) by confirming


Menu option	Description	Possible actions
		the menu item with the  key.

## 8.4 Changing remote control settings

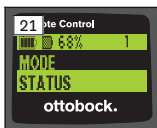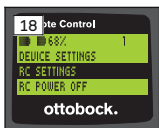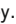- 1) Use the ▲, ▼ keys to select the menu option "RC settings" in the main menu and confirm this with the  key.
- 2) Use the ▲, ▼ keys to select from the following settings:



### Changing time and date (menu option "16:22/06.02.14 or PM 4:22/02-06-14")

- 1) Press the  key.
- 2) Use the ◀, ▶ keys to select the appropriate option. You can use the ▲, ▼ keys at the beginning of the line to toggle between 12-hour mode and 24-hour mode.
- 3) Use the ▲, ▼ keys to change the option.
- 4) Press the  key to save the changes and leave the menu.

### Establishing connection to prosthesis or setting up additional connections (menu option "Pairing")

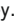Up to four prostheses can be managed using this remote control (see Page 35).

### Change font size on remote control display (menu option "Zoom")

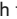- 1) Press the  key.
- 2) Use the ◀, ▶ keys to select the font size you want (see fig. 20):  
 "1" – normal font  
 "2" – large font
- 3) Press the  key to save your changes.  
 To leave the menu without saving changes, use the ▼ key to select the  symbol and confirm this with the  key.

### **Deleting connection to prosthesis (menu option "Delete connection")**

Prosthesis connections that were already stored can be deleted. See next section.

### **Calling up remote control information (menu option "Info")**

This menu is for informational purposes only. No changes can be made.

The menu can be used to retrieve information on the firmware version of the remote control and the signal strength of the Bluetooth connection to the prosthesis.

- 1) Press the **■** key.
- 2) Use the **▼**, **▲** keys to select from the following options:  
"RC version: 1.0.0.41.0.0.4" – Firmware version of remote control  
"Signal:" – Signal strength of Bluetooth connection (distance from remote control unit to prosthesis)  
\*: low signal strength/\*\*: medium signal strength/\*\*\*: good signal strength
- 3) To leave the menu without saving changes, use the **▼** key to select the **▲** symbol and confirm this with the **■** key.

## **8.5 Managing prostheses**

Connections with up to four different prostheses can be stored in this remote control. However, a prosthesis can only be connected to one remote control at any given time.

### **INFORMATION**

The connection to the original remote control can no longer be established if the prosthesis is connected to another remote control. As a result, the message "Device disconnected" appears in the display of the original remote control when the connection is established.

### **INFORMATION**

An already existing connection to a prosthesis is terminated when a search is executed for available prostheses. This connection must be re-established manually after the search is completed. It is not possible to automatically connect to the prosthesis.

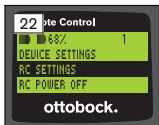
## Adding new (prosthesis) connection (menu option "Pairing")

### INFORMATION

The prosthesis serial number is needed to connect the remote control to the prosthesis. This begins with the letters SN and is found in the following position depending on the prosthesis:

Knee joints: On the inside of the frame.

Prosthetic feet: On the spherical cap.

- 1) Use the ▼, ▲ keys to select the menu option "RC settings" in the main menu and confirm this with the ■ key (see fig. 22).
- 2) Use the ▼, ▲ keys to select the menu option "Pairing", then confirm this with the ■ key to start the prosthesis search (see fig. 23).
  - If a prosthesis was found within the range, its serial number appears in the display (see fig. 24).
- 3) Compare the serial number that is displayed to the serial number of the prosthesis you want to connect to.  
If the serial number 000000000 is displayed (multiple prostheses within the range), use the ◀, ▶, ▲, ▼ keys to enter the serial number of the prosthesis you want to connect to.
- 4) Press the ■ key.
  - Once the connection has been established, the data are read from the prosthesis (see fig. 25). This process may take a few minutes. The blue LED will flash during the data transfer process.
- 5) Use the ▼, ▲ keys to select one of the four items to which the prosthesis should be saved and confirm this with the ■ key.



**INFORMATION: In addition, a prosthesis (item) that already exists can be overwritten.**

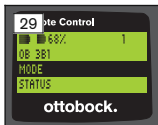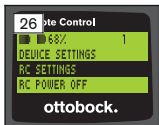- Data are read from the prosthesis and saved to the selected item.
- The prosthesis is then displayed on the main menu.

## Delete existing connection (menu option "Delete connection")

### INFORMATION

The already existing connection can also be deleted. This connection must be re-established manually after the search is completed.

- 1) Use the ▼, ▲ keys to select the menu option "RC settings" in the main menu and confirm this with the ■ key (see fig. 26).
- 2) Use the ▼, ▲ keys to select the menu option "Delete connection" and confirm this with the ■ key (see fig. 27).  
To leave the menu without saving changes, use the ▼ key to select the ⬅ symbol and confirm this with the ■ key.
- 3) Use the ▼, ▲ keys to select the connection you want to delete and confirm this with the ■ key (see fig. 28).
  - The connection is deleted.
  - The main menu is then displayed (see fig. 29).

## 8.6 Changing connections

If more than one prosthesis is stored, a user can toggle between the stored prostheses.

### 8.6.1 Changing connections (with remote control switched off)

- 1) Use the ■ key to activate the remote control.
- 2) Press the ▼ key to start a prosthesis search.
  - All prostheses that are already stored are displayed. The currently connected prosthesis is already selected (see fig. 30).
- 3) Use the ▼, ▲ keys to select the prosthesis required.
- 4) Confirm with the ■ key (see fig. 31).

- Once the connection has been established successfully, the data are read from the prosthesis (see fig. 32). This process may take a few minutes.

Once the connection has been established successfully, the main menu appears with the name of the connected prosthesis (see fig. 33).

### 8.6.2 Changing connections (with remote control switched on)

- 1) Select the name of the currently connected prosthesis in the main menu, using the ▲ key.
- 2) Confirm with the ■ key (see fig. 34).  
→ All prostheses that have already been stored are displayed. The currently connected prosthesis is already selected.
- 3) Use the ▼, ▲ keys to select the prosthesis required.
- 4) Confirm with the ■ key (see fig. 35).  
→ Once the connection has been established successfully, the data are read from the prosthesis. This process may take a few minutes (see fig. 36).

Once the connection has been established successfully, the main menu appears with the name of the connected prosthesis (see fig. 37).

### 8.7 Turning Bluetooth on the prosthesis on/off

Depending on the respective prosthesis, Bluetooth can be turned on/off using the remote control. Further information on the Bluetooth function and whether it can be turned on/off is found in the instructions for use of the respective prosthesis.

- 1) When a connection is established, use the ▲, ▼ keys to select the "Status" menu option in the main menu and confirm this with the ■ key.
- 2) Use the ▲, ▼ keys to select the menu option "Bluetooth". The current status is displayed.
- 3) By pressing the ■ key, Bluetooth is either turned on permanently (On) or turned off (Off).  
If Bluetooth is to be switched off, the remote control must be switched off after the ■ key has been pressed.
- 4) To leave the menu without saving changes, use the ▼ key to select the ⏪ symbol and confirm this with the ■ key.

## 9 Cleaning

- 1) Clean the remote control with a damp cloth and mild soap (e.g. Ottobock 453H10=1 Derma Clean) when needed.  
Ensure that no liquid penetrates into the remote control.
- 2) Dry the remote control with a lint-free cloth and allow it to air dry fully.

## 10 Legal Information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 10.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

### 10.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

### 10.3 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

This product meets the requirements of the European Directive 1999/5/EC for radio equipment and telecommunications terminal equipment. The conformity assessment was drawn up by the manufacturer in accordance with Annex IV of the directive.

### 10.4 Local Legal Information

Legal information that applies **exclusively** to specific countries is written in the official language of the respective country of use in this chapter.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1) This device may not cause harmful interference, and
- 2) This device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/ TV technician for help.

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**Caution: Exposure to Radio Frequency Radiation.**

This device must not be co-located or operating in conjunction with any other antenna or transmitter.

Responsible party:

Otto Bock Health Care, LP  
3820 West Great Lakes Drive  
Salt Lake City, Utah 84120-7205 USA  
Phone + 1-801-956-2400  
Fax + 1-801-956-2401

This device complies with RSS 210 of Industry Canada.

Operation is subject to the following two conditions:

- (1) this device may not cause interference, and

(2) this device must accept any interference, including interference that may cause undesired operation of this device.

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**Caution: Exposure to Radio Frequency Radiation.**

The installer of this radio equipment must ensure that the antenna is located or pointed such that it does not emit RF field in excess of Health Canada limits for the general population; consult Safety Code 6, obtainable from Health Canada's website

<http://www.hc-sc.gc.ca/rpb>.

Responsible party:

Otto Bock Healthcare Canada Ltd.

5470 Harvester Road

L7L 5N5 Burlington, Ontario

Canada

Phone + 1-800-665-3327

---

**Caution: Federal law (USA) restricts this device to sale by or on the order of a practitioner licensed by law of the State in which he/she practices to use or order the use of the device.**

## 11 Appendices

### 11.1 Symbols on remote control

Legal manufacturer

LOT:PPPP YYYY WW

Lot number

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe

the instructions of your national authority pertaining to return and collection.

Declaration of conformity according to the applicable European directives and the number of the notified body (0681)

Non-ionising radiation

Compliance with the requirements according to "FCC Part 15" (USA)

Compliance with the requirements according to the "Radiocommunications Act" (AUS)

## 11.2 Operating states/error signals

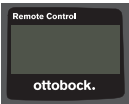
### 11.2.1 Error messages while establishing a connection

Error message	Cause	Correction
Device not found	A previously saved connection to the prosthesis could not be established.	Check the following points: <ul style="list-style-type: none"><li>• Distance from the prosthesis to the remote control</li><li>• Battery charge level of the prosthesis</li><li>• If multiple prostheses were stored in the remote control, was the correct prosthesis selected?</li></ul> <ol style="list-style-type: none"><li>1) Connect the battery charger to the prosthesis, then charge the battery for at least 3 hours. You can try to establish connection again while the battery is charging.</li><li>2) Use the ■ key to confirm the error message and try to establish the connection again.</li></ol>

Error message	Cause	Correction
		3) See section "Managing prostheses" (see Page 35) for further information.
Device disconnected	A current connection to the prosthesis was interrupted.	<p>Check the following points:</p> <ul style="list-style-type: none"> <li>• Distance from the prosthesis to the remote control</li> <li>• Was the prosthesis connected to another remote control?</li> </ul> <p>The connection that was already saved in the remote control must be deleted in order to re-connect the prosthesis to this remote control. A new connection must then be established from the prosthesis to this remote control (see Page 35).</p> <ul style="list-style-type: none"> <li>• Use the <b>■</b> key to confirm the error message and try to establish the connection again.</li> </ul>
No new device found	No new prosthesis could be found since this prosthesis is already stored.	<ol style="list-style-type: none"> <li>1) Confirm error message with the <b>■</b> key.</li> <li>2) Use the <b>▲</b>, <b>▼</b> keys to select the menu option "Select device" and confirm this with the <b>■</b> key.</li> <li>3) Use the <b>▲</b>, <b>▼</b> keys to select the prosthesis required and confirm this with the <b>■</b> key.</li> </ol>

## 11.2.2 Warnings/error signals

### Error while charging the remote control

LED on power supply	Display on the remote control which is shut off	Error	Resolution
○		Country-specific plug adapter not fully engaged on power supply	Check whether the country-specific plug adapter is fully engaged on the power supply.
		Non-functional socket	Check socket with another electric appliance.
		Defective power supply	The power supply must be inspected by an authorised Ottobock Service Centre.
●		Connection from power supply to remote control disrupted	Check whether the charging cable plug is fully engaged on the remote control.
		Defective remote control	The battery charger and remote control must be inspected by an authorised Ottobock Service Centre.

## 11.3 Technical data

Environmental conditions	
Transportation in original packaging	-25°C/-13°F to +70°C/+158°F
Transportation without packaging	-25°C/-13°F to +70°C/+158°F Max. 93% relative humidity, non-condensing
Storage (≤3 months)	-20°C/-4°F to +40°C/+104°F Max. 93% relative humidity, non-condensing

<b>Environmental conditions</b>	
Long-term storage (>3 months)	-20°C/-4°F to +20°C/+68°F Max. 93% relative humidity, non-condensing
Operation	-10°C/+14°F to +60°C/+140°F Max. 93% relative humidity, non-condensing
Charging the battery	+10°C/+50°F to +45°C/+113°F

<b>Remote control</b>	
Article number	4X350
Service life of remote control unit	2 years
Battery type	Li-Ion
Charging time until battery is fully charged	4 hours
Behaviour of remote control unit while being charged	<ul style="list-style-type: none"> <li>• When the remote control unit is switched off, the current charge level of the battery is shown on the display.</li> <li>• When the remote control unit is switched on, the current charge level of the battery is also shown in place of the start-up screen.</li> <li>• Remote control unit functional without restriction.</li> </ul>
Operating time with fully charged battery	approx. 2 months with average use

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Otto Bock Healthcare Products GmbH