
743A9

DE Gebrauchsanleitung (Benutzer / Techniker)	3
EN Instructions for Use (User / Technician)	4
FR Mode d'emploi (Utilisateur / Technicien)	5
IT Istruzioni d'uso (utilizzatore / tecnici)	6
ES Instrucciones de uso (usuario / técnico)	7
NL Gebruiksaanwijzing (gebruiker / technicus)	8
SV Bruksanvisning (användare / tekniker)	9
PT Manual de instruções (Usuário / Técnico)	10
DA Brugsvejledning (bruger / tekniker)	11
NO Bruksanvisning (bruker / tekniker)	12
PL Instrukcja użytkowania (Użytkownik / Technik)	13
CS Návod k použití (pro uživatele / techniky)	14
TR Kullanım kılavuzu (Kullanıcı / Teknisyen)	15
EL Οδηγίες χρήσης (χρήστης / τεχνικός)	16
RU Руководство по применению (для пользователей / технического персонала)	17
JA 取扱説明書 (ユーザー用 / 技術者用)	18

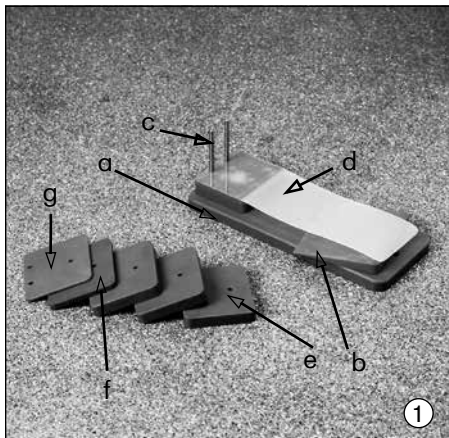INFORMATION

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INFORMATION

Date of the last update: 2019-09-25

- Please read this document carefully.
- Follow the safety instructions.

1 Application

The 743A9 Foot casting aid that facilitates negative plaster cast taking of feet, with different heel heights and adjustable toe pitch in the forefoot.

Field of application: e.g. AFO, KAFO, insoles etc.

2 Components (fig. 1)

- Base plate
- Forefoot wedge
- Guide pin
- Foot support 5 mm, incl. Trolen,
- Heel height plate 20 mm (4 pieces), plastic
- Heel height plate 10 mm (1 piece), plastic
- Heel height plate 5 mm (1 piece), plastic

3 Function and Description

The foot casting aid serves the precise determination of heel height and toe pitch in the forefoot during negative plaster cast taking.

Select different heel height plates to determine the heel height.

INFORMATION

The foot support already has a height of 5 mm!

Maximal heel height is 100 mm (10 cm)

Determine the toe pitch and heel height before taking the plaster cast (fig. 2).

Slightly lubricate the foot support (fig. 3).

Take the plaster cast of the feet in the usual manner (fig. 4-5).

4 Cleaning

If necessary, all components can be cleaned with soap solution and water and then wiped dry. From time to time, lubricate the guide pins with 633G6 Ottobock Special Lubricant.

5 Liability

The manufacturer's warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use.

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