

# ottobock.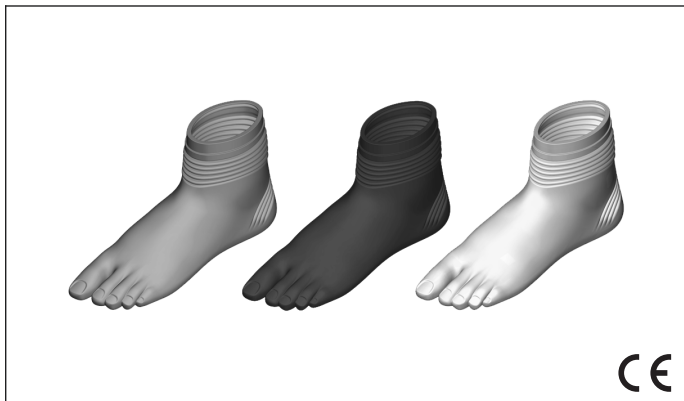


## 2C7

<b>DE</b> Gebrauchsanweisung .....	5
<b>EN</b> Instructions for use .....	12
<b>FR</b> Instructions d'utilisation .....	19
<b>IT</b> Istruzioni per l'uso .....	27
<b>ES</b> Instrucciones de uso .....	35
<b>PT</b> Manual de utilização .....	42
<b>NL</b> Gebruiksaanwijzing .....	50
<b>SV</b> Bruksanvisning .....	57
<b>DA</b> Brugsanvisning .....	65
<b>NO</b> Bruksanvisning .....	72
<b>FI</b> Käyttöohje .....	79
<b>PL</b> Instrukcja użytkowania .....	87
<b>HU</b> Használati utasítás .....	94
<b>CS</b> Návod k použití .....	101
<b>SK</b> Návod na používanie .....	108
<b>TR</b> Kullanma talimatı .....	116
<b>RU</b> Руководство по применению .....	123
<b>JA</b> 取扱説明書 .....	131


1




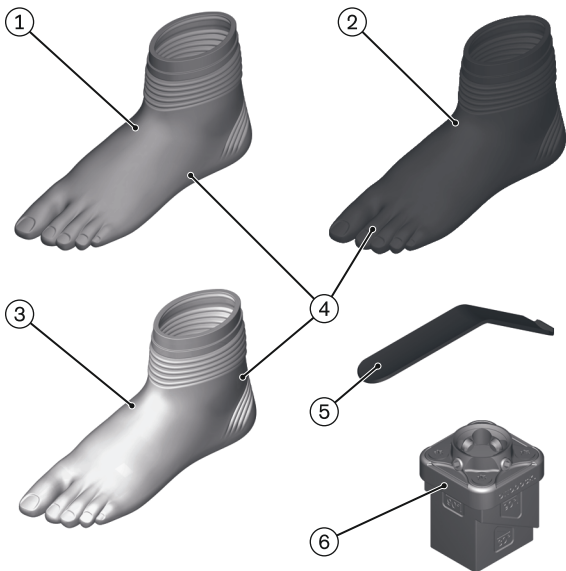








2

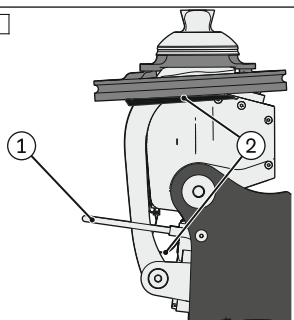3
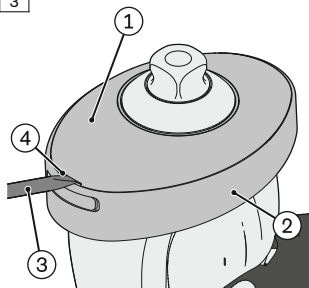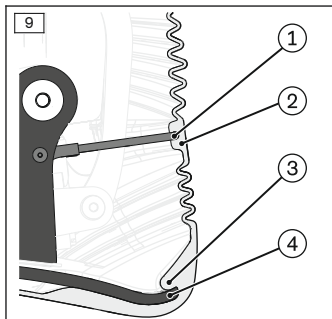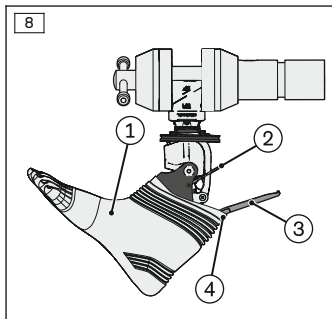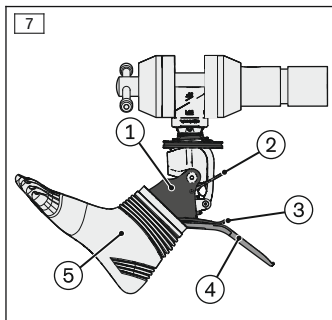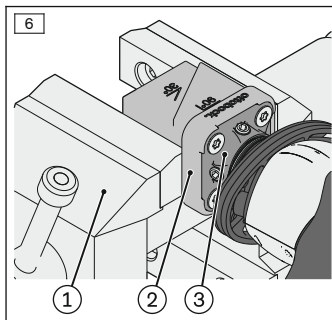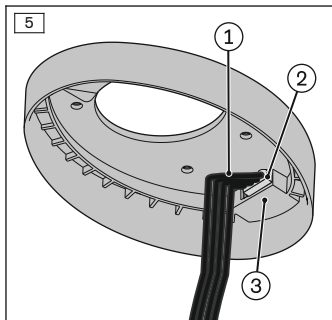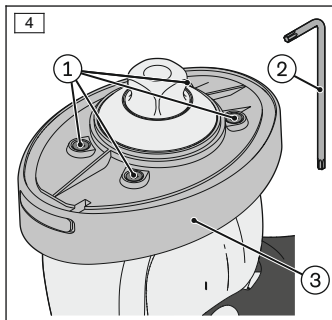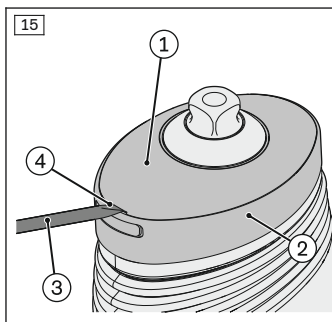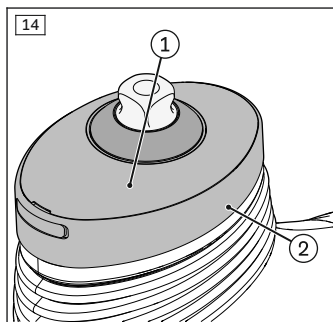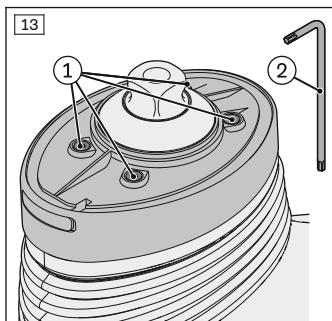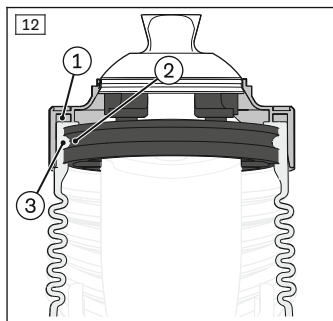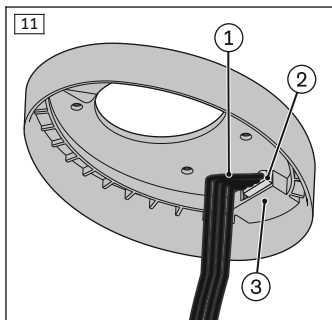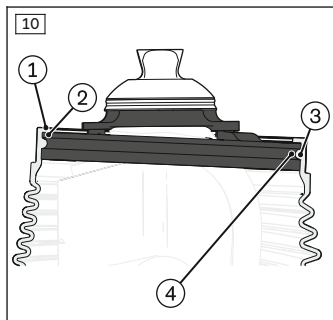
# 1 Produktbeschreibung

## INFORMATION

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### INFORMATION

Date of last update: 2020-04-01

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

### 1.1 Construction and Function

The 2C7\* Footshell is a protective cover for a prosthetic foot.

The footshell was developed especially for the 1B1\* Prosthetic Foot and adapts to all movements of the prosthetic foot.

The product features an abducted big toe.

The toenails are **not** paintable.

## 1.2 Combination possibilities

Designation	Reference number
Prosthetic foot	1B1*

## 2 Intended use

### 2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

### 2.2 Environmental conditions

Allowable environmental conditions
Cleaning agent: damp cloth and pH-neutral soap (e.g. 453H10 Derma Clean)
Barefoot use only in the household environment
Unallowable environmental conditions
Contact with skin care agents (e.g. skin cream, lotion, sunscreen), acids, unallowable cleaning agents, abrasive media (e.g. sand), damp soil, dust, dirt, highly staining materials (e.g. printing ink, new textiles), solvents (e.g. thinner, acetone, benzene), fire, embers and unallowable heat, objects and surfaces that have sharp edges or are pointy, rough or slippery
Perspiration, urine, salt water, acids

### 2.3 Service life

The product has a service life of one year when professionally installed and properly used.

## 3 Safety

### 3.1 Explanation of warning symbols

 <b>CAUTION</b>	Warning regarding possible risks of accident or injury.
 <b>NOTICE</b>	Warning regarding possible technical damage.

### 3.2 General safety instructions

 <b>CAUTION</b>
<b>Unallowable combination of prosthetic components</b>
Risk of injury due to breakage or deformation of the product

- ▶ Only combine the product with prosthetic components that are approved for that purpose.
- ▶ Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

**⚠ CAUTION**

**Exceeding the service life and reuse on another patient**

Risk of injury due to loss of functionality as well as damage to the product

- ▶ Ensure that the approved service life is not exceeded.
- ▶ Only use the product on a single patient.

**⚠ CAUTION**

**Contact with heat, embers or fire**

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

**NOTICE**

**Use under unallowable environmental conditions**

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions.
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop, etc.).

**NOTICE**

**Contact with salt water or water containing chlorine as well as abrasive substances (e. g. sand)**

Damage and premature product wear

- ▶ Following any contact with the substances identified above, promptly clean the product in accordance with the section "Cleaning".
- ▶ **Inform the patient.**

**NOTICE****Mechanical damage to the product**

Change in or loss of functionality due to damage

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

**Signs of changes in or loss of functionality during use**

Among other factors, changes in functionality can be indicated by an altered gait pattern and by the development of noises.

**4 Scope of delivery**

The following single components and accessories are included in the scope of delivery in the specified quantities and are available for reordering as single components (■), single components with minimum order quantities (▲) or single-component packs (●):

Fig.		Item no.	Quantity	Designation	Article number
1	-	④	1	Footshell	-
-	■	-	1	Instructions for use	647G1522=all_INT
<b>The following parts are for assembly only (not included in the scope of delivery!)</b>					
1	■	⑤	0	Footshell replacement tool	2C101
1	■	⑥	0	Clamping tool	704G30

**5 Preparing the product for use****⚠ CAUTION****Incorrect alignment or assembly**

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

## NOTICE

### Grinding the footshell

Premature wear resulting from damage to the footshell

- ▶ Do not grind the footshell.
- ▶ **Inform the patient.**

## 5.1 Installing the footshell

### > Tools required:

Vice, 704G30 clamping tool, 2C101 footshell replacement tool, flat screwdriver, T10 Torx screwdriver (see fig. 4 – ②)

### 1) **NOTICE! Proceed carefully during installation.**

Do not damage the spacer lug ① and cables ② with the footshell replacement tool (see fig. 2).

2) Insert a flat screwdriver into the groove ④ and push the cover plate ① out of the cover cap ② (see fig. 3).

3) Unscrew the screws ① (see fig. 4).

4) Remove the cover cap ③.

5) **NOTICE! Do not damage the plug connection and cable. Do not pull on the cable.**

Pull the plug ② of the cable ① out of the receptacle ③ in the cover cap (see fig. 5).

6) **NOTICE! To protect the pyramid adapter, use the 704G30 clamping tool.**

Clamp the clamping tool ② into the vice ① (see fig. 6).

7) Mount the pyramid adapter of the prosthetic foot in the pyramid receiver ③ of the clamping tool.

8) Pull off the old footshell, if any, according to the section "Removing the footshell" (see Page 17).

9) Bring the prosthetic foot to maximum plantar flexion (see the instructions for use of the prosthetic foot).

10) Fold the spacer lug ② in the proximal direction (see fig. 7).

11) Slide the footshell ⑤ as far as possible onto the prosthetic foot ①.

12) Insert the tool ④ into the footshell along the sole ③.

13) **NOTICE! Do not excessively deform the footshell while pulling it on. Do not puncture the folds in the footshell with the tool.**

Pull the posterior edge ④ of the footshell over the sole with the tool ③ (see fig. 8).

14) Engage the sole ④ in the adapter ③ of the footshell (see fig. 9).

15) Engage the spacer lug ① in the adapter ② of the footshell.

- 16) Press the bulge ③ of the footshell into the groove ④. (see fig. 10).
- 17) Press the proximal edge ① of the footshell against the edge ② of the adapter.
- 18) **NOTICE! Do not damage the plug connection and cable.**  
Insert the plug ② into the receptacle ③ of the cover cap (see fig. 11).
- 19) Install the cover cap ①. Do not allow the footshell ③ to slide off the adapter ② during this process. (see fig. 12).
- 20) Tighten the screws ① (see fig. 13).
- 21) Perform the charging process to verify that the plug connection is properly connected (see the instructions for use of the prosthetic foot).
- 22) Press the cover plate ① into the cover cap ② (see fig. 14).

## 5.2 Removing the footshell

### > Tools required:

Vice, 704G30 clamping tool, 2C101 footshell replacement tool, flat screwdriver, T10 Torx screwdriver (see fig. 13 – ②)

- 1) **NOTICE! Proceed carefully during removal.**  
Do not damage the spacer lug ① and cables ② with the footshell replacement tool (see fig. 2).
- 2) Insert a flat screwdriver ③ into the groove ④ and push the cover plate ① out of the cover cap ② (see fig. 15).
- 3) Unscrew the screws ① (see fig. 4).
- 4) Remove the cover cap ②.
- 5) **NOTICE! Do not damage the plug connection and cable. Do not pull on the cable.**  
Pull the plug ② of the cable ① out of the receptacle ③ in the cover cap (see fig. 5).
- 6) **NOTICE! To protect the pyramid adapter, use the 704G30 clamping tool.**  
Clamp the clamping tool ② into the vice ① (see fig. 6).
- 7) Mount the pyramid adapter of the prosthetic foot in the pyramid receiver ③ of the clamping tool.
- 8) Bring the prosthetic foot to maximum plantar flexion (see the instructions for use of the prosthetic foot).
- 9) Pull the spacer lug ① out of the adapter ② of the footshell (see fig. 9).
- 10) Pull the sole ④ out of the adapter ③ of the footshell.
- 11) Fold the spacer lug ② in the proximal direction (see fig. 7).

- 12) **NOTICE! Do not excessively deform the footshell while pulling it off. Do not puncture the folds in the footshell with the tool.**

Pull the posterior edge ④ of the footshell over the sole with the tool ③ (see fig. 8).

- 13) Remove the footshell ⑤ from the prosthetic foot ① (see fig. 5).  
14) Pull on a new footshell according to the section "Installing the footshell" (Installing the footshell).

## 6 Use

### 6.1 Information for use

#### CAUTION

##### **Use of the product without shoes**

Falling due to slipping on slippery surfaces

- ▶ Do not use the product on slippery surfaces.

### 6.2 Cleaning

#### CAUTION

##### **Use of unsuitable cleaning agents or disinfectants**

Impairment of functionality and damage due to incorrect cleaning agents or disinfectants

- ▶ Only clean the product with the approved cleaning agents.
- ▶ Only disinfect the product with the approved disinfectants.
- ▶ Observe the instructions for cleaning and care.

#### **INFORMATION**

- ▶ Only clean the product when needed.
- ▶ Observe the cleaning instructions for all prosthetic components.

- 1) Clean the product with a damp cloth and mild soap (e.g. Ottobock 453H10=1-N Derma Clean) when needed.

Ensure that no liquid penetrates into the system component(s).

- 2) Dry the product with a lint-free cloth and allow it to air dry fully.

## 7 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible

authorities in your country regarding return, collection and disposal procedures.

## 8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 8.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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