## 452K7, 452K6

<b>DE</b> Gebrauchsanweisung .....	2
<b>EN</b> Instructions for use .....	5
<b>FR</b> Instructions d'utilisation .....	9
<b>IT</b> Istruzioni per l'uso .....	12
<b>ES</b> Instrucciones de uso .....	16
<b>PT</b> Manual de utilização.....	19
<b>NL</b> Gebruiksaanwijzing .....	23
<b>SV</b> Bruksanvisning .....	26
<b>DA</b> Brugsanvisning.....	30
<b>NO</b> Bruksanvisning .....	33
<b>CS</b> Návod k použití.....	36
<b>RO</b> Instrucțiuni de utilizare.....	40
<b>SL</b> Navodila za uporabo.....	43
<b>RU</b> Руководство по применению .....	47

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Last update: 2020-04-07

- Please read this document carefully before using the product and observe the safety notices.
- Instruct the user in the safe use of the product.
- Please contact the manufacturer if you have questions about the product or in case of problems.
- Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- Please keep this document in a safe place.

## 1 Application

English

The 452K6/452K7 Suspension Sleeve should be used **solely** for the fitting of transtibial prostheses.

## 2 Safety

### 2.1 Explanation of warning symbols

 **WARNING**

Warning regarding possible serious risks of accident or injury.

**⚠ CAUTION**

Warning regarding possible risks of accident or injury.

**NOTICE**

Warning regarding possible technical damage.

## **2.2 General safety instructions**

**⚠ WARNING**

**Product is pulled over mouth and nose**

Danger of suffocation due to lack of oxygen supply

1. Store the product away from the reach of children.

**⚠ CAUTION**

**Reuse on other patients and improper cleaning**

Skin irritation, formation of eczema or infections due to contamination with germs

1. Only use the product for a single patient.
2. Clean the product regularly.

**⚠ CAUTION**

**Incorrect or excessively tight application of the product to the body**

Risk of pressure points and constriction of blood vessels and nerves due to improper application

1. Ensure that the product is applied properly and fits correctly.
2. Do not continue to use the product if there are problems with the correct fit.

**⚠ CAUTION****Use of the product on damaged skin**

Skin irritation, formation of eczema or infections due to contamination with germs

1. Only use the product on healthy skin.

**⚠ CAUTION****Contact with heat, embers or fire**

Injuries (e.g. burns) caused by melting of the material

1. Keep the product away from open flame, embers and other sources of heat.

### 3 Indication

For transtibial amputees.

### 4 Description

The Suspension Sleeve provides improved suspension for transtibial prostheses, while permitting freedom of movement of the knee.

### 5 Instructions for use

When applying the Suspension Sleeve, make sure it fits snugly and there are no wrinkles.

### 6 Care instructions

**NOTICE****Use of improper cleaning agents**

Damage to the product due to use of improper cleaning agents

1. Only clean the product with the approved cleaning agents.

After use, hand wash the Suspension Sleeve in approx. 40 °C/ 104 °F warm water using a mild detergent. Do not wring.

Rinse carefully under clear, running water.

Blot the Sleeve with a towel. Let the sleeve completely air dry on a rack, or lay flat on a towel.

## 8 Available sizes

452K6 with special internal silicone band.

Article No.	Length	Circumference		
		Below Knee	Knee	Above Knee
452K6=4	28 cm/11"	28.0 cm/11.0"	31.0 cm/12.21"	41.0 cm/16.15"
452K6=5	28 cm/11"	29.5 cm/11.5"	32.5 cm/12.8"	42.5 cm/16.75"
452K6=6	28 cm/11"	31.0 cm/12.2"	34.0 cm/13.4"	44.0 cm/17.3"
452K6=7	28 cm/11"	32.5 cm/12.8"	35.5 cm/14.0"	45.5 cm/18.0"

Article No.	Length	Circumference		
		Below Knee	Knee	Above Knee
452K7=4	25 cm/9.85"	28.0 cm/11.0"	31.0 cm/12.21"	41.0 cm/16.15"
452K7=5	25 cm/9.85"	29.5 cm/11.5"	32.5 cm/12.8"	42.5 cm/16.75"
452K7=6	25 cm/9.85"	31.0 cm/12.2"	34.0 cm/13.4"	44.0 cm/17.3"
452K7=7	25 cm/9.85"	32.5 cm/12.8"	35.5 cm/14.0"	45.5 cm/18.0"
452K7=8	25 cm/9.85"	34.0 cm/13.4"	37.0 cm/14.5"	47.0 cm/18.5"

## 9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

### 9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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