9E381=L/R




9E408=L/R







9E381=L/R, 9E408=L/R

DE	Gebrauchsanweisung	4
EN	Instructions for use	8
FR	Instructions d'utilisation	12
IT	Istruzioni per l'uso	16
ES	Instrucciones de uso	20
PT	Manual de utilização.....	24
NL	Gebruiksaanwijzing.....	27
SV	Bruksanvisning	33
DA	Brugsanvisning.....	37
PL	Instrukcja użytkowania.....	41
EL	Οδηγίες χρήσης	45

9E381=L/R

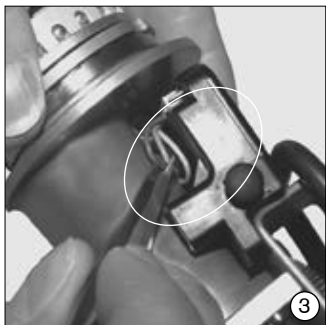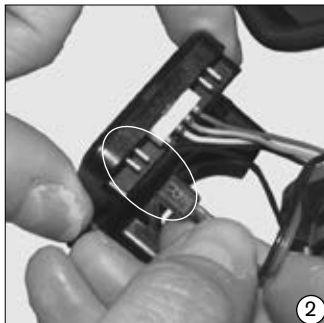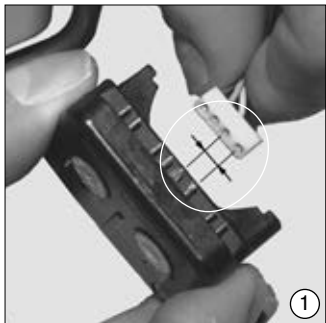9E408=L/R

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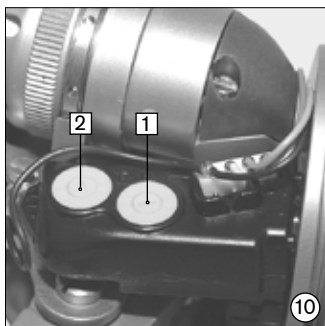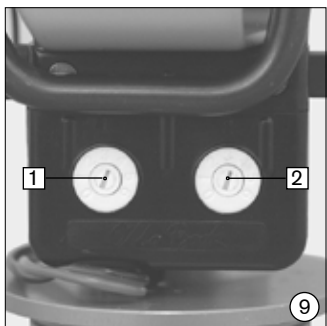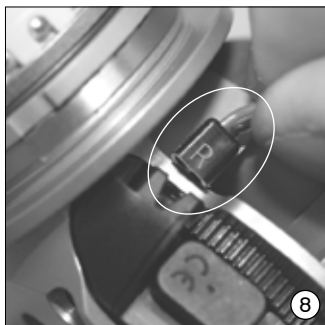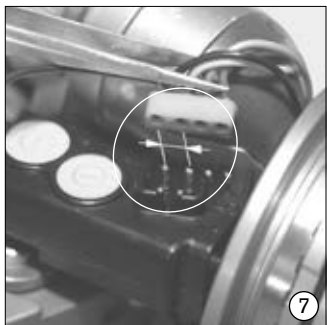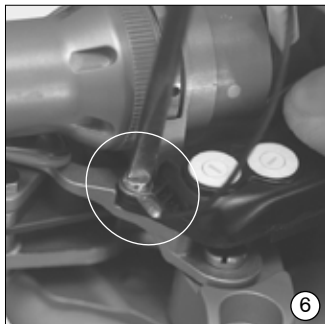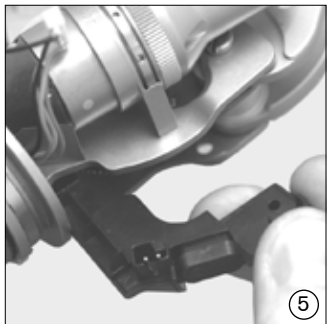English

Date of the last update: 2021-04-01

- Please read this document carefully before using the product and observe the safety notices.
- Please contact the manufacturer if you have questions about the product or in case of problems.
- Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- Please keep this document for your records.

Note:

Failure to follow the safety instructions given below can lead to damage to or malfunction of the product.

Only start up the product in accordance with the information contained in the accompanying documents supplied.

Instruct the patient in the proper use and care of the product. Handover to the patient without prior instruction is not permitted.

Attention:

The System Electric Hand may be opened or repaired only by an authorized Ottobock Service Department.

1 Disassembling the electronic control unit

Remove the lock ring of the System Inner Hand, then remove the System Inner Hand. Remove the bushings from the Electronic Control Unit. Disassemble the defective Electronic Control Unit.

2 Checking current consumption

Note:

Check the current consumption only with a fully charged battery!

Check the System Electric Hand's consumption of current with the 757T1 System Tester and 757P2=4 Test Cable by opening the System Electric Hand to the maximum. The current consumption should be **> 1.3 A**.

If the current consumption is lower, please contact the Ottobock Myo-Service.

3 Assembling the electronic control unit

3.1 9E381=L/R Electronic Control Unit (Fig. 1 to Fig. 4)

- 1) Plug the connection piece into the Electronic Control Unit (Fig. 1).

Note:

Different gaps between the contacts will help to find the correct position. Contacts and bushings must be lubricated with 633F11 Silicone Grease!

- 2) Plug motor cable bushing into the Electronic Control Unit. The letter "L" must remain visible with a left System Electric Hand. The letter "R" must remain visible with a right System Electric Hand (Fig. 2).
- 3) Position Electronic Control Unit on System Electric Hand.

Note:

Do not squeeze the cables! Use tweezers to lay open cables (Fig.3).

- 4) Tighten solder lug with ground cable and Electronic Control Unit (Fig. 4.)
- 5) Restore original control mode (13E185 Function Plug).

3.2 9E408=L/R Electronic Control Unit (Fig. 5 to Fig. 8)

The 9E408=L/R Electronic Control Unit is composed of two parts that have to be assembled separately.

- 1) Insert Electronic Control Unit (with on/off button) and fix with counter-sunk head screw (Fig.5).
- 2) Attach Electronic Control Unit (with potentiometer). Tighten solder lug with ground cable and Electronic Control Unit (Fig. 6.)
- 3) Plug the connection piece into the Electronic Control Unit (Fig. 7).

**Note:**

Different gaps between the contacts will help to find the correct position. Contacts and bushings must be lubricated with 633F11 Silicone Grease!

- 4) Plug motor cable bushing into the Electronic Control Unit. The letter "L" must remain visible with a left System Electric Hand. The letter "R" must remain visible with a right System Electric Hand (Fig. 8).
- 5) Restore original control mode (13E185 Function Plug).

4 Adjusting maximum grip force (Fig. 9, Fig. 10)

Pull on System Inner Hand. Use dial 1 (Fig. 9 and Fig. 10, item 1) to set the power shut-off.

- 1) Attach 757P5 Test Plug to 757T1 System Tester and to the System Electric Hand.
- 2) Simulate opening and closing the System Electric Hand with the 757T1 System Tester.

**Note:**

The open and close modes of the System Electric Hand must match the open and close signals on the 757T1 System Tester.

- 3) Adjust dial 1 so that a maximum grip force of approximately 90 N is reached. Use the 743F3 Pinch Gauge for checking and adjusting grip force.

5 Setting the upper threshold (Fig. 9, Fig. 10)

Set dial 2 (Fig. 9 and 10, item 2) to the same value as on the disassembled Electronic Control Unit.

Use the Ottobock 757T13 MyoSelect to reset the system electric hand to operation with the 757B15 X-ChangePack. Seal System Inner Hand with 633F11 Silicone Grease and fix with the lock ring.

For further details about the operating modes of the Digital Twin Control, please refer to the 647H327 Instructions for Use for the System Electric Hand with Digital Twin Control.

These products may not be disposed of with household waste in some jurisdictions. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in your country regarding return and collection processes.

6 Symbols used

Medical device

7 Liability

Otto Bock Healthcare Products GmbH, hereafter referred to as manufacturer, assumes liability only if the user complies with the processing, operating and maintenance instructions as well as the service intervals. The manufacturer explicitly states that this product may only be used in combination with components that were authorized by the manufacturer (see instructions for use and catalogs). The manufacturer does not assume liability for damage caused by component combinations which it did not authorize.

The product may only be opened and repaired by authorized Ottobock technicians.

8 Trademarks

All denotations within this accompanying document are subject to the provisions of the respective applicable trademark laws and the rights of the respective owners, with no restrictions.

All brands, trade names or company names may be registered trademarks and are subject to the rights of the respective owners.

Should trademarks in this accompanying document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

9 CE conformity

Otto Bock Healthcare Products GmbH hereby declares that the product is in compliance with applicable European requirements for medical devices. The full text of the regulations and requirements is available at the following Internet address: <http://www.ottobock.com/conformity>.

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