



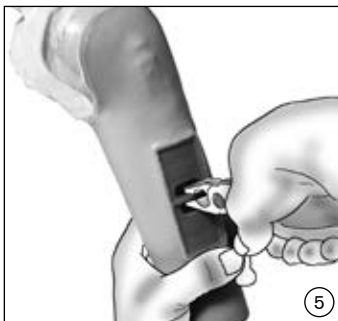
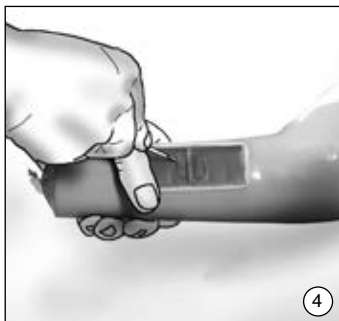
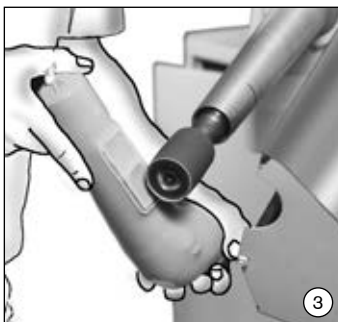
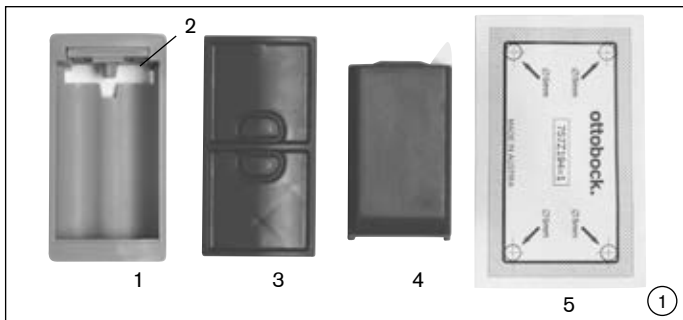
# ottobock.

757Z184=1 / 757Z190=1
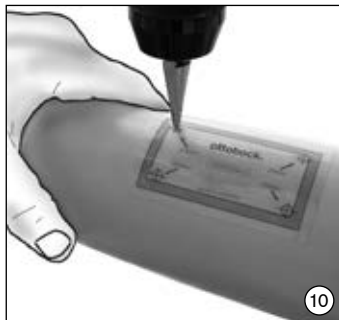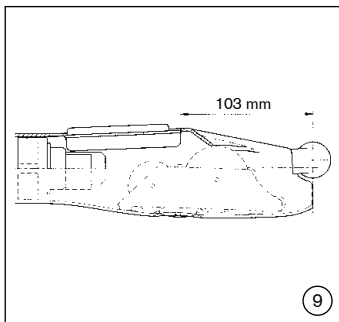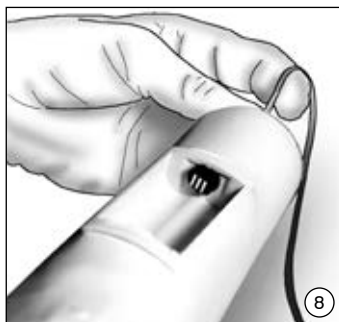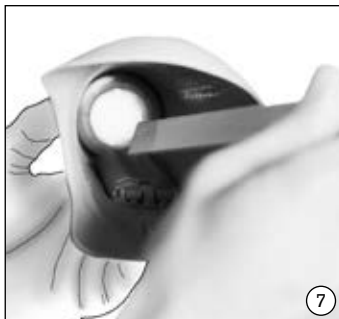
<b>DE</b>	Gebrauchsanweisung .....	4
<b>EN</b>	Instructions for use .....	6
<b>FR</b>	Instructions d'utilisation .....	9
<b>IT</b>	Istruzioni per l'uso .....	12
<b>ES</b>	Instrucciones de uso .....	14
<b>PT</b>	Manual de utilização.....	17
<b>NL</b>	Gebruiksaanwijzing.....	20
<b>SV</b>	Bruksanvisning.....	23
<b>DA</b>	Brugsanvisning.....	25
<b>HU</b>	Használati utasítás .....	28
<b>CS</b>	Návod k použití.....	30
<b>EL</b>	Οδηγίες χρήσης .....	33

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English

Date of the last update: 2020-11-30

- Please read this document carefully before using the product and observe the safety notices.
- Please contact the manufacturer if you have questions about the product or in case of problems.
- Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- Please keep this document for your records.

## 1 Application

The 757Z184=1 / 757Z190=1 Ottobock Battery Mounting Set is to be used **solely** for installation into an arm prosthesis.

## 2 Safety tips

- Do not allow foreign particles or liquids to get into the Ottobock Battery Mounting Set.
- The Ottobock Battery Mounting Set should not be subjected to intense smoke, dust, mechanical vibrations, shocks or high temperatures.

## 3 Scope of delivery (fig. 1)

	<b>757Z184=1</b> for EnergyPack 757B20	<b>757Z190=1</b> for EnergyPack 757B21
(1)	757Z185=1 Battery Receptacle	757Z191=1 Battery Receptacle
(2)	757Z186 Engaging Lever	757Z186 Engaging Lever
(3)	757Z187=1 Lamination Pattern	757Z192=1 Lamination Pattern
(4)	757Z188=1 Spacer Block	757Z193=1 Spacer Block
(5)	757Z189=1 Pattern	757Z194=1 Pattern

## 4 Installation instructions

### 4.1 Installing into a Laminated Socket

Fabricate the inner socket as usual, enclose the socket with foam and rough-grind the foam.

Set the lamination pattern into the rigid foam, making sure the edge extends out of the foam by **at least 2 mm**.

**Attention!** The surface below the lamination dummy must be flush.

Create a contoured edge on the socket by applying Plastilin around the lamination pattern. Be careful not to damage the edges of the lamination pattern (fig. 2).

Pull over a PVA bag and laminate the forearm shell in the usual manner.

Once the lamination form has set, grind the protruding edge of the lamination pattern so that it's flush (fig. 3).

Cut out the middle of the lamination pattern (fig. 4) and remove (fig. 5). Insert the spacer block into the battery receptacle (fig. 6) and install into the

forearm shell so that the engaging lever is positioned proximally. Next attach the battery receptacle using sealing resin.

**Attention!** Be careful when applying sealing resin in the area of the engaging lever (fig. 7).

Once the resin has set, remove the spacer block and mount the 13E188=\* Battery Connection Cable in the battery receptacle (fig. 8).

#### **4.2 Installation into the Forearm of an Ottobock Elbow Component**

Use the 757Z189=1 Pattern to position the battery receptacle.

**Attention!** When using an elbow component with AFB (Automatic Forearm Balance) together with an electric wrist rotator, allow a minimum distance of 103 mm between the elbow joint and battery receptacle to avoid possible collision (fig. 9).

Drill four holes within the shell of the pattern and saw out space for the battery receptacle (fig. 10).

Insert the spacer block into the battery receptacle (fig. 6) and install into the shell so that the engaging lever is positioned proximally. Next, attach the battery receptacle using sealing resin.

**Attention!** Be careful when applying sealing resin in the area of the engaging lever (fig. 7).

Once set, remove the spacer block and mount the 13E188=\* Battery Connection Cable in the battery receptacle (fig. 8).

## **5 Liability**

Otto Bock Healthcare Products GmbH, hereafter referred to as manufacturer, assumes liability only if the user complies with the processing, operating and maintenance instructions as well as the service intervals. The manufacturer explicitly states that this product may only be used in combination with components that were authorized by the manufacturer (see instructions for use and catalogs). The manufacturer does not assume liability for damage caused by component combinations which it did not authorize.

The product may only be opened and repaired by authorized Ottobock technicians.

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