
4R104=60, 4R104=75


DE Gebrauchsanweisung	4	CS Návod k použití.....	46
EN Instructions for use	7	RO Instrucțiuni de utilizare.....	49
FR Instructions d'utilisation.....	10	HR Upute za uporabu	53
IT Istruzioni per l'uso	14	SL Navodila za uporabo	56
ES Instrucciones de uso	17	SK Návod na používanie	59
PT Manual de utilização.....	20	BG Инструкция за употреба	62
NL Gebruiksaanwijzing.....	23	TR Kullanma talimatı	66
SV Bruksanvisning	27	EL Οδηγίες χρήσης	69
DA Brugsanvisning	30	RU Руководство по применению	73
NO Bruksanvisning	33	JA 取扱説明書.....	76
FI Käyttöohje.....	36	ZH 使用说明书.....	79
PL Instrukcja użytkownika.....	40	KO 사용 설명서.....	82
HU Használati utasítás	43		


1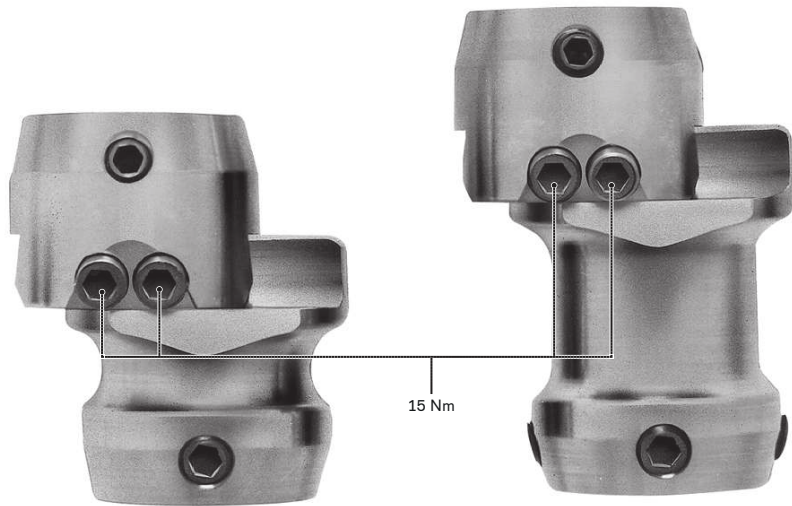

2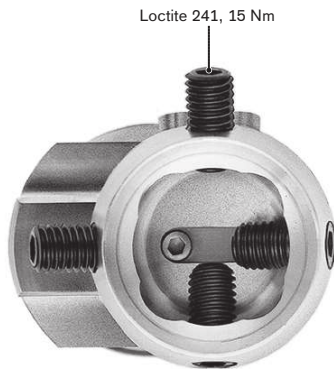

3

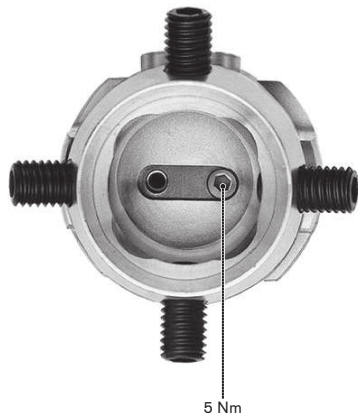1 Produktbeschreibung

Deutsch

INFORMATION

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INFORMATION

Date of last update: 2021-07-05

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.

- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

1.1 Construction and Function

The 4R104=60 and 4R104=75 double adapters, sliding, are used to connect 2 pyramid adapters. The pyramid receivers make it possible to make distal and proximal angle adjustments of the prosthetic components in the frontal and sagittal planes. The dovetail guide makes it possible to shift the prosthetic components in the frontal and sagittal planes.

1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

2 Intended use

2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

2.2 Area of application

Approved for a body weight of **up to 100 kg**.

2.3 Environmental conditions

Storage and transport

Temperature range -20 °C to +60 °C (-4 °F to +140 °F), relative humidity 20 % to 90 %, no mechanical vibrations or impacts

Allowable environmental conditions

Temperature range: -10 °C to +45 °C (14 °F to 113 °F)

Moisture: relative humidity: 20% to 90%, non-condensing

Unacceptable environmental conditions

Chemicals/liquids: fresh water, salt water, perspiration, urine, acids, soapsuds, chlorine water

Solids: dust, sand, highly hygroscopic particles (e. g. talcum)

2.4 Lifetime

This product was tested by the manufacturer with 3 million load cycles. Depending on the user's activity level, this corresponds to a maximum lifetime of 5 years.

3 Safety

3.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

3.2 General safety instructions

 **CAUTION!**

Risk of injury and risk of product damage

- ▶ Note the combination possibilities/combination exclusions in the instructions for use of the products.
- ▶ Comply with the product's field of application and do not expose it to excessive strain (see page 7).
- ▶ Do not expose the product to prohibited environmental conditions.
- ▶ Check the product for damage if it has been exposed to prohibited environmental conditions.
- ▶ Do not use the product if it is damaged or in a questionable condition. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).
- ▶ To avoid the risk of injury and product damage, do not use the product beyond the tested lifetime.

- ▶ To avoid the risk of injury and product damage, only use the product for a single patient.
- ▶ To prevent mechanical damage, use caution when working with the product.
- ▶ If you suspect the product is damaged, check it for proper function and readiness for use.
- ▶ Do not use the product if its functionality is restricted. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	Double adapter, sliding	–
	Pre-assembled:	
8	Set screw	506G3=M8x16
1	Set screw	506G3=M5x8
2	Cap screw	501Z16

5 Preparing the product for use

 **CAUTION**

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

CAUTION

Improper assembly of the screw connections

Risk of injury due to breakage or loosening of the screw connections

- ▶ Clean the threads before every installation.
- ▶ Apply the specified torque values.
- ▶ Follow the instructions regarding the length of the screws and about how to secure the screws.

5.1 Installation in the modular prosthesis

Adjusting the stop

The set screw in the dovetail guide limits sliding of the adapter in one direction. For sliding in the opposite direction, the adapter can be turned or the set screw can be moved.

> **Required tools:** 710D20 torque wrench

- ▶ **CAUTION! Only use the adapter with the set screw screwed into the dovetail guide.**

Make sure that the set screw (see fig. 3) is screwed correctly into the dovetail guide (tightening torque: **5 Nm**).

Connecting the pyramid adapter and pyramid receiver

The pyramid adapter is fixed with the set screws of the pyramid receiver.

> **Required materials:** torque wrench (e.g. 710D20), 636K13 Loctite 241

1) **Fitting:**

Screw in the set screws.

Use the torque wrench to tighten the set screws (**10 Nm**).

2) **Definitive assembly:**

Secure the set screws with Loctite.

Screw in the set screws.

Pre-tighten the set screws with the torque wrench (**10 Nm**) and then tighten them (**15 Nm**).

- 3) Replace any set screws that are protruding or are recessed too much with appropriate set screws (see selection chart).

Selection table for set screws

Reference number	Length (mm)
506G3=M8X12-V	12
506G3=M8X14	14
506G3=M8X16	16

Replacement and disassembly

The set position of the prosthetic component can be maintained during replacement or disassembly. In order to do this, unscrew the two set screws that are screwed in the furthest and located next to each other.

5.2 Adjusting adapter

> **Required tools:** 710D20 torque wrench

> **Prerequisite:** No load on the prosthesis and the prosthesis is not being worn.

- 1) Loosen the cap screws of the dovetail guide (see fig. 1).
- 2) Slide the adapter to the desired position.
- 3) Tighten the cap screws (installation torque: **15 Nm**).

6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

7 Maintenance

- ▶ A visual inspection and functional test of the prosthetic components should be performed after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.

8 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

9.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

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