## 14Y1, 14Y5 Skeo Up

<b>DE</b> Gebrauchsanweisung .....	3
<b>EN</b> Instructions for use .....	6
<b>FR</b> Instructions d'utilisation .....	10
<b>IT</b> Istruzioni per l'uso .....	13
<b>ES</b> Instrucciones de uso .....	17
<b>PT</b> Manual de utilização .....	20
<b>NL</b> Gebruiksaanwijzing .....	24
<b>SV</b> Bruksanvisning .....	27
<b>DA</b> Brugsanvisning .....	30
<b>NO</b> Bruksanvisning .....	34
<b>FI</b> Käyttöohje .....	37
<b>PL</b> Instrukcja użytkowania .....	41
<b>HU</b> Használati utasítás .....	44
<b>CS</b> Návod k použití .....	48
<b>RO</b> Instrucțiuni de utilizare .....	51
<b>HR</b> Upute za uporabu .....	54
<b>SL</b> Navodila za uporabo .....	58
<b>SK</b> Návod na používanie .....	61
<b>BG</b> Инструкция за употреба .....	64
<b>TR</b> Kullanma talimatı .....	68
<b>EL</b> Οδηγίες χρήσης .....	71
<b>RU</b> Руководство по применению .....	75
<b>JA</b> 取扱説明書 .....	79
<b>ZH</b> 使用说明书 .....	82
<b>KO</b> 사용 설명서 .....	85

## INFORMATION

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### INFORMATION

Date of last update: 2020-04-03

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- ▶ Please keep this document for your records.

### 1.1 Function

The 14Y1 liner and 14Y5 Skeo Up are arm liners made of silicone. Together with the 14A1 lock set and a custom-made prosthetic socket, they form the connection between the residual limb and the arm prosthesis. With a silicone arm liner, the arm prosthesis is more comfortable for the user to wear. A pin is used to establish the connection to the lock built into the prosthetic socket. The 14Y5 Skeo Up features an anti-friction coating on the outside to make donning easier.

## 2 Application

### 2.1 Application

The product is intended exclusively for exoprosthetic fittings of the upper limbs. Use of this product is only permitted for contact with healthy skin.

### 2.2 Area of Application

In residual limb sockets for arm prostheses from a residual limb length of 10 cm. The liner is available in the sizes =110, =140, =160, =180, =200 and =220. The liner is not suitable for compression therapy.

### 2.3 Service Life

The product is a wear part, which means it is subject to normal wear and tear.

## 3 Safety

### 3.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

### 3.2 General safety instructions

 **CAUTION**

#### **Incorrect or excessively tight application of the product to the body**

Risk of pressure points and constriction of blood vessels and nerves due to improper application

- ▶ Ensure that the product is applied properly and fits correctly.
- ▶ Do not continue to use the product if there are problems with the correct fit.

 **CAUTION**

#### **Contact with cold, heat, embers or fire**

Injuries (e.g. burns) caused by melting of the material and damage to the product

- ▶ Keep the product away from open flames, embers and other sources of heat.
- ▶ Do not expose the product to temperatures above +80 °C or below -20 °C.

 **CAUTION**

#### **Reuse on other persons and improper cleaning**

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may be used by one person only.
- ▶ Clean the product regularly.

 **CAUTION**

#### **Use of the product on damaged skin**

Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ Only use the product on healthy skin.

## 4 Scope of Delivery

Quantity	Designation	Reference number
1	Instructions for use	-
1	Liner	-

## 5 Preparing the product for use

### 5.1 Size selection

- 1) Measure the residual limb circumference **2 cm** to **3 cm** proximal to the end of the residual limb.
- 2) Depending on the soft tissue situation, reduce the circumference measurement by **1 cm** to **2 cm**.
- 3) Select the corresponding liner (size chart see packaging).

### 5.2 Adapting the Liner

#### INFORMATION

When using a myoelectric prosthesis:

- ▶ Tailor the liner so that the electrodes of the myoelectric prosthesis rest on the skin.

#### > Required tools:

719S5 Toothed Scissors or 719S7 Sizing Scissors

- ▶ Use scissors to adapt the liner to the condition of the patient's residual limb.

### 5.3 Installing the Pin

#### > Required materials:

14A1 Lock Set

- ▶ Screw the pin into the thread at the distal end of the liner.

## 6 Use

#### ⚠ CAUTION

#### Use of powder (e.g. baby powder or talcum)

Skin irritation on the residual limb and loss of function of prosthetic components due to clogging with particles or deprivation of lubricant

- ▶ Keep the product away from powders.

#### NOTICE

#### Mechanical overload

Impaired functionality due to mechanical damage

- ▶ Check the product for damage prior to each use.
- ▶ Do not use the product if its functionality has been impaired.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

### 6.1 Putting on and taking off the liner

#### ⚠ CAUTION

#### Pin not fully inserted into the lock

Risk of injury due to loss of connection to the prosthesis

- ▶ Guide the pin into the lock in accordance with the instructions.
- ▶ Always verify that the pin is engaged in the lock before using the prosthesis.

#### INFORMATION

The outside of the 14Y5 SkeoUp is provided with an anti-friction coating that makes donning easier. The liner does not need to be rolled up and no donning spray is required.

- ▶ Turn the liner inside out and place it on the end of the residual limb.
- ▶ Slide the liner onto the residual limb so that it has no wrinkles or air inclusions, and without shifting the soft tissue.

### Donning

1) **INFORMATION: The liner must be dry to put it on.**

Roll up the liner and place it on the end of the residual limb. Align the pin along the axis of the residual limb.

2) Apply the liner to the end of the residual limb and roll it over the residual limb so that it has no wrinkles or air inclusions, and without shifting the soft tissue. Briefly lift the proximal part to relieve stress.

3) **INFORMATION: Do not fold the liner over the edge of the prosthetic socket.**

Slide into the prosthetic socket with the liner until the pin is engaged in the lock.

4) Before using the prosthesis, verify that the pin is fully engaged in the lock.

### Doffing

1) Roll up the liner from the open end to the closed end and remove it from the residual limb.

2) **Recommendation:** clean the liner according to the "Cleaning and care" section.

## 6.2 Cleaning and Care

### NOTICE

#### Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

### INFORMATION

Clean the liner and skin more frequently in case of increased perspiration. Wearing a liner requires daily intensive skin care. Ottobock recommends the use of the Derma skin care system.

- 1) Turn the liner inside out.
- 2) Hand wash the liner in warm water (**30 °C**) using neutral detergent (e.g. Ottobock 453H10=1 Derma Clean).
- 3) Thoroughly rinse the liner with clear water.
- 4) Put a hand towel into the liner and leave the liner to dry lying flat on a drying rack.

## 7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 7.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 7.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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