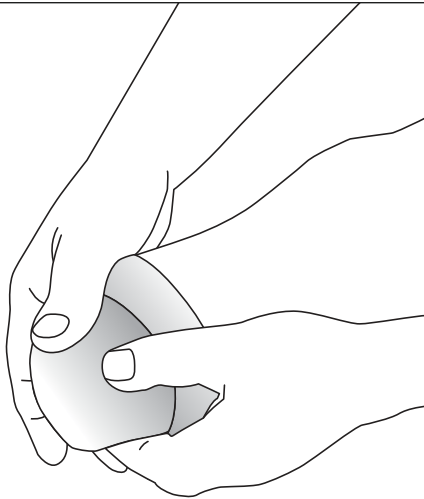
## 453D4, 453D5, 453D7 Derma Seal

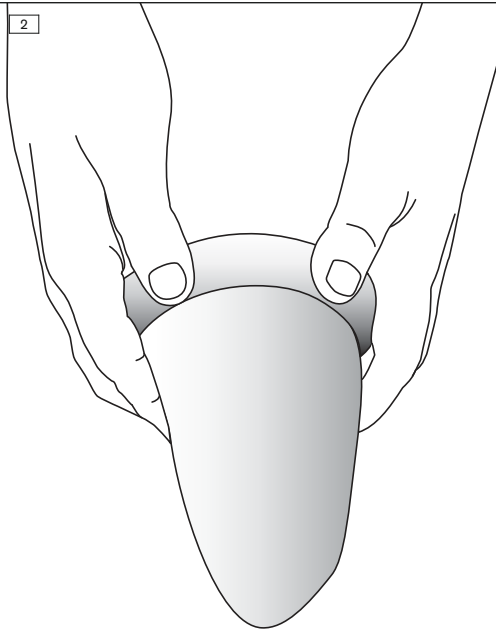
<b>DE</b> Gebrauchsanweisung .....	3	<b>CS</b> Návod k použití.....	37
<b>EN</b> Instructions for use .....	5	<b>RO</b> Instrucțiuni de utilizare.....	40
<b>FR</b> Instructions d'utilisation.....	8	<b>HR</b> Upute za uporabu .....	43
<b>IT</b> Istruzioni per l'uso .....	11	<b>SL</b> Navodila za uporabo .....	45
<b>ES</b> Instrucciones de uso .....	13	<b>SK</b> Návod na používanie .....	48
<b>PT</b> Manual de utilização.....	16	<b>BG</b> Инструкция за употреба .....	50
<b>NL</b> Gebruiksaanwijzing.....	19	<b>TR</b> Kullanma talimatı .....	53
<b>SV</b> Bruksanvisning .....	22	<b>EL</b> Οδηγίες χρήσης .....	56
<b>DA</b> Brugsanvisning .....	24	<b>RU</b> Руководство по применению .....	59
<b>NO</b> Bruksanvisning .....	27	<b>JA</b> 取扱説明書.....	62
<b>FI</b> Käyttöohje.....	29	<b>ZH</b> 使用说明书.....	64
<b>PL</b> Instrukcja użytkowania.....	32	<b>KO</b> 사용 설명서.....	67
<b>HU</b> Használati utasítás .....	35		

1

2

## 1 Produktbeschreibung

Deutsch

### INFORMATION

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### INFORMATION

Date of last update: 2021-12-16

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

### 1.1 Function

The 453D4, 453D5 and 453D7 Derma Seal residual limb socks are made from a polymer gel-coated fabric and are used for transtibial residual limbs. They are used to protect the skin of the residual limb and reduce stress caused by pressure and chafing. The medical-grade mineral oil in the polymer gel cares for the skin of the residual

limb when it is worn. The different Derma Seal residual limb socks are designed as follows:

Reference Number	Design
453D7 Derma Seal	Nylon fabric with inner polymer gel coating
453D4 Derma Seal forte	Coolmax® fabric with inner polymer gel coating
453D5 Derma Seal double forte	2 Coolmax® fabric layers with a polymer gel coating in-between

## 2 Intended use

### 2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

### 2.2 Area of application

The Derma Seal residual limb socks are applied to fittings of Münster supracondylar beddings and PTB prostheses (Patella-Tendon-Bearing).

## 3 Safety

### 3.1 Explanation of warning symbols

 Warning regarding possible risks of accident or injury.

 Warning regarding possible technical damage.

### 3.2 General safety instructions

#### Risk of severe injuries

- ▶ Store the product out of reach of children. There is a risk of suffocation if the product is pulled over the mouth and nose.

#### Risk of injury and risk of product damage

- ▶ Keep the product away from open flames, embers and other sources of heat.
- ▶ The product may only be used by one person; reuse on another person is prohibited.
- ▶ Observe the cleaning instructions.
- ▶ Make sure the product is applied properly and fits correctly. Improper or excessively tight application can cause pressure points and constriction of blood vessels and nerves.
- ▶ Do not continue to use the product if it does not fit properly.

#### Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ Only use the product on intact skin.

## 4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	Residual limb sock	–

## 5 Use

### 5.1 Donning and doffing the residual limb sock

#### Donning

- > **Requirement:** Remove all sharp objects from the hands and wrists (such as rings, bracelets or watches). Do not allow sharp fingernails to come into contact with the product.
- ▶ **INFORMATION: Put only dry residual limb socks on.** Roll up the residual limb sock, place it on the end of the residual limb and unroll it onto the residual limb with no wrinkles (see fig. 1).

## Doffing

- 1) Roll the residual limb sock up from the open end to the closed end and remove from the residual limb (see fig. 2).
- 2) **Recommendation:** clean the residual limb sock according to the section "Cleaning and Care" (see page 7).

## 5.2 Cleaning and Care

- 1) **NOTICE! In order to avoid damage, do not wring out when washing.**  
Hand wash the residual limb sock in warm water (30 °C) with mild detergent.
- 2) Thoroughly rinse the residual limb sock with clear water.
- 3) **INFORMATION: Using a hand towel speeds up the drying process.**  
Leave the residual limb sock to dry lying flat on a washing stand or a hand towel.

## 6 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

## 7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 7.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

## 7.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

## 7.3 Warranty

The manufacturer warrants this device from the date of fitting. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

## 8 Technical Data

The selection of the sock size depends on the length and circumference of the residual limb. The residual limb length is measured over the condyles. The length of the gel of the residual limb sock should exceed that of the prosthetic socket.

453D4				
Reference Number	Sock length [cm]	Gel length [cm]	Distal circumference [cm]	Proximal circumference [cm]
453D4=1	30	25	15 to 22	20 to 32
453D4=2	30	25	20 to 26	28 to 42
453D4=3	40	30	20 to 26	28 to 45
453D4=10	70	25	15 to 22	20 to 32
453D4=20	75	30	20 to 26	28 to 42

453D5				
Reference Number	Sock length [cm]	Gel length [cm]	Distal circumference [cm]	Proximal circumference [cm]
453D5=1	30	25	15 to 20	20 to 28
453D5=2	30	25	20 to 24	28 to 40

453D5				
Reference Number	Sock length [cm]	Gel length [cm]	Distal circumference [cm]	Proximal circumference [cm]
453D5=3	40	30	20 to 24	28 to 43
453D5=10	70	25	15 to 20	20 to 28
453D5=20	75	30	20 to 24	28 to 40

453D7				
Reference Number	Sock length [cm]	Gel length [cm]	Distal circumference [cm]	Proximal circumference [cm]
453D7=1	30	20	16 to 22	16 to 25
453D7=2	40	25	18 to 26	18 to 30
453D7=3	40	25	20 to 31	20 to 35
453D7=4	45	33	20 to 31	20 to 35
453D7=5	45	25	23 to 35	23 to 40
453D7=6	50	33	23 to 35	23 to 40
453D7=7	50	33	27 to 40	27 to 45
453D7=8	50	33	30 to 48	30 to 53

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