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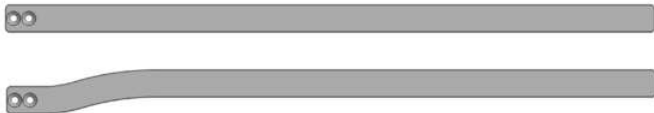
DE Gebrauchsanweisung	3
EN Instructions for use	9
FR Instructions d'utilisation	14
IT Istruzioni per l'uso	20
ES Instrucciones de uso	26
PT Manual de utilização	32
NL Gebruiksaanwijzing	38
SV Bruksanvisning	44
DA Brugsanvisning	50
NO Bruksanvisning	56
PL Instrukcja użytkowania	61
HU Használati utasítás	67
CS Návod k použití	73
TR Kullanma talimatı	79
EL Οδηγίες χρήσης	84
RU Руководство по применению	91

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INFORMATION

Date of last update: 2021-05-07

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide you with important information on the processing of the 17B4*, 17B5*, 17B6*, 17B38*, 17B39*, 17B40*, 17B41*, 17B84*, 17B85*, 17Y128* and 17Y129* side bars and lamination bars.

2 Product description

2.1 Available sizes

Side bars		
Article number	Designation	Width in mm
17B4*, 17B5*, 17B6*, 17B38*, 17B39*	Side bar	16, 20
17B40*, 17B41*	Side bar, extra long	16, 20
17B84*, 17B85*	Side bar, half round	16, 20
Lamination bars		
Article number	Designation	Width in mm
17Y128*	Lamination bar	16, 20
17Y129=L/R*	Lamination bar with contoured calf	16, 20

3 Intended use

3.1 Indications for use

The 17B4*, 17B5*, 17B6*, 17B38*, 17B39*, 17B40*, 17B41*, 17B84*, 17B85*, 17Y128* and 17Y129* bars are intended **exclusively** for bilateral use. All bars are to be used **exclusively** for providing a patient with a lower limb orthosis.

The side bars are to be used **exclusively** for the thermoplastic or bar/clamp technique.

The lamination bars are to be used **exclusively** for the lamination or prepreg technique.

3.2 Combination possibilities

Bar	Orthotic ankle joint	Orthotic knee joint	Orthotic hip joint
17B4*, 17B5*, 17B6*, 17B38*, 17B39*, 17B40*, 17B41*, 17B84*, 17B85*, 17Y128*, 17Y129*	17B62*, 17B54*, 17B57*, 17B63*, 17B53*, 17B59*, 17B98*	17B26*, 17B47*, 17B3*, 17B43*, 17B17*, 17B46*, 17B71*, 17B44*, 17B33*, 17B91*, 17B23*, 17B45*, 17B92*, 17B20*, 17B42*, 17B21*, 17B95*, 17B96*, 17B105*, 17B97*	17B82*

3.3 Indications

- Partial or total paralysis of the leg muscles
- Orthopaedic diseases of the lower limbs

Indications must be determined by the physician.

3.4 Lifetime

The product is designed for a lifetime of **3 years** when used as intended and assembled professionally.

3.5 Qualification

Patients may be fitted with the product only by trained qualified personnel. The qualified personnel must be familiar with the handling of the various techniques, materials, machines and tools.

4 Safety

4.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

4.2 Safety instructions

 **CAUTION**

Excessive strain on load-bearing components

Injuries due to changes in or loss of functionality

- ▶ Only use the product for the defined area of application.
- ▶ If the product has been exposed to extreme strain (e.g. due to falling), take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

 **CAUTION**

Incorrect alignment or assembly

Injuries due to changes in or loss of functionality

- ▶ The product may only be installed by trained, qualified personnel.
- ▶ Observe the alignment and assembly instructions.

 **CAUTION**

Mechanical damage to the product

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

 **CAUTION**

Excessive strain due to use on more than one patient

Risk of injury and loss of functionality as well as damage to the product

- ▶ Use the product on only one patient.
- ▶ Observe the maintenance recommendations.

NOTICE

Exposure of the product to unsuitable environmental conditions

Damage, brittleness or destruction due to improper handling

- ▶ Avoid storage in condensing ambient humidity.
- ▶ Avoid contact with abrasive substances (e.g. sand, dust).
- ▶ Do not expose the product to temperatures below $-10\text{ }^{\circ}\text{C}$ ($14\text{ }^{\circ}\text{F}$) or above $+60\text{ }^{\circ}\text{C}$ ($140\text{ }^{\circ}\text{F}$) (e.g. sauna, excessive sunlight, drying on a radiator).

5 Preparing the product for use

⚠ CAUTION

Repeated bending at the same position

Risk of injury due to breakage of the bar, change in or loss of functionality

- ▶ Avoid repeated bending at the same position.

⚠ CAUTION

Mechanical damage to the product

Risk of injury due to breakage of the bars, change in or loss of functionality

- ▶ Do not carry out any heat treatment on steel or titanium bars at temperatures above **$300\text{ }^{\circ}\text{C}$** .
- ▶ Do not carry out any heat treatment on aluminium bars at temperatures above **$150\text{ }^{\circ}\text{C}$** .

NOTICE

Heat treatment for shaping

Risk of injury due to breakage of the bar, change in or loss of functionality

- ▶ Avoid heat treatment for shaping.

Shortening side bars

- > **Required tools:** 702B9 hole gauge, 726S9=90x11.5 countersink, measuring tape, pen
- 1) Check the alignment of the side bars on the model.
- 2) **Optional:** Shorten the side bars and adapt the edges to the insertion zone of the orthotic joint.
- 3) Use the 702B9 hole gauge for the bores in the insertion zone of the side bar.
- 4) Countersink and deburr the bores for the oval countersunk screws with the countersink.

Shaping the bars

- > **Required tools:** 711S5 bending irons, 711S3 horizontal bending irons, 711S2 bending pusher set.
- 1) Use the 711S5 bending irons for bending.
 - 2) Use the 711S3 horizontal bending irons for horizontal contouring.
 - 3) Use a bending radius over 10 mm (see fig. 1).
 - 4) Remove grooves and burrs by grinding.
 - 5) **Optional:** For better adhesion of the reinforcement, roughen this area on the bar, at most up to 20 mm from the insertion zone. **NOTICE! To avoid a risk of breakage, do not modify the insertion zone on the bar.**
 - 6) During assembly of the bar, secure the screws in the insertion zone with Loctite 241.

Improving corrosion protection

Polish or sinter the product for improved corrosion protection. Ottobock recommends the 618T40* sintering powder for surface coating.

- Do not heat the material longer than **5 minutes** at max. **150 °C** for surface coating.

Gluing in bars

Required materials: 636W28* special adhesive, degreasing cleaner

The bars must be glued in the insertion zones to ensure stability.

- 1) Clean the connecting surfaces with a degreasing cleaner.
- 2) Apply the special adhesive to the connecting surfaces.
- 3) Insert the bars.
- 4) Secure with the enclosed screws.
- 5) Allow to cure for at least **4 hours**.

INFORMATION: Final bonding strength will be reached after 16 hours.

6 Cleaning

Promptly clean the product after contact with:

- Water containing salt, chlorine or soap, perspiration, urine or splashed water (e.g. after rain) and soiling.
- 1) Rinse the product with clear fresh water.
 - 2) Dry the product with a cloth or allow it to air dry. Avoid exposure to direct heat (e. g. from an oven or radiator).

7 Maintenance

INFORMATION

The product may be exposed to increased loads by the patient.

- ▶ Shorten the maintenance intervals according to the expected loads.

The manufacturer requires at least a semi-annual inspection of the product to verify functionality and check for wear.

8 Disposal

Dispose of the product in accordance with national regulations.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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