17B99*, 17B100*, 17B101*, 17B113*, 17B114*

DE	Gebrauchsanweisung	3
EN	Instructions for use	8
FR	Instructions d'utilisation	13
IT	Istruzioni per l'uso	19
ES	Instrucciones de uso	24
PT	Manual de utilização	30
NL	Gebruiksaanwijzing	35
SV	Bruksanvisning	40
DA	Brugsanvisning	45
NO	Bruksanvisning	50
PL	Instrukcja użytkowania	55
CS	Návod k použití	61
TR	Kullanma talimatı	66
EL	Οδηγίες χρήσης	71
RU	Руководство по применению	76
JA	取扱説明書	82
ZH	使用说明书	87

1

17B99

2

17B100

3

17B101

4

17B113

5

17B114

INFORMATION

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INFORMATION

Date of last update: 2021-06-30

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide you with important information on the use of the 17B99*, 17B100*, 17B101* system lamination foot stirrups and 17B113*, 17B114* system foot stirrups.

2 Product description

2.1 Available sizes

17B99*, 17B100*, 17B101* System lamination foot stirrups

Article number	System width	Thick-ness	Length from joint centre	Material
17B99=16	16 mm	2.5 mm	63 mm	Stainless steel
17B99=20	20 mm	3 mm		
17B100=16	16 mm	2.5 mm		
17B100=20	20 mm	3 mm		
17B101=16	16 mm	2.5 mm		
17B101=20	20 mm	3 mm		

17B113*, 17B114 System foot stirrup

Article number	Material
17B113	Stainless steel
17B114	

3 Intended use

3.1 Indications for use

The product is intended **exclusively** for orthotic fittings of the lower limbs.

3.2 Indications

- Partial or total paralysis of the leg muscles
- Orthopaedic diseases of the lower limbs

Indications must be determined by the physician.

3.3 Lifetime

The product is designed for a lifetime of **3 years** when used as intended and assembled professionally.

3.4 Qualification

Patients may be fitted with the product only by trained qualified personnel. The qualified personnel must be familiar with the handling of the various techniques, materials, machines and tools.

4 Safety

4.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

4.2 Safety instructions

 **CAUTION**

Excessive strain on load-bearing components

Injuries due to changes in or loss of functionality

- ▶ Only use the product for the defined area of application.
- ▶ If the product has been exposed to extreme strain (e.g. due to falling), take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

 **CAUTION**

Mechanical damage to the product

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

 **CAUTION**

Excessive strain due to use on more than one patient

Risk of injury and loss of functionality as well as damage to the product

- ▶ Use the product on only one patient.
- ▶ Observe the maintenance recommendations.

 **CAUTION**

Use under restricted environmental conditions

Risk of injury due to damage to the product

- ▶ Do not expose the product to restricted environmental conditions.
- ▶ If the product has been exposed to restricted environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.

- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop etc.).

NOTICE

Exposure of the product to unsuitable environmental conditions

Damage, brittleness or destruction due to improper handling

- ▶ Avoid storage in condensing ambient humidity.
- ▶ Avoid contact with abrasive substances (e.g. sand, dust).
- ▶ Do not expose the product to temperatures below -10 °C (14 °F) or above +60 °C (140 °F) (e.g. sauna, excessive sunlight, drying on a radiator).

NOTICE

Thermal overloading of the product

Damage due to improper thermal treatment

- ▶ Do not carry out any heat treatment at temperatures above **300 °C (570 °F)**.
- ▶ Prior to thermal treatment, remove all temperature-critical components (such as plastic parts).

NOTICE

Thermal overloading of product components

Loss of function due to improper thermal treatment

- ▶ Prior to the thermal treatment of the product, remove all temperature-critical components (e.g. plastic parts).

5 Preparing the product for use

CAUTION

Incorrect alignment or assembly

Injuries due to changes in or loss of functionality

- ▶ The product may only be installed by trained, qualified personnel.
- ▶ Observe the alignment and assembly instructions.

CAUTION

Repeated bending at the same position

Risk of injury due to breakage of the bar, change in or loss of functionality

- ▶ Avoid repeated bending at the same position.

Bending the foot stirrups

- > **Required tools:** Vice, 711S* bending irons, grinding machine
- 1) Position the foot stirrup in the orthotic ankle joint and mark the insertion zone of the foot stirrup in the orthotic ankle joint.
 - 2) Use the 711S* bending irons for contouring the foot stirrup.
 - 3) Clamp a bending iron in a vice and position the foot stirrup.
 - 4) **INFORMATION: To prevent the risk of breakage, do not bend or otherwise modify the insertion zone.** Adapt the foot stirrup with the bending irons.
 - 5) Remove grooves and burrs by grinding.

Improving corrosion protection

Polish or sinter the product for improved corrosion protection. Ottobock recommends the 618T40* sintering powder for surface coating.

- Do not heat the material longer than **5 minutes** at max. **150 °C** for surface coating.

6 Cleaning

Promptly clean the product after contact with:

- Water containing salt, chlorine or soap, perspiration, urine or splashed water (e.g. after rain) and soiling.
- 1) Rinse the product with clear fresh water.
 - 2) Dry the product with a cloth or allow it to air dry. Avoid exposure to direct heat (e.g. from an oven or radiator).

7 Maintenance

INFORMATION

The product may be exposed to increased loads by the patient.

- Shorten the maintenance intervals according to the expected loads.

The manufacturer requires at least a semi-annual inspection of the product to verify functionality and check for wear.

Only use 633F7 special lubricant.

Replacing the bushing

Article number	System width	Material
17Y17=7X9X2.4	16 mm	Brass
17Y17=9X11X2.9	20 mm	

- Remove the brass bushing from the foot stirrup and replace it with a new bushing.

8 Disposal

Dispose of the product in accordance with national regulations.

9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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