## RS1 Renegade®

DE	Gebrauchsanweisung .....	3
EN	Instructions for use .....	6
FR	Instructions d'utilisation.....	9
ES	Instrucciones de uso.....	12
PT	Manual de utilização .....	15
NL	Gebruiksaanwijzing .....	17

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Artikelnummer: RS1

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Product Number: RS1

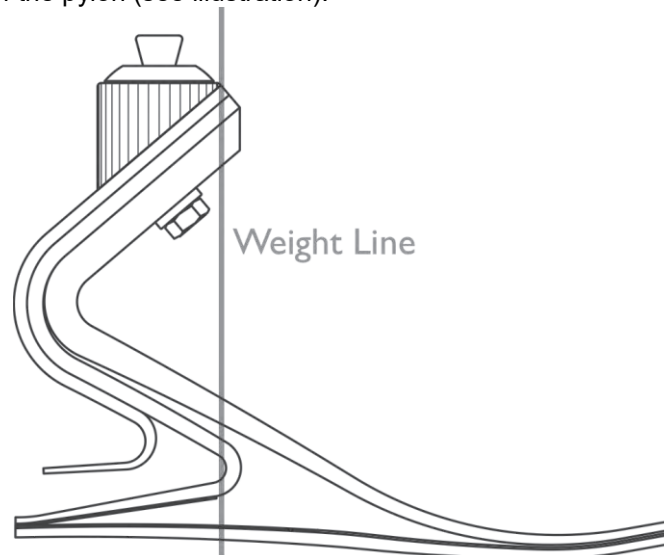
## Assembly

The Renegade® foot module is pre-assembled consisting of a graphite component (keel), sole plate, Spectra™ sock, and foot shell. Foam *stiffening bumpers* for increasing heel stiffness are provided. After dynamic alignment, torque pyramid adjustment screws to the manufacturer's specifications. Secure pyramid adjustment screws with a thread locking adhesive (i.e., Loctite 242).

## Bench Alignment

Prior to donning the prosthesis:

- Plantarflex/Dorsiflexion foot to match the shoe heel height.
- Adduct/Abduct socket to provide appropriate frontal plane angle.
- Flex/Extend socket to provide appropriate sagittal plane angle.
- Move the socket linearly to ensure the weight line falls along the anterior edge of the pylon (see illustration).

## Dynamic Alignment

During *loading response*, the sole plate and Z-Shock store energy and release it during *midstance*. This action provides momentum for the keel to store energy and release it during *terminal stance*. To optimize the heel to toe rollover motion, adjust the following variables:

- Anterior/posterior foot placement
- Dorsiflexion/plantarflexion
- Heel stiffness

## Troubleshooting

Heel too soft

### Symptoms

- Foot flat occurs too rapidly
- Toe feels excessively stiff
- Knee hyperextension

### Solutions

- Shift socket anteriorly in relation to the foot
- Attach foam *stiffening bumpers*

Heel too hard	<p><i>Symptoms</i></p> <ul style="list-style-type: none"> <li>• Rapid knee flexion, instability</li> <li>• Heel to toe progression to rapid</li> <li>• Lack of energy return sensation</li> </ul> <p><i>Solutions</i></p> <ul style="list-style-type: none"> <li>• Shift socket posteriorly in relation to the foot</li> <li>• Verify appropriate foot module category</li> </ul>
Foot module too stiff	<p><i>Symptoms</i></p> <ul style="list-style-type: none"> <li>• Flat spot in rollover motion at slow cadences</li> </ul> <p><i>Solutions</i></p> <ul style="list-style-type: none"> <li>• Consider a lower category foot module</li> </ul>
Foot module too soft	<p><i>Symptoms</i></p> <ul style="list-style-type: none"> <li>• Clicking noise at <i>initial contact</i></li> <li>• Excessive toe deflection with high impact activity</li> </ul> <p><i>Solutions</i></p> <ul style="list-style-type: none"> <li>• Consider a higher category foot module</li> </ul>
<b>Stiffening Bumpers</b>	<p>Foam <i>stiffening bumpers</i> are included to adjust the heel stiffness during <i>loading response</i>. The bumpers may be temporarily attached with tape to the inferior aspect of the Z-Shock. If the bumpers provide excessive heel stiffness, exclude or trim as appropriate. For permanent placement, adhere bumpers using Barge or similar contact cement.</p>
<b>Spectra™ Sock</b>	<p>A Spectra™ sock is provided to protect the foot shell and minimize noise. Spectra™ socks must be replaced at intervals appropriate to the user's activity level. Failure to inspect and replace the Spectra™ socks may prematurely wear the foot module, and will void the warranty.</p>
<b>Foot Shell</b>	<p>When removing or installing the foot shell, use the Foot Shell Removal Tool (ACC-00-10300-00) to prevent damage to the foot module.</p>
<b>Renegade®</b>	<p>Minimum clearance: 162mm-200mm  Maximum user weight: 166 kg (365 lbs)  Available sizes: 22cm-31cm  Heel height: 10mm</p>
<b>Maintenance</b>	<p>The Renegade® foot module requires periodic maintenance.</p> <ul style="list-style-type: none"> <li>• Inspect the foot module every six months. If the user is more active, more frequent inspection may necessary. Service as necessary. Replace Spectra™ sock and/or foot shell if worn to prevent damage to the graphite components.</li> <li>• The foot module may be cleaned and/or disinfected with soap and warm water. If the foot is exposed to water, remove the foot shell to clean and disinfect its interior.</li> </ul>
<b>Warnings</b>	<p>Failure to adhere to the guidelines of the <i>Instructions for Use</i> will void the warranty.</p> <ul style="list-style-type: none"> <li>• Never use the foot module without a <i>foot shell</i>. Failure to comply may cause premature wear, loss of function, and/or product failure.</li> <li>• Always use the foot module with a sock and shoe. Failure to comply may cause premature wear, loss of function, and/or product failure.</li> </ul>

- Never allow aggregates such as sand to remain in the *foot shell*. Upon exposure to aggregates, immediately disassemble foot module and rinse with water. The abrasive properties of aggregates will quickly wear the graphite components of the foot module.
- Never insert the *sole plate* inside the Spectra™ sock of the Renegade® foot module. This will lead to premature failure of the foot module.
- Never use the Renegade® foot module without the *sole plate*. *Sole plate* removal will adversely affect the performance of the foot module. The *sole plate* fills the mid-foot void between the Z-Shock and toe lever, preventing potential user injury.
- Freedom Innovations foot modules are manufactured to fit industry standard pyramids and receivers. It is the prosthetist's responsibility to select and/or fabricate properly fitting attachment components.
- Never attempt to loosen the bolt affixing the pyramid connector.
- Discontinue use and consult your prosthetist if any part of the prosthesis starts to make noise.
- Inform your prosthetist if you lose or gain a significant amount of weight.
- Freedom Innovations foot products are manufactured and tested for a particular weight and activity impact level. Use by another user for whom it was not originally manufactured may cause injury and shall void any written or implied warranty.

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Advena Limited. Tower Business Centre, 2nd Flr., Tower