
2C3-1, 2C15


DE Gebrauchsanweisung	3	CS Návod k použití.....	45
EN Instructions for use	6	RO Instrucțiuni de utilizare.....	49
FR Instructions d'utilisation.....	9	HR Upute za uporabu	52
IT Istruzioni per l'uso	13	SL Navodila za uporabo	55
ES Instrucciones de uso	16	SK Návod na používanie	58
PT Manual de utilização.....	19	BG Инструкция за употреба	61
NL Gebruiksaanwijzing.....	22	TR Kullanma talimatı	65
SV Bruksanvisning	26	EL Οδηγίες χρήσης	68
DA Brugsanvisning	29	RU Руководство по применению	71
NO Bruksanvisning	32	JA 取扱説明書.....	75
FI Käyttöohje.....	35	ZH 使用说明书.....	78
PL Instrukcja użytkowania.....	39	KO 사용 설명서.....	81
HU Használati utasítás	42		

1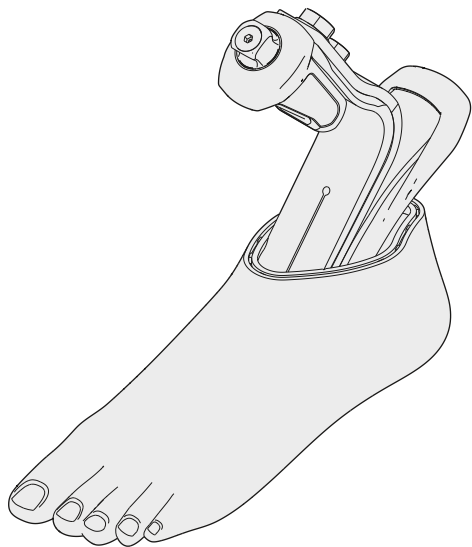

2

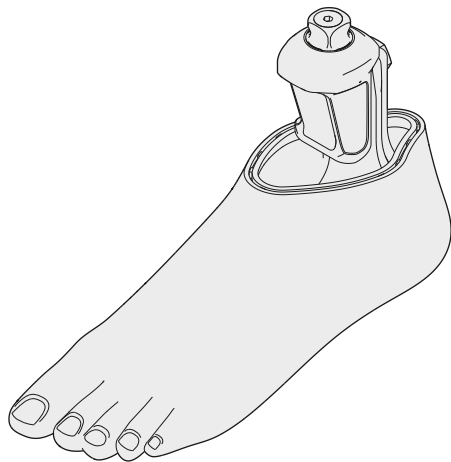1 Produktbeschreibung

Deutsch

INFORMATION

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INFORMATION

Date of last update: 2021-05-26

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.

- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

1.1 Construction and Function

Footshells are pulled over compatible prosthetic feet. They are required for using the prosthetic feet and give them their natural exterior shape. This document applies for the following footshells: 2C3-1, 2C15.

The toenails are **not** paintable.

1.2 Combination possibilities

- Observe the prosthetic foot instructions for use.

Footshell	2C3*	2C15
Connection cover	2C10	2C19
	2C20	2C20

2 Intended use

2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

2.2 Environmental conditions

Observe the prosthetic foot's environmental conditions.

Other environmental conditions	
UV light	Not resistant
Hydrolysis	Not resistant

2.3 Lifetime

The product is a wear part, which means it is subject to normal wear and tear.

3 Safety

3.1 Explanation of warning symbols

CAUTION Warning regarding possible risks of accident or injury.

NOTICE Warning regarding possible technical damage.

3.2 General safety instructions

CAUTION

Improper combination of prosthetic components

Risk of injury due to breakage or deformation of the product

- ▶ Only combine the product with prosthetic components that are approved for that purpose.
- ▶ Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

CAUTION

Use of a worn product and reuse on another patient

Risk of injury due to loss of functionality as well as damage to the product

- ▶ Inspect the product regularly for wear. Do not continue using a worn product.
- ▶ Only use the product for a single patient.

CAUTION

Contact with heat, embers or fire

Risk of injury (such as burns) and risk of product damage.

- ▶ Keep the product away from open flames, embers and other sources of heat.

NOTICE

Use under unallowable environmental conditions

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions.
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop, etc.).

NOTICE

Contact with salt water or water containing chlorine as well as abrasive substances (e.g. sand)

Damage and premature product wear and tear

- ▶ Promptly clean the product following any contact with the substances identified above.

NOTICE

Mechanical damage to the product

Change in or loss of functionality due to damage

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern and by the development of noises.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	-
1	Footshell	-
1	Connection cover	-

5 Preparing the product for use

⚠ CAUTION

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

NOTICE

Grinding the footshell

Premature wear resulting from damage to the footshell

- ▶ Do not grind the footshell.
- ▶ Inform the patient.

5.1 Applying/removing the footshell

Putting on the footshell

> **Recommended tools:** 2C100 changing device

- 1) Slide the prosthetic foot into the footshell.
- 2) Press the heel of the prosthetic foot into the footshell until it engages.

Removing the footshell

> **Recommended tools:** 2C100 changing device

- 1) Push the lock of the footshell to the rear and pull up the heel of the prosthetic foot.
- 2) Remove the prosthetic foot from the footshell.

6 Use

⚠ CAUTION

Use of the product without shoes

Risk of injury due to slipping on slippery surfaces

- ▶ Do not use the product on slippery surfaces.

6.1 Cleaning

⚠ CAUTION

Use of unsuitable cleaning agents or disinfectants

Impairment of functionality and damage due to incorrect cleaning agents or disinfectants

- ▶ Only clean the product with the approved cleaning agents.
- ▶ Only disinfect the product with the approved disinfectants.
- ▶ Observe the instructions for cleaning and care.

> **Recommended cleaning agent:** pH neutral soap (e.g. 453H10 Derma Clean)

- 1) Clean the product with clear fresh water and a pH neutral soap.
- 2) Rinse the soap away with clear fresh water.
- 3) Dry the product with a soft cloth.
- 4) Allow to air dry in order to remove residual moisture.

7 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the

responsible authorities in your country regarding return, collection and disposal procedures.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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