## 4R44=L, 4R50

<b>DE</b> Gebrauchsanweisung .....	4	<b>CS</b> Návod k použití.....	50
<b>EN</b> Instructions for use .....	7	<b>RO</b> Instrucțiuni de utilizare.....	53
<b>FR</b> Instructions d'utilisation.....	11	<b>HR</b> Upute za uporabu .....	57
<b>IT</b> Istruzioni per l'uso .....	15	<b>SL</b> Navodila za uporabo .....	61
<b>ES</b> Instrucciones de uso .....	18	<b>SK</b> Návod na používanie .....	64
<b>PT</b> Manual de utilização.....	22	<b>BG</b> Инструкция за употреба .....	67
<b>NL</b> Gebruiksaanwijzing.....	25	<b>TR</b> Kullanma talimatı .....	71
<b>SV</b> Bruksanvisning .....	29	<b>EL</b> Οδηγίες χρήσης .....	75
<b>DA</b> Brugsanvisning .....	33	<b>RU</b> Руководство по применению .....	78
<b>NO</b> Bruksanvisning .....	36	<b>JA</b> 取扱説明書.....	82
<b>FI</b> Käyttöohje.....	39	<b>ZH</b> 使用说明书.....	86
<b>PL</b> Instrukcja użytkownika.....	43	<b>KO</b> 사용 설명서.....	89
<b>HU</b> Használati utasítás .....	46		

1

2

3

# 1 Produktbeschreibung

Deutsch

## INFORMATION

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### INFORMATION

Date of last update: 2021-06-07

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

### 1.1 Construction and Function

The 4R44=L and 4R50 adapters serve as connecting elements in modular lower limb prostheses. They permit length compensation and rotation adjustment. The 4R44=L adapter is a pyramid receiver with exterior thread, the 4R50 adapter is a pyramid adapter with interior thread.

## 1.2 Combination possibilities

This prosthetic component is compatible with Ottobock's system of modular connectors. Functionality with components of other manufacturers that have compatible modular connectors has not been tested.

- The 4R44=L adapter can either be connected to a lamination anchor or with the 4R50 adapter.

## 2 Intended use

### 2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

### 2.2 Area of application

Approved for a body weight of **up to 150 kg**.

### 2.3 Environmental conditions

#### Storage and transport

Temperature range  $-20\text{ °C}$  to  $+60\text{ °C}$  ( $-4\text{ °F}$  to  $+140\text{ °F}$ ), relative humidity 20 % to 90 %, no mechanical vibrations or impacts

#### Allowable environmental conditions

**Temperature range:**  $-10\text{ °C}$  to  $+45\text{ °C}$  ( $14\text{ °F}$  to  $113\text{ °F}$ )

**Moisture:** relative humidity: 20% to 90%, non-condensing

#### Unacceptable environmental conditions

**Chemicals/liquids:** fresh water, salt water, perspiration, urine, acids, soapsuds, chlorine water

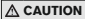
**Solids:** dust, sand, highly hygroscopic particles (e. g. talcum)

### 2.4 Lifetime

This product was tested by the manufacturer with 3 million load cycles. Depending on the user's activity level, this corresponds to a maximum lifetime of 5 years.

## 3 Safety

### 3.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

### 3.2 General safety instructions

 **CAUTION!**

#### Risk of injury and risk of product damage

- ▶ Comply with the product's field of application and do not expose it to excessive strain (see page 8).
- ▶ Note the combination possibilities/comboination exclusions in the instructions for use of the products.
- ▶ Do not expose the product to prohibited environmental conditions.
- ▶ Check the product for damage if it has been exposed to prohibited environmental conditions.
- ▶ Do not use the product if it is damaged or in a questionable condition. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).
- ▶ Observe the maximum lifetime of the product.
- ▶ To prevent mechanical damage, use caution when working with the product.
- ▶ If you suspect the product is damaged, check it for proper function and readiness for use.
- ▶ Do not use the product if its functionality is restricted. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).

### Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

## 4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	Adapter	–
<b>For 4R44=L</b>		
4	Set screw	506G3=M8x12-V
<b>For 4R50</b>		
1	Cap screw	501Z2=M6x20

## 5 Preparing the product for use

### CAUTION

#### Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

### CAUTION

#### Improper assembly of the screw connections

Risk of injury due to breakage or loosening of the screw connections

- ▶ Clean the threads before every installation.
- ▶ Apply the specified torque values.
- ▶ Follow the instructions regarding the length of the screws and about how to secure the screws.

## 5.1 Adjusting the adapter

The threaded connector of the 4R44=L adapter is shortened according to the patient measurements. A metal saw or lathe can be used to shorten the adapter.

- 1) Clamp the adapter so that the thread being used remains undamaged (minimum thread depth: **8 mm** for lamination anchor, **11 mm** for 4R50).
- 2) **NOTICE! Shorten the adapter at right angles to the longitudinal axis.**  
Shorten the thread of the adapter.
- 3) Deburr the cut edge without grinding a bevel on the thread.

## 5.2 Installation in the modular prosthesis

After shortening, the 4R44=L adapter is either screwed into the thread of a lamination anchor or connected to the 4R50 adapter. Then the remaining prosthesis components are assembled.

### 4R44=L with lamination anchor

> **Recommended tools:** 710D20 torque wrench, 636K13 Loctite®

- 1) Screw the adapter completely into the thread of the lamination anchor.
- 2) The adapter may be unscrewed by a **maximum of 1/4 turn** out of the thread in order to align it.
- 3) Secure the cap screw of the threaded clamping using Loctite® as described in the lamination anchor instructions for use.

### 4R44=L with 4R50

> **Recommended tools:** 710D20 torque wrench

- 1) Align the pyramid receiver of the 4R44=L adapter distally.
- 2) Fully screw the 4R50 adapter onto the thread of the 4R44=L adapter.
- 3) The adapter may be unscrewed by a **maximum of 1/4 turn** out of the thread in order to align it.
- 4) Align the threaded clamping to the anterior and tighten the cap screw (torque value: **13 Nm**).

## Connecting the pyramid adapter and pyramid receiver

The pyramid adapter is fixed with the set screws of the pyramid receiver.

> **Required materials:** torque wrench (e.g. 710D20), 636K13 Loctite 241

### 1) Fitting:

Screw in the set screws.

Use the torque wrench to tighten the set screws (**10 Nm**).

### 2) Definitive assembly:

Secure the set screws with Loctite.

Screw in the set screws.

Pre-tighten the set screws with the torque wrench (**10 Nm**) and then tighten them (**15 Nm**).

### 3) Replace any set screws that are protruding or are recessed too much with appropriate set screws (see selection chart).

**Selection table for set screws**

Reference number	Length (mm)
506G3=M8X12-V	12
506G3=M8X14	14
506G3=M8X16	16

## Alignment

The set screws in the pyramid receiver can be used to make static adjustments during alignment, trial fittings and after the prosthesis is finished.

## Replacement and disassembly

The set position of the prosthetic component can be maintained during replacement or disassembly. In order to do this, unscrew the two set screws that are screwed in the furthest and located next to each other.

## 6 Cleaning

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.

- 3) Allow to air dry in order to remove residual moisture.

## 7 Maintenance

- ▶ A visual inspection and functional test of the prosthetic components should be performed after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.

## 8 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

## 9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

### 9.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

## 10 Technical data

Reference number	4R44=L	4R50
Weight [g]	210	70
System height [mm]	Min. 31 to max. 91	-1
Build height [mm]	Min. 22	17
Overall height [mm]	87	-
Material	Stainless steel	Titanium
Max. body weight [kg]	150	

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