




711M62


DE	Gebrauchsanweisung	4
EN	Instructions for Use	5


1

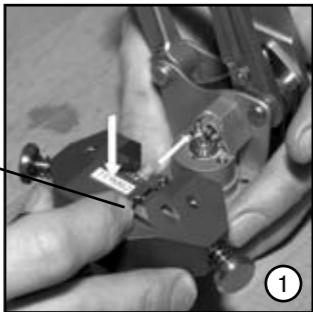

2

3

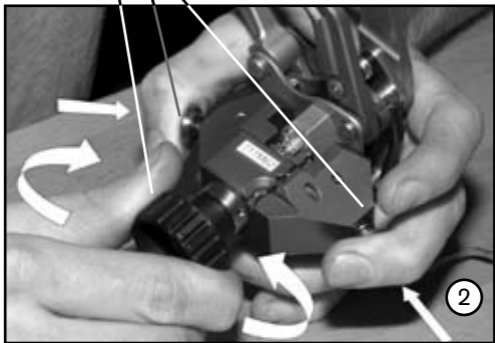

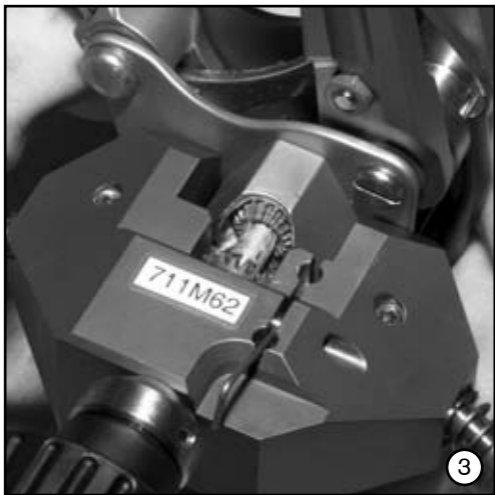

INFORMATION

Datum der letzten Aktualisierung: 2013-01-15
> Lesen sie das Dokument aufmerksam durch.

1 Bedeutung

Die Justiervorrichtung für Schaltgetriebe 711M62 ist ausschließlich zur Justage des Umschaltpunktes der Schaltgetriebe für Myobock System-Elektrohände geeignet.

2 Bedienung (Abb. 1, Abb. 2, Abb. 3)

- Drücken Sie den Hebel an der Justiervorrichtung (Abb. 1, Pos. 1) hinunter, und setzen Sie die Justiervorrichtung auf die System-Elektrohand bis zum Anschlag (Abb. 3) auf.
- Drücken Sie die beiden Knöpfe (Abb. 2, Pos. 3) bis zum Anschlag und halten Sie die Knöpfe, um die Justiervorrichtung in dieser Position zu fixieren.
- Durch Verdrehen des Rades (Abb. 2, Pos. 2) rastet der Fixierbolzen in der Justierscheibe des Schaltgetriebes ein.
- Führen Sie durch weiteres Verdrehen des Rades (Abb. 2, Pos. 2) in beliebiger Richtung die Justage durch.
- Drücken Sie den Hebel (Abb. 1, Pos. 1), um den Fixierbolzen zu lösen, lassen Sie beide Knöpfe (Abb. 2, Pos. 3) los, und ziehen Sie die Justiervorrichtung von der System-Elektrohand ab.

INFORMATION

Date of last update: 2013-01-15

> Please read this document carefully.

1 Application

The 711M62 Adjustment Device for Transmission Gear is to be used exclusively for the adjustment of the shift point of the MyoBock System Electric Hands change gear.

2 Handling (Fig. 1, Fig. 2, Fig. 3)

- Press the lever on the Adjustment Device (Fig. 1, Pos. 1) down and position the Adjustment Device on the System Electric Hand as far as it will go (Fig. 3).
- Press both buttons (Fig. 2, Pos. 3) as far as they will go and hold the buttons to fix the Adjustment Device in position.
- By twisting the wheel (Fig. 2, Pos. 2) the attachment bolt locks into the adjustment disk of the transmission gear.
- Carry out the adjustment by twisting the wheel (Fig. 2, Pos. 2) further in any direction.
- Press the lever (Fig. 1, Pos. 1) to disengage the bolt lock, let go of the two buttons (Fig. 2, Pos. 3), and remove the Adjustment Device from the System Electric Hand.

Notes

Notes

Otto Bock Healthcare Products GmbH
Kaiserstraße 39 · 1070 Wien · Austria
T +43-1 523 37 86 · F +43-1 523 22 64
info.austria@ottobock.com · www.ottobock.com

Ottobock has a certified Quality Management System
in accordance with ISO 13485.