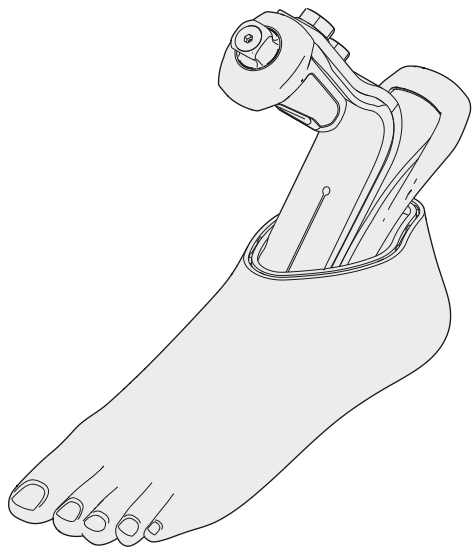
2C1, 2C2, 2C3-1, 2C4, 2C5, 2C6, 2C15

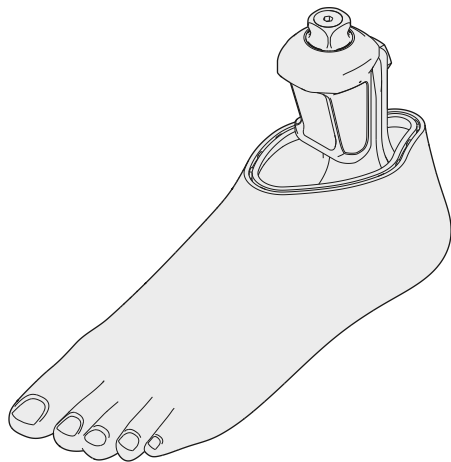
DE Gebrauchsanweisung	3	CS Návod k použití.....	46
EN Instructions for use	6	RO Instrucțiuni de utilizare.....	49
FR Instructions d'utilisation.....	9	HR Upute za uporabu	52
IT Istruzioni per l'uso	13	SL Navodila za uporabo	55
ES Instrucciones de uso	16	SK Návod na používanie	59
PT Manual de utilização.....	19	BG Инструкция за употреба	62
NL Gebruiksaanwijzing.....	23	TR Kullanma talimatı	65
SV Bruksanvisning	26	EL Οδηγίες χρήσης	69
DA Brugsanvisning	29	RU Руководство по применению	72
NO Bruksanvisning	32	JA 取扱説明書.....	76
FI Käyttöohje.....	36	ZH 使用说明书.....	79
PL Instrukcja użytkowania.....	39	KO 사용 설명서.....	82
HU Használati utasítás	42		

1

2

1 Produktbeschreibung

Deutsch

INFORMATION

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INFORMATION

Date of last update: 2021-08-17

- ▶ Please read this document carefully before using the product and observe the safety notices.

- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

1.1 Construction and Function

Footshells are pulled over compatible prosthetic feet. They are required for using the prosthetic feet and give them their natural exterior shape. This document applies for the following footshells: 2C1, 2C2, 2C3, 2C4, 2C5, 2C6.

The toenails are **not** paintable.

1.2 Combination possibilities

- Observe the prosthetic foot instructions for use.

Footshell	2C1	2C2	2C3*	2C4	2C5	2C6	2C15
Connection cover	2C19	-	2C10	2C11	2C10	2C19	2C19
	2C20		2C20			2C20	2C20

2 Intended use

2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

2.2 Environmental conditions

Observe the prosthetic foot's environmental conditions.

Other environmental conditions	
UV light	Not resistant
Hydrolysis	Not resistant

2.3 Lifetime

The product is a wear part, which means it is subject to normal wear and tear.

3 Safety

3.1 Explanation of warning symbols

CAUTION Warning regarding possible risks of accident or injury.

NOTICE Warning regarding possible technical damage.

3.2 General safety instructions

CAUTION!

Risk of injury and risk of product damage

- ▶ Note the combination possibilities/combinations exclusions in the instructions for use of the products.
- ▶ Inspect the product regularly for wear and tear. Do not continue using a worn product.
- ▶ The product may only be used by one person; reuse on another person is prohibited.
- ▶ Keep the product away from open flames, embers and other sources of heat.

NOTICE!

Risk of product damage and limited functionality

- ▶ Do not expose the product to prohibited environmental conditions.
- ▶ Check the product for damage if it has been exposed to prohibited environmental conditions.
- ▶ Do not use the product if it is damaged or in a questionable condition. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).

- ▶ Clean the product promptly each time after it comes into contact with salt water, chlorinated water or abrasive media (such as sand).
- ▶ To prevent mechanical damage, use caution when working with the product.
- ▶ If you suspect the product is damaged, check it for proper function and readiness for use.
- ▶ Do not use the product if its functionality is restricted. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern and by the development of noises.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	–
1	Footshell	–
1	Connection cover	–

5 Preparing the product for use

CAUTION

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

NOTICE

Grinding the footshell

Premature wear resulting from damage to the footshell

- ▶ Do not grind the footshell.
- ▶ **Inform the patient.**

5.1 Applying/removing the footshell

Putting on the footshell

- > **Recommended tools:** 2C100 changing device
- 1) Slide the prosthetic foot into the footshell.
- 2) Press the heel of the prosthetic foot into the footshell until it engages.

Removing the footshell

- > **Recommended tools:** 2C100 changing device
- 1) Push the lock of the footshell to the rear and pull up the heel of the prosthetic foot.
- 2) Remove the prosthetic foot from the footshell.

6 Use

⚠ CAUTION

Use of the product without shoes

Risk of injury due to slipping on slippery surfaces

- ▶ Do not use the product on slippery surfaces.

6.1 Cleaning

⚠ CAUTION

Use of unsuitable cleaning agents or disinfectants

Impairment of functionality and damage due to incorrect cleaning agents or disinfectants

- ▶ Only clean the product with the approved cleaning agents.
- ▶ Only disinfect the product with the approved disinfectants.
- ▶ Observe the instructions for cleaning and care.

- > **Recommended cleaning agent:** pH neutral soap (e.g. 453H10 Derma Clean)
- 1) Clean the product with clear fresh water and a pH neutral soap.
- 2) Rinse the soap away with clear fresh water.
- 3) Dry the product with a soft cloth.

- 4) Allow to air dry in order to remove residual moisture.

7 Disposal

In some jurisdictions it is not permissible to dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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