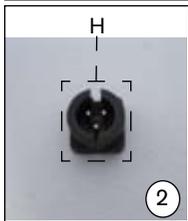
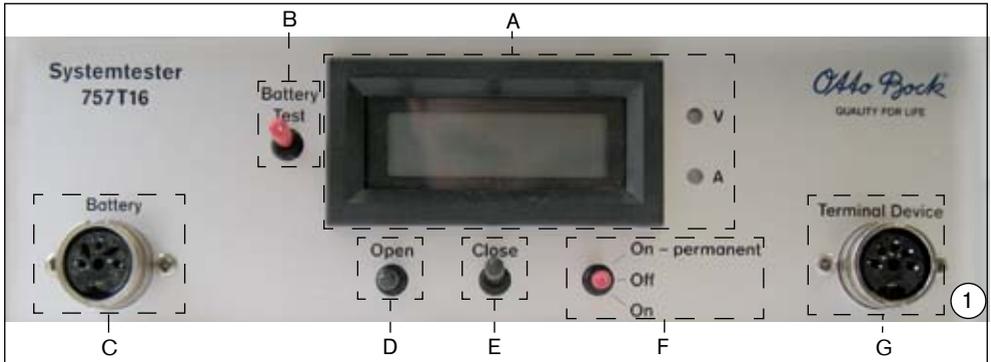
Systemtester 757T16

<input type="checkbox"/> DE	Gebrauchsanweisung (Fachpersonal)	2
<input type="checkbox"/> EN	Instructions for Use (Qualified Personnel)	10

1 Verwendung

Deutsch

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INFORMATION

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1.1 Intended Use

INFORMATION

The handling and use of this product is reserved exclusively for trained specialists.

The Ottobock System Tester 757T16 is intended exclusively for testing the functionality of individual components and modules of the MyoBock®-System.

Explanation of the ANSI symbols

⚠ WARNING

Warnings regarding possible risks of severe accident or injury.

⚠ CAUTION

Warnings regarding possible risks of accident or injury.

NOTICE

Warnings regarding possible technical damage.

INFORMATION

Additional information on the fitting/use.

INFORMATION

Please read the Instructions for Use carefully. Pay special attention to the safety instructions!

2 Safety Instructions

NOTICE

Destruction of the product through improper use of an AC adapter.

Only use the supplied AC adapter.

NOTICE

Penetration of substances.

Penetration of liquids and smoke leads to product damage.

Only use the product for service and production purposes.

NOTICE

Mechanical loads.

Impacts and vibration lead to malfunctions.

Only use the product for service and production purposes.

NOTICE

Thermal loads.

Temperatures outside of the specified operating temperatures result in damage to the product.

Only use the product for service and production purposes.

NOTICE

Use by unauthorized personnel.

Opening and calibration by unauthorized personnel results in damage to the product. Opening and calibration is reserved exclusively for Ottobock measuring and test equipment monitoring.

NOTICE

Visible damages.

Visible damage to the product results in faulty test results. In case of damage please contact Ottobock measuring and test equipment monitoring.

NOTICE

Manipulation.

Independent changes or modifications lead to malfunction of the product. Opening and calibration is reserved exclusively for Ottobock measuring and test equipment monitoring.

INFORMATION

The handling and use of this product is reserved exclusively for trained specialists.

INFORMATION

Observe the calibration intervals of the product according to the maintenance plan.

INFORMATION

Before functionality testing of the individual components, you have to connect the corresponding battery of the respective system.

INFORMATION

Before starting the testing process, you have to place all switches in the centre position.

INFORMATION

Before starting the component test, you have to place the enable switch for the functionality testing 1F(On-permanent/ Off/ On) in the upper position.

3 Delivery

3.1 Scope of Delivery

- System tester 757T16
- 757L16-2 Universal AC adapter
- AC adapter Euro FRIWO
- AC adapter USA for FRIWO
- Instructions for Use

3.2 Options

Test cables for the System Tester 757T16 are available for the components and modules of the MyoBock® System. An overview of the test cables and their applications is provided below.

		Test cable									
		757P2=*					757P35=*		757P5	757P28	
		4	6	8	9	10	1	2			
											
Battery	757B8		•								
	757B13					•					
	757B15		•								
	757B20						•				
	757B21							•			
	757B35					•					
Battery	13E51=2	•		•	•						
	13E51=3	•		•	•						
	13E51=4	•		•	•						
	13E132			•					•		
Connecting cable	13E188			•					•		
	13E200								•		
	13E202								•		
MyoBock®	Electrode								•		
									•		
Digital Twin®	System Electric Hand	8E38=7	•						•		
		8E39=7	•						•		
		8E41=7	•							•	
		8E44=7	•							•	
	System Electric Gripper	8E33=7								•	
		8E34=7								•	
DMC plus®	System Electric Hand	8E38=6	•							•	
		8E39=6	•							•	
		8E41=6	•								•
		8E44=6	•								•
	System Electric Gripper	8E33=6									•
		8E34=6									•
SensorHand Speed®	8E38=8									•	
	8E39=8									•	
	8E41=8									•	
DMC VariPlus®	System Electric Gripper	8E33=9								•	
		8E34=9								•	
	Electric Wrist Rotator	10S17				•					

4 Establishing serviceability

4.1 Setup

- 1 A Display for voltage and current measurements with range display (V, A) through LEDs
- 1 B Battery Test key
- 1 C Connector for battery connection cable
- 1 D Function key (open)
- 1 E Function key (close)
- 1 F Enable key for functionality testing (On-permanent/ Off/ On)
- 1 G Connector for test cable or test plug (terminal device)
- 2 H Connector for 757L16-2 Universal AC Adapter

Operation

4.2 Function select

4.2.1 Battery test

INFORMATION

The handling and use of this product is reserved exclusively for trained specialists.

INFORMATION

Before starting the testing process, you have to place all switches in the centre position.

The nominal battery voltages should be tested using the tables

Battery		Nominal voltage
757B8	Interchangeable battery	6 V
757B13	Interchangeable battery	4.8 V
757B15	X-ChangePack	6 V
757B20	EnergyPack	7.2 V
757B21	EnergyPack	7.2 V
757B35	MyoEnergy Integral	7,4 V

1. Connect test cable to respective battery.
2. Connect System Tester 757T16 to the test cable via connector 1C (battery).
3. Press System Tester 757T16 key 1B (Battery Test) for 3 seconds.
4. Read voltage value from display 1A.

4.2.2 Battery connecting cable test

INFORMATION

The handling and use of this product is reserved exclusively for trained specialists.

INFORMATION

Before starting the testing process, you have to place all switches in the centre position.

The battery connecting cables are connected to the batteries according to the following table.

Battery connecting cable	Battery
13E51=2	757B15/ 757B8
13E51=3	757B15/ 757B8
13E51=4	757B15/ 757B8
13E132	757B13
13E188	757B20/ 757B21
	757B35

1. Connect battery connecting cable to the respective battery via the cable tracks.
2. Connect the battery connecting cables to the respective test cables.
3. Connect the test cable to the System Tester 757T16 via connector 1C (Battery).
4. Move the battery connecting cable
5. Read voltage value from display 1A and check for discontinuity.

4.2.3 Component test

INFORMATION

The handling and use of this product is reserved exclusively for trained specialists.

INFORMATION

Before starting the component test, you have to place the enable switch for the functionality testing 1F(On-permanent/ Off/ On) in the upper position.

1. Connect test cable to respective battery.
2. Connect System Tester 757T16 to the test cable via connector 1C (battery).
3. Connect components with the respective test cable or test connector.
4. Use the test cable or test connector to connect the System Tester 757T16 to connector 1G (Terminal Device).
5. Activate the System Tester 757T16 function key 1E (close).
6. Activate the System Tester 757T16 function key 1D (open).

5 Maintenance

5.1 Maintenance

5.1.1 Calibration

NOTICE

Use by unauthorized personnel.

Opening and calibration by unauthorized personnel results in damage to the product.

Opening and calibration is reserved exclusively for Ottobock measuring and test equipment monitoring.

NOTICE

Manipulation.

Independent changes or modifications lead to malfunction of the product.

Opening and calibration is reserved exclusively for Ottobock measuring and test equipment monitoring.

INFORMATION

The handling and use of this product is reserved exclusively for trained specialists.

The product must be calibrated by Ottobock measuring and test equipment monitoring at 3 year intervals.

6 Disposal

6.1 Disposal Information

This product may not be disposed of with regular domestic waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in your country regarding return and collection processes.

7 Legal Information

7.1 Liability statement

In case of the culpable breach of so-called cardinal obligations, Ottobock is liable for the reimbursement of expenses and the payment of compensation (jointly referred to as “compensation” in Section 5) according to the applicable legal regulations. Cardinal obligations in this sense include all obligations where a breach would compromise the ability to fulfil the purpose of this contract, or where performance is required in order to properly fulfil this contract and where you may rely on regular performance. However, to the extent that breaches of cardinal obligations are due to slight negligence and do not result in death, physical injury, or the impairment of health, your claims to compensation are limited to the amount of the typically foreseeable loss.

Ottobock is also liable to you according to the provisions of the Product Liability Act; in case of intent or gross negligence; for the loss of life, physical injury, or the impairment of health; where Ottobock issues any guarantees; and in other cases of mandatory legal liability.

Any other claims for compensation by you against Ottobock – no matter on what legal grounds and especially due to material defects, defective title, and / or the breach of other obligations arising from the contract on the part of Ottobock, its legal representatives, employees, or assistants or due to illegal acts – are excluded.

To the extent that the liability of Ottobock is limited or excluded by the preceding provisions, the same applies to the personal liability of the legal representatives, employees, or assistants of Ottobock.

The preceding provisions do not affect the burden of proof to your detriment. Claims for compensation on your part expire within the legal period of limitation, starting at the beginning of said period as established by law.

7.2 CE Conformity

The System Tester 757T16 fulfills the requirements of the guideline 2004 / 108 / EU for electromagnetic compatibility. The declaration of conformity was created by Ottobock according to appendix II of the guidelines.

The System Tester 757T16 also fulfills the requirements of the guideline 2006/95/EG for electrical equipment to be used within certain voltage limits (low voltage guideline).

A copy of the declaration of conformity can be requested from the manufacturer (see last page).

7.3 Registered trademarks

MyoBock®, Digital Twin®, DMC plus® and DMC VariPlus® are registered trademarks of Ottobock Healthcare Products GmbH.

8 Appendixes

8.1 Technical Data

Size (WxHxD)	210x225x72 mm
Weight (without AC adapter)	approx. 700 g
Operating temperature	0-50°C
AC adapter operating voltage	100-240V 50 - 60 Hz
AC adapter output voltage	12V DC

Ottobock Healthcare Products GmbH
Kaiserstraße 39 · 1070 Wien · Austria
T +43-1 523 37 86 · F +43-1 523 22 64
info.austria@ottobock.com · www.ottobock.com

Ottobock has a certified Quality Management Systems in accordance with ISO 13485.