

## 17PA1=\*-WR

<b>DE</b>	Gebrauchsanweisung .....	4
<b>EN</b>	Instructions for use .....	12
<b>FR</b>	Instructions d'utilisation .....	19
<b>IT</b>	Istruzioni per l'uso .....	28
<b>ES</b>	Instrucciones de uso .....	36
<b>PT</b>	Manual de utilização .....	45
<b>NL</b>	Gebruiksaanwijzing .....	53
<b>SV</b>	Bruksanvisning .....	61
<b>DA</b>	Brugsanvisning .....	69
<b>NO</b>	Bruksanvisning .....	76
<b>CS</b>	Návod k použití .....	84
<b>TR</b>	Kullanma talimatı .....	92
<b>RU</b>	Руководство по применению .....	100


1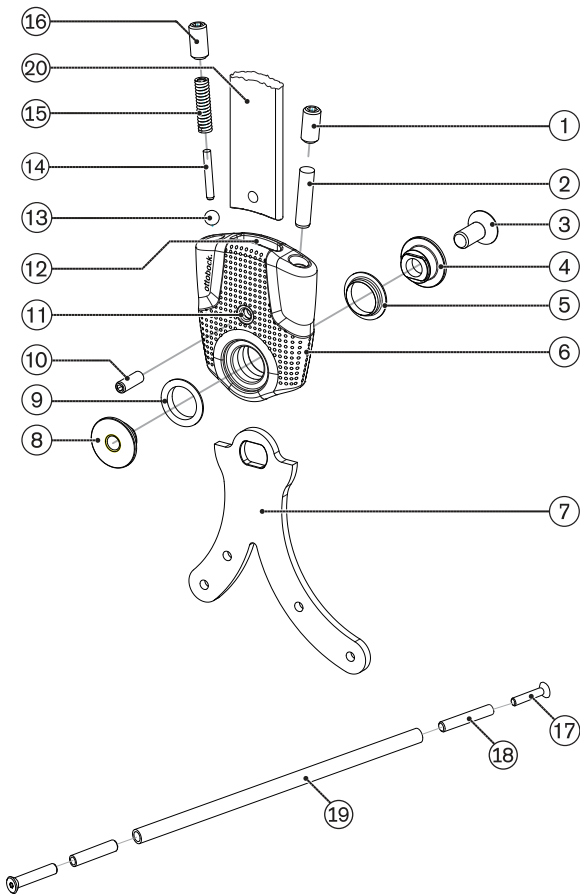

2

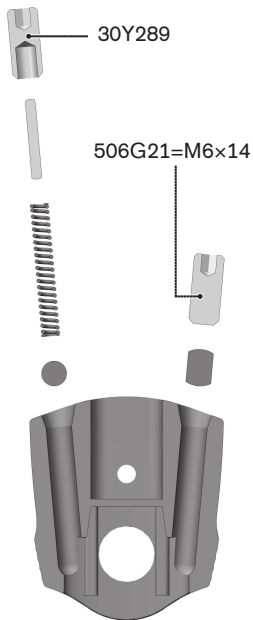3
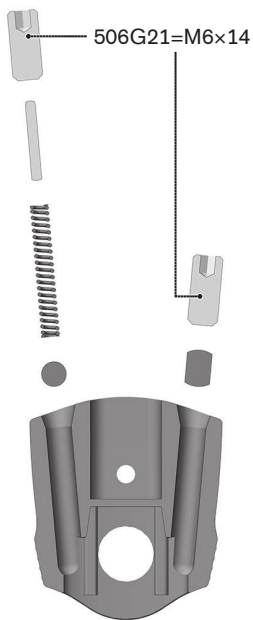
## INFORMATION

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## INFORMATION

Date of last update: 2022-08-16

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

These instructions for use provide you with important information on the processing of the 17PA1=\*-WR waterproof orthotic ankle joints made of plastic.

## 2 Product description

### 2.1 Available sizes

Reference number	Size	Insertion zone in mm	Material
17PA1*-WR	*=14	14	Plastic
17PA1*-WR	*=20	20	

### 2.2 Components/design

Scope of delivery (see fig. 1)				
Item	Quantity [Piece(s)]	Designation	Article number	
			17PA1=14	17PA1=20
1	2	Set screw	506G21=M6x14	
2	2	Cylinder pin	506A5=5x-M6x6	506A27=5-xM6x20
3	1	Joint screw	29PA1=14	29PA1
4	1	Joint bolt		
5	1	Bushing		
6	1	Ankle joint		
8	1	Joint nut	29PA1=14	29PA1
9	1	Bushing		
10	1	Set screw	506G21=M4x12	

Scope of delivery (see fig. 1)					
Item	Quantity [Piece(s)]	Designation	Article number		
				17PA1=14	17PA1=20
11	1	Bore hole for set screw			
12	1	Insertion zone for bar			
13	2	Ball		509Y1=5.0	
14	2	Cylinder pin		506A8=2x-16	506A8=2.-5x18
15	2	Compression spring		513D87=-0.75x2.95-x2	513D18=-4.7x31-2
16	2	Set screw		30Y289	506G21=-M6x14
17	2	Screw		501S84=-M4x25	501T28=-M6x35
18	2	Adapter for parallel alignment tube		29PK4=14	
19	1	Parallel alignment tube		30Y216	

Not included in the scope of delivery (see fig. 1)			
Item	Quantity [Piece(s)]	Designation	Reference number
7	1	Foot stirrup	17PF1*
20	1	Bar	651P4*, 605P8*

### 3 Intended use

#### 3.1 Indications for use

The waterproof orthotic joints made of plastic are to be used **exclusively** in pairs for lower limb orthotic fittings in waterproof walking devices.

The waterproof walking aid is intended for use in wet areas, e.g. transfer into a pool, and is not a replacement for the initial orthosis.

Article number	Max. body weight
17PA1=14-WR	45 kg
17PA1=20-WR	100 kg

### 3.2 Indications

- Partial or total paralysis of the leg muscles
- Orthopaedic diseases of the lower limbs

Indications must be determined by the physician.

### 3.3 Further Usage Restrictions

The product is not suitable for use in sports.

### 3.4 Lifetime

The product is designed for a lifetime of **3 years** when used as intended and assembled professionally.

### 3.5 Qualification

Patients may be fitted with the product only by trained qualified personnel. The qualified personnel must be familiar with the handling of the various techniques, materials, machines and tools.

### 3.6 Combination possibilities

The 17PA1\* orthotic ankle joints can be combined with the 17PK1\* orthotic knee joints.

## 4 Safety

### 4.1 Explanation of warning symbols

 <b>CAUTION</b>	Warning regarding possible risks of accident or injury.
 <b>NOTICE</b>	Warning regarding possible technical damage.

### 4.2 Safety instructions

 <b>CAUTION</b>
<b>Excessive strain on load-bearing components</b> Injuries due to changes in or loss of functionality ▶ Only use the product for the defined area of application.

- ▶ If the product has been exposed to extreme strain (e.g. due to falling), take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

**⚠ CAUTION**

**Incorrect alignment or assembly**

Injuries due to changes in or loss of functionality

- ▶ The product may only be installed by trained, qualified personnel.
- ▶ Observe the alignment and assembly instructions.

**⚠ CAUTION**

**Mechanical damage to the product**

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ Check the product for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, discontinue use of the product and have it checked by authorised, qualified personnel.

**⚠ CAUTION**

**Excessive strain due to use on more than one patient**

Risk of injury and loss of functionality as well as damage to the product

- ▶ Use the product on only one patient.
- ▶ Observe the maintenance recommendations.

**⚠ CAUTION**

**Unallowable use of lubricants**

Risk of injury and loss of functionality as well as damage to the product

- ▶ The product is designed to be free of lubricants. Avoid contact with lubricants (e.g. grease, oil).
- ▶ After contact with lubricants, promptly clean the product or parts of the product with a degreasing cleaning agent (e.g. acetone or isopropyl alcohol).

**NOTICE**

**Contact with saltwater or water containing chlorine/soap, fresh water or abrasive substances (e.g. sand)**

Damage and premature product wear

- ▶ Following any contact with the substances identified above, promptly clean the product in accordance with the section "Cleaning".

#### NOTICE

#### **Exposure of the product to unsuitable environmental conditions**

Damage, brittleness or destruction due to improper handling

- ▶ Avoid storage in condensing ambient humidity.
- ▶ Avoid contact with abrasive substances (e.g. sand, dust).
- ▶ Do not expose the product to temperatures below -10 °C (14 °F) or above +60 °C (140 °F) (e.g. sauna, excessive sunlight, drying on a radiator).

## 5 Preparing the product for use

#### INFORMATION

- ▶ Check that the orthotic joints are aligned in the parallel position before delivering the orthosis.

#### INFORMATION

An adjustment aid is included with each pair of orthotic joints so the orthotic joints can be aligned parallel to one another. The adjustment aid consists of two screws, one parallel alignment tube and, depending on the size of the orthotic joints, two adapter pieces.

### **Mounting the foot stirrup**

- 1) Slide the bearing bushings onto the orthotic joint.
- 2) Secure the foot stirrup with the joint bolt and joint nut.
- 3) Screw the joint nut together with the joint screw.
- 4) Clean the threads of the screw connections using a degreasing cleaner and secure them with Loctite 241.

### **Mounting the compression spring**

- > **Prerequisite:** The foot stirrup is mounted.
- 1) Insert the ball, compression spring and cylinder pin into the threaded hole (see fig. 2, see fig. 3).
- 2) Screw the set screw into the threaded hole and set the spring force of the ankle joint.
- 3) Clean the threads of the screw connections using a degreasing cleaner and secure them with Loctite 241.

## Mounting the stop pin

> **Prerequisite:** The foot stirrup is mounted.

- 1) Insert the stop pin into the threaded hole (see fig. 2, see fig. 3).
- 2) Screw the set screw into the threaded hole and set the angle of the orthotic joint.
- 3) Clean the threads of the screw connections using a degreasing cleaner and secure them with Loctite 241.

## Mounting the bar

> **Required tools:** 624S14=4 drill bit, 726S9=90x11.5 counterbore.

- 1) Insert the bar into the insertion zone of the orthotic joint and mark the position of the bore hole.
- 2) Use the 624S14\* drill bit to make the bore hole.
- 3) Deburr the bore with the 726S9\* counterbore.
- 4) Secure the bar in the insertion zone with the 506G21\* set screw.

**INFORMATION: The bars are glued in the insertion zone to complete the assembly.**

## Gluing in bars

**Required materials:** 636W28\* special adhesive, degreasing cleaner

The bars must be glued in the insertion zones to ensure stability.

- 1) Clean the connecting surfaces with a degreasing cleaner.
- 2) Apply the special adhesive to the connecting surfaces.
- 3) Insert the bars.
- 4) Secure with the enclosed screws.
- 5) Allow to cure for at least **4 hours**.

**INFORMATION: Final bonding strength will be reached after 16 hours.**

## Removing the bar

- 1) Unscrew the set screw.
- 2) Tap out and remove the bar.

## 17PA1\*

The 17PA1\* orthotic ankle joints are adapted to the needs of the patient through the individual combination of the ball, stop pin and compression spring with the set screws.

The following combinations are possible:

Combination		Intended use
Installation position Anterior	Installation position Posterior	
Stop pin	Stop pin	e.g. ICP or spina bifida
Compression spring	Stop pin	Support for dorsal extension, e.g. in case of lower leg muscle weakness, to prevent hyper-extension of the knee joint or for energy return during toe-off
Compression spring	Compression spring	Dorsal and plantar support, e.g. in case of lower leg muscle weakness
Stop pin	Compression spring	Dorsiflexion assist with adjustable dorsal stop, e.g. in case of peroneal paralysis with or without own muscular knee joint protection

## 6 Cleaning

Promptly clean the product after contact with:

- Water containing salt, chlorine or soap, perspiration, urine or splashed water (e.g. after rain) and soiling.
  - 1) Rinse the product with clear fresh water.
  - 2) Dry the product with a cloth or allow it to air dry. Avoid exposure to direct heat (e.g. from an oven or radiator).

Clean the product immediately after contact with:

- Lubricants (e.g. oil and grease)
  - 1) Wet a cloth with a degreasing cleaner.
  - 2) Wipe the orthotic joints or components with the cloth.

## 7 Maintenance

### INFORMATION

**The product may be exposed to increased loads by the patient.**

- ▶ Shorten the maintenance intervals according to the expected loads.

The manufacturer requires at least a semi-annual inspection of the product to verify functionality and check for wear.

Spare parts are listed in the section "Components/design".

## Replacing the bushing

- 1) Loosen the joint screw and remove the joint bolt and joint nut from the orthotic joint.
- 2) Disassemble the orthotic joint.
- 3) Insert the new bushing into the orthotic joint.
- 4) Assemble the orthotic joint with the joint bolt, joint nut and joint screw.
- 5) Clean the threads of the screw connections using a degreasing cleaner and secure them with Loctite 241.

## 8 Disposal

Dispose of the product in accordance with national regulations.

## 9 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

### 9.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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