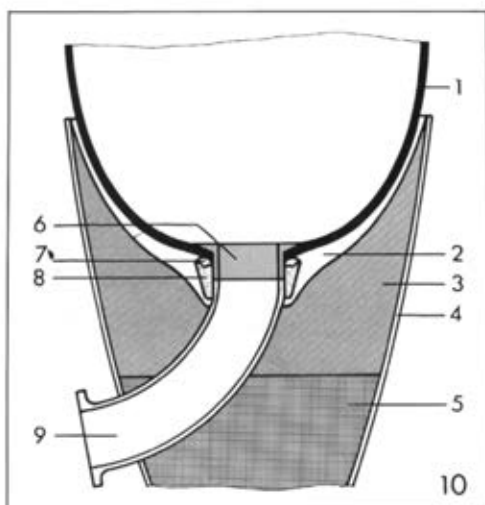
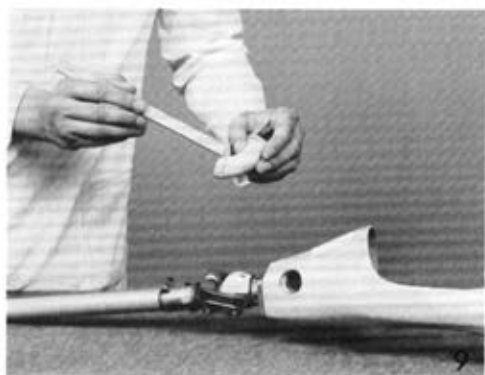
## 21Y97

<b>DE</b> Gebrauchsanweisung .....	4
<b>EN</b> Instructions for use .....	6
<b>FR</b> Instructions d'utilisation .....	8
<b>IT</b> Istruzioni per l'uso .....	10
<b>ES</b> Instrucciones de uso .....	12
<b>PT</b> Manual de utilização .....	14
<b>NL</b> Gebruiksaanwijzing .....	16
<b>SV</b> Bruksanvisning .....	18
<b>DA</b> Brugsanvisning .....	20
<b>PL</b> Instrukcja użytkowania .....	22
<b>HU</b> Használati utasítás .....	24
<b>CS</b> Návod k použití .....	26
<b>EL</b> Οδηγίες χρήσης .....	28
<b>ZH</b> 使用说明书 .....	30

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---

**NOTICE** Warnings regarding possible technical damage.

**INFORMATION** Important information on the fitting/use.

## INFORMATION

Last update: 2020-04-16

- Please read this document carefully before using the product and observe the safety notices.
- Instruct the user in the safe use of the product.
- Please contact the manufacturer if you have questions about the product or in case of problems.
- Report each serious incident in connection with the product, in particular a worsening of the state of health, to the manufacturer and to the relevant authority in your country.
- Please keep this document in a safe place.

The 21Y97 Valve provides the detachable connection between the flexible socket and the distal portion of the socket frame (for example in the ISNY Socket). The valve kit comprises all parts including patterns which are required for adequate installation.

For lateral installation use 21Y105 Valve Kit (without tube).

## 1 Parts (Fig. 1)

- (1) 21Y101 Pattern for Vacuum Forming, with wood screw
- (2) 21Y102 Pattern for Laminating, with countersunk screws M5
- (3) 21Y99=10 Insert Ring
- (4) 21Y98 Lamination Collar with 21Y103 Seal Ring
- (5) 21Y40=40 Flat Rubber Valve with 21Y77 Tube and Seat Ring
- (6) 4Z8 Screw-in Aid

## 2 Safety instructions

### NOTICE

**Risk of damage if using powder.** Any kind of powder on the inner side of the socket or on the skin can restrict or disable the functionality of the valve during use. Avoid using powder in areas close to the valve.

## 3 Instruction for installation

The valve can be provided at the distal end, centrally or laterally, depending on the length of the residual limb. Detailed instruction for the fabrication of an ISNY Socket using the 21Y97 Valve is given in the Technical Information bulletins 2.3.5 and 2.3.6.

Hold the 21Y101 Pattern for Vacuum Forming in position at the distal end of the plaster cast using a wood screw (Fig. 2).

After vacuum forming clear the cylindrical part of the pattern. Remove the wood screw and attach the 21Y102 Pattern for Laminating by a countersunk screw (Fig. 3).

After lamination clear the area of the pattern by grinding. Remove the countersunk screw and pull the vacuum formed socket out of the lamination socket (Fig. 4).

Press the insert ring from the inside into the socket using the enclosed screw-in aid, and attach the lamination collar from the outside (Fig. 5).

Fill Plastilin clay into the valve opening at the lamination socket, add PEDILEN foam to the socket attachment area and subsequently drill a hole through it, using a Forstner drill, diam. 28 mm (Fig. 6).

Apply sealing resin putty to the conical surface of the lamination collar. Press the flexible socket into the frame. Make sure it fits accurately in the valve area (Fig. 7).

After shaping the foam, fit in the connection tube of the valve (Fig. 8).

Apply sealing resin to the valve tube, insert it into the valve opening in such a way that tube and lamination collar are bonded to each other (Fig. 9).

Cross-cut of a socket with installed valve (Fig. 10):

- (1) Flexible Socket
- (2) Plastic Socket Frame
- (3) PEDILEN Rigid Foam H300
- (4) Outer Laminant
- (5) 5R1 Socket Attachment Block
- (6) 21Y99=10 Insert Ring\*
- (7) 21Y103 Seal Ring
- (8) 21Y98 Lamination Collar
- (9) Connection Tube with Seat Ring

\* An insert ring with longer thread (21Y99=14) which can be shortened for individual requirements is available.

## 4 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### 4.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

### 4.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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