
755Y50 Wasserabschneider-Set / water separator set

DE Montageanleitung	3
EN Assembly Instructions.....	3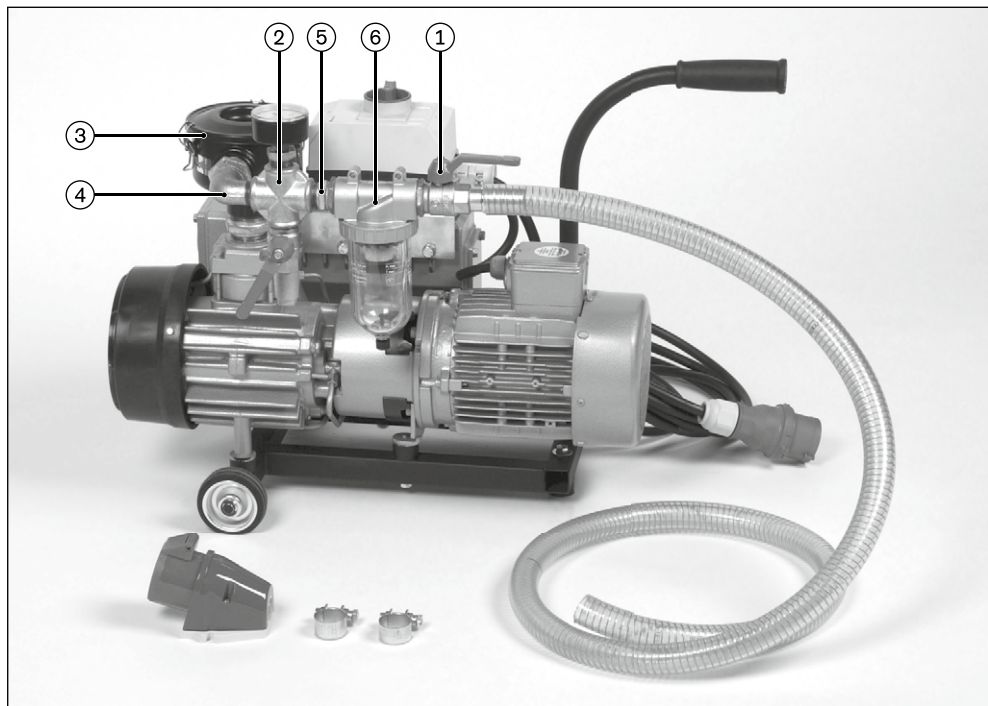

- | | |
|---|---|
| 1 | Kugelhahn / ball valve |
| 2 | Entlüftungseinheit / admixed air unit |
| 3 | Luftfilter / airfilter |
| 4 | 755X109=3/4" Messingwinkel / brass angle connector |
| 5 | 755Y25=3/4"x3/4" Doppelgewindenippel / double threaded nipple |
| 6 | 755Y43=3/4" Wasserabschneider / water separator |

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English

The Water Separator Set 755Y50 consists of the following component parts:

- 1 piece 755Y 43=3/4" water separator
- 2 pieces 755Y25=3/4"x 3/4" double threaded nipple
- 1 piece 755X109=3/4" brass angle connector
- 1 piece 627B14 sealing tape

1. Remove the ball valve (Pos. 1) and the admixed air unit (Pos. 2) from the vacuum pump.
2. Turn the air filter (Pos. 3) clockwise by 90°.
3. Screw the brass angle connector (Pos. 4) horizontally on the front piece of the air filter and screw the double threaded nipple (Pos. 5) in item (Pos. 4).
4. Screw the admixed air unit (Pos. 2) on the double threaded nipple (Pos. 5).
5. Screw the double threaded nipple (Pos. 5) in the admixed air unit (Pos. 2).
6. Screw the water separator (Pos. 6) on the double threaded nipple (Pos. 5).
Please pay attention to the direction of the arrow! <= towards the pump.
7. Screw the ball valve (Pos. 1) which was removed in the first step in the water separator (Pos. 6).

Sealing tape 627B14 has to be used for all connections. Let escape water through the water outlet valve of the water separator at regular intervals in order to guarantee a trouble-free operation of the water separator.

Otto Bock HealthCare GmbH
Max-Näder-Straße 15 · 37115 Duderstadt/Germany
T +49 5527 848-0 · F +49 5527 72330
healthcare@ottobock.de · www.ottobock.com

Ottobock has a certified Quality Management System in accordance with ISO 13485.