99B116 SoftTouch

DE	Gebrauchsanweisung	2
EN	Instructions for use	4
FR	Instructions d'utilisation	6
IT	Istruzioni per l'uso	8
ES	Instrucciones de uso	10
PT	Manual de utilização.....	12
NL	Gebruiksaanwijzing.....	14
SV	Bruksanvisning.....	16
DA	Brugsanvisning.....	18
PL	Instrukcja użytkowania.....	20
HU	Használati utasítás	22
CS	Návod k použití.....	24
RO	Instrucțiuni de utilizare.....	26
HR	Upute za uporabu	28
TR	Kullanma talimatı.....	30
EL	Οδηγίες χρήσης	32
RU	Руководство по применению	34
JA	取扱説明書.....	36
ZH	使用说明书.....	38

INFORMATION

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INFORMATION

Last update: 2016-11-15

- Please read this document carefully before using the product.
- Follow the safety instructions to avoid injuries and damage to the product.
- Instruct the user in the proper and safe use of the product.
- Please keep this document in a safe place.

Explanation of symbols

 **CAUTION** Warnings regarding possible risks of accident or injury.

 **INFORMATION** Additional information on the fitting / use.

CAUTION

Non-observance of safety instructions. Non-observance of the following safety instructions can lead to product malfunction or defects and result in a risk of injury to the patient.

1 Description

1.1 Application

99B116 SoftTouch Protective Outer Surface Cover is to be used solely as a protective covering for lower limb prostheses. They are pulled over the cosmetic foam cover to provide a final protective covering for an endoskeletal prosthesis.

1.2 Features

The outer surface covers feature high protection properties against environmental hazards such as moisture, soiling, UV rays and abrasion.

2 Use

2.1 Pulling on the cover

Before applying the Cover, remove any rings, bracelets, watches, etc. to avoid damaging the surface. To put on the Cover, turn it inside out down to the toe. Pull the foot section over the toes and instep until the heel section fits correctly. Pull the Cover up one section at a time, being careful to avoid creating wrinkles.

2.2 Note

CAUTION

Danger due to fire. Avoid contact with open flames or other sources of heat, as the product could catch fire.

INFORMATION

Make sure the inside of the footwear is smooth and free from defects. Rough edges along the shoe lining increase friction and wear on the stockings.

The SoftTouch Cover can be worn three months or longer with average use. Duration of use depends on the condition of the prosthesis and the type(s) of footwear worn and of the patient's activity.

2.3 Care

Clean lightly soiled Cover by hand washing with mild hand soap and water. Very dirty Covers can be hand washed in 30 °C / 86 °F warm water with gentle detergent.

3 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

3.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

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Otto Bock HealthCare GmbH