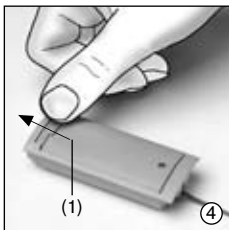
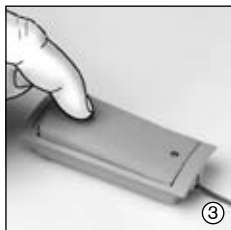
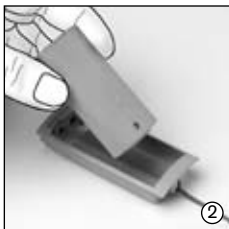
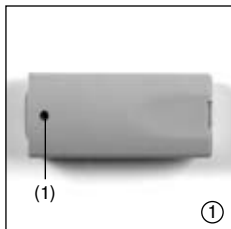
757B20/ 757B21

DE	Gebrauchsanweisung	3
EN	Instructions for Use	7
FR	Instructions d'utilisation	12
IT	Istruzioni per l'uso	16
ES	Instrucciones de uso	21
PT	Manual de utilização.....	25
NL	Gebruiksaanwijzing	30
SV	Bruksanvisning	34
CS	Návod k použití	39
EL	Οδηγίες χρήσης	43

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English

Date of the last update: 2013-11-06

- Please read this document carefully.
- Follow the safety instructions and the precautions specified in this accompanying document.
- If assistance is required during the start-up, use or maintenance of the product, as well as in the event of unexpected operating behaviour or circumstances, contact the manufacturer (see manufacturer's address on the back of this accompanying document).

Follow the safety instructions and the precautions specified in this accompanying document.

1 Application

The 757B20/757B21 EnergyPack is to be used **solely** as a power supply for upper extremity prostheses.

The 757B20/757B21 EnergyPack may be used only in combination with expressly authorized components. Please observe the Instructions for Use of the components in question, or ask your Ottobock Myo-Service.

Only start up the product in accordance with the information contained in the accompanying documents supplied. Instruct the patient in the proper use and care of the product. The product may not be transferred to the patient without prior instruction.

2 Safety Precautions

Inappropriate handling of the EnergyPack may be harmful to the user.

- The EnergyPack may only be charged with the Ottobock 757L20 Lithium-Ion Battery Charger.
- If any damage is visible, do not use the EnergyPack. If you discover any damage to the device, send the EnergyPack to an Ottobock service facility.
- Do not allow objects or liquids to fall onto or spill into the EnergyPack.
- Do not expose the EnergyPack charger to excessive heat (avoid direct sunlight), excessive smoke, mechanical vibrations or shock.

This product may not be disposed of with regular domestic waste. Disposal that does not comply with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions provided by the responsible authorities in your country regarding return and collection processes.

3 Scope of Delivery

- 1 pc. 757B20 EnergyPack and/or
- 1 pc. 757B21 EnergyPack
- 1 pc. 647H356 Instructions for Use

4 Description and Function

For installing the EnergyPack into the prosthetic socket, you need 757Z184=* or 757Z190=* Battery Mounting Set. For detailed information, please refer to the Installation Instructions provided with the set.

Unlike NiCad batteries, the lithium-ion battery EnergyPack has the benefits of a lower self-discharge, a higher cell potential and a higher energy density. Lithium-ion technology offers higher capacities and has no memory effect.

The EnergyPack's integrated **electronic safety system** provides increased safety. If there is a short circuit, the safety system automatically shuts off the EnergyPack.

As soon as a certain voltage level is exceeded while **charging**, the EnergyPack automatically shuts off. This protects the EnergyPack from overcharging.

Once the EnergyPack is reinserted into the prosthesis, it will be reset.

As soon as the voltage falls below the final **discharge** voltage, the EnergyPack will shut off automatically. After shut-down or if there is a short circuit, the EnergyPack can be reset by inserting it into the Ottobock 757L20 Lithium-Ion Battery Charger. Doing so will re-activate the charging process.

The EnergyPack has a capacity indicator (fig. 1, pos. 1) indicating the approximate capacity of the EnergyPack. The capacity indicator is activated for a few seconds as soon as the EnergyPack is inserted into the prosthetic shell.

Indicator	Capacity
green	> 50%
green and red	5% – 50%
red	< 5%

5 Inserting the EnergyPack

Insert the EnergyPack into the battery receptacle (fig. 2) until it snaps in (fig. 3). The mechanical latch also provides the electric connection.

Note!

Only clean, grease-free and undamaged components facilitate the flawless supply of electricity to the components. Use a Q-tip and some alcohol for this reason to clean the electrical contact points of chargers, EnergyPacks and battery receptacles regularly. The coating of the contact surfaces must not be damaged by pointed or sharp objects under any circumstances: penetrating humidity and especially body sweat could cause serious oxidation!

6 Changing the EnergyPack

The EnergyPack is designed so that it can easily be removed for recharging. To do this, push the engaging lever (fig. 4, pos. 1) away from the EnergyPack.

7 Technical Data

	757B20	757B21
Capacity	900 mAh	800 mAh
Charging time (full charge)	ca. 3,5 h	ca. 3,0 h
Nominal voltage (average)	7, 2 V	7, 2 V
Weight	65 g/2 ¹ / ₄ oz.	51g/1 ³ / ₄ oz.
Operating temperature	0–50 °C	0–50 °C
Service life	2 years	2 years
Operating time	depending on level of usage, approx. one day	

8 Liability

Otto Bock Healthcare Products GmbH, hereafter referred to as manufacturer, assumes liability only if the user complies with the processing, operating and maintenance instructions as well as the service intervals. The manufacturer explicitly states that this product may only be used in combination with components that

were authorized by the manufacturer (see instructions for use and catalogs). The manufacturer does not assume liability for damage caused by component combinations which it did not authorize.

The product may only be opened and repaired by authorized Ottobock technicians.

9 CE conformity

This product meets the requirements of the 93/42/EWG guidelines for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Ottobock with sole responsibility according to appendix VII of the guidelines.

10 Trademarks

All names mentioned in this accompanying document are subject without limitation to the provisions of the applicable trademark law and the rights of the respective owners.

All brands, trade names or company names mentioned herein may be registered trademarks and are the property of the respective owners. Should trademarks used in this accompanying document fail to be explicitly identified as such, this does not justify the conclusion that the respective designation is free of third-party rights.

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