9E264=7

DE	Gebrauchsanweisung	3
EN	Instructions for Use.....	5
FR	Instructions d'utilisation	7
IT	Istruzioni per l'uso	10
ES	Instrucciones de uso	12
NL	Gebruiksaanwijzing.....	14
SV	Bruksanvisning.....	17

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English

Date of the last update: 2014-06-25

- Please read this document carefully.
- Follow the safety instructions and the precautions specified in this accompanying document.
- If assistance is required during the start-up, use or maintenance of the product, as well as in the event of unexpected operating behaviour or circumstances, contact the manufacturer (see manufacturer's address on the back of this accompanying document).

1 General Description

The Electronic Control Unit is to be used solely for the operation of an Electric Greifer with Digital Twin control (8E33=7 or 8E34=7) in combination with a 757B20 and 757B21 Ottobock EnergyPack or a 757B8 Ottobock Interchangeable Battery. As soon as the electronic control unit

recognizes that a fully charged EnergyPack has been installed, it automatically switches to the operational mode "EnergyPack". To change back to the operational mode "757B8 Ottobock Interchangeable Battery" the electronic control unit must be sent to an Ottobock Myo-Service facility. As delivered, the electronic control unit is in the operational mode "757B8 Ottobock Interchangeable Battery".

This product may not be disposed of with regular domestic waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in your country regarding return and collection processes.

2 Changing the Electronic Control Unit

Attention!

Observe the correct order for changing the electronic control unit as described in these instructions for use.

Do not touch or bend the contact springs of the electronic control unit. Direction arrows located on the housing indicate the direction for assembly and disassembly. Assembling or disassembling the electronic control unit in the wrong direction will destroy the unit's contact springs.

3 Disassembly

- 3.1** Close the Electric Greifer.
- 3.2** Set the on/off switch to the "On" position. Bring the quick-disconnect wrist to mid-position. Unscrew the screw on the bottom of the coaxial bushing and remove the coaxial bushing (Fig. 1).
- 3.3** Turn the quick-disconnect wrist to the right up to the stop. Next, bring the on/off switch to the position "Off" (Fig. 2).
- 3.4** Use a thin screwdriver or similar tool to carefully lift the two snap locks. Push the electronic control unit off the Electric Greifer by applying the tool laterally to one of the snap locks (Fig. 3 and Fig. 4).

4 Assembly

- 4.1** Set the on/off switch to the "Off" position. Turn the quick-disconnect wrist to the left up to the stop. Carefully insert the electronic control unit into the Electric Greifer until you hear it click in place (Fig. 5).
- 4.2** Bring the quick-disconnect wrist to mid-position. Set the on/off switch to the "On" position. Mount the coaxial bushing and make sure that it's correctly positioned (Fig. 6).

5 Adjusting the Electrical Power Shut-Off

- 5.1** Position the quick-disconnect wrist so that the dial becomes visible.
- 5.2** Adjust the dial 1 so to achieve a maximal grip force of 160N.
- 5.3** With a new Electric Greifer, the shut-off current has been set.

For further information on the operational modes of the Digital Twin control, please refer to the 647H382 Instructions for Use of the Ottobock System Electric Greifer with Digital Twin control.

6 Liability

Otto Bock Healthcare Products GmbH, hereafter referred to as manufacturer, assumes liability only if the user complies with the processing, operating and maintenance instructions as well as the service intervals. The manufacturer explicitly states that this product may only be used in combination with components that were authorised by the manufacturer (see instructions for use and catalogues). The manufacturer does not assume liability for damage caused by component combinations which it did not authorise. The product may only be opened and repaired by authorised Otto Bock technicians.

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