
ottobock.

CE

757B15

DE	Gebrauchsanweisung (Fachpersonal)	4
EN	Instructions for use (qualified personnel)	13
FR	Instructions d'utilisation (Personnel spécialisé)	21
IT	Istruzioni per l'uso (Personale tecnico specializzato)	31
ES	Instrucciones de uso (Personal técnico especializado) ..	39
PT	Manual de utilização (Pessoal técnico)	49
NL	Gebruiksaanwijzing (Vakmensen)	58
SV	Bruksanvisning (Fackpersonal)	67
DA	Brugsanvisning (Faguddannet personale)	75
FI	Käyttöohje (Ammattihenkilöstö)	84
PL	Instrukcja użytkowania (Personel fachowy)	92
HU	Használati utasítás (szakszemélyzet)	101
RO	Instrucțiuni de utilizare (Personal de specialitate)	110
EL	Οδηγίες χρήσης (Τεχνικό προσωπικό)	119

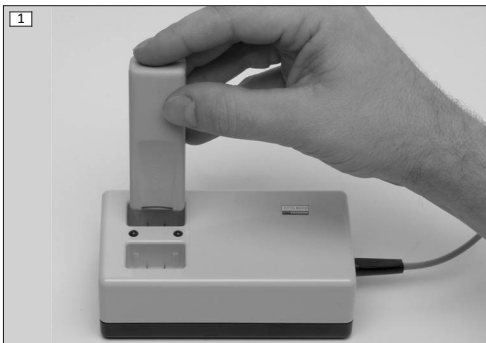INFORMATION

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INFORMATION

Last update: 2015-03-12

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product (e.g. regarding the start-up, use, maintenance, unexpected operating behaviour or circumstances). Contact information can be found on the back page.
- ▶ Please keep this document in a safe place.

These instructions for use provide you with important information on the use, adaptation and handling of the product.

The 757B15 X-ChangePack is referred to simply as the product/rechargeable battery below.

Instruct the patient in the proper use and care of the product. The product may not be transferred to the patient without prior instruction.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

2 Product Description

2.1 Construction and Function

The rechargeable battery is a power supply system. The battery receptacle is required for mounting the battery into the prosthetic socket. The electronic contact will be established automatically by inserting the battery into the battery receptacle.

2.2 Combination Possibilities

The product is to be combined exclusively with components from the Ottobock MyoBock system. Components from the Axon-Bus prosthetic system or components from other manufacturers cannot be used with this product.

3 Application

3.1 Indications for use

The product is to be used **exclusively** as a power supply for upper limb MyoBock system components.

3.2 Area of Application

The product is intended for use with Myo system hands.

3.3 Conditions of use

The product is intended **exclusively** for use on **one** patient. Use of the product by another person is not approved by the manufacturer. The product was developed for everyday use and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

4 Safety

4.1 Explanation of Warning Symbols

 WARNING	Warning regarding possible serious risks of accident or injury.
 CAUTION	Warning regarding possible risks of accident or injury.
NOTICE	Warning regarding possible technical damage.

4.2 Structure of the safety instructions

CAUTION

The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in case of failure to observe the hazard
- > E.g.: Consequence 2 in case of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

4.3 General Safety Instructions

WARNING

Fitting of children

Injury due to swallowing small parts.

- ▶ Never leave children unattended during the fitting.

CAUTION

Changing prosthetic components when switched on

Injury due to faulty control or malfunction of the prosthetic system.

- ▶ Before changing prosthetic components (e.g. gripping components) switch the product off.

CAUTION

Improper use

Injury due to faulty operation or malfunction of the product.

- ▶ Instruct the patient in the proper use of the product.

4.4 Patient information

CAUTION

Penetration of dirt and humidity into the product

Injury due to unexpected behaviour of the prosthetic system.

- ▶ Ensure that neither solid particles nor liquids can penetrate into the product.
- ▶ Do not use the product for bathing prostheses.

⚠ CAUTION

Use of the product when battery charge level is too low

Injury due to unexpected product behaviour.

- ▶ Check the current charge level before use and charge the product if required.
- ▶ Note that the operating time of the product may be reduced at low ambient temperatures or due to ageing of the battery.

⚠ CAUTION

Mechanical stress on the product

Injury due to faulty control or malfunction of the product.

- ▶ Do not subject the product to mechanical vibrations or impacts.
- ▶ Check the product for visible damage before each use.

⚠ CAUTION

Remaining in areas outside the allowable temperature range

Injury due to faulty control or malfunction of the product.

- ▶ Avoid remaining in areas outside the allowable temperature range (see Page 21).

⚠ CAUTION

Charging the product with soiled or damaged contacts

Injury due to unexpected behaviour of the product caused by insufficient charging.

- ▶ Make sure that the contacts are always clean and free of grease.

- ▶ Clean the electrical contacts of the charging plug and charging receptacle regularly using cotton swabs and a mild soap solution.
- ▶ Take care to avoid damaging the contacts with pointed or sharp objects.

⚠ CAUTION

Independent user manipulation of system components

Injury due to faulty control or malfunction of the product.

- ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.
- ▶ The battery may only be handled by Ottobock authorised, qualified personnel (no replacement by the user).
- ▶ The product and any damaged components may only be opened and repaired by authorised, qualified Ottobock personnel.

NOTICE

Use of incorrect power supply unit/battery charger

Damage to product due to incorrect voltage, current or polarity.

- ▶ Use only power supply units/battery chargers approved for this product by Ottobock (see instructions for use and catalogues).

5 Scope of Delivery and Accessories

Scope of Delivery

- 1 pc. 757B15 X-ChangePack
- 1 pc. 647H482 instructions for use (qualified personnel)

Required accessories for operation

- 1 pc. 757L14 battery charger
- 1 pc. 757Z103=1/757Z103=2 battery mounting set

6 Handling

6.1 Charging the battery

NOTICE

Use of incorrect power supply unit/battery charger

Damage to product due to incorrect voltage, current or polarity.

- ▶ Use only power supply units/battery chargers approved for this product by Ottobock (see instructions for use and catalogues).

NOTICE

Deep discharge of the battery

Irreparable damage to the product and resulting loss of functionality in case of irregular and incomplete charging of the battery.

- ▶ In case of long-term storage, fully charge the battery every 4 to 6 months.

The following points must be observed when charging the battery:

- The capacity of a fully charged battery is sufficient for an average day's use.
- We recommend charging the product overnight when used by the patient on a daily basis.
- Use the 757L16* power supply and 757L14 battery charger to charge the battery.

6.2 Inserting the battery into the battery receptacle

- ▶ Insert the battery in the battery receptacle and allow to engage.

INFORMATION: The electrical contact will be established by engaging the battery.

6.3 Removing the battery from the battery receptacle

- 1) Push the engaging lever of the battery receptacle forward.
- 2) Remove the battery from the battery receptacle.

7 Cleaning and Care

NOTICE

Improper product care

Damage to the product due to the use of incorrect cleaning agents.

► Only clean the product and the contacts with a damp cloth and mild soap (e.g. 453H10=1 Ottobock DermaClean).

- 1) Clean the product with a damp cloth and mild soap (e.g. Ottobock 453H10=1 Derma Clean) when needed.
Ensure that no liquid penetrates into the system component(s).
- 2) Dry the product with a lint-free cloth and allow it to air dry fully.

8 Disposal

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

9 Legal Information

9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

9.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

9.3 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

10 Appendices

10.1 Symbols Used

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

Declaration of conformity according to the applicable European directives

LOT.PPPP YYYY WW

Lot number

Legal manufacturer

10.2 Technical data

Rechargeable battery	757B15
Battery type	NiMH
Charging time until battery is fully charged	10 h – 12 h
Capacity	550 mAh
Nominal voltage (average)	6 V
Weight	77 g
Operating, storage and transport temperature in packaging	-20°C/-4°F to +60°C/140°F
Operating, storage and transport temperature without packaging	-20°C/-4°F to +60°C/140°F
Service life	2 years

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Otto Bock Healthcare Products GmbH