

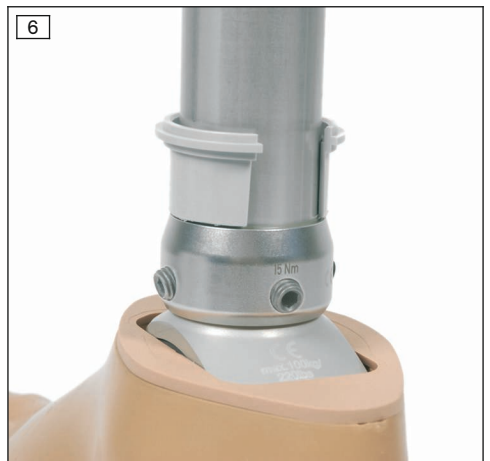
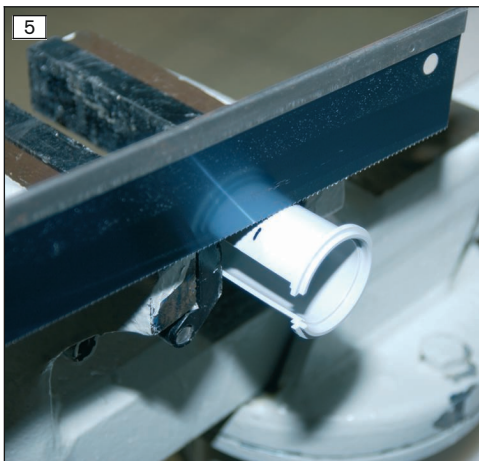
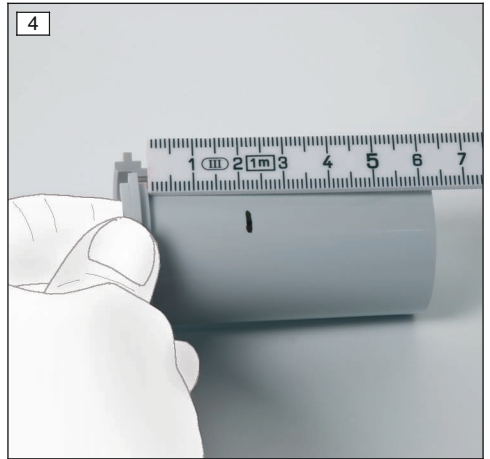
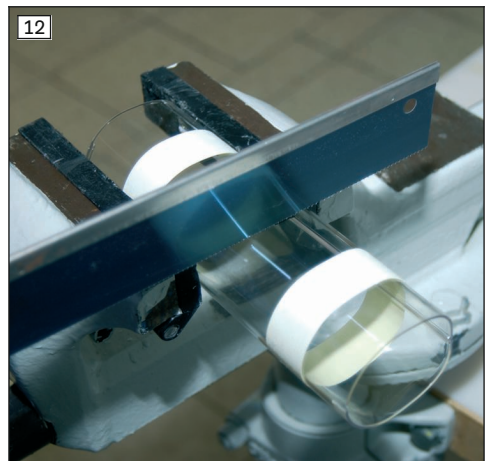
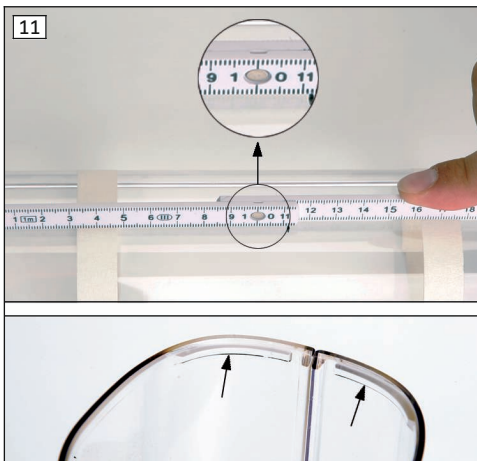
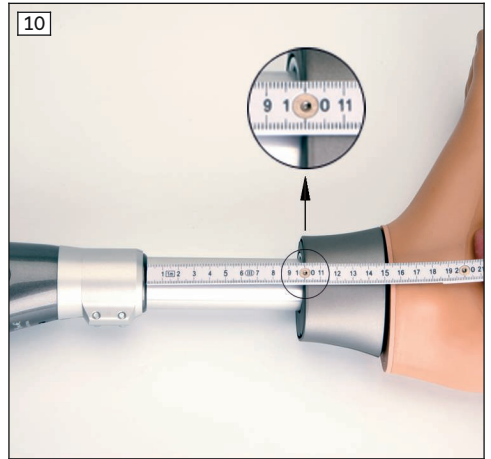
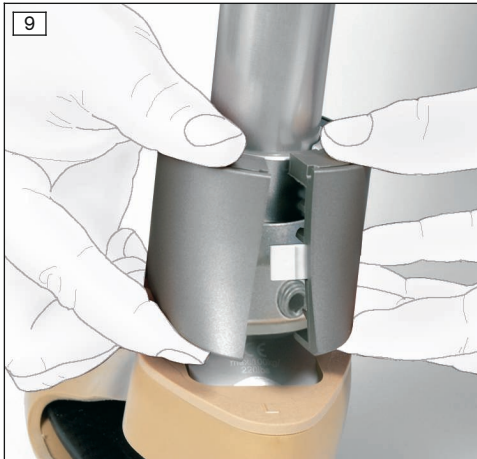
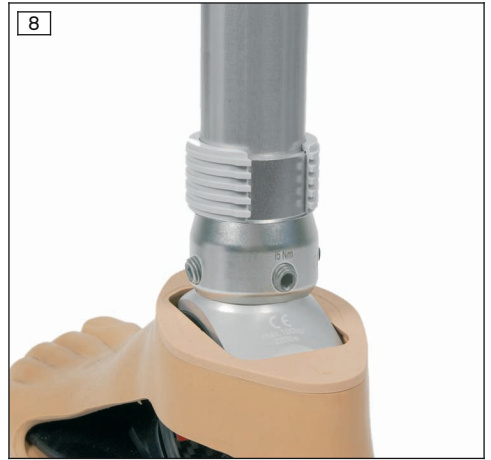
4X160=\*

<b>DE</b> Gebrauchsanweisung (Fachpersonal) .....	5
<b>EN</b> Instructions for use (qualified personnel) .....	12
<b>FR</b> Instructions d'utilisation (Personnel spécialisé) .....	19
<b>IT</b> Istruzioni per l'uso (Personale tecnico specializzato) .....	26
<b>ES</b> Instrucciones de uso (Personal técnico especializado) .....	33
<b>PT</b> Manual de utilização (Pessoal técnico) .....	41
<b>NL</b> Gebruiksaanwijzing (Vakmensen) .....	48
<b>SV</b> Bruksanvisning (Fackpersonal) .....	55
<b>DA</b> Brugsanvisning (Faguddannet personale) .....	62
<b>NO</b> Bruksanvisning (Fagpersonell) .....	69
<b>PL</b> Instrukcja użytkowania (Personel fachowy) .....	76
<b>HU</b> Használati utasítás (szakszemélyzet) .....	83
<b>CS</b> Návod k použití (Odborný personál) .....	90
<b>HR</b> Upute za uporabu (Stručno osoblje) .....	97
<b>EL</b> Οδηγίες χρήσης (Τεχνικό προσωπικό) .....	104
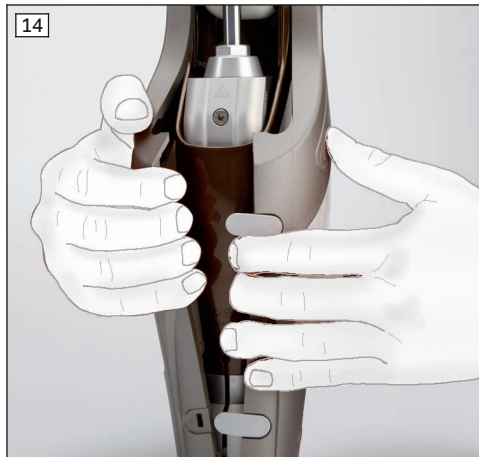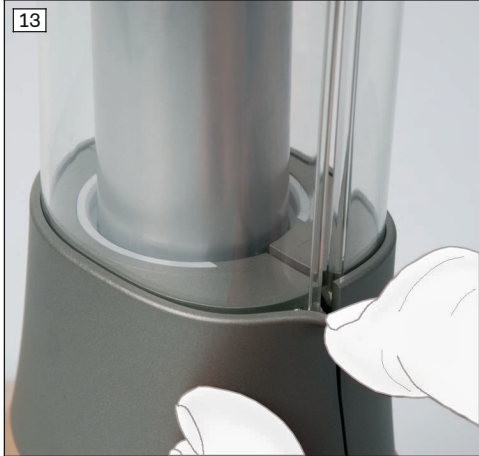
## INFORMATION

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### INFORMATION

Last update: 2017-03-13

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.
- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product (e.g. regarding the start-up, use, maintenance, unexpected operating behaviour or circumstances). Contact information can be found on the back page.
- ▶ Please keep this document in a safe place.

The 4X160 =\* C-Leg Protector is referred to simply as the protector/product below.

These instructions for use provide you with important information on the use, adaptation and handling of the product.

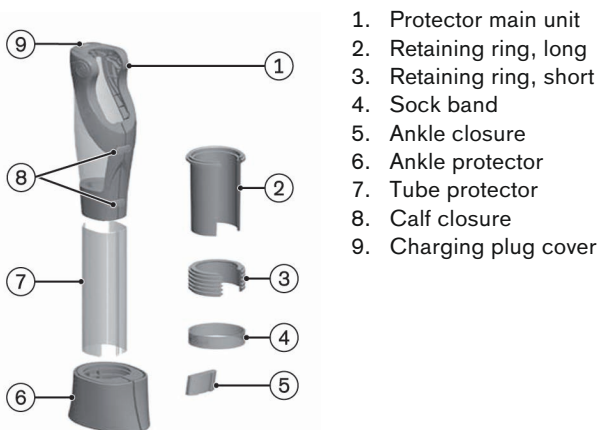
Instruct the patient in the proper use and care of the product. The product may not be transferred to the patient without prior instruction.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

## 2 Product description

### 2.1 Design

The product consists of the following components:

## 2.2 Function

The protector provides a means of enclosing the C-Leg electronic knee joint, including the tube adapter, to enhance its appearance and function. The housing and plug contacts of the joint are protected against external influences by the protector.

## 2.3 Combination possibilities

Prosthetic components that are suited for combination with the product are listed below.

### Knee Joint Combination Possibilities

Designation	Reference number
C-Leg	3C98/ 3C88 3C98-1/ 3C88-1 3C98-2/ 3C88-2
C-Leg compact	3C96/ 3C86 3C96-1/ 3C86-1
Ottobock Compact	3C95/ 3C85

## 3 Application

### 3.1 Intended Use

The product is to be used **solely** for lower limb exoprosthesis fittings.

### 3.2 Conditions of use

The product was developed for everyday use and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

## 4 Safety

### 4.1 Explanation of warning symbols

 <b>WARNING</b>	Warning regarding possible serious risks of accident or injury.
 <b>CAUTION</b>	Warning regarding possible risks of accident or injury.
 <b>NOTICE</b>	Warning regarding possible technical damage.

### 4.2 Structure of the safety instructions

 <b>CAUTION</b>
<p><b>The heading describes the source and/or the type of hazard</b></p> <p>The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows in case more than one consequence is possible:</p> <ul style="list-style-type: none"><li>&gt; E.g.: Consequence 1 in case of failure to observe the hazard</li><li>&gt; E.g.: Consequence 2 in case of failure to observe the hazard</li><li>▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.</li></ul>

### 4.3 General Safety Instructions

 <b>NOTICE</b>
<p><b>Clamping the product in the vice too hard</b></p> <p>Deformation or breakage of the product.</p> <ul style="list-style-type: none"><li>▶ Clamp the product in the vice with protective wedges for the purpose of shortening only as hard as is necessary.</li></ul>

**NOTICE****Improper assembly/disassembly of the product**

Damage to the product.

- ▶ Observe the process steps, instructions and required tools in the section "Preparation for use".
- ▶ Instruct the patient in the proper disassembly/ assembly of the product.

**NOTICE****Independent user changes or modifications made to the product**

Damage to the product.

- ▶ Manipulations to the product other than the tasks described in these instructions for use are not permitted.

**4.4 Patient information****NOTICE****Independent user changes or modifications made to the product**

Damage to the product.

- ▶ Have any changes or modifications to the product carried out only by authorised Ottobock qualified personnel.

**⚠ CAUTION****Penetration of dirt and humidity into the prosthesis**

Injury due to unexpected prosthesis behaviour or malfunction.

- ▶ Should moisture have penetrated the prosthesis, remove the product from the prosthesis and allow both to dry. Observe safety information in the 646D551=\* C-Leg or 646D179=\* C-Leg compact instructions for use.
- ▶ The product and the prosthesis must be inspected by an authorised Ottobock Service Centre.

**5 Scope of delivery**

Item	Quantity	Product	Article number
1	1 pc	Protector main unit	-
2	1 pc	Retaining ring, long	-
3	1 pc	Retaining ring, short	-
4	1 pc	Sock band	4X202
5	1 pc	Ankle closure	-
6	1 pc	Ankle protector	-
7	1 pc	Tube protector	-
8	2 pc	Calf closure	-
9	1 pc	Charging plug cover	-
no illustr.	1 pc	Instructions for use	647H506

## 6 Preparation for use

### Required Tools

Designation	Reference number
Saw	-

### 6.1 Preparing the retaining ring

#### 6.1.1 Selecting the retaining ring

Selecting the correct retaining ring depends on the chosen tube adapter and prosthetic foot. The table below shows the required retaining ring.

Tube adapter	Prosthetic foot	Retaining ring
2R82	All models	long (item 2)
2R81	$h > 10 \text{ mm}$ 	short (item 3)
	$h < 10 \text{ mm}$ 	long (item 2)

#### 6.1.2 Cutting the retaining ring to length

##### INFORMATION

The short retaining ring (item 3) must not be cut to length.

The long retaining ring (item 2) has to be cut to length.

- 1) Place the ankle protector around the tube adapter and slide it up to the joint plane of the foot (see fig. 1).
- 2) Mark the tube adapter at the top of the ankle protector (see fig. 2).
- 3) Remove the ankle protector.
- 4) Measure the distance from the mark on the tube adapter to the top of the tube adapter (see fig. 3).
- 5) Transfer the measured length onto the long retaining ring.  
Measure from the underside of the circumferential sleeve (see fig. 4).
- 6) Cut the retaining ring to length at the mark using a saw (see fig. 5).
- 7) Deburr the cut edge.

### 6.2 Assembling the ankle protector

#### 6.2.1 Retaining ring, long

- 1) Carefully stretch the retaining ring slightly and place it over the tube adapter (see fig. 6).  
Align the cut edge of the retaining ring in the direction of the foot adapter.  
Dorsally position the slot opening of the retaining ring.
- 2) Apply a thin, double-sided adhesive tape between the retaining ring and the tube adapter to prevent gliding movements between the retaining ring and the tube adapter.
- 3) Slide the retaining ring down until it rests on the top edge of the tube adapter.

- 4) Carefully bend open the ankle protector and place it over the retaining ring (see fig. 7).  
Make sure the circumferential sleeve of the retaining ring is engaged in the recess of the ankle protector.
- 5) Close the ankle closure (see fig. 9).

### 6.2.2 Retaining ring, short

- 1) Carefully stretch the retaining ring slightly and place it over the tube adapter (see fig. 8).  
Dorsally position the slot opening of the retaining ring.
- 2) Apply a thin, double-sided adhesive tape between the retaining ring and the tube adapter to prevent gliding movements between the retaining ring and the tube adapter.
- 3) Slide the retaining ring down until it rests on the top edge of the tube adapter.
- 4) Carefully bend open the ankle protector and place it over the retaining ring (see fig. 7).  
Make sure that one of the circumferential sleeves on the retaining ring is engaged in the recess of the ankle protector.  
Depending upon the sleeve you select, the distance of the ankle protector to the joint plane of the foot can be adjusted to approx. 5 mm.
- 5) Close the ankle closure (see fig. 9).

### 6.3 Cutting the tube protector to length

Adjusting the length of the tube protector is required because of different lower leg lengths of patients.

- 1) Measure the distance between the ankle protector and bottom of the joint (see fig. 10).
- 2) Add 12 mm to this measurement.
- 3) Transfer the total measurement onto the tube protector (see fig. 11).  
Transfer the total measurement from the tube end which shows locking sleeves (see fig. 11, arrows).
- 4) Use linen adhesive tape to clamp the tube protector together at both ends in order to ensure the tube will remain stable when cut off.
- 5) Cut the tube protector to length at the marks using a saw (see fig. 12).
- 6) Deburr the cut edge.

### 6.4 Assembling the tube protector and joint protector

#### INFORMATION

Remove the protective elastomer cap from the joint before assembling the joint protector on the C-Leg compact or Ottobock Compact (see 647G170=\* instructions for use).

#### INFORMATION

Before assembling the joint protector on the C-Leg, remove the fixed plug cover and insert the red protective cover into the communication port (see 647G750 instructions for use).

- 1) Stretch the tube protector and place it over the tube adapter.
- 2) Insert the tube protector into the groove of the ankle protector until it engages (see fig. 13).
- 3) Stretch the joint protector and slide it over the joint (see fig. 14).
- 4) Place the joint protector over the tube protector.  
Make sure the tube is enclosed completely by the recess of the joint protector.
- 5) Close the joint protector by pressing in both calf closures (see fig. 15).

### 6.5 Positioning the sock band

The sock band serves to fix the sock onto the tube protector.

- 1) Remove the shoe from the prosthetic foot.

- 2) Pull the sock band over the prosthetic foot and place it on the tube protector at an appropriate height.  
Only stretch out the sock band as far as necessary!
- 3) Put on the socks by pulling the collar of the sock over the sock band.

## 7 Handling

### INFORMATION

Advise your patient of the information in this section.

### 7.1 Protector Installation

- 1) Stretch the tube protector and place it over the tube adapter.
- 2) Insert the tube protector into the groove of the ankle protector until it engages (see fig. 13).
- 3) Stretch the joint protector and slide it over the joint (see fig. 14).
- 4) Place the joint protector over the tube protector. Make sure the tube is enclosed completely by the recess of the joint protector.
- 5) Close the joint protector by pressing in both calf closures (see fig. 15).

#### With use of the sock band

- 1) Remove the shoe from the prosthetic foot.
- 2) Pull the sock band over the prosthetic foot and place it on the tube protector at an appropriate height (see fig. 16).

### 7.2 Removing the Protector

- 1) Remove the shoe from the prosthetic foot.
- 2) Stretch the sock band and take it off over the prosthetic foot.
- 3) Open the calf closures on the joint protector.
- 4) Stretch open the joint protector and remove it from the knee joint.
- 5) Pull the tube protector out of the groove of the ankle protector.
- 6) Stretch open the tube protector take it off the tube adapter.

### INFORMATION

Removing the ankle protector from the prosthetic foot is not possible. The ankle protector stays on the prosthetic foot even when the protector is not being used.

## 8 Maintenance

- ▶ During the standard inspection of the prosthesis components, also inspect the product.

### 8.1 Cleaning and Care

#### NOTICE

#### Improper product care

Damage to the product due to the use of incorrect cleaning agents.

- ▶ Only clean the product with a damp cloth and mild soap (e.g. 453H10=1 Ottobock DermaClean).
- 1) Clean the product with a damp cloth and mild soap (e.g. Ottobock 453H10=1 Derma Clean) when needed.  
Ensure that no liquid penetrates into the system component(s).
  - 2) Dry the product with a lint-free cloth and allow it to air dry fully.

## 9 Disposal

In some jurisdictions it is not permissible to dispose of these products with unsorted household waste. Disposal that is not in accordance with the regulations of your country may have a detrimental impact on health and the environment. Please observe the instructions of your national authority pertaining to return and collection.

## 10 Legal information

### 10.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

### 10.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

### 10.3 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

## 11 Appendices

### 11.1 Technical data

General	
Reference number	4X160=*
Service life	Wear part subject to normal wear and tear

### 11.2 Accessories

#### 4X177 Protector closure kit

Piece(s)	Product	Article number
1 pc	Charging plug cover	-
2 pc	Calf closure	-
1 pc	Ankle closure	-

#### 4X178 Protector tube kit

Piece(s)	Product	Article number
1 pc	Tube protector	-
1 pc	Retaining ring, long	-
1 pc	Retaining ring, short	-
1 pc	Sock band	4X202

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Otto Bock Healthcare Products GmbH