Genium Protector 4X880=*

DE Gebrauchsanweisung (Benutzer)	3
EN Instructions for use (user)	7
FR Instructions d'utilisation (Utilisateur)	11
IT Istruzioni per l'uso (Utilizzatore)	15
ES Instrucciones de uso (Usuario)	19
PT Manual de utilização (Usuário)	23
NL Gebruiksaanwijzing (Gebruiker)	27
SV Bruksanvisning (Användare)	31
DA Brugsanvisning (Bruger)	35
NO Bruksanvisning (Bruker)	38
FI Käyttöohje (Käyttäjä)	42
PL Instrukcja użytkowania (Użytkownik)	46
CS Návod k použití (Uživatel)	50
HR Upute za uporabu (Korisnik)	54
TR Kullanma talimatı (Kullanıcı)	58
EL Οδηγίες χρήσης (Χρήστης)	62
RU Руководство по применению (Пользователь)	66
ZH 使用说明书 (用户)	71
KO 사용 설명서 (사용자)	74

INFORMATION

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INFORMATION

Date of last update: 2020-11-24

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Obtain instruction from the qualified personnel in the safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

The "4X880=* Genium Protective Cover" is referred to simply as the Protective Cover/product below.

These instructions for use provide you with important information on the use, adaptation and handling of the product.

Only put the product into use in accordance with the information contained in the accompanying documents supplied.

2 Product description

2.1 Design

The product consists of the following components:

The Protective Cover consists of the following components:

1. Protective Cover main unit
2. Upper closure
3. Lengthwise closure
4. Protector foot cuff for Protective Cover

2.2 Function

The Protective Cover provides a means of enclosing the Genium electronic knee joint, including the tube adapter, to enhance its appearance and function. The housing of the knee joint is protected against external influences by the Protective Cover, e.g. scratching and surface contamination. The Protective Cover is not required for the proper and safe functioning of the joint.

3 Intended use

3.1 Indications for use

The product is to be used **solely** for lower limb exoprosthetic fittings.

3.2 Conditions of use

The product was developed for everyday use and must not be used for unusual activities such as extreme sports (free climbing, paragliding, etc.).

3.3 Contraindications

- All conditions which contradict or go beyond the specifications listed in the section on "Safety" and "Indications for use".

4 Safety

4.1 Explanation of warning symbols

CAUTION Warning regarding possible risks of accident or injury.

NOTICE Warning regarding possible technical damage.

4.2 Structure of the safety instructions

CAUTION

The heading describes the source and/or the type of hazard

The introduction describes the consequences in case of failure to observe the safety instructions. Consequences are presented as follows if more than one consequence is possible:

- > E.g.: Consequence 1 in case of failure to observe the hazard
- > E.g.: Consequence 2 in case of failure to observe the hazard
- ▶ This symbol identifies activities/actions that must be observed/carried out in order to avert the hazard.

4.3 General safety instructions

CAUTION

Non-observance of safety instructions

Personal injury/damage to the product due to using the product in certain situations.

- ▶ Observe the safety instructions and the stated precautions in this accompanying document.

CAUTION

Improper disassembly, assembly and handling of the product

- > Injury due to pinching of body parts.
- > Possible injury in case of parts with sharp edges.
- ▶ Use increased caution during disassembly/assembly and handling of the product.

CAUTION

Changes or modifications to the product made independently

Injury due to faulty operation or malfunction of the product.

- ▶ Have any changes or modifications to the product carried out only by authorised, qualified Ottobock personnel.

INFORMATION

When the Protective Cover is installed, the battery charger cable has to point to the upper closure. A correct knee joint charging process is only ensured with this alignment.

5 Scope of delivery

- 1 pc. 4X880=* Genium Protective Cover
- 1 pc. 4P880=* Protector foot cuff
- 1 pc. Instructions for use (user)

6 Handling

INFORMATION

Removing the foot cuff from the prosthetic foot is not possible. The foot cuff stays on the prosthetic foot even when the Protective Cover is not being used.

6.1 Protective Cover Installation

- 1) Open or remove the closures.
- 2) Spread open the Protective Cover and apply it to the knee joint.
 - Thread the Protective Cover into the foot cuff from the front (see fig. 1).
 - The retainers of the Protective Cover have to engage on the frame element of the knee joint (see fig. 2).
- 3) Fasten the upper closure.
 - The retainer of the upper closure has to engage on the frame element of the knee joint (see fig. 3).
 - Slightly squeeze the Protective Cover together with both hands.
 - The upper closure has to be pushed down after engaging.
 - Push down the upper closure with the thumb (see fig. 4).
- 4) Fasten the lengthwise closure (see fig. 5).

NOTICE

Loss of stability due to inadequately installed retainers

No hold of the Protective Cover on the knee joint.

- ▶ Ensure that all retainers and closures are properly engaged and/or fastened.

6.2 Removing the Protective Cover

- 1) Pull down the textile material of the foot cuff until the Protective Cover is exposed.
- 2) Open or remove the closures.
- 3) Spread open the Protective Cover and remove it from the knee joint.
- 4) Fasten the two closures after removing the Protective Cover.

7 Maintenance

- ▶ During the standard inspection of the prosthesis components, also inspect the product.

7.1 Cleaning and Care

NOTICE

Improper product care

Damage to the product due to the use of incorrect cleaning agents.

- ▶ Only clean the product with a damp cloth and mild soap (e.g. 453H10=1-N Ottobock DermaClean).

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

8 Legal information

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

9 Technical data

Ambient conditions	
Storage and transport in original packaging	-25 °C/-13 °F to +70 °C/+158 °F
Storage and transport without packaging	-25 °C/-13 °F to +70 °C/+158 °F max. 93% relative humidity, non-condensing
Operation	-10 °C/+14 °F to +40 °C/+104 °F max. 93% relative humidity, non-condensing

General	
Reference number	4X880=*
Product service life	Wear part subject to normal wear and tear
Protective Cover weight (with closures)	450 g
Foot cuff weight	approx. 60 g

10 Appendices

10.1 Symbols Used

 Lot number (PPPP YYYY WW)
PPPP – plant
YYYY – year of manufacture
WW – week of manufacture

Manufacturer

Declaration of conformity according to the applicable European directives

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Otto Bock Healthcare Products GmbH