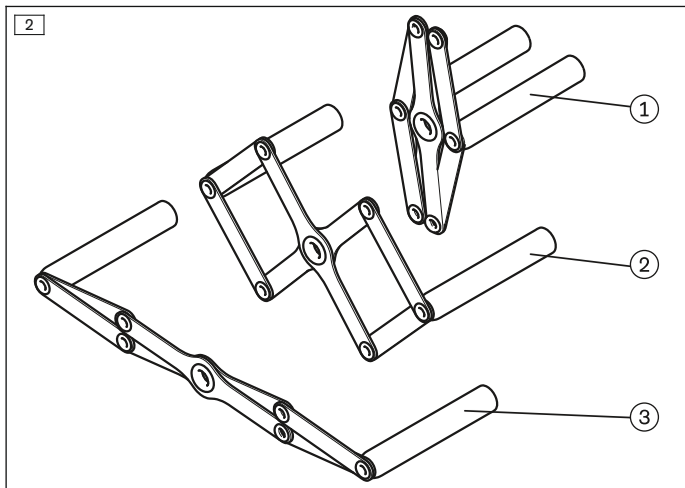
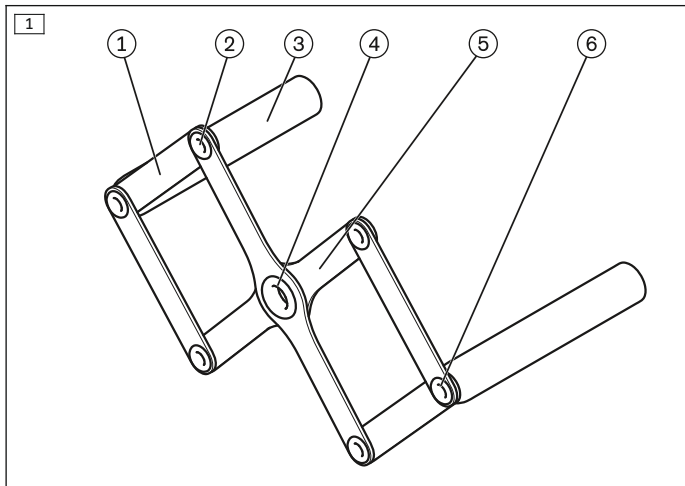
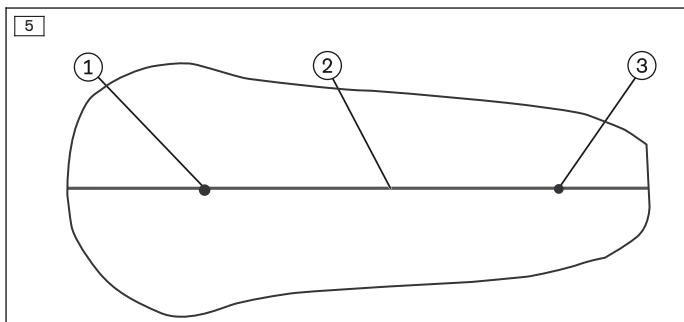
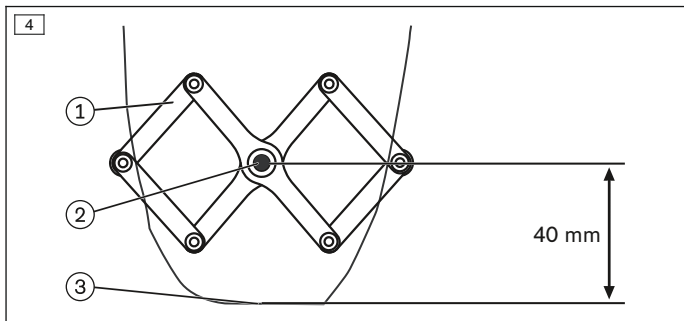
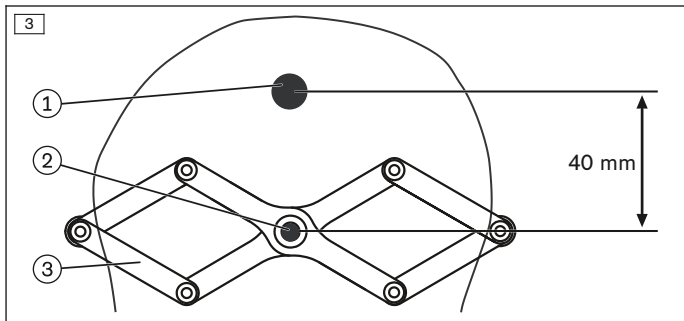
743A80

DE Gebrauchsanleitung (Benutzer / Techniker)	4
EN Instructions for use (user / technician)	5
FR Mode d'emploi (Utilisateur / Technicien)	7
ES Instrucciones de uso (usuario / técnico)	8

INFORMATION

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INFORMATION

Date of the last update: 2020-08-31

► Please read this document attentively.

1.1 Construction and Function

The 743A80 50:50 gauge was designed for the purpose of precisely determining the lateral centre line of a prosthetic socket. Adapting it to prosthetic sockets of various sizes is possible thanks to the movable legs (1 + 5) (see Figure 1).

2 Scope of Delivery

The scope of delivery for the product is shown on page 2 (Figure 1).

The following components are included in the scope of delivery in the specified quantities and / or are available for reordering as single components (■), single components with minimum order quantities (▲), single-component packs (●) or accessories:

743A80 50:50 Gauge					
Fig.		Item no.	Quantity	Designation	Reference number
–	■	–	1	Instructions for Use	647W002=all_INT
1	–	1	4	Leg, small	–
	–	2	4	Eyelet, small	–
	–	3	2	Positioning sleeve	–
	–	4	1	Eyelet, large	–
	–	5	2	Leg, large	–
	–	6	2	Eyelet, medium	–

3 Use

3.1 Handling

- > Have the prosthetic socket with the marked trochanter position ready.
- 1) Extend the product (see Figure 2, item 3).
 - 2) Position the product 40 mm below the trochanter mark (1) on the prosthetic socket and push it together (see Figure 3).
 - 3) Mark the centre of the prosthetic socket through the centre eyelet (2) (see Figure 3).
 - 4) Position the product on the prosthetic socket, 40 mm over the distal end of the prosthetic socket (3), and push it together (see Figure 4).
 - 5) Mark the centre of the prosthetic socket through the centre eyelet (2) (see Figure 4).
 - 6) Draw the lateral centre line of the prosthetic socket (2) with the help of the two marked points (1 + 3) (see Figure 5).

3.2 Cleaning and Care

- 1) Clean with a damp cloth and soap after every use.
 - 2) Rinse the product with clear fresh water.
 - 3) Dry the product with a soft cloth.
- If the product does not operate easily, slightly grease the eyelets (see Figure 1, item 2 + 4 + 6).

4 Liability

The manufacturer's warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use.

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