
## 743A8

<b>DE</b>	Gebrauchsanleitung (Benutzer/Techniker) .....	3
<b>EN</b>	Instructions for use (user/technician) .....	4
<b>FR</b>	Mode d'emploi (Utilisateur/Technicien) .....	5
<b>IT</b>	Istruzioni d'uso (utilizzatore/tecnici) .....	6
<b>ES</b>	Instrucciones de uso (usuario/técnico) .....	7
<b>PT</b>	Manual de instruções (Usuário/Técnico) .....	8
<b>NL</b>	Gebruiksaanwijzing (gebruiker/technicus) .....	9
<b>SV</b>	Bruksanvisning (användare/tekniker) .....	10
<b>DA</b>	Brugsvejledning (bruger/tekniker) .....	11
<b>NO</b>	Bruksanvisning (bruker/teknikker) .....	12
<b>PL</b>	Instrukcja użytkowania (Użytkownik/Technik) .....	12
<b>CS</b>	Návod k použití (pro uživatele/techniky) .....	14
<b>TR</b>	Kullanım kılavuzu (Kullanıcı/Teknisyen) .....	15
<b>EL</b>	Οδηγίες χρήσης (χρήστης/τεχνικός) .....	16
<b>RU</b>	Руководство по применению (для пользователей/ технического персонала) .....	17

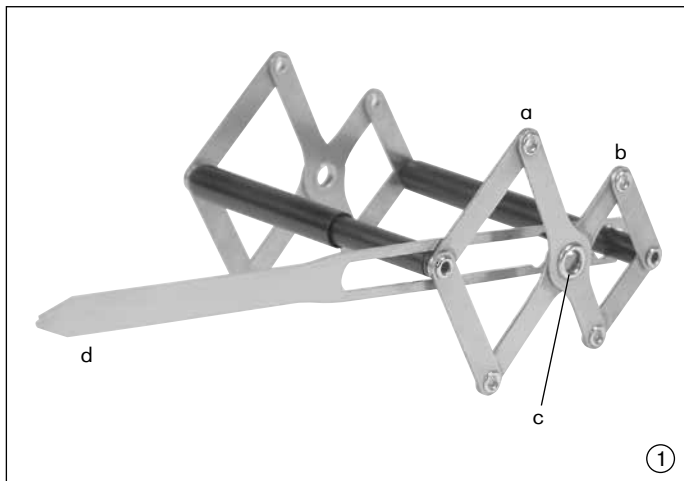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Date of the last update: 2020-10-16

- Please read this document carefully.
- Follow the safety instructions.

## 1 Components (Fig. 1)

- Long arm
- Short arm
- Pivot joint
- Rotation pointer

## 2 Indications for use

The compromise center of rotation for accurate location of orthosis joints based on Nietert can be marked directly on the leg. The a-p knee width is divided by the proportion 60:40 at a distance of 25 mm above the gap between tibia and femur.

## 3 Application and use

Unfold the knee pivot gauge and pull it onto the knee. The long arm is in the front and the black sleeves are located on the patella center, or possibly, placed in the popliteal space created when the leg is slightly flexed (Fig. 2). Adapt the knee pivot gauge to the knee size by pressing it together. Mark off the compromise point of rotation medially and laterally (Fig. 3). Use the rotation pointer to visually check the rotation of the determined joint axis in relation to the longitudinal axis of the leg.

## 4 Cleaning and care

Clean the knee pivot gauge by washing with water. If the gauge is used frequently for plaster impression, grease the arm pivot points slightly.

## 5 Liability

The manufacturer's warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use.

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