
2C3-1, 2C8, 2C15, 2C15=*-L, 2C24


DE Gebrauchsanweisung	3	CS Návod k použití.....	38
EN Instructions for use	5	RO Instrucțiuni de utilizare.....	41
FR Instructions d'utilisation.....	8	HR Upute za uporabu	44
IT Istruzioni per l'uso	11	SL Navodila za uporabo	46
ES Instrucciones de uso	13	SK Návod na používanie	49
PT Manual de utilização.....	16	BG Инструкция за употреба	52
NL Gebruiksaanwijzing.....	19	TR Kullanma talimatı	54
SV Bruksanvisning	22	EL Οδηγίες χρήσης	57
DA Brugsanvisning	24	RU Руководство по применению	60
NO Bruksanvisning	27	JA 取扱説明書.....	63
FI Käyttöohje.....	30	ZH 使用说明书.....	65
PL Instrukcja użytkowania.....	33	KO 사용 설명서.....	68
HU Használati utasítás	35		

1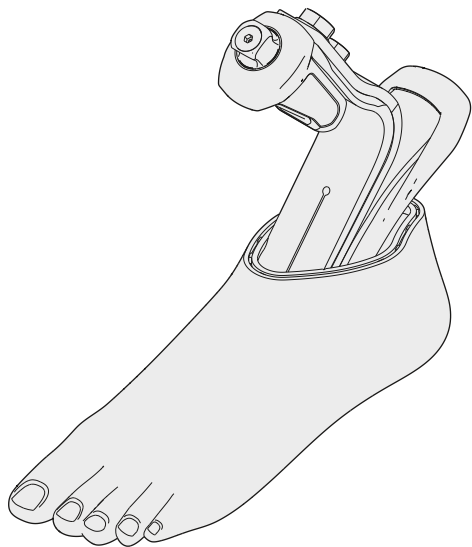

2

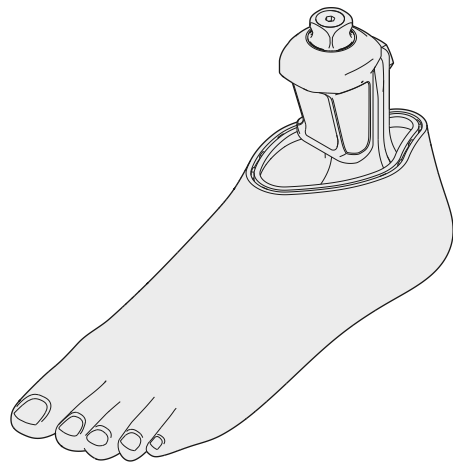1 Produktbeschreibung

Deutsch

INFORMATION

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INFORMATION

Date of last update: 2024-04-23

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

Footshells are pulled over compatible prosthetic feet. They are required for using the prosthetic feet and give them their natural exterior shape. This document applies for the following footshells: 2C3-1, 2C8, 2C15, 2C24.

The toenails are **not** paintable.

2 Intended use

2.1 Indications for use

The product is intended exclusively for lower limb exoprosthetic fittings.

2.2 Area of application

Observe the prosthetic foot's area of application.

2.3 Combination possibilities

- Observe the prosthetic foot instructions for use.

Footshell	2C3*	2C8	2C15	2C15=-L	2C24
Connection cover	2C10 2C20	2C21	2C19 2C20	None	

2.4 Environmental conditions

Observe the prosthetic foot's environmental conditions.

Other environmental conditions	
UV light	Not resistant
Hydrolysis	Not resistant

- ▶ Keep the product away from open flames, embers and other sources of heat.
- ▶ Clean the product promptly each time after it comes into contact with salt water, chlorinated water or abrasive media (such as sand).

2.5 Lifetime

The product is a wear part, which means it is subject to normal wear and tear.

3 General safety instructions

CAUTION! Risk of injury and risk of product damage

- ▶ To prevent mechanical damage, use caution when working with the product.
- ▶ Note the combination possibilities/combination exclusions in the instructions for use of the products.

- ▶ Observe the maximum lifetime of the product.
- ▶ Check the product for damage and readiness for use prior to each use.
- ▶ Do not expose the product to excessive strain (section: "Area of application" see page 6) or to prohibited environmental conditions (see page 6).
- ▶ Do not use the product if it is damaged or in a questionable condition. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).
- ▶ The product may only be used by one person; reuse on another person is prohibited.

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern and by the development of noises.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	-
1	Footshell	-
1	Connection cover	-

5 Preparing the product for use

⚠ CAUTION

Incorrect alignment or assembly

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

NOTICE

Changing the footshell

Premature wear due to footshell damage

- ▶ Do not modify the footshell.

5.1 Applying/removing the footshell

Putting on the footshell

- > **Recommended tools:** 2C101 changing device
- 1) Slide the prosthetic foot into the footshell.
 - 2) Press the heel of the prosthetic foot into the footshell until it engages.

Removing the footshell

- > **Recommended tools:** 2C101 changing device
- 1) Push the lock of the footshell to the rear and pull up the heel of the prosthetic foot.
 - 2) Remove the prosthetic foot from the footshell.

6 Use

⚠ CAUTION

Use of the product without shoes

Risk of injury due to slipping on slippery surfaces

- ▶ Do not use the product on slippery surfaces.

6.1 Cleaning

- ▶ Observe the cleaning instructions for the prosthetic foot.

7 Disposal

Do not dispose of the product with unsorted household waste. Improper disposal can be harmful to health and the environment. Observe the information provided by the responsible authorities in your country regarding return, collection and disposal procedures.

8 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

8.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this

document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

8.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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