

ParaGolfer

EN Instructions for use



Quality for life

CE

Table of contents

1	General information	5	2.8.3	Interference immunity of this power wheelchair	21
1.1	Preface	5	2.9	Safety functions.....	21
1.2	Intended use	6	2.10	Signage on the product	22
1.3	Field of application	6	2.11	Warning symbols and type plates	23
1.4	Technical service	7	3	Product description	25
2	Safety instructions	7	4	Delivery and preparation for use	26
2.1	Explanation of symbols	7	4.1	Delivery	26
2.2	Standards and directives.....	8	4.2	Settings.....	27
2.3	General safety instructions	8	4.3	Initial operation.....	27
2.4	Safety requirements for transportation, storage and assembly.....	9	4.4	Handing over the product.....	28
2.5	Safety requirements for operation.....	11	5	Transportation and storage	29
2.6	Safety requirements for care, maintenance and disposal.....	16	5.1	Transportation	29
2.7	Requirements for the user.....	18	5.2	Storage.....	30
2.8	Effects of electromagnetic interference on the product and on the user	18	6	Operation	31
2.8.1	General information on electromagnetic interference	18	6.1	Setting options	31
2.8.2	Warnings regarding effects of electromagnetic interference	20	6.1.1	Seat depth	33
			6.1.2	Seat width.....	34
			6.1.3	Lower leg length.....	35
			6.1.4	Backrest	35
			6.1.5	Armrests	36
			6.1.6	Control panel	38

6.2	Getting into and out of the sports wheelchair.....	39	7.2.1	Control panel holder, swing-away in parallel.....	59
6.3	Fixation	39	7.3	Decorative rim covers.....	60
6.3.1	Lap belt.....	40	7.4	Rain cover for storage compartment.....	60
6.3.2	Knee belt with knee pad	40	8	Malfunctions/troubleshooting	61
6.3.3	Chest belt	41	8.1	Warning	61
6.4	Control unit	42	8.2	Error.....	62
6.4.1	Control panel	42	8.3	Defect/failure.....	65
6.4.2	Switching on and off.....	43	9	Maintenance, cleaning and disinfection	65
6.4.3	Driving function.....	44	9.2	Maintenance intervals.....	66
6.4.4	"Battery charge level" indicator	46	9.3	Changing the fuse	68
6.4.5	Range	47	9.4	Wheel replacement	68
6.4.6	Drive-away lock.....	47	9.5	Cleaning and care.....	70
6.5	Stand-up function	48	9.5.1	Disinfection	70
6.6	Releasing and locking the brake.....	51	10	Disposal	71
6.7	Batteries	53	11	Information on re-use	71
6.7.1	Charging.....	53	12	Legal information	72
6.7.2	Battery charger	55	12.1	Liability.....	72
6.7.3	Battery disconnect switch.....	57	12.2	CE Conformity	72
7	Accessories.....	57	12.3	Warranty.....	72
7.1	Golf bag holder	57	12.4	Trademarks	72
7.2	Control panel holder.....	59	13	Technical data.....	73

1 General information

INFORMATION

Date of the last update: 2016-09-21

- Please read this document carefully before using the product.
- Follow the safety instructions to avoid injuries and damage to the product.
- Have yourself instructed by qualified personnel in the proper and safe use of the product.
- Please keep this document in a safe place.

INFORMATION

- New information regarding product safety and product recalls can be obtained from the Customer Care Center (CCC) at oa@ottobock.com or from the manufacturer's service department (see inside back cover or back page for addresses).
- You can request this document as a PDF file from the Customer Care Center (CCC) at oa@ottobock.com or from the manufacturer's service department (see inside back cover or back page for addresses). It is possible to increase the display size of the PDF document.

- For further questions about the instructions for use, please contact the authorised personnel who issued the product to you.

1.1 Preface

These instructions for use provide the user as well as his or her attendants with all the required knowledge on the design, function, operation and maintenance of the ParaGolfer sports wheelchair from Ottobock. The instructions contain all the information that is needed to use the sports wheelchair safely, to determine the possible causes of a malfunction and to help eliminate it.

Knowledge of these instructions for use is absolutely necessary to ensure safe use of the sports wheelchair. Therefore, the user and his or her attendants must read the instructions for use thoroughly, especially the section "Safety Instructions", before using the sports wheelchair. This ensures the functionality of the sports wheelchair is fully utilised.

1.2 Intended use

The sports wheelchair is designed solely for persons who are unable to work or have a walking impediment for doing indoor or outdoor sports that require an upright body position. The ParaGolfer was specially designed for users who are able to independently move in a sports wheelchair.

Any other use is considered improper use. The manufacturer is not liable for any personal injury or damage to property resulting from improper use; in such cases, the user has sole liability.

The ParaGolfer has been developed in accordance with the requirements of the wheelchair standard EN 12184. The values required there (Category C) for safe inclination, obstacle height, speed, etc. are reached or exceeded.

The sports wheelchair may only be combined with the options mentioned in these instructions for use and vice versa. Ottobock assumes no liability for combinations with medical devices and/or accessories from other manufacturers outside of the modular system.

The sports wheelchair may only be used by properly trained persons. Training users and attendants to use the sports wheelchair is required in order to protect persons from dan-

ger and to ensure that the sports wheelchair is operated safely and correctly.

The operational safety of the sports wheelchair can only be ensured if it is used properly in accordance with the information contained in these instructions for use. The user is ultimately responsible for accident-free operation.

1.3 Field of application

The functions of the sports wheelchair make it suitable for users who have walking impediments or walking disabilities due to the following diseases:

- Paraplegia
- Spina bifida
- Cerebral palsy
- Multiple sclerosis
- Muscular dystrophy
- Polio

1.4 Technical service

INFORMATION

Service and repairs on the sports wheelchair must be carried out exclusively by experts at specialist dealers who have been authorised and trained by Ottobock. Should any problems arise, please contact your sports wheelchair supplier.

Should you have any questions or a problem that cannot be resolved despite using the instructions for use, please contact Ottobock Customer Service (see back side of cover for address).

Ottobock endeavours to support their customers in all respects in order to keep them satisfied with their product for a long time.

2 Safety instructions

2.1 Explanation of symbols

WARNING

Warnings regarding possible risks of severe accident or injury.

CAUTION

Warnings regarding possible risks of accident or injury.

NOTICE

Warnings regarding possible technical damage.

INFORMATION

Information regarding operation. Information for the user and for service personnel.

2.2 Standards and directives

All information on safety contained in these instructions for use refers to the currently valid national laws and regulations of the European Union. In other countries, compliance with the applicable laws and national regulations is required.

In addition to the safety instructions contained in these instructions for use, the user is required to observe and comply with the BGV (Employer's Liability Insurance Association regulations), UVV (accident prevention regulations) and environmental protection regulations. All information contained in these instructions for use must be complied with at all times without restrictions.

The sports wheelchair has been constructed in accordance with state-of-the-art technology and is safe to operate. The safety of the ParaGolfer has been confirmed by CE certification and the declaration of conformity.

2.3 General safety instructions

WARNING

Risk of suffocation. Packaging materials must be kept out of reach of children.

WARNING

Risk of accidents and injury due to improper use in road traffic. The sports wheelchair has not been approved by Ottobock for driving in public road traffic.

CAUTION

Risk of accidents and injury by failure to comply with the safety instructions. All safety instructions contained in these instructions for use and in all other applicable documents must be observed and complied with. The instructions for use must be available to the user at all times.

CAUTION

Risk of accidents and injury due to improper use. The sports wheelchair may only be used in accordance with the guidelines for proper use. The sports wheelchair may only be used by properly trained persons. The sports wheelchair may only be used to transport one person at a time.

⚠ CAUTION

Risk of burns in the proximity of fire. The back upholstery and seat cushion of the sports wheelchair are flame retardant but nevertheless flammable. Therefore utmost caution near any sources of open flame or sparks, especially lit cigarettes, is required.

⚠ CAUTION

Risk of hypothermia or burns on wheelchair parts. Parts can become extremely hot or cold due to extreme temperatures.

Do not expose the product to any extreme temperatures (e.g. direct sunlight, sauna, extreme cold) in order to prevent injuries by coming into contact with the parts.

INFORMATION

Only use original manufacturer's options. The optional components may be mounted only as described here. Failure to comply will void the warranty.

2.4 Safety requirements for transportation, storage and assembly

Transportation and storage

⚠ WARNING

Risk of accidents and injury due to improper use for transportation in motor vehicles for the disabled. The sports wheelchair has not been approved by Ottobock for use as a seat in motor vehicles for the transportation of disabled persons.

NOTICE

Risk of damage due to improper transportation. Only use hoisting devices that have a sufficient capacity and the designated carry handles. The sports wheelchair must be secured in accordance with the regulations for the transportation device used. During transport on lifting platforms or in lifts, the wheelchair controls must be switched off. Engage the brake. Ensure that the sports wheelchair is centred on the lifting platform. None of the sports wheelchair's components, such as its anti-tippers or other components, may be in the danger area.

NOTICE

Battery damage due to deep discharge. Remove the fuse for shipping or when the sports wheelchair is not being used for an extended period of time.

INFORMATION

The tyres of the sports wheelchair contain chemical substances that can react with other chemical substances (such as cleaning agents, acids). Black tyres contain soot particles that can cause discolouration and black marks. Therefore a suitable surface is required when parking the wheelchair for extended periods of time.

Assembly

 **WARNING**

Risk of accidents and injury when loosening plug or screw connections. For all setting works or when preparing the wheelchair for transportation, screw and/or plug connections have to be loosened. This can cause uncontrolled movements of wheelchair components. To avoid injuries at pinch and shear points, carry out such works with the support of another person. Make sure to always keep parts of the body (e. g. hands or head) out of the danger zone.

 **CAUTION**

Risk of accidents due to unsecured screw connections. When screw connections with thread locks are loosened, they must be replaced by new ones or secured with a medium-strength thread lock substance (e. g. EuroLock A24.20).
After any settings on the sports wheelchair are established or changed, the mounting screws and / or nuts must be retightened securely. During tightening observe torques when specified.

INFORMATION

Prior to using the sports wheelchair, all the necessary mechanical adaptations (e. g. mounting special controls) and software settings (e. g. programming the control) must be made to comply with the individual requirements and abilities of the user. The settings may only be made by trained specialists who have been authorised by Ottobock.

INFORMATION

The mounting position of the stand-up device unit and/or seat must not be modified.

2.5 Safety requirements for operation

⚠ WARNING

Risk of accidents and injury due to defective safety functions. The user is obligated to ensure that the sports wheelchair and its safety functions are in safe and proper condition before every use.

The sports wheelchair may only be operated if all safety functions, e.g. the automatic brakes, are functional. Brake failure can result in serious accidents with fatal injuries.

⚠ WARNING

Risk of accidents and injury due to incorrect configuration settings. Modified parameter settings in the configuration lead to changes in driving characteristics. In particular, changes to the speed, acceleration, braking or joystick settings can lead to unexpected and therefore uncontrollable operating performance with a risk of accidents. Always test the driving characteristics of the sports wheelchair after configuration / programming is complete. Programming must only be completed by authorised personnel. Neither Ottobock nor the control unit manufacturer are liable for damages (especially in combination with special controls) caused by programming that was not properly / professionally adapted to the abilities of the wheelchair user.

⚠ WARNING

Risk of accidents and injury due to a lack of brake functionality when the brake is disengaged. Note the lack of brake functionality when the brake is disengaged, especially when moving the sports wheelchair on slopes.

⚠ WARNING

Risk of injury due to tipping during operation. The ParaGolfer has been approved for ascending or descending inclines of up to **17°** or **30 %**. Navigating inclines above this value is not permitted.

The critical obstacle height of the ParaGolfer is **100 mm / 3.9"**. It is not permitted to cross obstacles higher than **100 mm / 3.9"**. When driving on inclines or slopes, it is not permissible to negotiate obstacles. Operating the power wheelchair on stairs is not permitted.

⚠ WARNING

Risk of accidents due to tipping during operation. Reduce the driving speed when driving downhill (e.g. set it to speed level 1). When driving on inclines or slopes, it is not permissible to negotiate obstacles. Avoid getting into or out of the wheelchair on slopes. Before driving up slopes or over obstacles, the stand-up device must be brought into its lowest position.

⚠ WARNING

Risk of tipping when driving on inappropriate ground. Driving on very smooth areas (e.g. icy surfaces) or extremely rough terrain (e.g. broken rock or rubble) is not permitted.

⚠ WARNING

Risk of tipping when using lifting platforms. When using lifting platforms, lifts, busses or trains, the controls of the power wheelchair must be switched off and the brake must be locked.

⚠ WARNING

Risk of tipping due to changes in the centre of gravity. The effects of changes in the centre of gravity on the behaviour of the sports wheelchair on inclines, side hills, or when crossing obstacles must be tested with the secure support of an attendant prior to initial use.

⚠ WARNING

Risk of pinching or crushing in the electric seat adjustment and stand-up device ranges. When operating the stand-up device, there are pinch and shear points in the area between the seat frame and wheelchair frame caused by the construction. All attendants must be informed about this. To avoid injuries, always make sure that no parts of the body, e. g. hands or feet get into the danger zone, that no interfering objects, e. g. clothing or obstacles are within the danger zone, and that no unauthorised persons are present in the danger zone.

⚠ WARNING

Risk of accidents and injury due to improper use of the stand-up function.

- The stand-up function may be used on level ground only.
- When using the stand-up device, no unauthorised persons may be present in the danger zone. There must not be any interfering objects or obstacles in the stand-up area.

- All attendants must be informed that there are pinch points caused by the construction in the area between the seat frame and wheelchair frame. Users and attendants must not reach into the danger zone.
- The stand-up function may only be used with applied chest belt, lap belt and knee belt.

⚠ CAUTION

Risk of accident when driving without experience. Driving without experience can lead to falls and other dangerous situations. Practice using the wheelchair on level, straightforward terrain before using it.

⚠ CAUTION

Risk of accidents and injury due to getting into or out of the wheelchair improperly. Switch off the wheelchair controls before getting into and out of the sports wheelchair. Please notice that the armrests are not capable of bearing full body weight and that they therefore must not be used for getting into or out of the wheelchair.

⚠ CAUTION

Risk of accidents and injury due to rolling. A released brake may result in uncontrolled rolling of the sports wheelchair. Therefore, ensure that the brake is engaged after parking the sports wheelchair.

⚠ CAUTION

Risk of accidents due to bad tyres. Visually inspect the tyres for sufficient tread depth and correct tyre pressure before each use.

Whenever possible, avoid parking the wheelchair outdoors and in direct sunlight (UV light), as it causes the tyres to age prematurely. As a result, the tread surface hardens and corner pieces break out of the tread. Ottobock recommends to replace the tyres every 2 years regardless of wear and tear.

When the wheelchair is not used for extended periods of time or if the tyres are heated strongly (e. g. near radiators or by sunlight shining through a window), permanent deformation of the tyres will result. Therefore always make sure that the wheelchair has sufficient distance from sources of heat, move your wheelchair from time to time or jack up the wheelchair when storing it.

NOTICE

Risk of damage due to improper use of the stand-up function. The power wheelchair is equipped with a stand-up device. The following particularities must be observed during operation:

- The seat function actuators are not intended for continuous use, but only for short-term limited operation (10 % use, 90 % idle time). General recommendation: At maximum load, 10 seconds of operation should be followed by 90 seconds of idle time. In this regard, the electric seat functions are considered independently of the driving function.
- The seat with stand-up device may be subjected to a maximum load of **140 kg / 308 lbs.**
- The stand-up device may not be operated in case of errors or malfunctions.
- If creep speed is not activated when the stand-up device is operated, please visit a specialist dealer immediately. The power wheelchair may only be used with the stand-up device in its lowest position until the fault is repaired.

NOTICE

Risk of damage due to excessive heat or cold. The ParaGolfer may only be operated in the temperature range from **-15 °C to +40 °C/5 °F to 104 °F**. It must not be operated at temperatures outside this range.

NOTICE

Risk of damage caused by overload/excessive strain. The maximum load for the ParaGolfer is **140 kg / 308 lbs**.

NOTICE

Damage to tyres due to excessive tyre pressure. Make sure that the tyre pressure indicated in section "Technical data" is not exceeded.

INFORMATION

It is strongly recommended that the user for his/her own safety always fastens the lap belt. The lap belt serves to stabilise the person sitting or standing in the sports wheelchair.

INFORMATION

Every time you switch on the control unit, it will return to the previously selected speed level. If desired, the starting speed level can be changed in the parameter settings. These settings may only be made by trained specialists who have been authorised by Ottobock.

INFORMATION

If the driving mode is changed while driving, the sports wheelchair will accelerate or decelerate.

INFORMATION

The controls of the sports wheelchair are protected according to protective system IP 54 and can thus be used in bad weather (e. g. rain). The controls are approved for indoor and outdoor operation and meet the requirements regarding climate and splashed water.

INFORMATION

After each emergency stop, the controls of the sports wheelchair must be turned on again. In the event of communication problems in the bus system of the controls, the system triggers an emergency stop and thus prevents any uncontrolled functions. If the driving function is still not available after switching the controls on again, unlock the brakes to activate the push mode. In this case, contact a specialist dealer as soon as possible.

INFORMATION

If the controls do not emit an error signal as soon as the brake lever is activated, this indicates a malfunction. The settings must be checked by a specialist dealer.

INFORMATION

Manoeuvring is only allowed at reduced speeds.

2.6 Safety requirements for care, maintenance and disposal

⚠ WARNING

Serious injuries due to exceeding the service life.

- Using the product beyond the specified expected service life (see the section “Service life”) leads to increased residual risk and should only take place subject to the due diligence and deliberations of qualified personnel.
- If the service life is reached, the user or a responsible attendant should contact the qualified personnel who fitted the product or the manufacturer’s servicing department (see inside rear cover or back page for address). Here the user can obtain information about known risks and the current options for refurbishing the product.

⚠ CAUTION

Risk of accidents and injury due to improper maintenance, repairs or adjustments. Only trained staff who have been authorised by Ottobock may do maintenance work on the sports wheelchair. This also applies to all repairs and settings on the brakes. Incorrect settings can lead to a loss of braking power.

⚠ CAUTION

Risk of injury due to explosive gases. Explosive gases can develop while the batteries are charging. Therefore the following safety precautions are required when charging the batteries:

- Switch the control off.
- Ensure sufficient ventilation when charging the batteries in closed rooms.
- Smoking and open flames are not permitted. Sparks must be avoided.
- The air vents in the trim must not be covered.

⚠ CAUTION

Risk of injury due to uncontrolled movements. The fuse must always be removed for any maintenance work where the battery cover is open.

NOTICE

Unauthorised battery replacement. The battery may only be replaced by a specialist dealer. The characteristic curve of the battery charger set at the factory corresponds to the battery provided and must not be changed. Setting the characteristic curve incorrectly can result in permanent damage to the battery.

NOTICE

Risk of damage to electronics due to short circuit. Always use insulated tools when doing any maintenance work on the batteries. Ensure correct polarity is used when connecting the battery cables.

NOTICE

Risk of damage to electronics due to water penetration. Never use a water jet or high-pressure cleaning apparatus for cleaning the sports wheelchair. Water must not come into direct contact with the electronics, motor or batteries.

NOTICE

Risk of damage due to uncontrolled movements. When jacking up, secure the wheelchair to prevent it from slipping or tilting to the side by placing a suitable base under the drive unit support.

INFORMATION

The functionality and operating safety of the sports wheelchair must be verified by an authorised specialist at least once per year.

INFORMATION

Defective batteries must be disposed of properly in accordance with country-specific regulations.

2.7 Requirements for the user

⚠ CAUTION

Risk of accidents and injury due to improper operation.

The sports wheelchair may only be operated by a properly trained user. For this purpose, the user and any attendants must receive instruction in the use of the sports wheelchair from specialists trained and authorised by Ottobock.

The user must have read and understood all the information in the instructions for use.

The operating company/institution is obligated to ensure that the user can operate the ParaGolfer safely and faultlessly.

The wheelchair may not be operated in cases of exhaustion or under the influence of alcohol or medications.

The user must not have any mental limitations which can temporarily or permanently restrict attentiveness and judgement.

2.8 Effects of electromagnetic interference on the product and on the user

INFORMATION

Risks of injury and product damage due to electromagnetic interference.

It is very important that you read this information regarding the possible effects of electromagnetic interference on your sports wheelchair.

Risks of injury and product damage due to electromagnetic interference.

It is very important that you read this information regarding the possible effects of electromagnetic interference on your sports wheelchair.

2.8.1 General information on electromagnetic interference

- Sports wheelchairs may be susceptible to electromagnetic interference (EMI), which is interfering electromagnetic energy (EM) emitted from sources such as radio stations, TV stations, amateur radio (HAM) transmitters, two-way radios, and cellular phones.
- The interference (from radio wave sources) can cause the sports wheelchair to release its brakes, move by itself, or move in unintended directions.

- It can also permanently damage the sports wheelchairs control system.
- The intensity of the interfering EM energy can be measured in volts per meter (V/m). Each sports wheelchair can resist EMI up to a certain intensity. This is called its “immunity level”. The higher the immunity level, the greater the protection. At this time, current technology is capable of achieving at least a 20 V/m immunity level, which would provide useful protection from the more common sources of radiated EMI.
- There are a number of sources of relatively intense electromagnetic fields in the everyday environment. Some of these sources are obvious and easy to avoid. Others are not apparent and exposure is unavoidable. However, we believe that by following the warnings listed below, your risk to EMI will be minimized.
- The sources of radiated EMI can be broadly classified into three types:
 1. Hand-held portable transceivers (transmitters-receivers) with the antenna mounted directly on the transmitting unit. Examples include: citizens band (CB) radios, "walkie talkie," security, fire, and police transceivers, cellular telephones, and other personal communication devices.

NOTICE Some cellular telephones and similar devices transmit signals while they are ON, even when not being used.

2. Medium-range mobile transceivers, such as those used in police cars, fire trucks, ambulances, and taxis. These usually have the antenna mounted on the outside of the vehicle;
 3. Long-range transmitters and transceivers, such as commercial broadcast transmitters (radio and TV broadcast antenna towers) and amateur (HAM) radios.
- Because EM energy rapidly becomes more intense as one moves closer to the transmitting antenna (source), the EM fields from hand-held radio wave sources (transceivers) are of special concern. It is possible to unintentionally bring high levels of EM energy very close to the sports wheelchair's control system while using these devices. This can affect sports wheelchair movement and braking.
 - Other types of hand-held devices, such as cordless phones, laptop computers, AM/FM radios, TV sets, CD players, and cassette players, and small appliances, such as electric shavers and hair dryers, so far as we know, are not likely to cause EMI problems to your sports wheelchair.

2.8.2 Warnings regarding effects of electromagnetic interference

INFORMATION

The warnings listed below are recommended to prevent possible interference with the control system of the sports wheelchair.

WARNING

Risk of accidents due to uncontrolled driving behaviour. Electromagnetic interference (EMI) from sources such as radio and TV stations, amateur radio (HAM) transmitters, two-way radios, and cellular phones can affect sports wheelchairs and motorized scooters. Following the warnings listed below should reduce the chance of unintended brake release or sports wheelchair movement which could result in serious injury.

- Do not turn ON hand-held personal communication devices, such as citizens band (CB) radios and cellular phones, while the sports wheelchair is turned ON.
- Be aware of nearby transmitters, such as radio or TV stations, and try to avoid coming close to them.

- If unintended movement or brake release occurs, turn the sports wheelchair OFF as soon as it is safe. In this case, please contact your authorised specialist dealer immediately.
- Be aware that adding accessories or components, or modifying the sports wheelchair, may make it more susceptible to interference from radio wave sources (Note: There is no easy way to evaluate their effect on the overall immunity of the sports wheelchair).
- Report all incidents of unintended movement or brake release to the authorised specialist dealer or the sports wheelchair manufacturer, and note whether there is a radio wave source nearby.

NOTICE

Interference from electromagnetic fields. The power wheelchair has been tested according to EMC regulations. The following particularities must be observed during operation:

- The power wheelchair may generate electromagnetic fields that can cause interference with other devices. Therefore the controls should be switched off when no function is required.

- Despite of compliance with all applicable EMC directives and standards, the power wheelchair can be affected by interference from other electric devices (e.g. department store EAS systems) or cause interference to such devices. If you notice such a behaviour, move your power wheelchair outside the interference range.

2.8.3 Interference immunity of this power wheelchair

INFORMATION

20 volts per meter (V/m) is a generally achievable and useful immunity level against interference from radio wave sources (as of May 1994) (the higher the level, the greater the protection).

INFORMATION

This sports wheelchair model as shipped, with no further modification, has an immunity level of **20 V/m**.

2.9 Safety functions

INFORMATION

In dangerous situations, the ParaGolfer can be turned off at any time using the on/off button. When the button is pressed, the sports wheelchair brakes immediately and the electric functions will be stopped.

If any malfunctions occur such as an insufficient supply of energy to the brake, the software will recognise them and trigger the emergency stop brake or reduce the speed of the sports wheelchair. At the same time, a warning signal will be emitted.

A tilt sensor is mounted in the centre of gravity of the ParaGolfer; if the allowable side tilt angle is exceeded, an acoustic and visual alarm signal is emitted.

2.10 Signage on the product



Fig. 1 Signage on the ParaGolfers

2.11 Warning symbols and type plates

Label/nameplate	Explanation
	<p>A Type designation</p> <p>B Read the instructions for use prior to using the product.</p> <p>C Symbol for separate collection of electric and electronic devices. Components of the power wheelchair and the batteries must not be disposed of like regular domestic waste.</p> <p>D The product has not been approved by the manufacturer for use as a seat in a wheelchair accessible vehicle.</p> <p>E European article number (EAN)</p> <p>F Serial number*</p> <p>G Product identification (UDI)</p> <p>H Product reference number</p> <p>I CE marking – product safety in conformity with EC Directives</p> <p>J Manufacturer / address</p> <p>K Maximum load capacity (see section “Technical data”)</p> <p>L Maximum climbing ability (see section “Technical data”)</p> <p>M Maximum speed (see section “Technical data”)</p>

VV = version; S = speed code; YY = year of manufacture; WW = week of manufacture; PP = production site; XXX = sequential production number

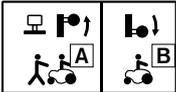
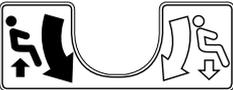
Label/nameplate	Explanation
	<p>A Manual pushing mode: unlock motor brake</p> <p>B Electric driving operation: lock motor brake</p>
	<p>Adjust the length of the lower leg (= distance between seat and foot rest) here. To do so, turn the adjustment spindle on the front of the seat frame in the clockwise or counter-clockwise direction.</p>
	<p>Warnings regarding possible risks of severe injury.</p> <p>Danger due to cutting. Do not reach into the danger area.</p>

Table 1 Signage on the ParaGolfer

3 Product description

The sports wheelchair ParaGolfer is a high-quality product which enables the user to do sports that require an upright body position. For this purpose, the sports wheelchair features an integrated standing function.

Operating the standing function using the control panel causes the seat bottom of the sports wheelchair to move from horizontal into the vertical position by means of a motor-driven lift seat mechanism. Along with the seat bottom adjustment, the angles of the backrest and footplate simultaneously adapt to the new position so that the user assumes an upright position. Thanks to the continuous linear drive, any position between the two end positions "sitting" and "standing" can be assumed.

The sports wheelchair distinguishes itself through its all-terrain mobility, for example for the use on a golf course. A high-performance drive system powered by two 12 V gel batteries and adequate tyres enable the ParaGolfer to overcome obstacles easily and offer safe driving characteristics.

Steering of the sports wheelchair is realised through differential control of the drive wheels. The rear wheel can rotate freely around the steering axle, what gives the sports wheel-

chair its good manoeuvrability and allows, for example, to turn on the spot.

The sports wheelchair is controlled by an enAble50 wheelchair control system. It includes a control panel to enter driving commands and display the current status as well as a controller that controls the drive motors and other electrical functions based on the input data. Data transmission is realised via a bus system.

Since the enAble50 can be programmed, it is possible to adapt the control unit to the personal requirements of the user; e. g. the speed, acceleration and deceleration values can all be adapted.

The special features of the sports wheelchair include:

- Individual adaptation possibilities of the stand-up seat to the user's body measurements
- Easy servicing due to good and clear accessibility of all component groups
- Adaptation of the speed, acceleration and deceleration values to individual wishes of the user

The modular design of the sports wheelchair makes it possible to add other modules and devices in addition to the main components (see fig. 2):

- Golf bag holder
- Decorative rim covers
- Lateral pads
- Seat cushion
- Grab handle
- Foil covering
- Control panel holder, swing-away in parallel
- Golf ball as joystick top
- Rain cover for storage compartment
- Bumper bar
- Kids package (consisting of kids footplate, kids seat cushion, kids chest belt, kids knee pads and kids side cushion)

4 Delivery and preparation for use

4.1 Delivery

WARNING

Risk of accidents and injury due to incorrect settings.
All later adaptation or setting work may only be carried out by authorised specialist dealers.

INFORMATION

The options included in the scope of delivery depend on the product configuration that has been selected for the ParaGolfer.
The ParaGolfer is delivered fully assembled and ready for operation.

Scope of delivery includes:

- Preset sports wheelchair with main components (see fig. 2)
- Tool kit
- Battery charger
- Instructions for use
- Options (see section 7)

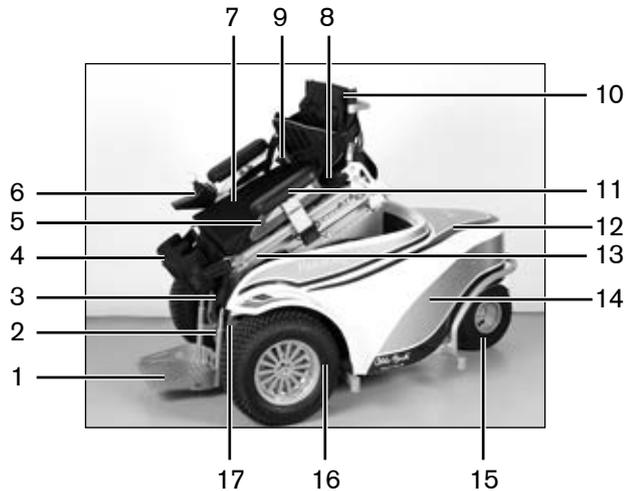


Fig. 2 Main components of the ParaGolfer

1 Footplate	10 Backrest
2 Charging receptacle	11 Armrest
3 Brake release lever	12 Storage compartment
4 Knee belt with knee pad	13 Stand-up device
5 Lateral pads	14 Cover
6 Control panel	15 Rear wheel
7 Seat cushion	16 Drive wheel
8 Lap belt	17 Chassis with motor, controller, battery and fuse
9 Chest belt	

4.2 Settings

The following tools are required to adjust the wheelchair to the requirements of the user and for maintenance work:

- Allen wrenches, sizes 3, 4, 5, 6
- Open-end or combination wrenches, sizes 8, 10
- Phillips screwdriver
- Screwdriver for pan head screws
- Tyre repair accessories

4.3 Initial operation

⚠ WARNING

Risk of suffocation. Packaging materials must be kept out of reach of children.

All components must be checked for completeness and functionality prior to initial operation.

Before the sports wheelchair can be switched on, the fuse must be inserted into the fuse holder which is located laterally underneath the seat, next to the left drive wheel (see fig. 3). To do so, remove the fuse from the protective cover on the control panel, open the cap of the fuse holder and insert the fuse. It is important to make sure that the fuse is inserted into the centre of the spring contacts and that it is not at an angle.

Close the cap of the fuse holder until you feel it snap back into place.

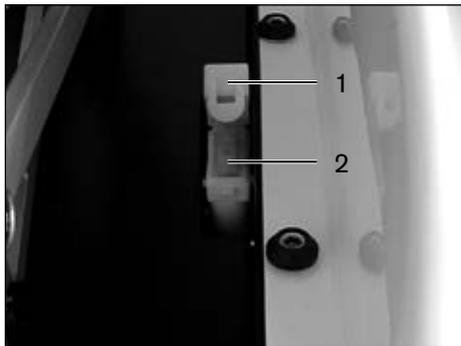


Fig. 3 Fuse

- 1 Cap of fuse housing, open
- 2 Fuse

4.4 Handing over the product

⚠ WARNING

Risk of injury due to lack of instruction. Instruct the user or the attendant in the proper use of the product when handing it over.

The following steps must be performed for the safe delivery of the wheelchair:

- Have the user get into the product and perform a trial seat fitting. Pay special attention to proper positioning according to medical considerations.
- The user and any attendants must be instructed in safe use of the product. In particular, these instructions for use are to be used.
- The instructions for use must be given to the user or an attendant upon delivery of the product.
- The user or attendant should acknowledge that they have been instructed in how to use the product and were informed of the residual risks.

5 Transportation and storage

⚠ WARNING

Risk of accidents and injury due to improper use for transportation in vehicle for transporting persons with reduced mobility. The ParaGolfer has not been approved by Ottobock for use as a seat in vehicles for transporting persons with reduced mobility.

⚠ WARNING

Risk of accidents and injury when loosening plug or screw connections. For all setting works or when preparing the wheelchair for transportation, screw and/or plug connections have to be loosened. This can cause uncontrolled movements of wheelchair components. To avoid injuries at pinch and shear points, carry out such works with the support of another person. Make sure to always keep parts of the body (e. g. hands or head) out of the danger zone.

⚠ CAUTION

Risk of injury if the ParaGolfer falls down. The empty weight of the ParaGolfer is 180 kg/396 lbs. For transportation, only the designated carry handles may be used.

⚠ CAUTION

Risk of injury due to insufficient restraints. Secure the sports wheelchair during transportation in another vehicle by tightening the tensioning straps properly.

5.1 Transportation

For transportation, the size of the ParaGolfer can be reduced to a height of 810 mm/31.9" and a total length of 1490 mm/58.7".



Fig. 4 Transportation size of the ParaGolfer

For this purpose, the seat bottom must be moved into the lowest position and the backrest be removed (see sections 6.1.1 and 6.1.4). The armrests (see section 6.1.5) as well as the seat cushion and lateral pads can optionally be removed and stowed separately. Swivel the caster wheel under the cover (see fig. 4). The control system of the ParaGolfer must be turned off (see section 6.4) and the brake must be locked (see section 6.11) for transportation.

The ParaGolfer must be secured with straps during transportation. At the front, place two straps around the drive unit sustainer, one along the right motor and one along the left motor. The two straps at the rear must be placed into the retainer bushes for the golf bag holder, which are located at the rear of the drive unit sustainer, below the cover. The angle at which the straps are tightened must be as flat as possible.

The ParaGolfer can be optionally provided with a rain repellent protective cover and transportation bows for transportation.

5.2 Storage

INFORMATION

If your sports wheelchair is not moved for several days, permanent colour changes may occur where the wheelchair comes into contact with the surface it is standing on. Therefore a suitable surface is required when parking the wheelchair for extended periods of time.

INFORMATION

Tyres contain chemical substances that can react with other chemical substances (such as cleaning agents, acids, etc.).

Black tyres contain soot particles. They can leave black traces on the ground.

For this reason, Ottobock recommend choosing grey tyres when using the power wheelchair mainly indoors.

INFORMATION

Direct sunlight/UV light causes the tyres to age prematurely. As a result, the tread surface hardens and corner pieces break out of the tread.

INFORMATION

Avoid parking the wheelchair outdoors whenever possible. Regardless of wear and tear, replace the tyres every 2 years.

INFORMATION

Remove the fuse for shipping or when the sports wheelchair is not being used for an extended period of time.

The sports wheelchair must be stored in a dry place. It must not be subjected to any damaging exterior influences, e.g. rain, snow, or strong solar radiation during storage.

For transportation and storage, an ambient temperature range from -15 °C to +40 °C / 5 °F to +104 °F must be observed.

6 Operation

⚠ WARNING

Risk of accidents and injury due to risk of tipping because of overloading. In case of overloading, the axle or caster can break which can result in tipping of the wheelchair. In addition, the backrest upholstery can tear and the seat tubes break, which can cause the user to fall out of the wheelchair. For this reason, the permissible total weight of maximally 140 kg/308 lbs must not be exceeded.

6.1 Setting options

⚠ WARNING

Risk of accidents and injury due to incorrect settings. All later adaptation or setting work may only be carried out by authorised specialist dealers. For quick and precise adaptation of the seating unit, all setting options can be made use of while the user is sitting in the wheelchair.

⚠ WARNING

Risk of accidents and injury when loosening plug or screw connections. For all setting works or for folding down the backrest for transportation, screw and/or plug connections have to be loosened. This can cause uncontrolled movements of wheelchair components.

Especially the backrest can fold to the front or rear when removing the cotter pin (double spring cotter) from the backrest. To avoid injuries at pinch and shear points, carry out such tasks with the support of another person. Make sure to always keep parts of the body (e. g. hands or head) out of the danger zone.

⚠ CAUTION

Risk of accidents due to unsecured screw connections. When screw connections with thread locks are loosened, they must be replaced by new ones or secured with a medium-strength thread lock substance (e. g. EuroLock A24.20).

After any settings on the sports wheelchair are established or changed, the mounting screws and / or nuts must be retightened securely. During tightening observe torques when specified.

⚠ CAUTION

Behaviour in cases of skin changes. As with every new seating support, the product can change the way a person sits. Users must continue to carry out regular activities to relieve pressure and must also check their skin condition. This must not be restricted to only those areas at which the product comes into contact with the user, but must also include the most important pressure areas such as the sacrum, legs, and buttocks.

If increased reddening or skin irritation develops, stop using the product and consult your doctor.

The seating unit of the ParaGolfer can be individually adapted to the user's body measurements. The seat depth, backrest height and lower leg length can be changed quickly and easily.

The backrest can be removed when required, and the armrests can be plugged out. Moreover, the control panel can be adapted to the arm length and can be mounted on the other side.

6.1.1 Seat depth

⚠ WARNING

Risk of injury due to tipping out of the wheelchair. To change the seat depth setting you first have to move the seat from the sitting position into the standing position. If you make the adjustment while the user is sitting in the wheelchair, all belts must be fastened first (see section 6.3).

INFORMATION

Remove the fuse for shipping or when the sports wheelchair is not being used for an extended period of time.

When adjusting the seat depth, the settings must be identical on both sides.

To adjust the seat depth, the position of the backrest can be changed in increments of **25 mm / 0.98"**.

To have sufficient space available for making adjustments, the seat must be moved into the standing position until the clamping screws are freely accessible. The clamping screws are located on the right and left seat rail and can be loosened with a size 8 Allen wrench (see section 5).



Fig. 5 Loosening the screws on the seat rail

Then the locking pins must be pulled out (see fig. 6) and turned by 90°, so that they remain in the unlocked position.

Now the backrest can be moved forward and backward along the seat rail. Move the backrest evenly with both hands and do not skew.



Fig. 6 Locking pins on the seat rail

Insert the locking pins into the corresponding holes to fix the desired position. Finally retighten the screws on the seat rail and move the seat back into the lowest position.

6.1.2 Seat width

Lateral pads can be inserted between the clothing protectors and seat cushion to adapt the seat width. Depending on the scope of delivery, the pads have a width of **20 mm / 0.79"** or **30 mm / 1.18"** and are attached to the clothing protectors through a hook and loop connection.



Fig. 7 Lateral pad

- | | | | |
|---|-------------|---|--------------------|
| 1 | Armrest | 3 | Clothing protector |
| 2 | Lateral pad | 4 | Seat cushion |

6.1.3 Lower leg length

The lower leg length, i.e. the distance between the seat and footplate, can be adjusted with a spindle. To turn the spindle use a size 8 Allen wrench. The spindle head is located at the front side of the seat frame (see fig. 8).

Turn the spindle clockwise to lower the seat bottom; turn it counter-clockwise to raise the seat bottom (see fig. 8). Since the footplate remains in the same position during adjustment, the seat height changes similarly to the distance between the seat and footplate.

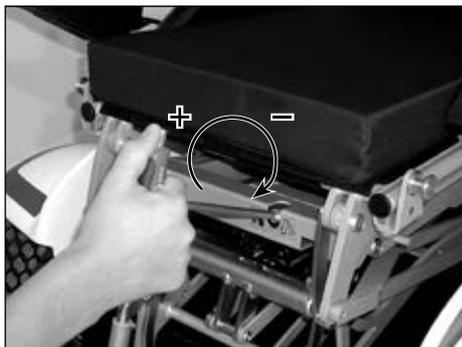


Fig. 8 Turning the spindle

6.1.4 Backrest

⚠ WARNING

Risk of injury due to tipping out of the wheelchair. To change the seat depth setting you first have to move the seat from the sitting position into the standing position. If you make the adjustment while the user is sitting in the wheelchair, all belts must be fastened first (see section 6.3).

⚠ WARNING

Risk of injury due to improper attachment of the backrest. To ensure sufficient stability, a minimum clamping range, identified by a red mark on the backrest tubes, must be observed when adjusting the backrest. The clamps must be attached below this range.

⚠ CAUTION

Risk of pinching or crushing the fingers. Do not reach into the danger zone between the clamp and the seat when adjusting the backrest.

The height of the backrest can be adjusted by **80 mm / 3.15"**. To do so, loosen the clamps on the right and left backrest tube using the clamping levers (see fig. 9a). Then the backrest can be moved up and down along the tube. Make sure to move the backrest evenly with both hands and do not skew (see fig. 9b). Do not forget to firmly retighten the clamping levers after adjustment.

In the same way, the backrest can be pulled off completely for transportation purposes.



Fig. 9a Loosening the clamps



Fig. 9b Adjusting the backrest height

6.1.5 Armrests

⚠ WARNING

Risk of injury due to tipping out of the wheelchair. To change the seat depth setting you first have to move the seat from the sitting position into the standing position. If you make the adjustment while the user is sitting in the wheelchair, all belts must be fastened first (see section 6.3).

The position of the armrests is variable. For adjustment, the two screws on the bottom side of the armrest holder have to be loosened with a size 5 Allen wrench (see fig. 10).

To have sufficient space available for making adjustments, the seat must be moved into the standing position until the screws are freely accessible. Now the armrests can be moved forward and backward along the seat.

Afterwards, firmly retighten the screws.



Fig. 10 Loosening the screws on the bottom side of the armrest holder

To remove the armrest, pull up the release pin which is located centrally underneath the armrest pad. This will cause the locking bolt to slide back into the receiver, and the armrest can be pulled upwards out of the holder (see fig. 11).

To plug in the armrest, you have to pull the release pin again. You may let go of the release pin once you have plugged in the armrest. Move the armrest down into the the holder until the locking bolt has engaged securely and the armrest can no longer be pulled upwards.

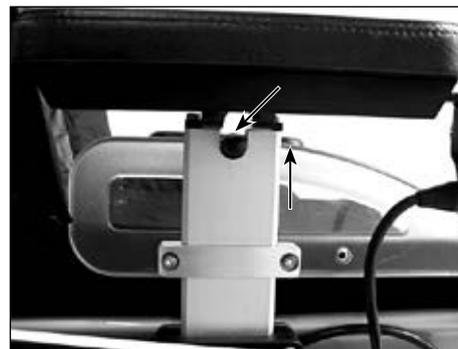


Fig. 11 Release pin on the armrest

6.1.6 Control panel

⚠ WARNING

Risk of accidents and injury due to uncontrolled driving characteristics. Should the connection plug come loose during driving, e. g. caused by vibrations, the connection to the control panel will get lost and the ParaGolfer trigger an emergency stop. When inserting the plug, make sure to turn the plug's lock ring to the right until it locks in place.

NOTICE

Risk of cable damage. Positioning the cable incorrectly can lead to pinching and thus damage to the cables. When changing the mounting side of the control panel, be sure to position the cable in the same way on the other side.

INFORMATION

If the control panel rail is too long, the protruding part can be cut off with a saw.

The control panel is attached to a rail which is located beneath the armrest.

To adapt the control panel to the arm length of the user, you must loosen the two screws on the underside of the armrest using a size 3 Allen wrench (see fig. 12). You can now slide the control panel forwards and backwards. Firmly retighten the screws after making adjustments.

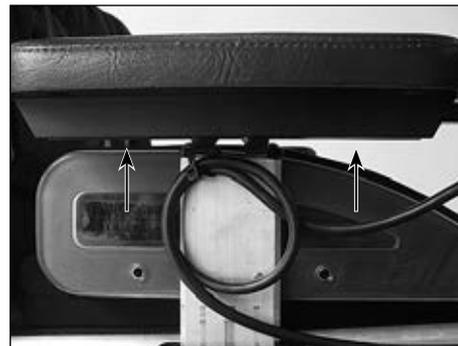


Fig. 12 Screws on the control panel rail on the underside of the armrest

The standard configuration has the control panel mounted on the right side. Depending on whether the wheelchair user is a right hander or left hander, it can be mounted to either armrest. To change the mounting position, the connection plug on the rear side of the control panel must be loosened (see fig. 13).

Turn the plug's lock ring to the left into the end position. Then you can disconnect the plug and pull off the control panel to the front.

After making adjustments, firmly retighten the screws and plug in the connection plug. When plugging in the plug, pay attention that the nose fits into the groove. The lock ring must be turned to the right until it engages and can then be tightened.

6.2 Getting into and out of the sports wheelchair

CAUTION

Risk of injury if the sports wheelchair starts rolling. Switch off the wheelchair controls before getting into and out of the sports wheelchair. This will automatically engage the motor brake.

NOTICE

Risk of armrest breakage due to overloading. Please notice that the armrests are not capable of bearing full body weight and that they therefore must not be used for getting into or out of the wheelchair.

INFORMATION

If the ParaGolfer is standing still, the mechanical brake is automatically activated.

Getting into and out of the ParaGolfer can be done by the user individually in a way that suits him or her best.

Getting into the wheelchair from the side requires to remove the right or left armrest depending on the side where the user will get into the wheelchair (see section 6.1.5). Then the user approaches the ParaGolfer laterally and can get into it by supporting him- or herself on the seat surface. Afterwards, firmly plug in the armrest again.

6.3 Fixation

WARNING

Risk of injury due to tipping out of the wheelchair. When using the standing function and when driving downhill, the user always has to fasten all belts that are mounted to the ParaGolfer.

⚠ CAUTION

Risk of injury due to defective belts and pads. Inspect the seams, belts/straps and pads for signs of wear once a week. If they are heavily worn, please contact a specialist dealer for proper repair or replacement. The product must under no circumstances be modified or repaired by unauthorised persons.

6.3.1 Lap belt

The lap belt runs at abdomen height and is attached to the backrest tubes. It must be applied tightly on the abdomen for fixation.

To do so, first insert the buckle into the lock until it engages and then pull the belt and the loose end of the belt so that the belt is tight and the buckle centred on the abdomen.

6.3.2 Knee belt with knee pad

The knee belt consists of a pre-shaped foam pad with a belt and 2 clevis type eyelets, which are hooked into the designated bushings on the right and left side of the seat (see fig. 13a).

The pad must be applied tightly on the legs in the area just below the knees for fixation. Pull the belt tight and close the hook and loop closure to fasten the belt (see fig. 13b).



Fig. 13a Hooking in the knee belt



Fig. 13b Closing the knee belt's hook and loop closure

6.3.3 Chest belt

⚠ WARNING

Risk of injury due to tipping out of the wheelchair. Accidentally unfastening the chest belt can cause the user to fall out of the wheelchair. If the movements or cognitive abilities of the user can cause the belt to be accidentally unfastened, then an attendant must be present at all times while in use. Ensure that the lap belt is used at all times.

The chest belt consists of a flat foam pad with belt and runs at chest height around the backrest tubes. It has to be threaded through the 2 belt retaining eyelets on the back.

It must be applied tightly on the chest for fixation. To do so, first insert the buckle into the lock until it engages (see fig. 14 a) and then pull both belt ends (see fig. 14 b), so that the belt is tight and the buckle centred on the chest.

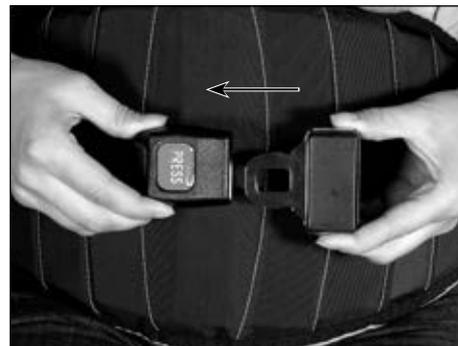


Fig. 14 a Fastening the chest belt



Fig. 14 b Tightening the chest belt

6.4 Control unit

NOTICE

Risk of impairing the driving performance of the sports wheelchair. The driving characteristics of the sports wheelchair can be affected by electromagnetic fields (mobile phones or other radiating devices). For this reason, please switch off all mobile devices when driving.

NOTICE

Risk of damage to other devices. The sports wheelchair can generate electromagnetic fields that can cause interference with other devices. Switch off the controls whenever you do not need them.

6.4.1 Control panel

The sports wheelchair is operated using the control panel (see fig. 15).

The control panel consists of a keypad, LCD display, and joystick. The programming receptacle and two inputs for external push buttons are located on the underside. The sports wheelchair is turned on and off using the keypad; driving commands can be entered and the status of certain functions and components can be displayed.

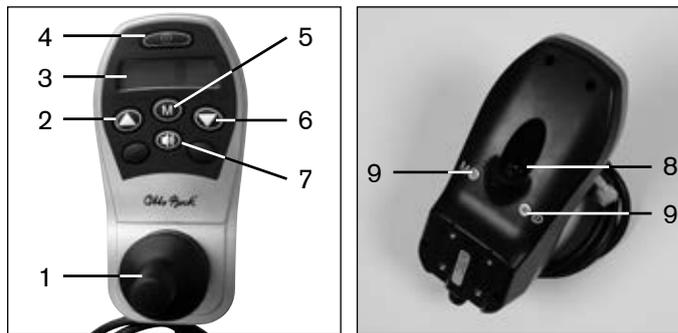


Fig. 15 Control panel (ParaGolfer)

- | | |
|-------------------------|------------------------------------|
| 1 Joystick | 6 Stand-up function: DOWN |
| 2 Stand-up function: UP | 7 Horn |
| 3 LCD display | 8 Programming receptacle |
| 4 On/off button | 9 Inputs for external push buttons |
| 5 Mode button | |

On/off button

The on/off button is used for switching the sports wheelchair on and off and for activating the drive-away lock.

Mode button

Briefly pressing the mode button increases the speed level. After reaching the maximum speed level, you can change back to speed level 1 by pressing the button again (1-2-3-4-5-1-...).

Horn

The horn will sound as long as the horn button is being pressed.

Up and down arrow buttons

The up and down arrow buttons are used to control the stand-up function. Press the up arrow button to raise the seat; press the down arrow button to lower the seat.

LCD display

The LCD display is the communication interface between the user and the control unit. It indicates the selected speed level, the remaining battery capacity, the status of electric options and special functions as well as warnings and errors. During the start phase, all display symbols are visible.



Fig. 16 LCD display with all symbols

6.4.2 Switching on and off

⚠ WARNING

Risk of death due to lack of brake functionality. The brake release lever must be locked when using the sports wheelchair. The automatic brakes must be operational and functional.

⚠ CAUTION

Risk of accidents due to incorrect tyre pressure. Visually inspect the tyres of the sports wheelchair for sufficient tread depth and sufficient tyre pressure before each use. Incorrect tyre pressure reduces the life of the tyres and impairs the driving characteristics.

The control unit of the sports wheelchair is switched on and off by pressing the on/off button (see fig. 15, item 5). If the control is not used for a certain time, which is adjustable, the wheelchair is switched off automatically. It is also possible to switch off the sports wheelchair with the on/off button during driving. In this case, the wheelchair brakes immediately until it stops.

INFORMATION

The parameter settings can be changed by the specialist dealer in accordance with the user's desire in what speed level or what menu he/she wants the sports wheelchair to be after switching on.

6.4.3 Driving function

CAUTION

Uncontrolled driving behaviour, unexpected sounds or odours. Falling, tipping, collision with persons or nearby objects due to defects.

- If any faults, defects or other hazards that can lead to personal injury are detected, the product must be taken out of service immediately. This includes uncontrolled movements as well as unexpected or previously not noted sounds or odours that deviate significantly from the state of the product at the time of delivery.
- Please contact your authorised dealer.

CAUTION

Risk of injury due to uncontrolled operation of the sports wheelchair. Switch off the controls on the sports wheelchair whenever you do not need them. This will prevent unintentional activation of the joystick.

CAUTION

Risk of injury due to tipping during operation. Observe the following safety information when operating the sports wheelchair:

- Operation of the wheelchair is limited to inclines up to a maximum of **30 %** or **17°**.
- When driving downhill, reduce speed in accordance with the degree of the incline.
- Do not cross differences in height of more than **100 mm / 3.9"**.
- Do not cross steps or curbs without braking first.

The sports wheelchair has been approved for ascending or descending inclines of up to 30 % or 17°. Navigating inclines above this value is not permitted.

To ensure safe downhill driving, the driving speed must be adapted to the degree of incline (e.g. set it to speed level 1).

The critical obstacle height of the sports wheelchair is 100 mm/3.9". It is not permitted to cross obstacles higher than 100 mm/3.9". Obstacles such as steps or curbs must be crossed at reduced speeds.

When travelling at maximum speed, sudden steering actions can cause the rear of the wheelchair to swerve. Steering actions must always be adapted to the travel speed.

If there are obstacles in the travel path, it is important to drive around them with a large margin of safety. On uneven ground, the driving behaviour of the wheelchair may get out of control. Therefore the speed must always be adapted to the ground conditions.

The battery indicator and speed level are shown in the driving menu on the LCD display.

For driving use the joystick. The further the joystick is moved from the centre position, the faster the sports wheelchair will travel in that direction.

The maximum speed with full deflection of the joystick depends on the selected speed level.

Releasing the joystick automatically activates the brake function, which brings the wheelchair to a halt. When standing still, the mechanical brakes are automatically activated so the sports wheelchair cannot roll away.

The sports wheelchair has a programmable number of speed levels (delivery condition = 5 speed levels). The "M" button is used to increase the speed level. After reaching the maximum speed level, the control unit switches back to speed level 1.

Adapting the driving characteristics

WARNING

Risk of accidents and injury due to incorrect configuration settings. Modified parameter settings in the configuration lead to changes in driving characteristics. In particular, changes to the speed, acceleration, braking or joystick settings can lead to unexpected and therefore uncontrollable operating performance with a risk of accidents. Always test the driving characteristics of the sports wheelchair after configuration / programming is complete. Programming must only be completed by authorised personnel. Neither Ottobock nor the control unit manufacturer are liable for damages (especially in combination with special controls) caused by programming that was not properly / professionally adapted to the abilities of the wheelchair user.

6.4.4 "Battery charge level" indicator

INFORMATION

Immediately after switching the sports wheelchair on, the battery indicator shows the battery capacity saved before the wheelchair was switched off the last time. The exact battery status is displayed after approximately 5 minutes.

INFORMATION

At temperatures of **<0 °C/32 °F** the battery capacity drops by up to **35 %** in relation to the capacity for an outside temperature of **20 °C/68 °F**. This shortens the range of the power wheelchair accordingly. Moreover at low temperatures the charge level displayed on the control panel can differ more significantly from the actual battery capacity.

The battery indicator in the LED display is divided into 10 segments and shows the current charge level.

A charge of 100 % corresponds to 7 segments on the battery symbol. As the remaining battery charge decreases, the LED segments turn off one by one (see Table 2).

When the last segment flashes, the battery must be charged immediately.

If all the segments have disappeared and the battery symbol is flashing, the battery is under voltage. Since any further use will lead to battery damage, a warning signal also appears. Then the sports wheelchair drives only at creep speed.

If all the segments of the battery symbol are flashing, this means that the battery is over voltage. Since this will lead to battery damage, a warning symbol also appears.

The charging process is indicated by the battery segments lighting up one after the other. When the battery is charging, the driving function is locked out.

Display symbol(s)	Information
	Driving menu with speed level and battery charge level
	Battery charging urgently required
	Charging process with drive lock
	Battery is under voltage, with warning symbol
	Battery is over voltage, with warning symbol

Display symbol(s)	Information
	Creep speed

Table 2 Battery indicator on the control panel

6.4.5 Range

INFORMATION

Information about the range of the power wheelchair can be found in the chapter 'TechnicalData'.

Note that the batteries reach their full capacity only after **20–30** charge cycles, and the power wheelchair can achieve the stated range only then.

Please note that the specified range was determined under defined conditions in accordance with ISO 7176-4. In practice the range can be reduced by up to **50 %**.

The following factors influence the range of a power wheelchair:

- Battery capacity
- Age of the batteries
- Ambient temperature

- Driving conditions (e.g. terrain profile, condition of surface)
- Charging method
- Use of power options
- Body weight of user

6.4.6 Drive-away lock

INFORMATION

The drive-away lock can be activated as described in the following, only if the "drive-away lock" parameter has been set to "On". At the factory, the drive-away lock parameter is set to "Off". It is also possible that the factory setting has been changed to "On" if requested with the order or by the specialist dealer. Please ask your specialist dealer about the setting selected for your wheelchair.

The control unit of the ParaGolfer sports wheelchair features an electric drive-away lock to disable the driving function.

Activation

1. With the control unit turned on, press and hold the mode button for at least 5 seconds.

2. A short beep confirms activation of the drive-away lock (if the feature that returns a beep with each push of a button is activated, then the second beep confirms activation of the drive-away lock).
3. The control unit is turned off automatically.

Activation of the drive-away lock is indicated by the key symbol on the display.

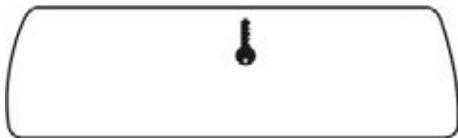


Fig. 17 Display of the drive-away lock on the control panel

Deactivation

1. Press the on/off button to turn on the control.
2. Move the joystick into the front end position until you hear a beep.
3. Move the joystick into the rear end position until you hear a beep.
4. Release the joystick.
5. Another beep confirms the successful deactivation of the drive-away lock.

6. The speed level and battery indicator appear on the LCD display, and the control unit is ready for driving. The drive-away lock is deactivated and the wheelchair can be driven.

INFORMATION

If the joystick sequence is not completed correctly, the drive-away lock remains engaged. Turn the control of the sports wheelchair off and on again and make another attempt at deactivating the drive-away lock.

6.5 Stand-up function

⚠ WARNING

Risk of injury due to improper use of the stand-up function. When the standing function is in use, there is the risk that the user tips out of the wheelchair. For this reason, the lower leg belt, lap belt and chest belt always have to be fastened when using the standing function (see section 6.3).

⚠ WARNING

Risk of injury if the ParaGolfer tips over. Creep speed is automatically activated when the standing function is operated. If this is not case, this represents a malfunction and the standing function must not be used. Visit a specialist dealer immediately to eliminate the fault.

⚠ WARNING

Risk of injury if the ParaGolfer tips over. The standing function may only be used on ground with an inclination of no more than **30 %**. For reasons of stability, the control reduces the speed (creep speed) as soon as the standing device is being used. With creep speed, only short distances can be covered, e. g. for manoeuvring.

⚠ CAUTION

Risk of injury if the sports wheelchair tips over. The sports wheelchair can tip over if the stand-up device is in the raised position and the user can fall out of the wheelchair. The following safety instructions must be followed under all circumstances:

- Do not lean out of the wheelchair when the stand-up device is in the raised position.

- Do not pull any objects towards you when the stand-up device is in the raised position.
- Only travel short distances when the stand-up device is in the raised position, e. g. when manoeuvring.
- The maximum load capacity of the stand-up device is **140 kg / 308 lbs.**
- Use the stand-up device only on solid level ground.
- Do not change the mounting position of the stand-up device and seating unit.

⚠ CAUTION

Risk of pinching in the stand-up zone. When using the stand-up function, there are shearing and crushing points in the area below the seat of the sports wheelchair. To avoid injuries, do not reach into the danger area. No unauthorised persons are permitted in the danger area. There must not be any interfering objects or obstacles in the stand-up area. There are crushing points caused by the construction in the area between the seat frame and wheelchair frame. Never let any parts of the body (hands, feet) or objects get under the raised seat. This risk of crushing must be explained to the attendants.

NOTICE

Risk of damage due to improper maintenance. Inspect the stand-up device for visible signs of damage at least once a month and ensure all screw connections are tight. Also maintain correct air pressure in the tyres.

NOTICE

Risk of damage during transportation. Always lower the stand-up device fully for loading or transportation.

INFORMATION

For optimal operation of the stand-up function, please also observe the instructions in the brochure "Richtig Stehen" (correct standing).

INFORMATION

Make sure a sufficient supply of power is maintained in order to guarantee proper operation of the stand-up device.

INFORMATION

The motor of the standing function is not intended for continuous use, but only for short-term limited operation (10 % use – 90 % idle time). General recommendation:

At a maximum load of **140 kg / 308 lbs**, **10** seconds of operation should be followed by approx. **90** seconds of idle time. In this regard, the stand-up device is considered independently of the driving function.

The drive function remains available even when the seat is raised to the standing position. As soon as the seat leaves the lowest position, the speed is reduced (creep speed) because the power wheelchair is less tip-resistant. This is indicated by a snail symbol on the control panel (see fig. 19).



Fig. 18 Display of the standing function



Fig. 19 Display of the creep speed on the control panel

6.6 Releasing and locking the brake

⚠ WARNING

Risk of death due to brake malfunction. Incorrect brake settings can lead to brake failure, and therefore to serious bodily injuries and even death. Repairs and adjustments to the brake may only be carried out by authorised specialist staff.

⚠ WARNING

Risk of accidents and injury due to lack of brake functionality.

- No brake functionality is available when the brake is unlocked (push mode).
- When moving the power wheelchair on an incline, the person pushing must provide the appropriate brake force.
- The brake function may only be released in the presence of an attendant.
- Should the user be unable to release the brake himself, the brake can be released by the attendant.

NOTICE

Risk of damage when parking the wheelchair with the brake deactivated. Releasing the brake may result in uncontrolled rolling of the sports wheelchair. Therefore, ensure that the brake is engaged after parking the sports wheelchair.

It is possible to push the sports wheelchair in case of control unit failure or insufficient battery charge level. To do so, the brake is released by mechanically unlocking it. The brake release mechanism is located on the right between the frame and seat.

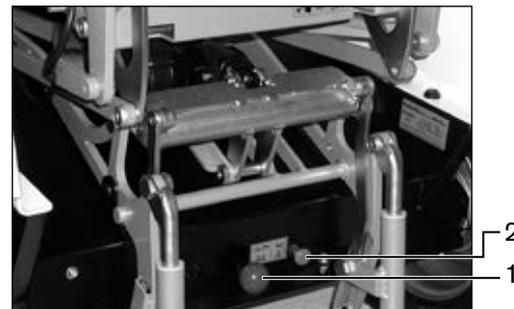


Fig. 20 Brake release lever and safety pin

1 Brake release bolt

2 Brake release lever

Releasing the brake

Pull the safety pin forward, and simultaneously pull the brake release lever up (see fig. 21).

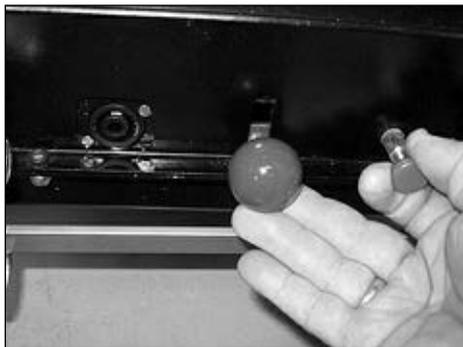


Fig. 21 Operating the brake release lever and safety pin

INFORMATION

When the joystick is activated while the brakes are unlocked, the control system emits an error signal on the control panel. If this is not the case, there is a malfunction that must be corrected immediately by a specialist dealer.

When the brake release lever is as far up as possible, the brakes are unlocked. In this position, the control unit will recognise that the brake has been released and automatically deactivate the driving function. As soon as the joystick is deflected, a warning will appear on the LCD display.

INFORMATION

Once the brake release lever has been released, all braking systems are deactivated.

INFORMATION

The control can be turned off while the wheelchair is being pushed.

Locking the brake

To activate the brake, push the brake release lever down until the brake release bolt engages. To reactivate the driving function, switch off the wheelchair controls and then switch them on again.

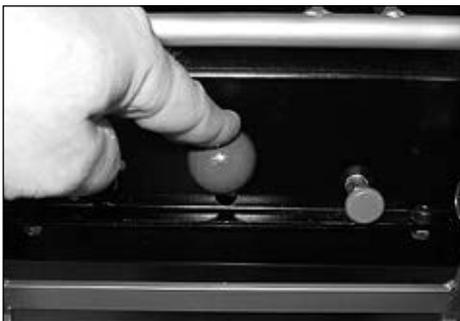


Fig. 22 Locking the brake

6.7 Batteries

NOTICE

Risk of damage to electronics due to short circuit. Always use insulated tools when doing any maintenance work on the batteries. Ensure correct polarity is used when connecting the battery cables.

INFORMATION

Before doing any maintenance work on the batteries, please read the warnings thoroughly!

The ParaGolfer is equipped with two rechargeable 12 V gel batteries with a capacity of 94 Ah (C5). The batteries are located underneath the seat, centred in the chassis of the ParaGolfer. They are maintenance-free.

6.7.1 Charging

⚠ WARNING

Risk of injury due to explosive gases. Explosive gases can develop while the batteries are charging. The following safety instructions must be followed under all circumstances:

- Ensure sufficient ventilation when charging the batteries in closed rooms.
- Smoking and fire are not permitted.
- Sparks must be avoided.
- Do not cover the air vents in the trim.

NOTICE

Battery damage due to deep discharge. Driving for longer periods with low battery charge level will result in deep discharge and damage to the battery. There is a risk that the power wheelchair may stop due to zero battery charge level and bring the user into a dangerous situation.

The range of the sports wheelchair is determined by the battery capacity. Many factors have an influence on the battery capacity. In addition to the temperature, age of the batteries and amount of use, the charging cycle has a pronounced effect on the capacity and therefore on the range.

The batteries reach their full capacity only after **20–30** charge cycles. Only if the full capacity of the batteries has been reached can the power wheelchair achieve the stated range.

At temperatures of **<0 °C/32 °F** the battery capacity drops by up to **35 %** in relation to the capacity for an outside temperature of **20 °C/68 °F**. This shortens the range of the power wheelchair accordingly. Moreover at low temperatures the charge level displayed on the control panel can differ more significantly from the actual battery capacity.

For an optimal charging frequency the following applies:

- The batteries can be charged at any time, regardless of the remaining charge.
- It takes about 10 hours until a discharged battery (lowest segment in the battery symbol is flashing) is completely charged. Subsequently, the sports wheelchair can remain connected to the charger without causing any problems; the charger has a programmed recharge

phase which ensures that maximum battery capacity is maintained.

- For daily use, it is recommended to connect the charger overnight so that the full capacity is available every day.
- If the wheelchair is not used for extended periods of time, the battery gradually discharges, and a charging cycle must therefore be carried out at least once a week to maintain the battery capacity. When the power wheelchair is not used for an extended period of time, we recommend removing the fuse.
- Total discharge of the batteries should be avoided.
- When the batteries are charging, the controls on the sports wheelchair must be switched off to allow the charging current to be fed completely into the battery.

The following must be observed when charging the batteries:

- The master switch must be on during the charging process.
- Only the battery charger provided by Ottobock for this purpose may be used for charging. Failure to comply with this requirement will render the warranty null and void.

- The voltage settings on the battery charger must correspond to the voltage used in your country.

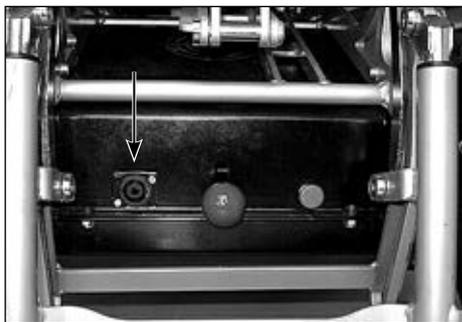


Fig. 23 Charging receptacle on the ParaGolfer

6.7.2 Battery charger

⚠ WARNING

Explosion hazard due to sparks. The battery charger must always be switched off and the mains plug be removed before disconnecting the battery.

NOTICE

Unauthorised battery replacement. The battery may only be replaced by a specialist dealer. The characteristic curve of the battery charger set at the factory corresponds to the battery provided and must not be changed. Setting the characteristic curve incorrectly can result in permanent damage to the battery.

NOTICE

Risk of damage to the battery charger / risk of damage caused by the battery charger. Please note the following when using the charger:

- Only use a battery charger supplied by Ottobock which has been tested and approved by Ottobock for the respective batteries (observe information on the charger). Failure to do so can result in a battery explosion and possible impairment of health due to contact with battery acid.
- The information on the nameplate of the charger must match the country-specific voltage of the respective power supply network.
- Only use the battery charger within the specified ranges of temperature and humidity.

- When setting the charger up close to a window, protect it against direct sunlight.
- Keep the charger from overheating. The vent openings on the back of the charger must not be covered.
- Turn off the control unit during the charging process so that the entire charging current flows to the battery.
- Avoid dust and dirt. Only clean with a dry cloth.

The battery charger is designed for gel batteries (maintenance-free batteries). The charger features the matching characteristic curve to ensure best possible charging of the respective kind of battery. Upon delivery of the sports wheelchair, the matching characteristic curve will be set at the factory. If the battery charger is to be used with another sports wheelchair or if new batteries are installed, the setting has to be checked.

To charge the batteries, proceed as follows:

1. Switch off the controls of the ParaGolfer.
2. Pull back the metal slide on the charging plug and connect the plug to the charging receptacle on the ParaGolfer. Turn the plug clockwise until it engages.
3. Connect the battery charger to a wall socket and turn it on. Charging will start automatically.
4. The current charge status is indicated on the LEDs of the battery charger (see Table 3).
5. When the charging process is complete, turn off the battery charger.
6. Disconnect the battery charger from the mains supply.
7. Pull back the metal slide on the charging plug, turn the charging plug counter-clockwise up to the stop and then pull it out.
8. Turn on the controls. The ParaGolfer is ready for driving.

INFORMATION

Please see the instructions for use supplied with the battery charger for further details on use and on the LED indicators

6.7.3 Battery disconnect switch

The ParaGolfer is equipped with a battery disconnect switch. This battery disconnect switch is located underneath the seat next to the fuse box.

If you pull off the battery disconnect switch, the entire control of the ParaGolfer is blocked.

To do so, turn the battery disconnect switch by 90° and then pull it off.

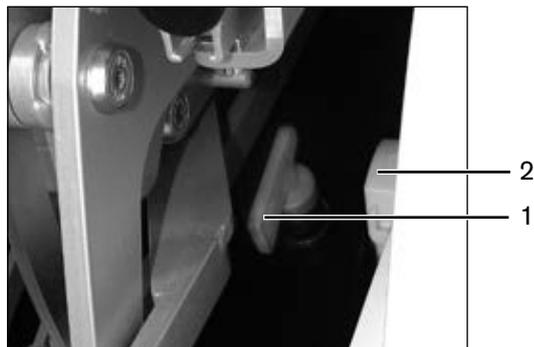


Fig. 24 Battery disconnect switch

- 1 Battery disconnect switch 2 Fuse box

7 Accessories

INFORMATION

Only original options provided by the manufacturer may be used. The optional components may be mounted only as described here. Failure to comply will void the warranty.

INFORMATION

All available optional components are contained on the order form and in the wheelchair accessories catalogue.

The complete range of options is listed on the order form and in the wheelchair accessories catalogue.

7.1 Golf bag holder

⚠ WARNING

Risk of injury if the ParaGolfer tips over. Attaching a golf bag will change the centre of gravity and thus the driving and braking characteristics of the ParaGolfer. In addition, the ParaGolfer will have larger dimensions. To avoid tipping and dangerous situations, driving with the different driving and braking characteristics must therefore be practiced first sufficiently.

⚠ WARNING

Risk of injury if the ParaGolfer tips over. Attaching a golf bag will change the loading proportion of the ParaGolfer. The weight of the user together with the weight of the golf bag must not exceed the permissible total weight of **140 kg / 308 lbs.**

The golf bag holder can be attached laterally to the ParaGolfer. First insert the two mounting brackets into the retainers bushes on the drive unit sustainer until they firmly engage with the locking pins (see fig. 25a/b).

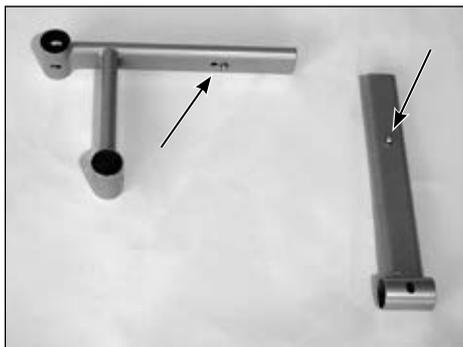


Fig. 25a Mounting brackets with locking pins



Fig. 25b Mounting brackets attached

The golf bag holder can now be inserted in the mounting brackets (see fig. 26). Again, it must be ensured that the locking pins engage.



Fig. 26 Golf bag holder attached

The golf bag is placed on the two supports and fixed with the straps. After loosening the clamping lever, the front support can be adapted to the length of the golf bag.

INFORMATION

The golf bag holder can be mounted on the right and left side of the ParaGolfer. The respective matching mounting bracket set is required for this purpose.

7.2 Control panel holder

⚠ WARNING

Risk of injury due to uncontrolled driving characteristics. If the control panel is not fully inserted, the driving behaviour of the wheelchair may get out of control. Therefore make sure when inserting the control panel to push the holding device (the spring plate) into the receiver up to the stop.

7.2.1 Control panel holder, swing-away in parallel

The manufacturer offers the option of equipping the power wheelchair with a removable control panel mounted on a swing-away control panel holder. This control panel holder makes it possible to drive the power wheelchair under a table or closer to an object.

The control panel holder can be rotated up to the armrest.

1. Apply slight pressure to push the control panel holder to the side.
→ The pivot element is unlocked.
2. Swing the control panel holder away to the side.

INFORMATION The pivot element locks in place again when the holder is rotated back to the original position.

Following control panel holder are available as an option:

- Control panel holder, standard, offset downwards
- Control panel holder, straight
- Control panel holder, swing-away, height-adjustable

7.3 Decorative rim covers

The rear wheels of the ParaGolfer can be equipped with decorative rim covers (see fig. 27). They are mounted with a central screw, which is screwed in using a size 6 Allen wrench.



Fig. 27 Decorative wheel cover

7.4 Rain cover for storage compartment

The storage compartment can be equipped with a rain cover (see fig. 28).



Fig. 28 Rain cover

8 Malfunctions/troubleshooting

CAUTION

Risk of accident and injury due to uncontrolled movements of the sports wheelchair. Uncontrolled movements can occur during the operation of the sports wheelchair as a result of malfunction. In this case, please contact your authorised dealer immediately.

INFORMATION

Should you encounter problems while troubleshooting or if you do not manage to completely eliminate a problem by following the measures described here, please contact your specialist dealer.

Malfunctions are indicated on the control panel LCD display. Table 4 describes the individual displays with the corresponding problem sources as well as the possible causes and measures to be taken.

If malfunctions cannot be fully corrected using the procedures described here, a specialist dealer has the ability to read the exact error code with a hand-held programming device and to carry out a targeted system analysis.

All problems that have ever occurred are saved in a list and can be retrieved, e.g. in case of a general overhaul of the sports wheelchair. The saved data can be used to determine future service and maintenance intervals, for example.

8.1 Warning

A warning indicates a status or malfunction of one or several components of the sports wheelchair. The function of components that have no errors is not restricted. For example, if the connection between the controller and seat motor malfunctions, this error will only be indicated if the user attempts to activate the motor. However, the driving function is still available.



8.2 Error

 **CAUTION**

Risk of injury if the sports wheelchair stops abruptly. In the event of communication problems in the bus system of the controls, the system triggers an emergency stop to prevent any uncontrolled functions. Depending on the source of the malfunction, it is possible to drive the wheelchair out of a danger zone after switching the control unit on again.

If the driving function is still not available after switching on the controls again, the brakes must be released (see section 6.6) to switch over to the pushing mode. After the sports wheelchair has triggered an emergency stop, contact a specialist dealer as soon as possible.

An error affects one or several functions of the sports wheelchair. The system is not fully operational until the error has been corrected.

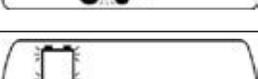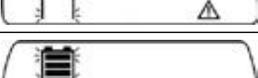Display symbol(s)	Warning/error	Cause / possible measure
	Controller temperature warning	Overheating due to excessive load Cool down phase
	Motor temperature warning	Overheating due to excessive load Cool down phase
	Joystick warning	Bring joystick to zero position before switching on
	Hand control device fault	Defective joystick
	Controller fault	Defective controller
	Communication error (alternating flashing signal)	Faulty connection between hand control device and controller/ defective cabling, software or hardware
	Battery under voltage	Battery deep discharge Charge as soon as possible
	Battery over voltage	Voltage too high (after full charge and driving downhill, for example)

Display symbol(s)	Warning/error	Cause / possible measure
	Stand-up device motor fault	Faulty cabling or plug connection; defective actuator
	Drive motor fault	Faulty cabling or plug connection; defective actuator
	Brake fault	Open brake release / defective brake
	Emergency stop	Severe fault caused by controller, hand control device, and / or drive motor malfunction

Table 3 Status and error messages

8.3 Defect/failure

A defect represents a serious failure of a system component. A failure is the most serious type of malfunction, which causes an immediate emergency stop of the system.

Defects / failures are shown by a continuous sequential light indicator with an audible signal.

After the malfunction has been corrected, the system is re-activated by restarting it.

9 Maintenance, cleaning and disinfection

9.1 Safety instructions

CAUTION

Insufficient maintenance. Severe user injuries, damage to the product due to failure to observe maintenance intervals.

- The power wheelchair may only be serviced by qualified personnel trained by the manufacturer. The functionality and operating safety of the power wheelchair must be verified and a service performed at least 1 x per year.
- In case of frequent user changes (children and growing youth) or changing clinical pictures, the power wheelchair should be inspected, adjusted and serviced 1 x every six months.

CAUTION

Uncontrolled driving behaviour, unexpected sounds or odours. Falling, tipping, collision with persons or nearby objects due to defects.

- If any faults, defects or other hazards that can lead to personal injury are detected, the product must be taken out of service immediately. This includes uncontrolled movements as well as sounds that are unexpected or previously not noted or odours that deviate significantly from the state of the product at the time of delivery.
- Please contact your authorised dealer.

NOTICE

Failure to inspect important product features. Damage to the product due to incorrect maintenance.

- Inspect the seat adjustment features for visible signs of damage at least 1 x per month and ensure all screw connections are tight.
- Maintain sufficient air pressure in the tyres. The correct tyre pressure is printed on the tyre casing and listed in the section "Technical data".

INFORMATION

Use the online spare parts catalogue on the Ottobock website ("Customers & Technicians" category) to order spare parts.

Only spare parts from Ottobock may be used. Failure to comply will render the warranty null and void.

INFORMATION

Contact your authorised specialist dealer if you encounter problems during maintenance. The operational safety of the power wheelchair must be checked by an authorised specialist dealer once a year.

9.2 Maintenance intervals

The correct function of the sports wheelchair should be checked every time before using it. The items listed in table 5 must be checked by the operating company/institution or user at the indicated intervals.

Component	Activity	Prior to every use	Weekly	Monthly
Tyres	Tyre pressure (see technical data)			X
	Sufficient tread depth			X
Electronics	Control system free of errors Battery charger does not show any error messages on the LEDs	X	X	
Batteries	Check battery charge level	X		
Brake	Activate brake lever while control unit is switched on Check for active brake function while the brakes are locked	X	X	
Back upholstery, belts and padding	Visual inspection, proper condition of seams, belts, and pads		X	
Wear parts, e.g. bearings, rails	Visual inspection			X
Screw and bolt connections, including wheel attachment	Check firm seating			X

Table 4 Maintenance measures and intervals

9.3 Changing the fuse

The fuse is located in a fuse holder laterally underneath the seat, next to the left drive wheel (see fig. 29). To replace the fuse, open the cover cap of the fuse holder, replace the fuse and close the cap again until you feel it engage.

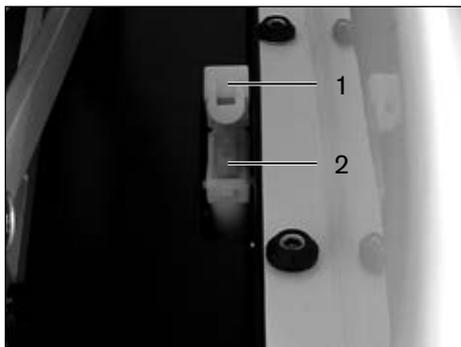


Fig. 29 Fuse holder with fuse

1 Cover cap, open

2 Main fuse, inserted

9.4 Wheel replacement

NOTICE

Risk of damage due to uncontrolled movements. When jacking up, secure the wheelchair to prevent it from slipping or tilting to the side by placing a suitable base under the drive unit support.

NOTICE

Risk of damage to tyres due to excessive tyre pressure. Make sure that the tyre pressure indicated in section "Technical data" is not exceeded.

INFORMATION

Direct sunlight (UV light) causes the tyres to age prematurely. As a result, the tread surface hardens and corner pieces break out of the tread.

INFORMATION

Avoid parking the wheelchair outdoors whenever possible. Regardless of wear and tear, replace the tyres every 2 years.

If the wheelchair is parked for an extended period of time or the tyres overheat (e.g. in the vicinity of radiators or in case of exposure to strong sunlight behind glass), the tyres may become permanently deformed. Therefore, sufficient clearance should always be maintained from sources of heat, and the wheelchair should be moved frequently or placed on blocks for storage.

If defective, both front drive wheels and the rear wheel can be replaced.

To disassemble a drive wheel, first loosen the central screw in the middle of the wheel using a size 5 Allen wrench (see fig. 30) to remove the decorative rim cover, if present. Then loosen the six wheel nuts with a size 10 Allen wrench (see fig. 31). Now you can pull off the wheel.



Fig. 30 Wheel cover



Fig. 31 Wheel nut

To disassemble the rear wheel you have to loosen the three axle screws using a size 6 Allen wrench. Now you can pull off the wheel.

9.5 Cleaning and care

NOTICE

Risk of damage to electronics due to water penetration. In order to avoid malfunctions, the electronic components, motors and batteries must not come into contact with water when cleaning the sports wheelchair.

NOTICE

Risk of damage to sports wheelchair components. Do not use any aggressive cleansing agents, solvents or hard brushes for cleaning the sports wheelchair. The sports wheelchair must never be cleaned using a water jet or high-pressure cleaning apparatus.

The sports wheelchair must be cleaned at regular intervals, depending on the amount of use and the degree of soiling.

The control panel, battery charger, armrests and cover can be cleaned with a damp cloth and mild cleaning solution. Use a dry brush to clean the seat and back upholstery as well as the seat cushion.

Use a damp plastic brush to clean the wheels and frame.

9.5.1 Disinfection

Wipe all parts of the wheelchair down with disinfectant.

Important information about disinfection

- Water based disinfectants should be used. Observe the usage information provided by the manufacturer.
- Prior to disinfection, clean the seat and back upholstery, seat cushion, control panel, and armrest.

10 Disposal

INFORMATION

If a wheelchair is to be disposed of, all components and materials of the sports wheelchair must be recycled or disposed of properly.

If the sports wheelchair is no longer in use, it must be disposed of properly in accordance with national regulations.

Please return defective batteries to your dealer when buying new ones.

11 Information on re-use

The ParaGolfer sports wheelchair is suitable for re-use.

Similar to second-hand machines or cars, products that are being re-used are subject to increased strain. Features and functions must not change in a way that could endanger patients or other persons within the product's life cycle.

Based on market observations and the current state of technology, the manufacturer has calculated that the ParaGolfer sports wheelchair can be used for a period of 5 years, provided that it is used properly and that the service and main-

tenance instructions are observed. Periods during which the product is stored at the dealer or with the third party payer are not included in this period. It should be clearly pointed out, however, that the ParaGolfer is a reliable product far beyond this defined period of time, provided that it is cared for and maintained appropriately.

In cases of re-use, the corresponding product must first be thoroughly cleaned and disinfected.

Subsequently, the product must be examined by an authorised specialist to check the condition, wear and possible damages.

Any worn and damaged components as well as components which do not fit or are not suitable for the new user must be replaced.

The service manual includes a service schedule for each model, detailed information and a list of the required tools.

12 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

12.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

12.2 CE Conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

The product meets the requirements under the RoHS Directive 2011/65/EU of the European Parliament and of the

Council of 8 June 2011 on the restriction of the use of certain hazardous substances in electrical and electronic equipment.

12.3 Warranty

Further information on the warranty terms and conditions can be obtained from the qualified personnel that has fitted this product or the manufacturer's service (see inside back cover for addresses).

12.4 Trademarks

All product names mentioned in this document are subject without restriction to the respective applicable trademark laws and are the property of the respective owners.

All brands, trade names or company names may be registered trademarks and are the property of the respective owners.

Should trademarks used in this document fail to be explicitly identified as such, this does not justify the conclusion that the denotation in question is free of third-party rights.

13 Technical data

Dimensions and weights	
Seat width	460 mm / 18.1" (can be reduced with lateral pads)
Seat depth	360 – 510 mm / 14.2" – 20.1" (25 mm / 0.98" increments)
Seat height	520 – 670 mm / 20.5" – 26.4" (continuous)
Seat inclination	0°
Armrest height	150 mm / 6"
Armrest length	260 mm / 10.2"
Lower leg length	430 – 560 mm / 16.9" – 22" (continuous) in combination with the seat height
Backrest height	290 – 350 mm / 11.4" – 13.8" (continuous)
Back angle	+5°
Total width	900 mm / 35.4"
Max. total height	Depending on the backrest setting
Total length	1530 mm / 60.2" (when travelling forward) 1490 mm / 58.7" (transportation size with the caster swivelled under the cover)
Turning radius	1900 mm / 74.8"
Tyre size	
Front:	390 mm / 16" (16x6.50-8)
Rear:	275 mm / 11" (11x6.00-5)

Dimensions and weights	
Tyre pressure	
Front:	3.9 bar
Rear:	4 bar
Empty weight	180 kg/396 lbs
Max. load capacity	140 kg/308 lbs
Necessary surface load bearing capacity	130 kg/m ² / 26.6 lbs/sq ft

Corrosion protection	
Corrosion protection	Coated frame

Electrical installation	
Operating voltage	24 V
Batteries: Gel batteries	2 x 12 V, 94 Ah (C5)
Control unit:	
Model	enAble50
Operating voltage	24 V DC
Max. output current per motor	100 A
Fuse	120 A
Battery charger	For more information, see the included battery charger instructions for use

Electrical installation	
Approx. distance range	35 km/ 21 miles

Driving data	
Speed	10 km/h/ 6.2 mph
Climbing ability	17° or 30 %
Maximum obstacle height	100 mm/ 3.9"
Wading depth	100 mm/ 3.9"

Drive (DC motor)	
Voltage	24 V DC
Nominal power	550 W
Nominal speed	162 U/min
Step-down gear	18.7 : 1
Maximum torque	130 Nm
Protection type	IP54
Operating temperature range	-15 °C to +40 °C/ +5 °F to +104 °F
Transport and storage temperature range	-15 °C to +40 °C/ +5 °F to +104 °F

Brake	
Voltage	24 V DC
Power consumption	20 W
Braking torque	5.5 Nm

* 3-point turn by 180°

** The specified weights vary according to the selected options and model.

*** The product meets all requirements under ISO 7176-14.

**** Depending on the individual, the continuous climbing ability can be significantly lower than the value specified.

***** The specified range was determined under defined conditions according to ISO 7176-4. In practice the range can be reduced by up to 50 %. For information on this, see the section "Range" in the instructions for use (user).

***** The braking distance can be correspondingly longer due to user weight, installed options and condition of the tyres, and due to weather and surface conditions.

Technical changes reserved.

Table 5 Technic data

Kundenservice/Customer Service

Europe

Otto Bock HealthCare
Deutschland GmbH
37115 Duderstadt · Germany
T +49 5527 848-3433
F +49 5527 848-1460
healthcare@ottobock.de

Otto Bock Healthcare
Products GmbH
1110 Wien · Austria
F +43 1 5267985
service-admin.vienna@otto-
bock.com

Otto Bock Adria d.o.o. Sarajevo
71000 Sarajevo
Bosnia-Herzegovina
T +387 33 255-405
F +387 33 255-401
obadria@bih.net.ba

Otto Bock Bulgaria Ltd.
1612 Sofia · Bulgaria
T +359 2 80 57 980
F +359 2 80 57 982
info@ottobock.bg

Otto Bock Suisse AG
6000 Luzern 16 · Suisse
T +41 41 455 61 71
F +41 41 455 61 70
suisse@ottobock.com

Otto Bock ČR s.r.o.
33008 Zruč-Senec
Czech Republic
T +420 377825044
F +420 377825036
email@ottobock.cz

Otto Bock Iberica S.A.
28760 Tres Cantos (Madrid)
Spain
T +34 91 8063000
F +34 91 8060415
info@ottobock.es

Otto Bock France SNC
91978 Courtaboeuf Cedex
France
T +33 1 69188830
F +33 1 69071802
information@ottobock.fr

Otto Bock Healthcare plc
Egham, Surrey TW20 0LD
United Kingdom
T +44 1784 744900
F +44 1784 744901
bockuk@ottobock.com

Otto Bock Hungária Kft.
1135 Budapest · Hungary
T +36 1 4511020
F +36 1 4511021
info@ottobock.hu

Otto Bock Adria d.o.o.
10431 Sveta Nedelja · Croatia
T +385 1 3361 544
F +385 1 3365 986
ottobockadria@ottobock.hr

Otto Bock Italia Srl Us
40054 Budrio (BO) · Italy
T +39 051 692-4711
F +39 051 692-4720
info.italia@ottobock.com

Otto Bock Benelux B.V.
5253 RC Nieuwkuijk
The Netherlands
T +31 73 5186488
F +31 73 5114960
info.benelux@ottobock.com

Industria Ortopédica
Otto Bock Unip. Lda.
1050-161 Lisboa · Portugal
T +351 21 3535587
F +351 21 3535590
ottobockportugal@mail.telepac.pt

Otto Bock Polska Sp. z o. o.
61-029 Poznań · Poland
T +48 61 6538250
F +48 61 6538031
ottobock@ottobock.pl

Otto Bock Romania srl
077405 Chitila, Jud. Ilfov
Romania
T +40 21 4363110
F +40 21 4363023
info@ottobock.ro

OOO Otto Bock Service
143441 Moscow
Region/Krasnogorskiy Rayon
Russian Federation
T +7 495 564 8360
F +7 495 564 8363
info@ottobock.ru

Otto Bock Scandinavia AB
60114 Norrköping · Sweden
T +46 11 280600
F +46 11 312005
info@ottobock.se

Otto Bock Slovakia s.r.o.
851 01 Bratislava 5
Slovak Republic
T +421 2 32 78 20 70
F +421 2 32 78 20 89
info@ottobock.sk

Otto Bock Sava d.o.o.
34000 Kragujevac
Republika Srbija
T +381 34 351 671
F +381 34 351 671
info@ottobock.rs

Otto Bock Ortopedi ve
Rehabilitasyon Tekniği Ltd. Şti.
34387 Mecidiyeköy-İstanbul
Turkey
T +90 212 3565040
F +90 212 3566688
info@ottobock.com.tr

Africa

Otto Bock Algérie E.U.R.L.
Mackie-Ben Aknoun · Alger
DZ Algérie
T +213 21 913863
F +213 21 913863
information@ottobock.fr

Otto Bock Egypt S.A.E.
Mohandessein · Giza · Egypt
T +20 2 37606818
F +20 2 37605734
info@ottobock.com.eg

Otto Bock South Africa (Pty) Ltd
Johannesburg · South Africa
T +27 11 564 9360
info-southafrica@ottobock.co.za

Americas

Otto Bock Argentina S.A.
Ciudad Autónoma de
Buenos Aires · Argentina
T +54 11 5032-8201 / 5032-8202
atencionclientes@
ottobock.com.ar

Otto Bock do Brasil
Tecnica Ortopédica Ltda.
CEP: 13.278-181, Valinhos-SP
Brasil
T +55 19 3729 3500
F +55 19 3269 6061
ottobock@ottobock.com.br

Otto Bock HealthCare Canada
Burlington, Ontario, L7L 5N5
Canada
T +1 289 288-4848
F +1 289 288-4837
infocanada@ottobock.com

Otto Bock HealthCare
Andina Ltda.
Bogotá · Colombia
T +57 1 8619988
F +57 1 8619977
info@ottobock.com.co

Otto Bock de Mexico S.A. de C.V.
C.P. 01180 México, D.F. · Mexico
T +52 55 5575 0290
F +52 55 5575 0234
info@ottobock.com.mx

Otto Bock HealthCare
Minneapolis, MN 55447 · USA
T +1 763 553 9464
F +1 763 519 6153
usa.customerservice
@ottobockus.com

Asia/Pacific

Otto Bock Australia Pty. Ltd.
Baulkham Hills NSW 2153
Australia
T +61 2 8818 2800
F +61 2 8814 4500
healthcare@ottobock.com.au

Beijing Otto Bock
Orthopaedic Industries Co., Ltd.
Beijing, 100015, P.R. China
T +8610 8598 6880
F +8610 8598 0040
news-service@ottobock.com.cn

Otto Bock Asia Pacific Ltd.
Kowloon, Hong Kong · China
T +852 2598 9772
F +852 2598 7886
info@ottobock.com.hk

Otto Bock HealthCare
India Pvt. Ltd.
Mumbai 400 021 · India
T +91 22 2274 5500 / 5501 / 5502
information@indiaottobock.com

Otto Bock Japan K. K.
Tokyo, 108-0023 · Japan
T +81 3 3798-2111
F +81 3 3798-2112
ottobock@ottobock.co.jp

Otto Bock Korea HealthCare Inc.
137-070 Seoul · Korea
T +82 2 577-3831
F +82 2 577-3828
info@ottobockkorea.com

Otto Bock
South East Asia Co., Ltd.
Bangkok 10900 · Thailand
T +66 2 930 3030
F +66 2 930 3311
obsea@ottobock.co.th

Other countries

Otto Bock HealthCare GmbH
37115 Duderstadt · Germany
T +49 5527 848-1590
F +49 5527 848-1676
reha-export@ottobock.de

Ihr Fachhändler/Your specialist dealer:



Otto Bock Mobility Solutions GmbH
Lindenstraße 13 · 07426 Königsee-Rottenbach/Germany
www.ottobock.com

