14Y4 Skeo Up Daily Assist

DE Gebrauchsanweisung	5
EN Instructions for use	8
FR Instructions d'utilisation	11
IT Istruzioni per l'uso	14
ES Instrucciones de uso	17
PT Instruções de utilização	20
NL Gebruiksaanwijzing	23
SV Bruksanvisning	27
DA Brugsanvisning	29
NO Bruksanvisning	32
FI Käyttöohje	35
PL Instrukcja użytkowania	38
HU Használati útmutató	41
CS Návod k použití	44
RO Instrucțiuni de utilizare	47
HR Upute za uporabu	50
SL Navodila za uporabo	53
SK Návod na použitie	56
BG Инструкция за употреба	59
TR Kullanım kılavuzu	62
EL Οδηγίες χρήσης	65
RU Руководство по применению	69
UK Інструкція щодо використання	72
JA 取扱説明書	75
ZH 使用说明书	78
KO 사용 설명서	80

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INFORMATION

Date of last update: 2024-08-07

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Instruct the user in the safe use of the product.
- ▶ Please contact the manufacturer if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

14Y4 Skeo Up Daily Assist is a passive hand prosthesis made of silicone. It is put on like a liner and enables the user to hold objects. Objects **up to a thickness of 4 cm** (e.g. toothbrushes, razors) can be clamped between a silicone contour, which corresponds to the four fingers, and an inserted plastic clamp adapter.

The product is particularly suitable for use in wet areas, for example, for personal hygiene.

INFORMATION: Colour deviations (dots or fine stripes) may occur during the manufacture of the product. This does not impair the quality of the product.

2 Intended use

2.1 Indications for use

The product is to be used solely for upper limb exoprosthetic fittings.

2.2 Area of application

Recommended for residual limb lengths of 10 cm and up. The product is available in the sizes (circumference in mm) =140, =160, =180, =200 and =220. Do **not** use the product for compression therapy.

The product is intended for use in wet areas with a daily lifetime of **2 hours**.

2.3 Lifetime

The product is a wear part, which means it is subject to normal wear and tear.

2.4 Indications

- Transradial amputation or dysmelia
- Unilateral or bilateral amputation

2.5 Contraindications

- Osseointegration
- Skin diseases, skin injuries, inflammation, prominent scars with swelling, reddening and hyperthermia in the treated body region

3 General safety instructions

CAUTION! Risk of injury and risk of product damage

- ▶ Check the product for damage and readiness for use prior to each use.
- ▶ Do not use the product if its functionality is restricted. Take suitable measures (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop).
- ▶ Make sure the product is applied properly and fits correctly. Improper or excessively tight application can cause pressure points and constriction of blood vessels and nerves.
- ▶ Do not continue to use the product if it does not fit properly.
- ▶ Keep the product away from open flames, embers and other sources of heat.
- ▶ Do not expose the product to temperatures above +80 °C and below -20 °C.

CAUTION! Skin irritation, formation of eczema or infections due to contamination with germs

- ▶ The product may only be used by one person; reuse on another person is prohibited.
- ▶ Observe the cleaning instructions.
- ▶ Only use the product on intact skin.

4 Scope of delivery

Quantity	Designation	Reference number
1	Instructions for use	-
1	Skeo Up Daily Assist	-
1	Clamp adapter	9S405=1

5 Preparing the product for use

5.1 Size selection

- 1) Measure the residual limb circumference **2 cm to 3 cm** proximal to the end of the residual limb.
- 2) Select the corresponding size (see table at the beginning of the document).

5.2 Trimming

> **Recommended tools:** 719S7 trimming scissors

- ▶ Trim the product for the user (see fig. 1). Make sure the cut edges are smooth so the material does not tear. The lines on the product can be used as a contour.

INFORMATION: Leave the product as long as possible without it causing discomfort to the user. This ensures a large surface area that rests on the skin.

6 Use

CAUTION

Use of powder (e.g. baby powder or talcum)

Skin irritation on the residual limb and loss of function of prosthetic components due to clogging with particles or deprivation of lubricant

- ▶ Keep the product away from powders.

INFORMATION

Wear the product directly on the skin. Fabrics between the skin and the product can cause skin irritation.

If an allergic reaction occurs, do not continue using the product and contact your doctor.

6.1 Putting on and taking off

NOTICE! Do not bring the product into contact with sharp-edged objects (e.g. jewellery or watches) and do not overstretch the silicone, in order to prevent damage to the product.

First use

When used for the first time, wear the product for a maximum of 30 minutes. Then inspect the residual limb for constriction or marks. If no problems occur, wear it for longer during each subsequent use.

Donning

- > **Prerequisite:** The product is dry, clean and undamaged.
- > **Tip:** Wet the residual limb with water to make it easier to put on.
- 1) Turn the silicone material inside out or roll it up (see fig. 2).
- 2) Place the product on the residual limb and align it, then push the end of the residual limb in (see fig. 3).
- 3) Unroll the silicone material over the residual limb without moving the soft tissue or creating air pockets (see fig. 4). Briefly lift the proximal edge to relieve skin tension.

Doffing

- 1) Turn the silicone material inside out and take the product off of the residual limb. In order to do so, carefully pull on the flap (see fig. 5, see fig. 6).
- 2) **Recommendation:** Clean the product (see page 10).

6.2 Inserting/removing the clamp adapter

The clamp adapter is inserted into the opening provided in the product. Its shape and adhesion to the silicone keep it in position. It can be easily pulled out for cleaning or replacement. The alignment of the clamp adapter can be freely selected depending on the desired function.

- 1) Insert the clamp adapter into the product (see fig. 7, see fig. 8).
- 2) Slowly pull out the clamp adapter to remove it.

6.3 Cleaning and Care

NOTICE

Use of improper cleaning agents

Damage to the product due to use of improper cleaning agents

- ▶ Only clean the product with the approved cleaning agents.

INFORMATION

If you perspire a lot, clean your skin and the liner/prosthesis more often. Wearing a prosthesis requires daily intensive skin care. Ottobock recommends the Derma care system for this.

- 1) Clean the inside and outside of the product with warm water, a cloth and neutral soap (e.g. 453H10=1 Ottobock Derma Clean) (see fig. 9, see fig. 10).
- 2) Rinse off soap thoroughly with clear water.
- 3) Insert a towel into the product and let it dry lying flat, e.g. on a drying rack.

NOTICE! To avoid material damage, do not place the product on a radiator or directly in the sun.

7 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

7.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregarding the information in this document, particularly due to improper use or unauthorised modification of the product.

7.2 CE conformity

The product meets the requirements of Regulation (EU) 2017/745 on medical devices. The CE declaration of conformity can be downloaded from the manufacturer's website.

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